ENGLISH

ProSep™ **Separating Agent**

Prior to use, carefully read the instructions for use

ProSep is a latex liquid intended for use with ProVeneer™.
Because it peels off, the use of ProSep will have no effect on the properties of ProVeneer.

HOW TO USE

1. Brush on a thin layer of ProSep on the relevant teeth.
2. Let dry (air or heat will speed up drying time). When dry, color will be clear.
3. Proceed with preparation of ProVeneer Preview Veneer according to IFU.
Once cured, feel comfortable to remove the preview-veneer without any concern.

REMOVAL OF PROSEP

Thoroughly clean any residue of ProSep left on teeth.
After Preview Veneer was set in the microwave, ProSep is easily peeled off by using an excavator.

STORAGE

Store in cool, dry place. Do not refrigerate.

SHELF LIFE

The lot number & expiry date are indicated on the product. Do not use after expiry date.
*If ProSep thickens as a result of water evaporating over time, simply add a few drops of distilled water back and shake.

CAUTION

This product contains natural rubber latex which may cause allergic reactions.

PRECAUTION

For Dental use only.
Further safety instructions: see SDS, available online at: www.silmetdental.com

WARRANTY

Silmet Ltd. will replace product that is proven to be defective. Silmet Ltd. does not accept liability for any damage or loss, direct or consequential, arising from the use of or inability to use the product described. It is the responsibility of the dentist to determine before use, the suitability of the product for its intended use. The dentist assumes all risk and liability in connection therewith.

SILMET Ltd.
12 Hassadna st., Or-Yehuda, 6022011, Israel
Tel: 972-3-7353000 Fax: 972-3-5331581 info@silmet.co.il; www.silmetdental.com
Diadent 2 bis chemin du Loup, 93290 Tremblay-en-France
Tel: 33-1-45913182 Fax: 33-1-48656723

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