GB

	Page
Hinge Rails (Powered & Manual)	2
Rail preparation and fixing	18
Carriage Preparation and fixing	26
Footrest Fitting	32
Chair fitting	34
Hand Change - 260 Chair	38
Rail Furniture	42
Radio Programming	48
LED Fault Diagnosis	52
Wiring Diagram 2609001	58
Wiring Diagram 2609011	59
Connection Diagram 2609106	60
Test Certificate	61
Test & Inspection Methods	62
Demonstration and Handover	68
Installation Tools & Special Tools	74
Warning Labels	80

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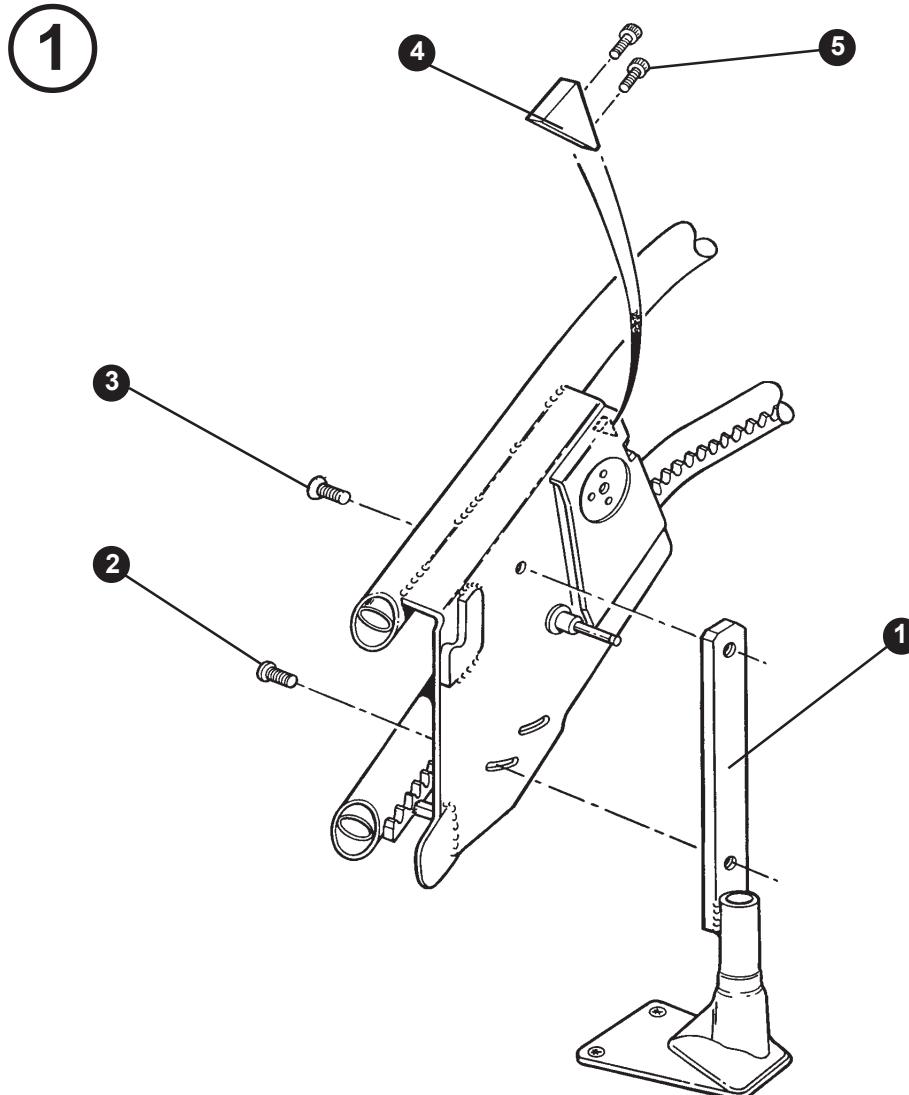
**GB****Preparation:**

- Position and secure the rail assembly referring to the details on the contract drawing. See Figure 1.
- Fit the hinge leg support (1) and secure using screws (2) & (3).
- Position and secure the hinge stop (4) to the fixed section of the hinge using screw (5) in the centre of the channel.

NOTE: Final adjustment to be carried out on completion of installation.

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
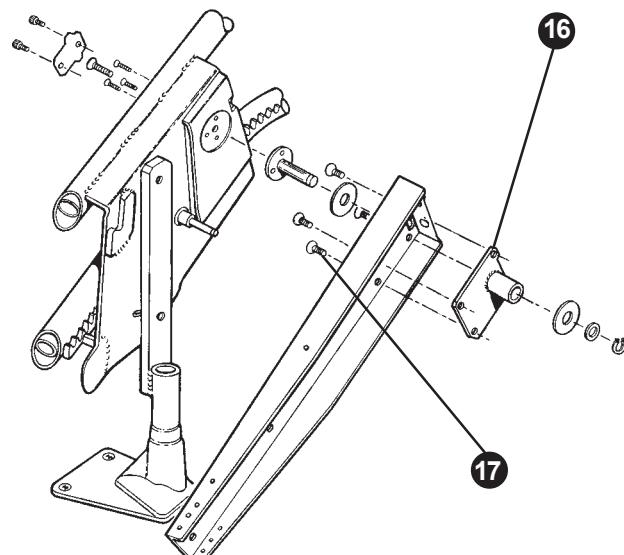
2B

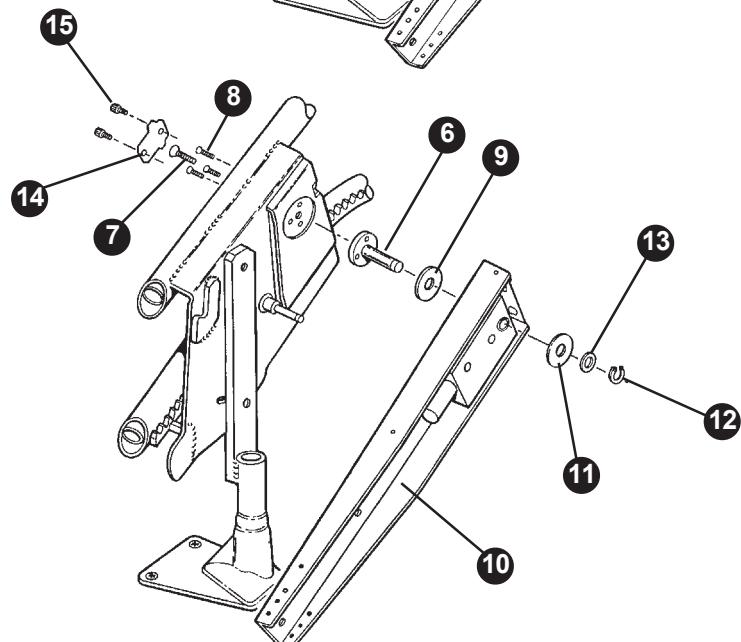GB

- Secure the hinge pivot pin (6) to the fixed section using screw (7) to a torque of 40Nm and three screws (8) to a torque of 10Nm.
- Fit the hinge pivot washer (9).
- Fit the hinge arm assembly (10) to the hinge pivot.

Note: Hinge arm fitted with dummy motor (16) and screws (17) for manual hinge refer to Figure 2A and for hinge arm fitted with motor for powered hinged refer to Figure 2B.

- Fit the washer (11) and the circlip (12) to the hinge pivot shaft.
- ⚠** Fit packing washers (13) as required to minimise lateral movements.
- Fit hinge screw lock plate (14) using screws (15).

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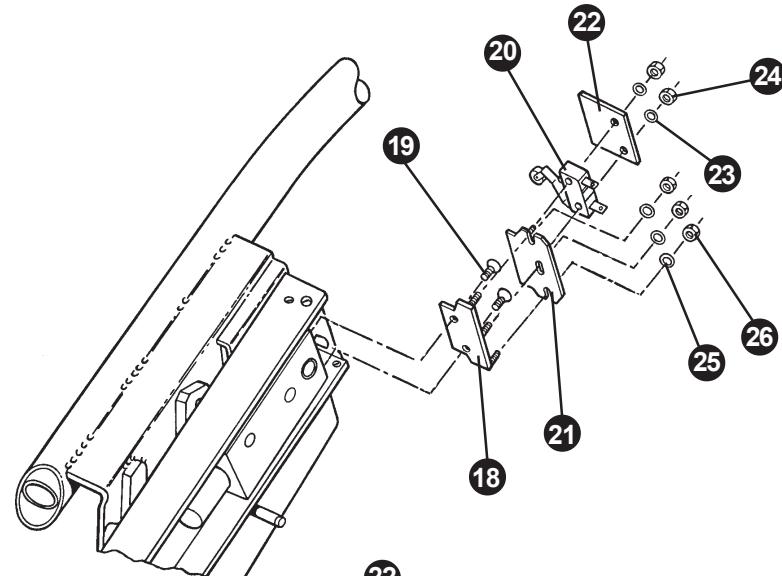
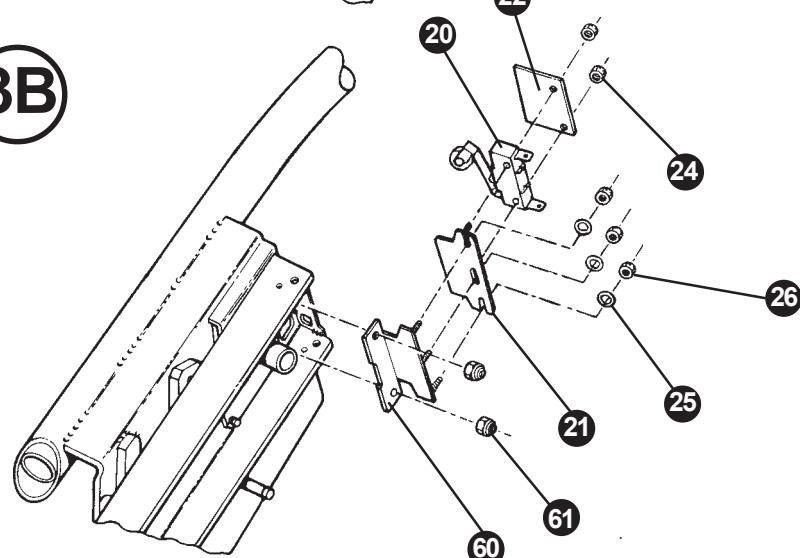
3A**3B****GB**

Powered Hinge

- Refer to Figure 3A.
- Fit the hinge arm safety switch mounting bracket (18) to the motor using screws (19).
- Fit the switch (20) to the switch mounting plate (21) using the switch guard (22), washers (23) and nuts (24).
- Fit the above assembly to the switch mounting bracket (18) using washers (25) and nuts (26).

Manual Hinge

- Refer to Figure 3B.
- Fit the hinge arm safety switch mounting bracket (60) to the dummy motor using nuts (61).
- Fit the switch (20) to the switch mounting plate (21) using the switch guard (22), washers (23) and nuts (24).
- Fit the above assembly to the switch mounting bracket (18) using washers (25) and nuts (26).

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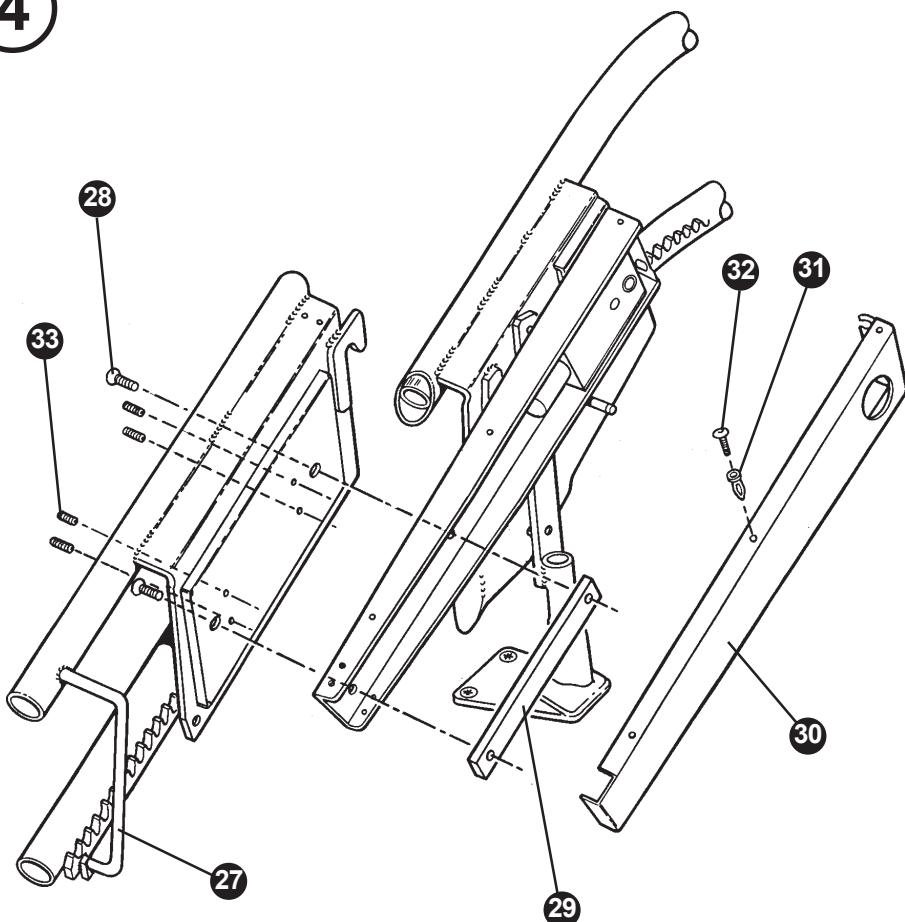
4

**GB**

- Position and secure the lower hinge section (27) to the hinge arm using screws (28) and hinge arm lock plate (29).
- Route 2 core cable (charge cable) to lower tube section.
- Fit hinge arm cover (30) and secure with fixings (31) & (32) after final adjustment of switch (20).
- Using hinge set screws (33) adjust as required to ensure correct alignment of the hinge and joint sections.

NOTE: Use G clamps and inverted piece of rack as required to temporarily align and secure hinge. (See page 8 for range of adjustments).

Screws (28) to be tightened to a torque of 40Nm.

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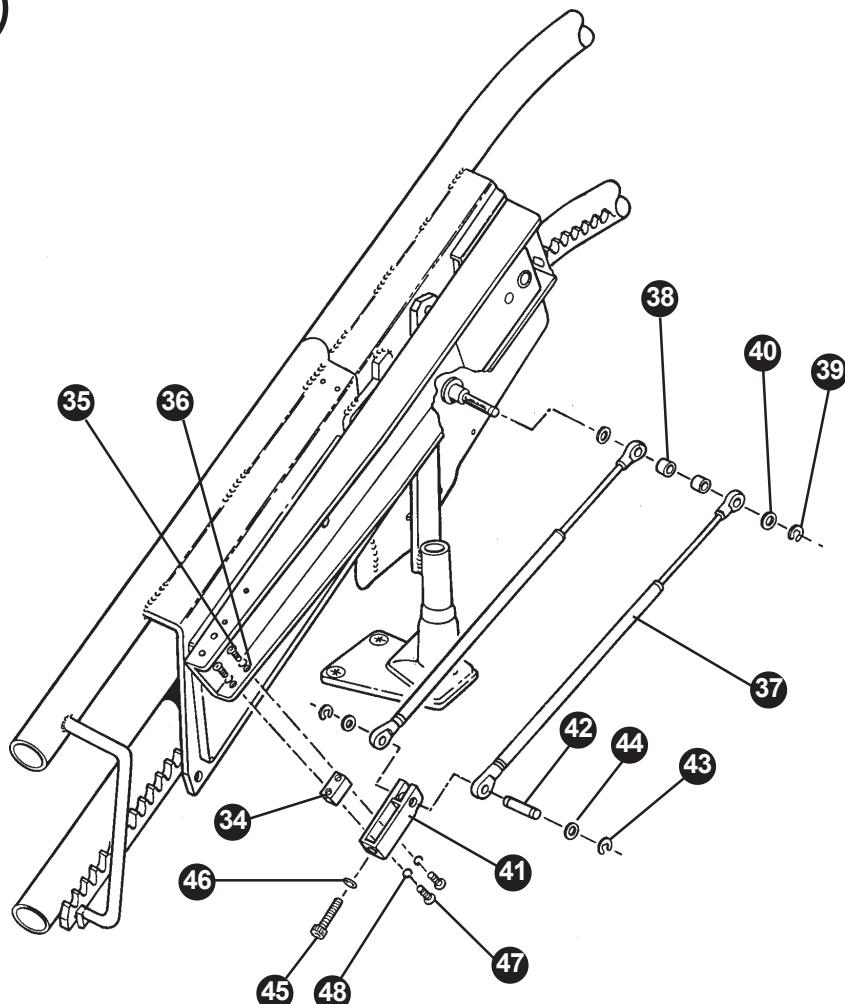
5

GB

- Lift the hinge assembly into the vertical position.
- Position and secure the gas spring compressor block (34) using screws (35) and spring washers (36).
- Fit the two gas springs (37) to the pin on the fixed section of hinge using the spacer (38) to separate the gas springs. Secure in place using circlip (39) and washer (40).

NOTE: Fit gas springs as shown in Fig 5.

- Fit the top sections of the two gas springs with the compressor assembly (41) and secure with the pivot pin (42) using circlips (43) and washers (44).
- Position and align the compressor assembly (41) over gas spring compressor block (34).
- Fit screw (45) and washer (46) and adjust as required.
- Secure compressor assembly (41) using screws (47) and washers (48).

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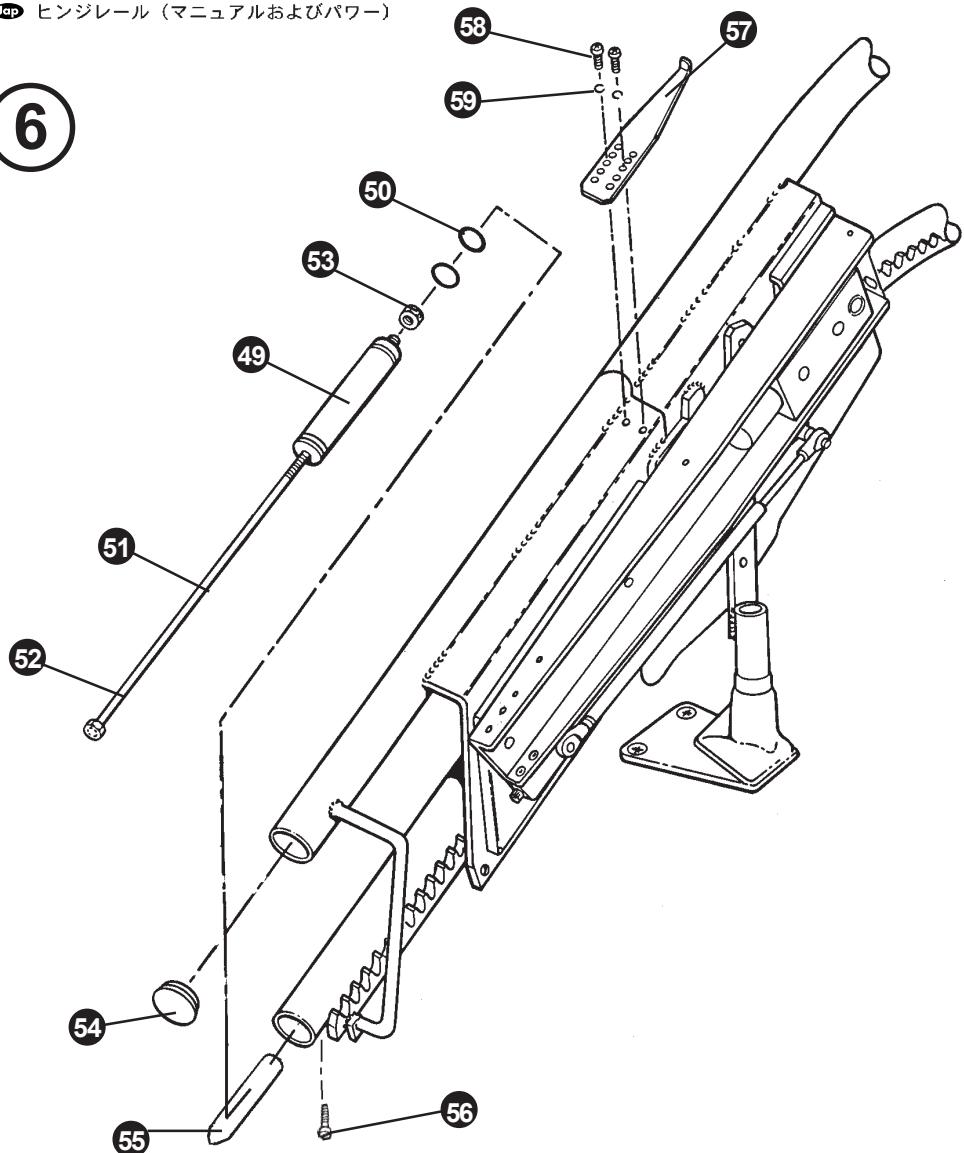
6

GB

- Temporarily fit counterbalance weight (49) without 2 x O rings (50) to the upper or lower tube section.

NOTE: Final adjustment of the counterbalance weight can be carried out using studding (51) and nut (52) prior to commissioning where the 2 x O rings (50) and securing nut (53) should be fitted on completion.

- 2 x rail end bungs (54) to be fitted on completion of adjustment.
- For hinges over 600mm fit hinge support leg (55) and tighten securing screw (56) and one rail end bung (54).
- Fit hinge safety bracket (57) using screws (58) and washers (59). Refer to chart for correct holes and angle of rail.

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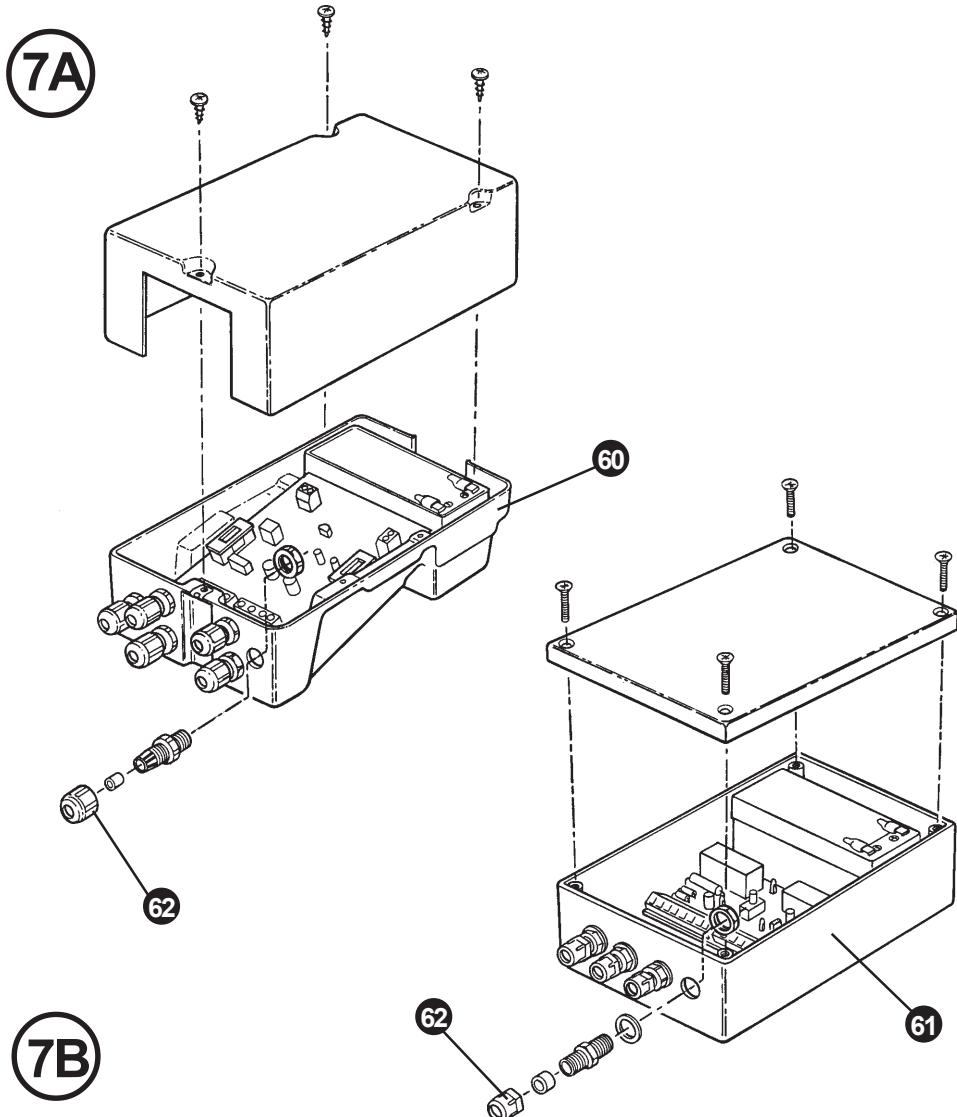
GB

- Carry out electrical wiring by referring to Electrical Connection Diagrams:
2609206 for Easy Loop (Fig 7A) powered hinge
2609230 for Easy Loop (Fig 7A) manual hinge
2609245 for Radio (Fig 7B).
- Terminate all wiring to appropriate junction box, (60) Figure 7A easy loop, (61) Figure 7B for radio.
- Finally secure using glands (62) as appropriate.
- Switch on and carry out functional tests of all operations.
- Make necessary adjustments as required and recheck.
- Fit and secure remaining rail furniture as per standard installation instructions.
- Position and affix relevant warning labels.
- Commission stairlift to BS 5776.
- Hand over and demonstration.
- Complete paperwork.

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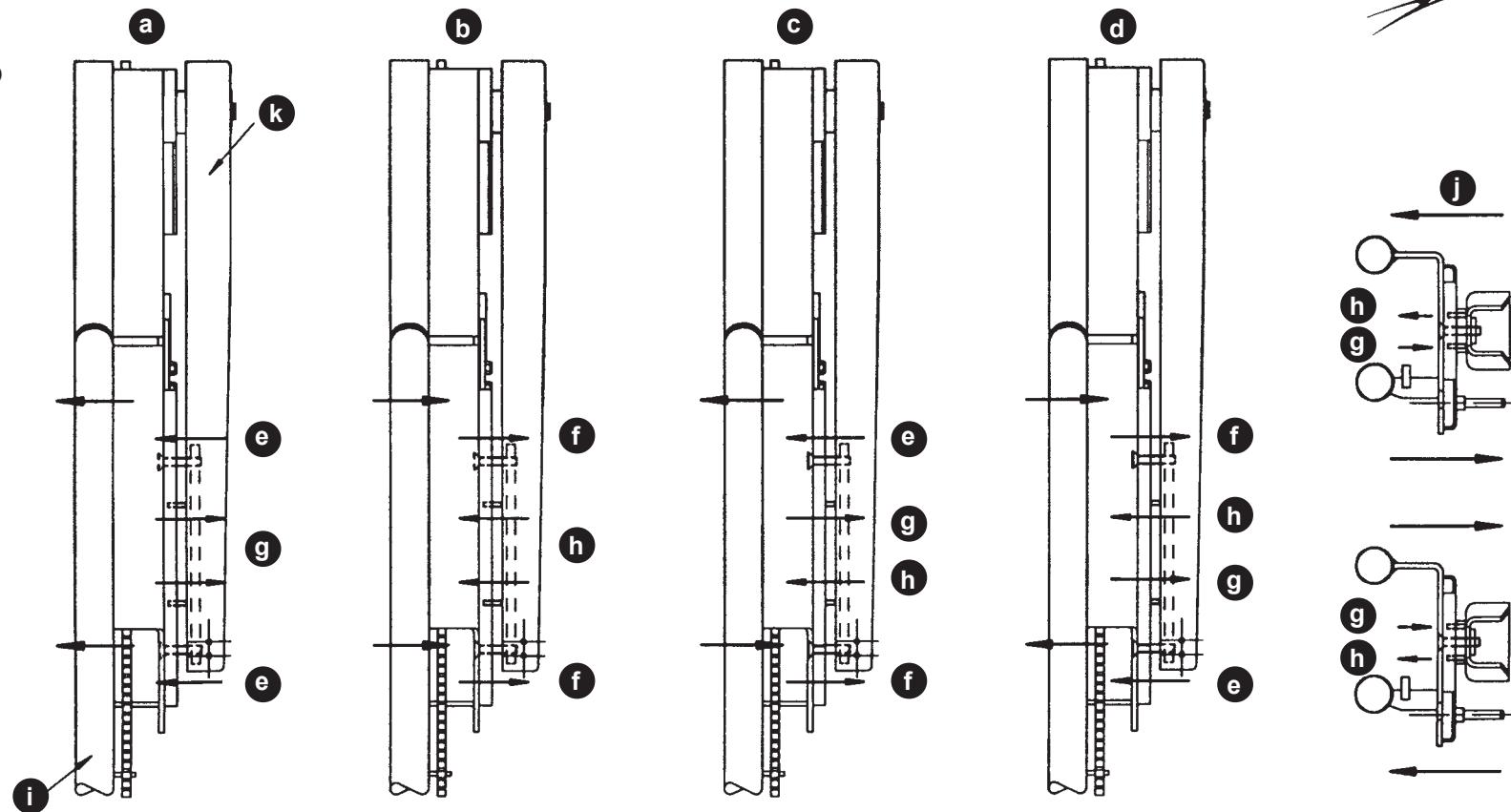
(8)

**GB**

- a To move hinge moving section away from hinge arm
 b To move hinge moving section closer to hinge arm
 c To move top of moving section away from hinge arm and bottom of hinge moving section closer to hinge arm
 d To move bottom of moving section away from hinge arm and top of hinge moving section closer to hinge arm
 e Loosen main screws
 f Tighten main screws
 g Tighten grub screws
 h Loosen grub screws
 i Hinge moving section
 j Adjusting rail tilt
 k Hinge arm

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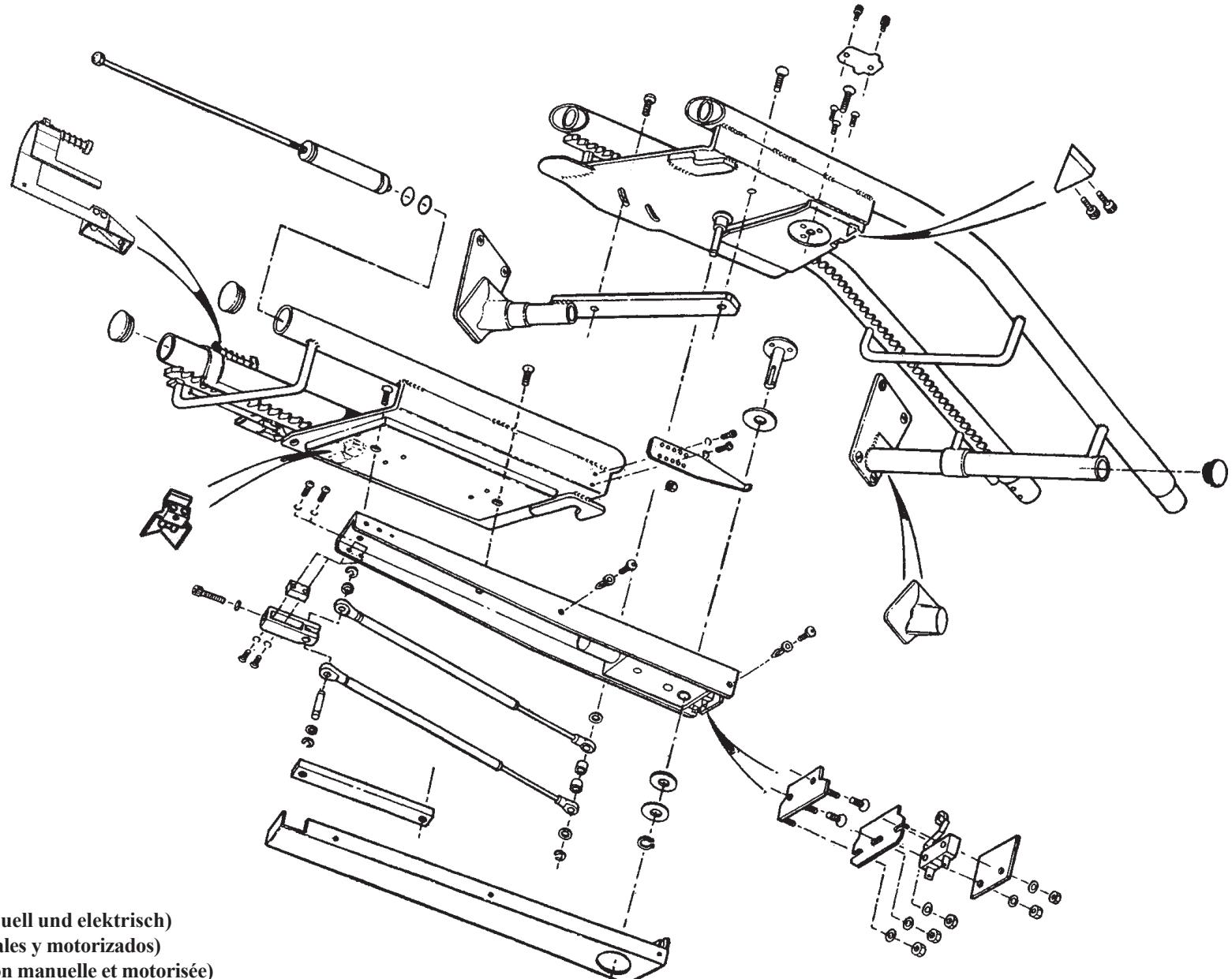
① Fit 2 joint pins on both rails for hinge joints

Fit 1 joint pin for all other rail joints

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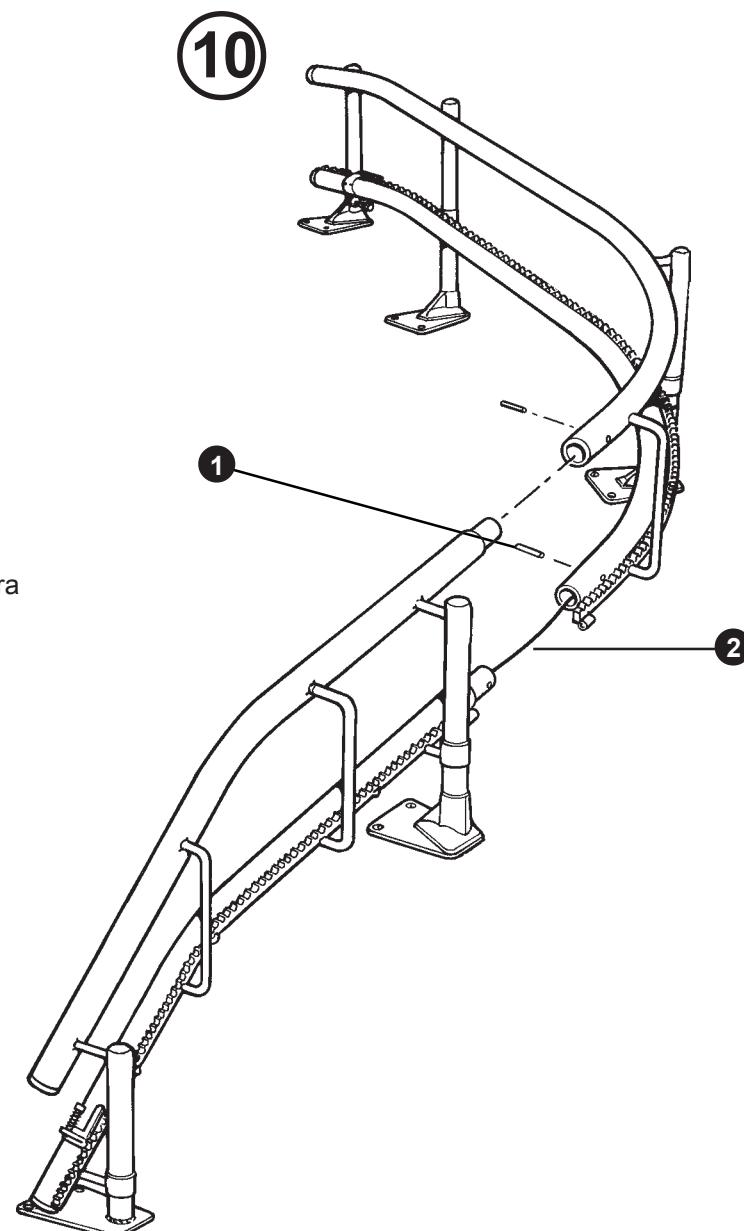
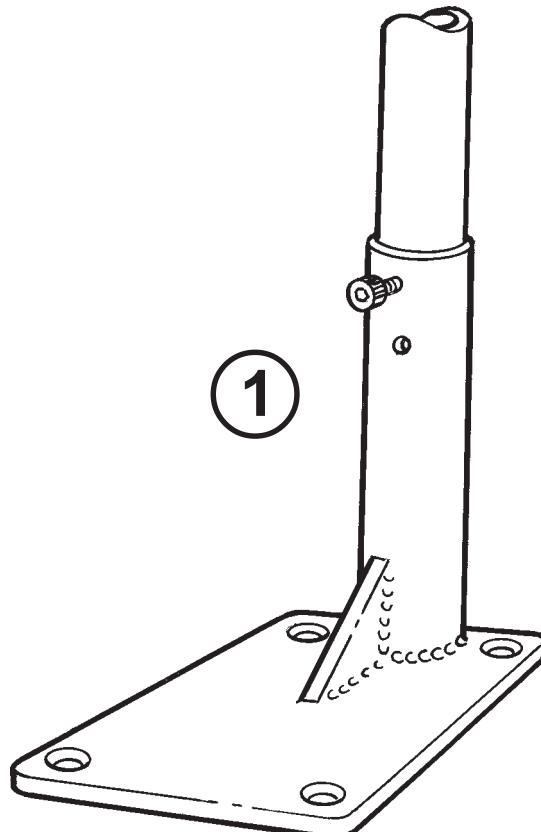
GB

- Remove packaging, protective wrapping and identification tags from rail sections.
- Discuss carpet cutting with customer if necessary.
 - . If carpet and underlay thickness combined is greater than 15mm, then the support leg must be fitted directly to the stair treads.
- Fix footplates to rail support legs using screws provided (Figure 1).

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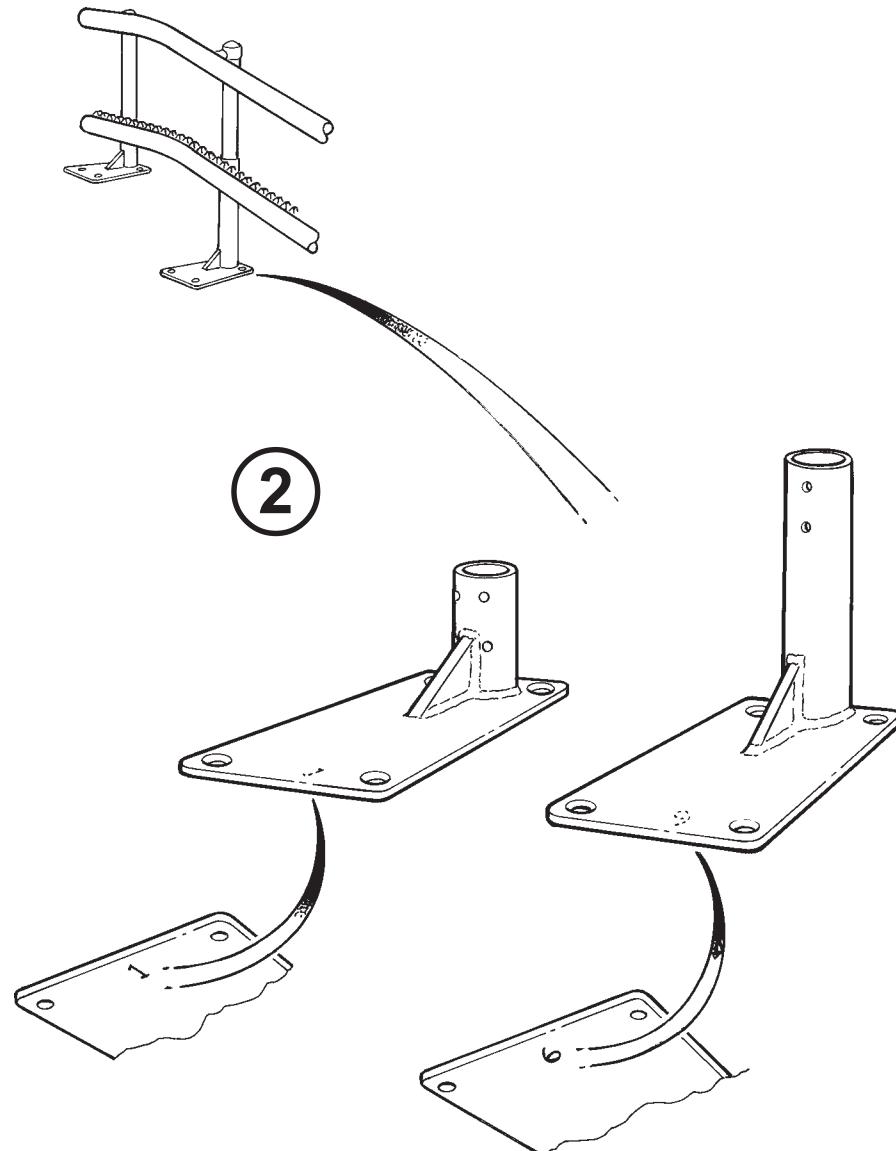
GB

- Footplates are numbered (Figure 2).
- Set individual leg heights as per table on Installation Drawing.
- Position rail sections on staircase as per Installation Drawing.
- Rail sections are marked with the Contract Number and a letter on the racking.
i.e.
Bottom Section Job Number A
Second Section Job Number B
Third Section Job Number C
- Thread 2-core cable through lower tube of each section of rail, working from top to bottom.
- Ensure a minimum of 150mm of cable at each end.

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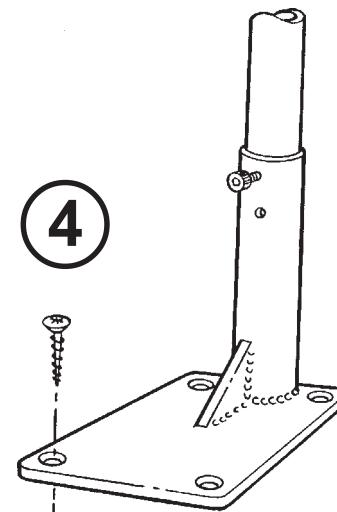
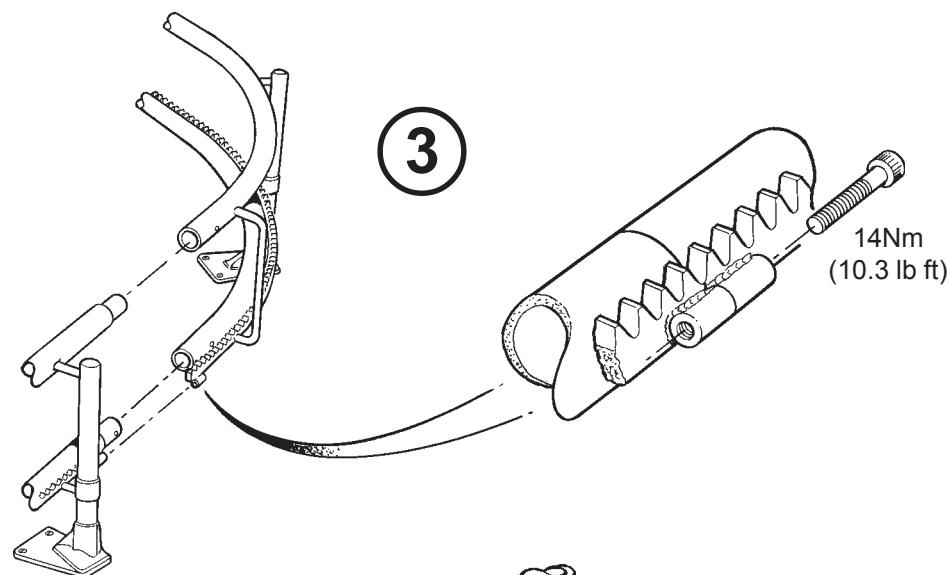
GB

- Assemble rail joints together working from bottom to top of stairs (Figure 3). Note: Offer top tubes first.
- Apply Superlube grease to male part of joint.
- Spirol pins should be flush with stair side of rail.
- Fit and tighten joint clamps (**torque 14Nm**) (10.3 lbf ft).
- Using the Installation drawing details:
 - Check top and bottom rail heights
 - Check rail angles
 - Check lower and upper rails are vertically in line.
- Adjust leg heights as required.
- Ensure rail to wall measurements match the Installation drawing.
- Secure plates to the staircase by the following method:
 - Loosen screw to leg (Figure 1)
 - Fix foot-plate with 4 screws (Figure 4)
 - Tighten screw to leg.
 - Repeat for each leg.

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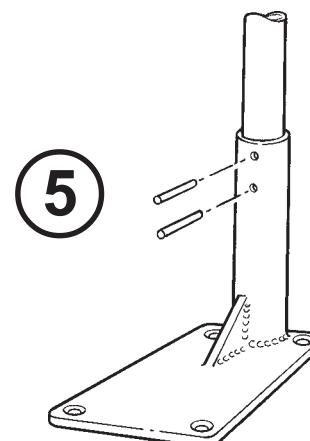
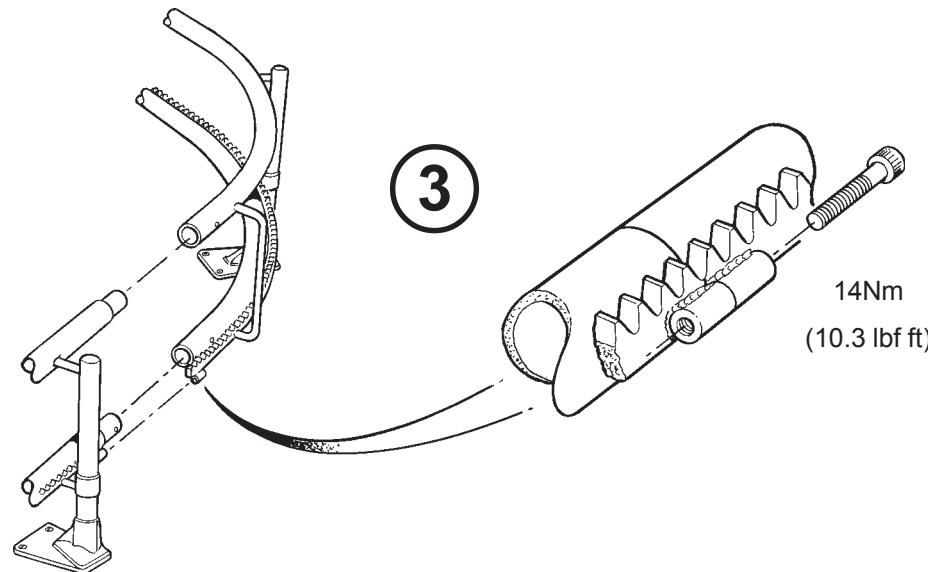
GB

The following fixings are recommended:

- Timber Staircase - 4 x 50mm screws.
- Concrete Staircase - 4 x 50mm screws and 4 x rawlplugs.
- Other Staircases - Fixings supplied with contract.
- Finally ensure all rail dimensions are in accordance with the rail installation drawing.
- Carry out any final adjustments as required.
- Drill and pin all rail supports (Figure 5).
 - Drill one hole with 6m drill bit.
 - Fit spirol pin.
 - Remove screw.
 - Drill second hole.
 - Fit spirol pin.
 - Remove burrs from back of holes with file.

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
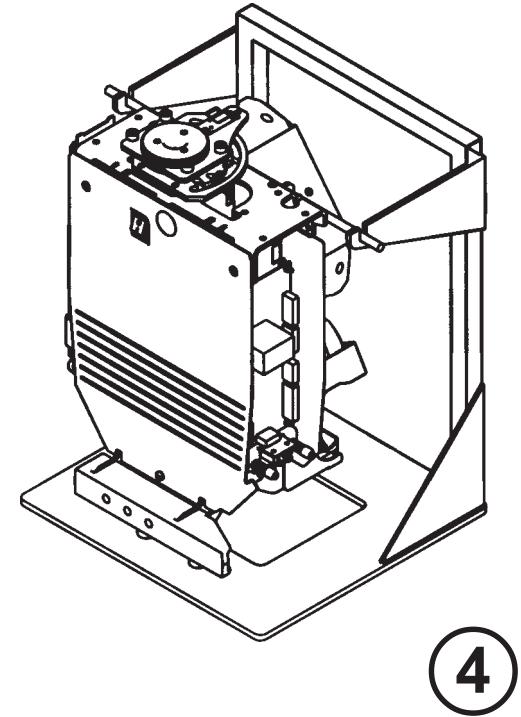
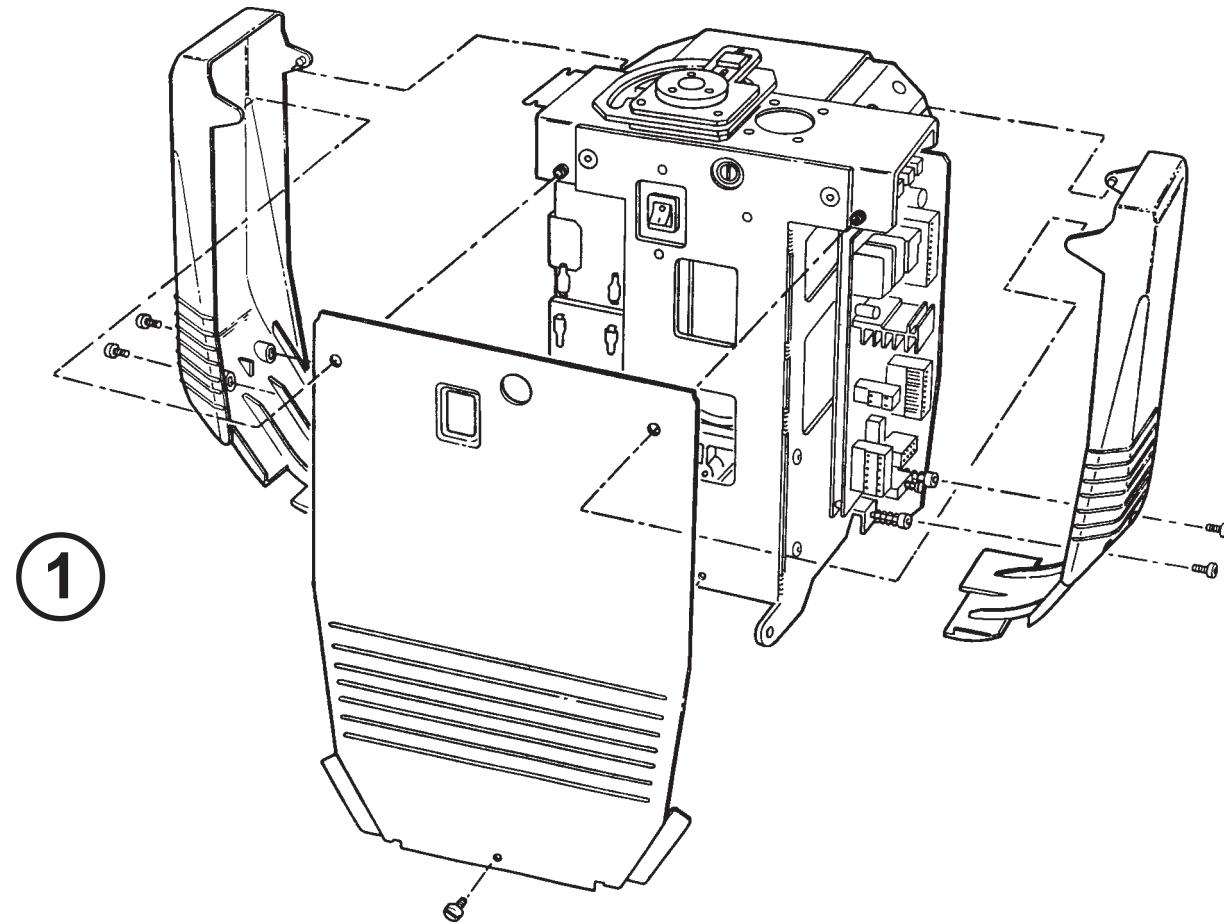
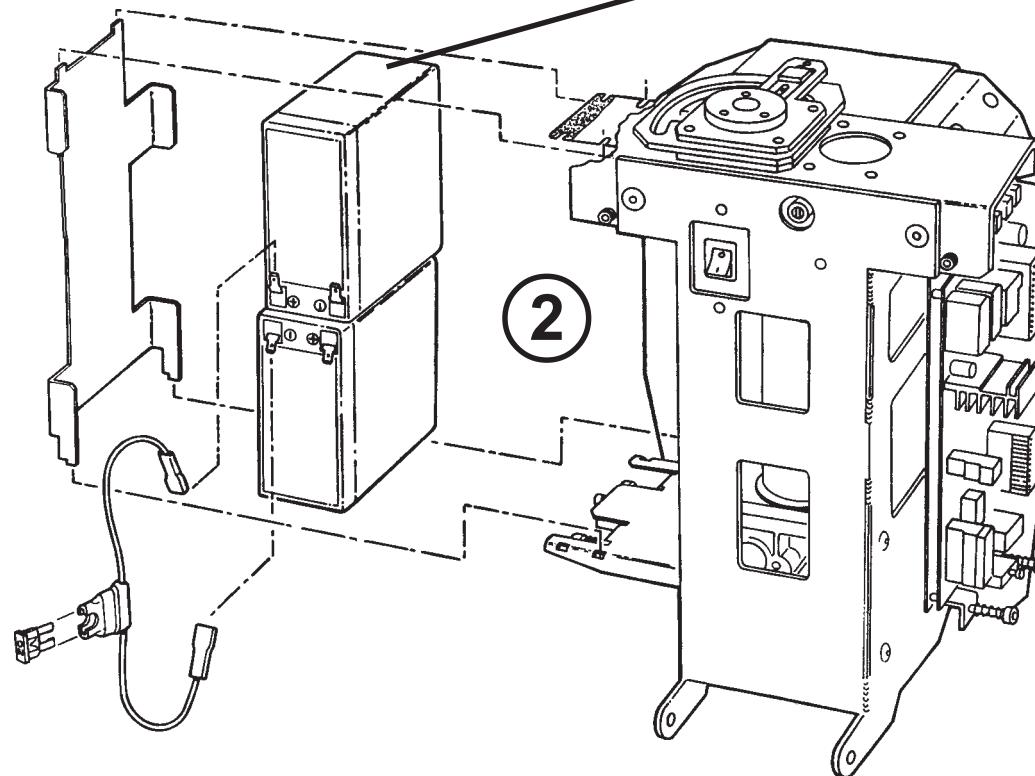
WARNING! BATTERIES MUST BE FITTED IN CARRIAGE AS SHOWN

- ACHTUNG! DIE BATTERIEN MÜSSEN AUF DIE ANGEGBENE WEISE IN DAS FAHRWERK EINGESETZT WERDEN
- ADVERTENCIA! LAS BATERIAS DEBEN INSTALARSE EN EL CARRO TAL Y COMO SE MUESTRA
- ATTENTION ! LES BATTERIES DOIVENT ETRE MONTÉES SUR LE CHARIOT COMME INDIQUE
- AVVERTENZA! LE BATTERIE DEVONO ESSERE MONTATE NEL CARRELLO COME ILLUSTRATO
- 警告! 電池は必ず図示のように キャリッジに取り付けること

- Anbringen und Befestigen des Fahrgestells
- Preparación e instalación del carro
- Préparation et fixation du chariot
- Preparazione e montaggio del carrello
- キャリッジの準備と取付け

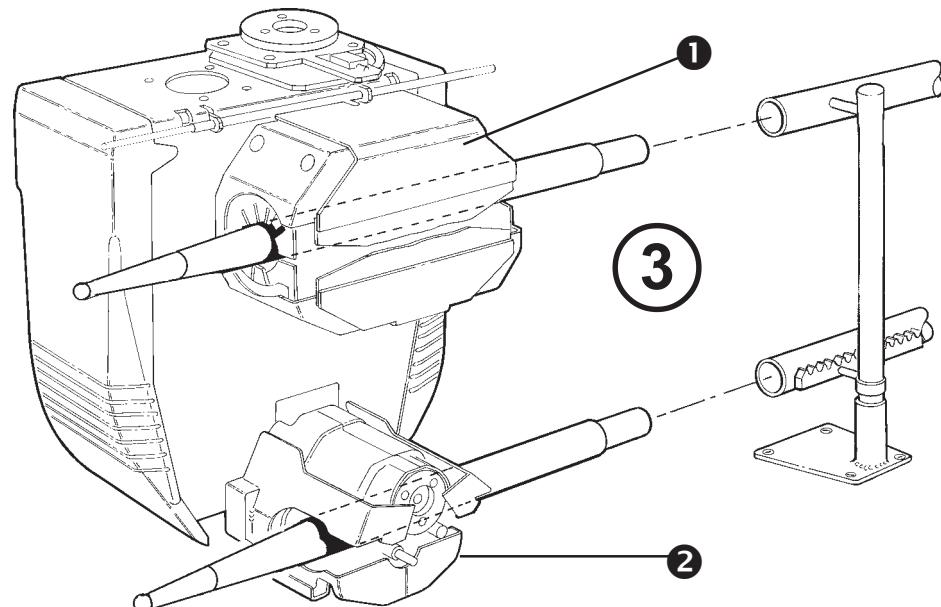- Anbringen und Befestigen des Fahrgestells
- Preparación e instalación del carro
- Préparation et fixation du chariot
- Preparazione e montaggio del carrello
- キャリッジの準備と取付け

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GB

The carriage should be carried by 2 people at all times.

- Remove packaging.
- Remove 2 main carriage covers - 2 screws each (Figure - 1).
- Remove front cover - 1 screw.
- Remove battery tray.
- Insert carriage lifting handle through footrest lever holes.
- Place carriage on service stand (Figure 4).
- Ensure battery isolator is switched off.
- Fit 2 batteries and fuse link, batteries **MUST** be fitted correctly (Figure 2 and Electrical Drawings 2609001 and 2609011 refers).
- Connect hand held controls to EP plug connector, removing existing plug fitted.
- Connect footrest link plug to CON AP.
- Switch on and test run. Switch off.

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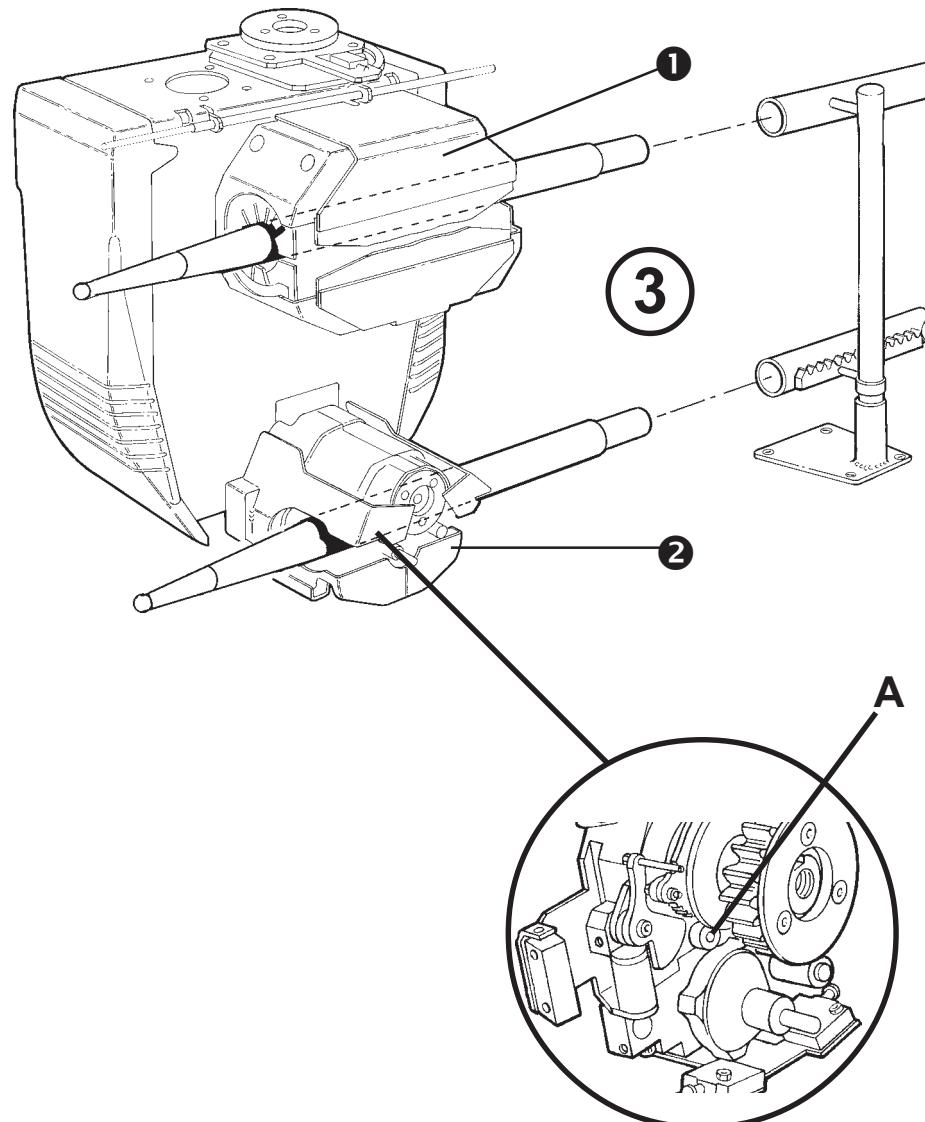
GB

- Place installation tube set bars through skate assembly and cluster assembly (Figure 3).
- Position carriage and insert bars into end of rail sections.
- Remove cable tie which is temporarily securing the skate assembly.
- Switch on and drive carriage to a convenient position on rail. Switch off.

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A Ensure the swinging arm assembly has not dropped behind the tyred roller assembly on the cluster.

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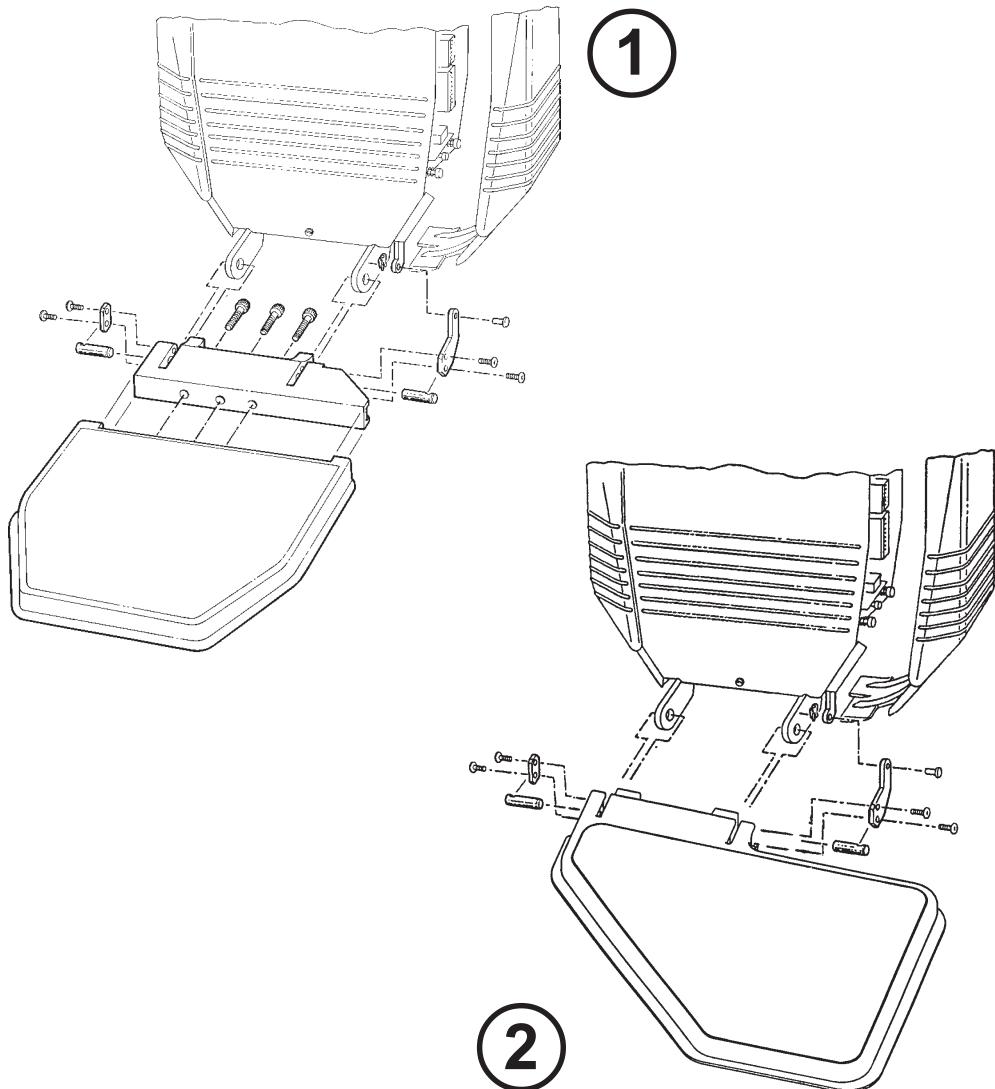
GB

Standard Footrest (Figure 1)

- Remove the safety pad extension cover.
- Fit footrest to carriage footrest block using 3 x bolts 35Nm (26 lbf ft).
- Fit safety pad extension to cover the lugs on the footrest block.
- Disconnect footrest link plug and connect footrest wiring loom.
- Remove carriage lifting handle.
- Place footrest lever through footrest lever holes.
- Fix lever to linkage arm using M5 x 15 btn head screws.

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Offset Footrest (Figure 2)

- Assemble the offset footrest as shown in Figure 2.

Note: The footrest mounting block already fitted to the carriage assembly is not required for offset footrest version and should therefore be removed from the carriage and discarded prior to assembly.

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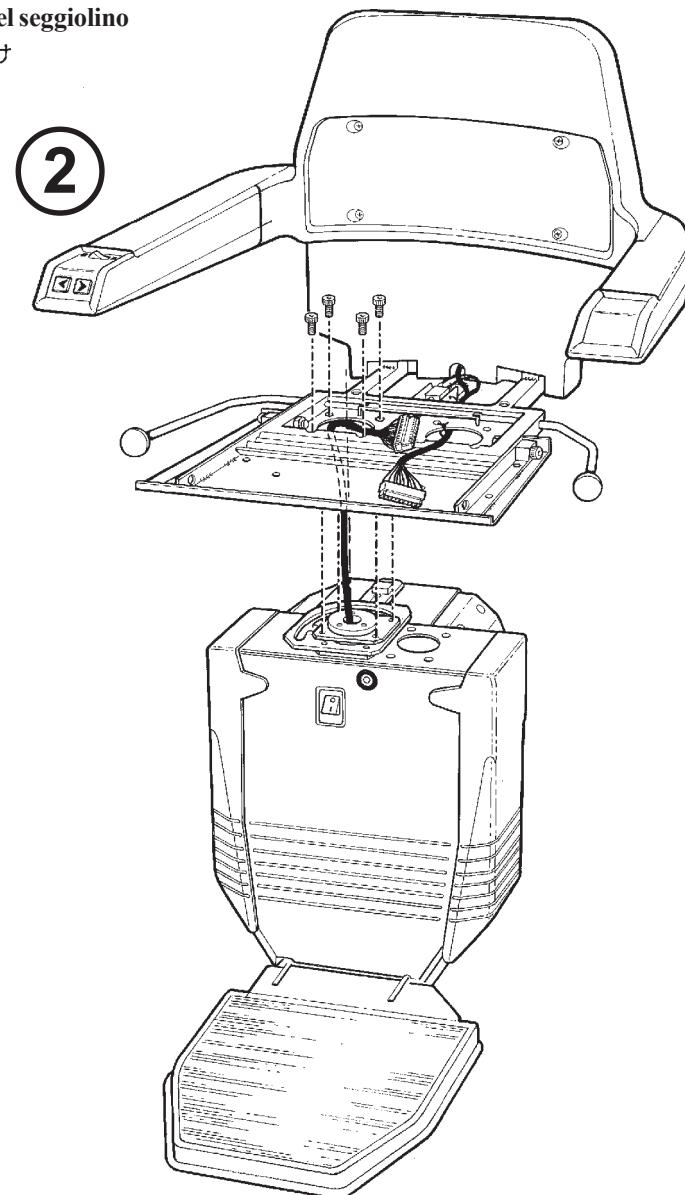
GB

- Remove key from chair.
- Fit chair to carriage boss assembly.
- Ensure swivel locking pin is located correctly in boss assembly.
- Secure the chair to boss with 4 x bolts 24Nm (17.71 lb fit) (Figure 2).
- Remove cable tie which is temporarily securing the swivel levers in the up position.
- Swivel chair and make sure chair locks securely in run, intermediate and swivel positions.
- Connect chair loom to carriage loom - ensure cable colours match (see Electrical Diagram 2609011).

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
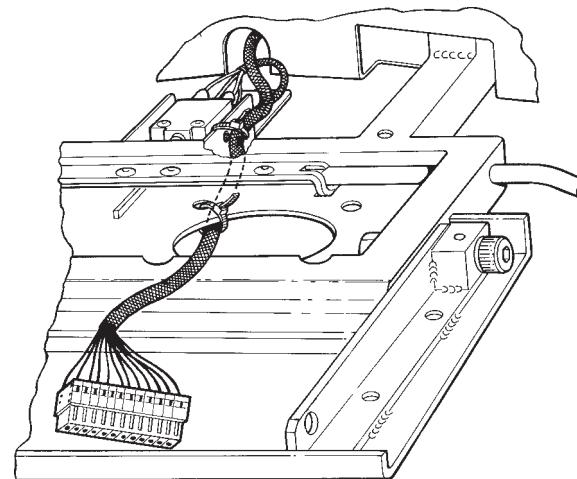
3

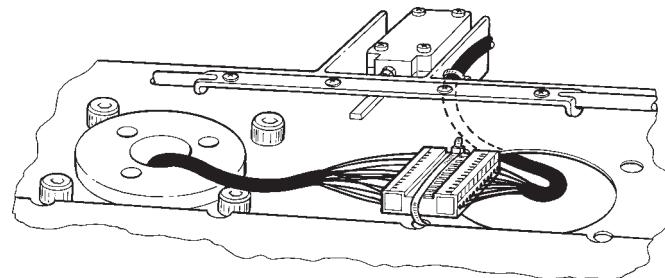GB

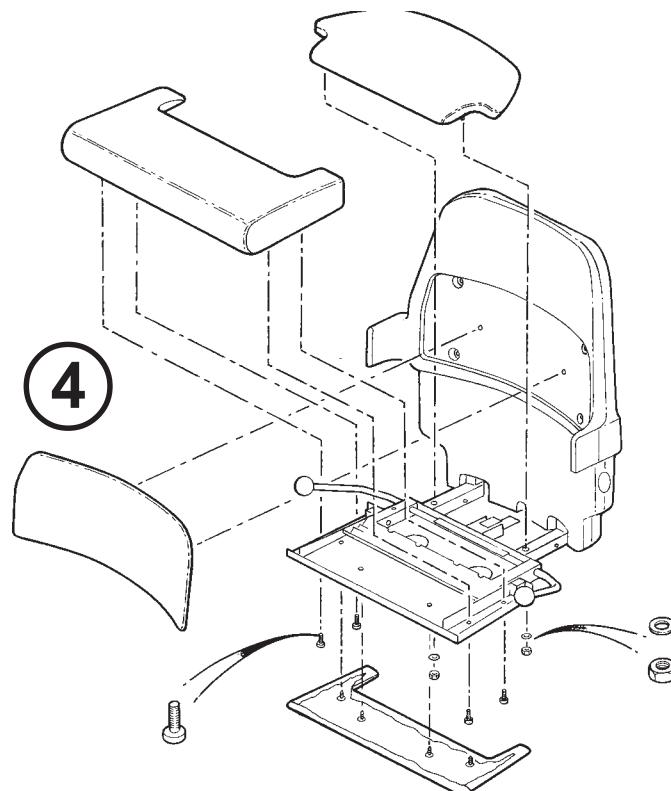
- Cable clip wiring loom under chair (Figure 3).
- Test operation of stairlift. Remember the stops are not yet fitted.
- Chairs are as standard, right hand. To make the chair buttons function correctly for a left hand, move jumper on push button PCB, i.e.
 - remove arm lower plastic moulding
 - move jumper to left hand position.
 - Replace arm lower plastic moulding
- Fit upholstery (Figure 4).

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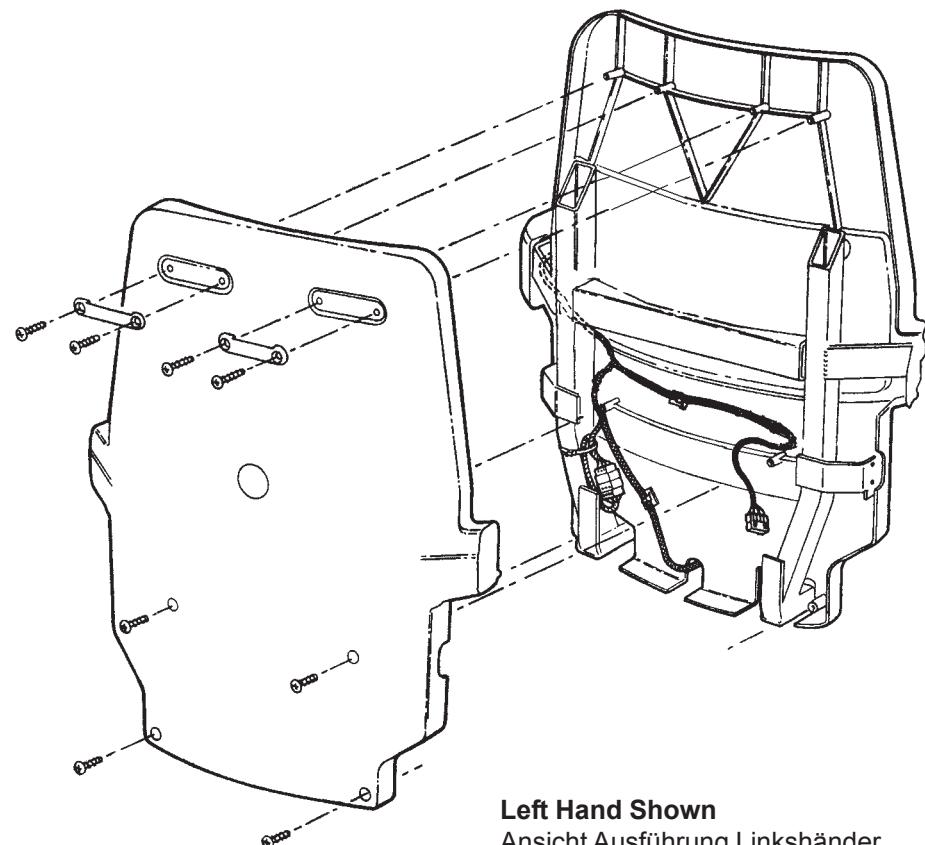
GB

If you do not need to change the handing on the control arm, go to page 42.

- Remove chair back.
- Remove both arm covers
 - Remove key
 - Remove key cover fixing screw
 - Remove captive nut (other arm)
- Remove PCB (2 screws)
- Remove Molex connector from PCB.
- Remove LED's - push forward - remove caps - pull back.
- Remove switch - push through. Disconnect wires.
- Remove key switch locking nut.
- Remove key switch and on/off switch from cables - note for re-wiring back.
- Remove cable clips as required to remove cable.
- Remove cable from swivel switch housing - leave swivel switch connected.

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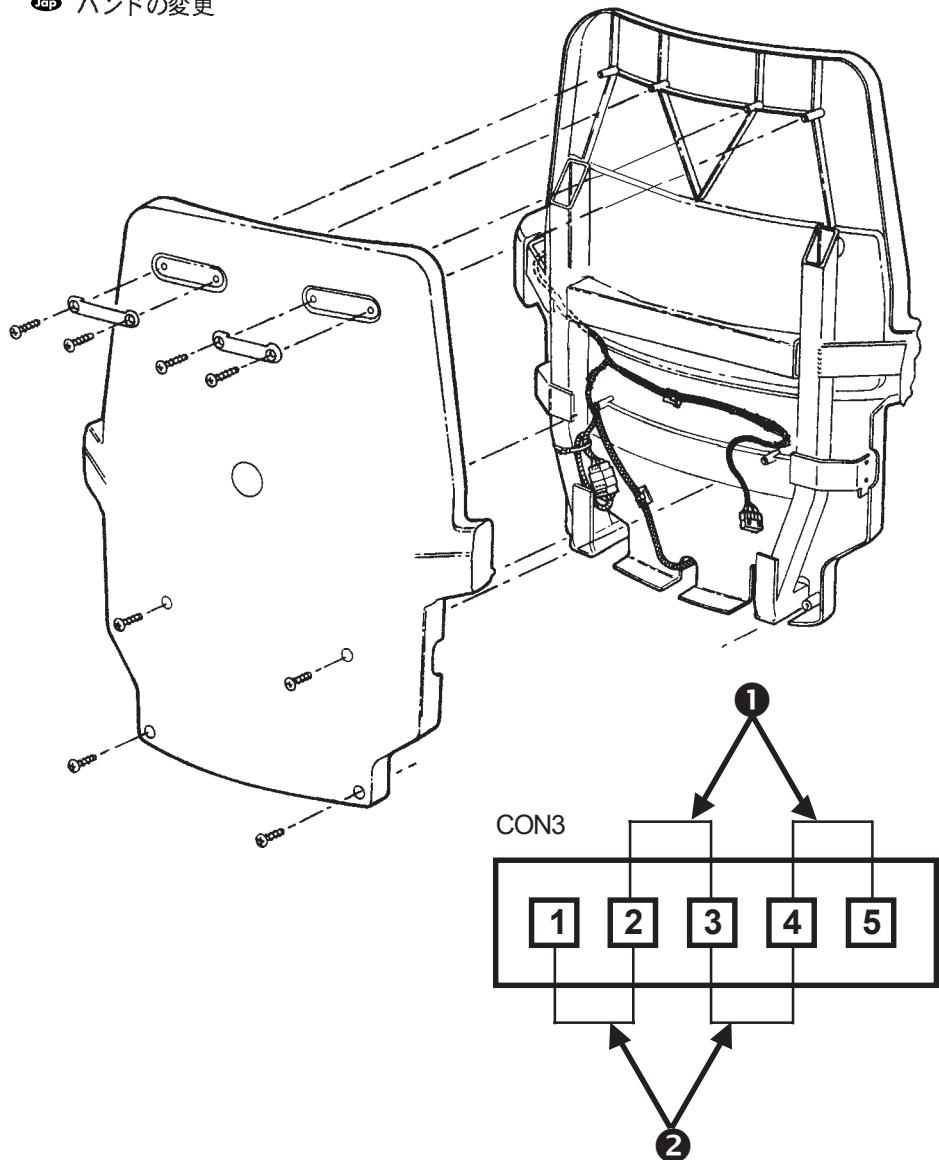
GB

- Cable tie cable to other side of swivel switch housing.
- Cable tie cable to bottom of seat.
- Remove arm controls blanking panel.
- Fit arm controls blanking panel to opposite arm.
- Refit cable up other side.
- Refit PCB, switch, key switch and LED's.
- Reconnect wiring as required.
- Check handing plugs on PCB.
- Cable clip as required.
- Fit captive nut to other side.
- Refit arm covers.
- Fit Radio board as required.
- Fit chair back after programming/testing.

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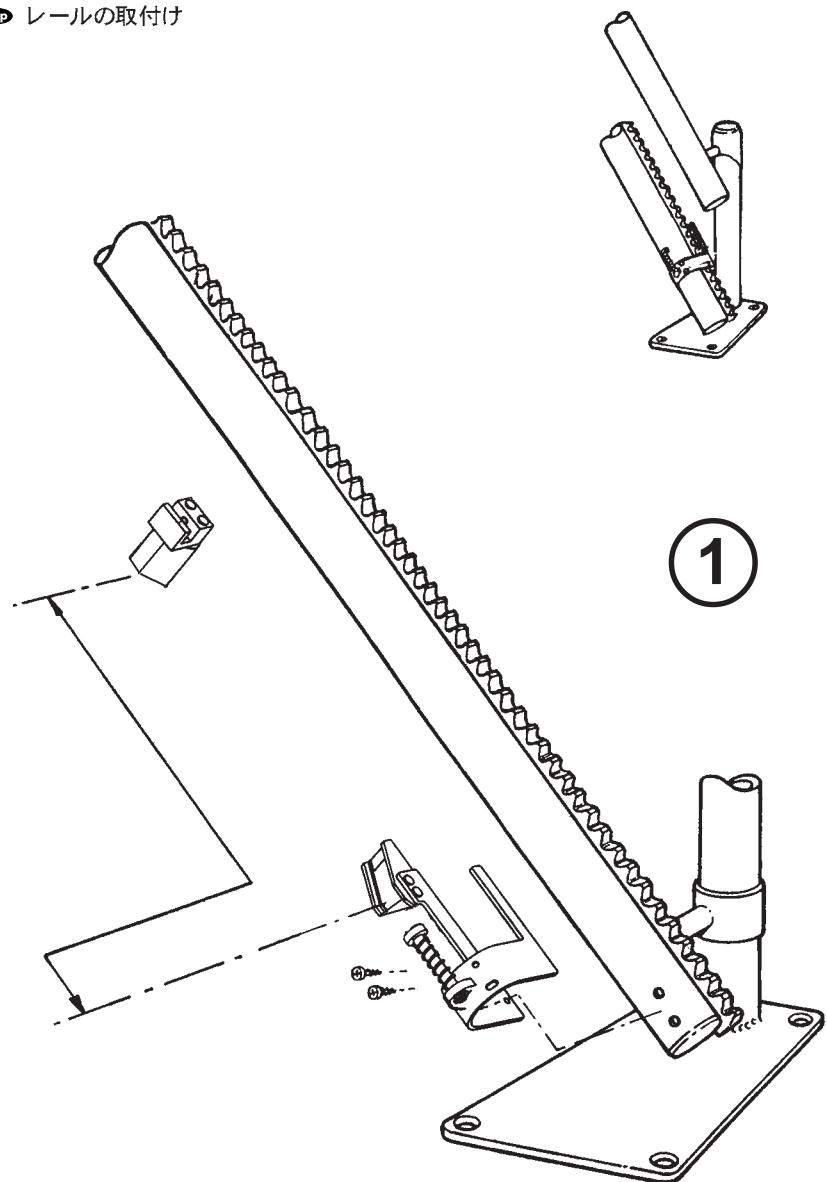
GB

- Fit the limit assemblies to the lower tube section.
 - Having obtained the correct footrest height move the limit assembly to the correct position and mark the rail for drilling.
- Note:** Ensure that the limit assembly at the **top** of the rail is fitted a minimum of 10mm from end of rail.
- Drill Ø3.7mm holes through the slot in the limit assembly and into the rail. This allows a small amount of adjustment.
 - Fix the limit assemblies to the rail using two pan head screw (Figure 1).
 - The charger will be situated either at the top or bottom of the stairs, depending on the position of the power point.
 - Connect the charger.
 - For intermediate charging and stopping, position the charging and slowing ramps according to Rail Drawing 4.
 - Ensure that when the carriage is run onto the charging ramp, it does not make contact with the ultimate stop position on the ramp.
 - Drill Ø3.7mm in rail using holes in brackets as a guide, secure using pan head screw.

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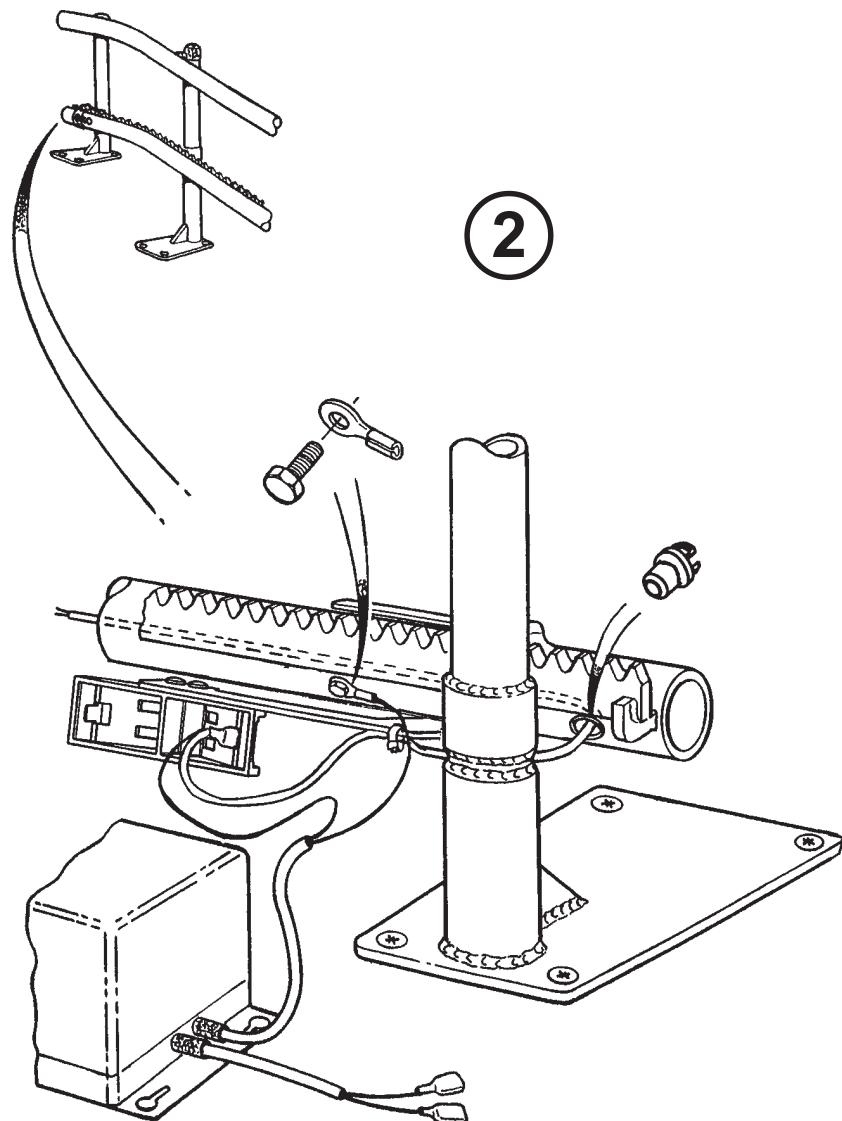
GB

- Attach the black (-) wire from the charger to the rail using a 2BA eyelet and No. 8 x $\frac{1}{2}$ " hex head self tap screw. There is a pre-drilled hole in the rail to be used for this purpose (Figure 2).
- Join the red(+) wire from the charger and the brown wire from the two core cable from the rail together, using a spade connector.
- Secure the two core cable from the rail using the cable tie on the charging assembly.
- Fit the spade connector to the charging ramp assembly.
- Connect the other end of the brown wire, from the two core cable, to the opposite charging ramp assembly observing polarity of connections.
- The charger should be connected into the power source in order to check that are approximately + 27 Volts DC at each charge point.
- Run the lift onto the charging assembly at either end of the rail to ensure that the charging plunger makes contact with the charging strip.

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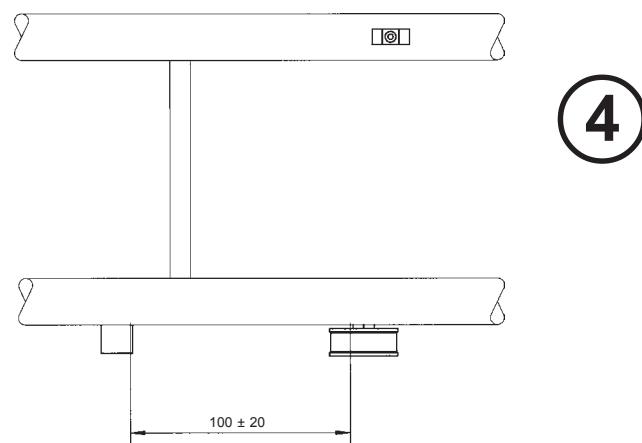
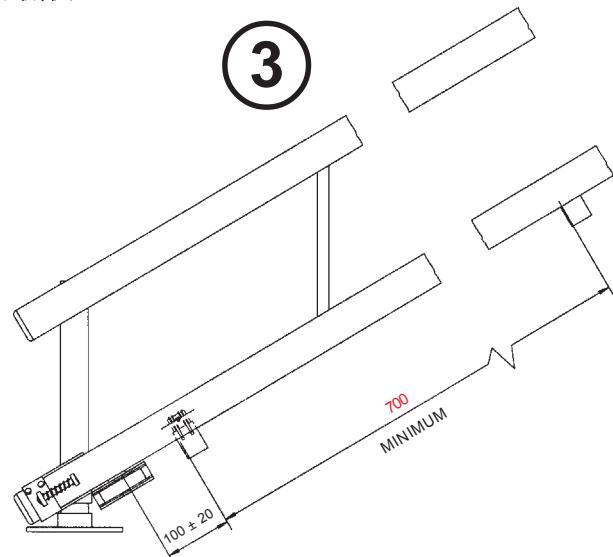
GB

- Fit the slowing ramp (Figure 3) using 3mm Allen key.
 - If slowing is not activated, the ramp can be adjusted as it is fitted on slots.
 - Check that when the stairlift is in the parked position, the yellow LED is illuminated to indicate the stairlift is charging.
 - To test, run the stairlift in a downward direction so that the carriage drives through the slowing ramp assembly at full speed without actually slowing down.
 - When the stairlift finally stops on the plunger ensure that the ultimate limit switch plunger is not activated.
 - Repeat this procedure for the top rail section.
- If any charging point is on a horizontal section of rail, a zero volt ramp must be fitted (Figure 4).**

Lubrication

- Superlube grease (630087) must be applied to rack at maximum 500mm intervals. Particular attention should be given to horizontal inside bends to ensure application of grease.

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GB
Radio System

Check correct frequency/label colour match on receiver and transmitters.

Wear antistatic/earth strap.

Transmitters

Connect a 9V PP3 battery to each transmitter.

Receivers-Carriage

Radio sets are supplied programmed and ready to use. However, if reprogramming is required, follows these steps:

- Switch off stairlift (carriage isolator)
- Connect CON1 on receiver (Drawing 2609106)
- Connect jumper plug to JP1
- Switch on stairlift (carriage isolator)
Green LED on receiver flashes once
- Press either button on transmitter 1 for 3 seconds, check receiver green LED flashes
- Press either button on transmitter 2 for 3 seconds, check receiver green LED flashes
- Switch off stairlift (carriage isolator)
- Remove JP1 jumper plug
- Switch on stairlift, and check transmitters and chair arm buttons
- Fix receiver to chair back with screws, use pilot holes as a guide.

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LED 2

GB

- LED's 1-9 will flash once when the ECU is powered on (LED check).

LED 1 - Failed (Red):

Will activate under the following conditions:

- Relay 3 (K3) contact weld
- Relay 1 and 2 (K1, K2) contact weld
- Direction input short circuit on control PCB (not wiring loom)
- Q1 failure
- Q2 failure
- Motor disconnected/Open circuit

Action:

- Reset board, re-check. Test for presence of short circuit. Change PCB
- Check motor connections.

LED 2 - Low Voltage LED (Red):

Will activate when supply voltage (when running under load) is <20V.

Action:

- Check batteries. Check cable resistance.

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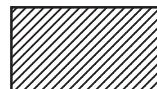
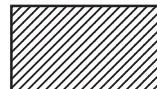
GB

LED 3 - Current Limit LED (Red):

Will activate when lift stalls or draws excessive run current (>18A).

Action:

- Check motor, rollers, weight of user, obstruction on rail.

LED 4 - Brake LED (Red) :

Will activate if there is a brake fault or it is not connected, or if Q6 is short circuit.

Action:

- Check brake circuits, check for output voltage. Change board.

LED 5 - RMU LED (Red):

Will activate when the RMU switch operates and will be continually on if the RMU has tripped.

Action:

- Check RMU operation
- Check output from ECU
- Check continuity of Reed switch
- Check continuity of wiring and connections
- Replace RMU switch.

LED 6 - Halfspeed Switch LED (Yellow):

Will activate when the lift is in half speed i.e. travelling on inside bend rail section.

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LED 10

GB

LED 7 - Slow Switch LED (Yellow):
Will activate when the slowing switch operates.

Action:

- Use for setting slowing ramps.

LED 8 - Call Exists (Yellow):

Will activate when a call has been placed and received at the control PCB.

Action:

- Lights when call button is pressed and the direction safety circuit is closed.

LED 9 - PIC Diagnostics LED (Green):

Will pulse once per second indicating correct software operation.

Action:

- No pulse, reset, re-check. Change board.

LED 10 - 5V LED (Green):

Indicates active 5V supply.

Action:

- No lights, check battery supply voltage, fuses and ultimate limit circuit on all on/off switches.

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

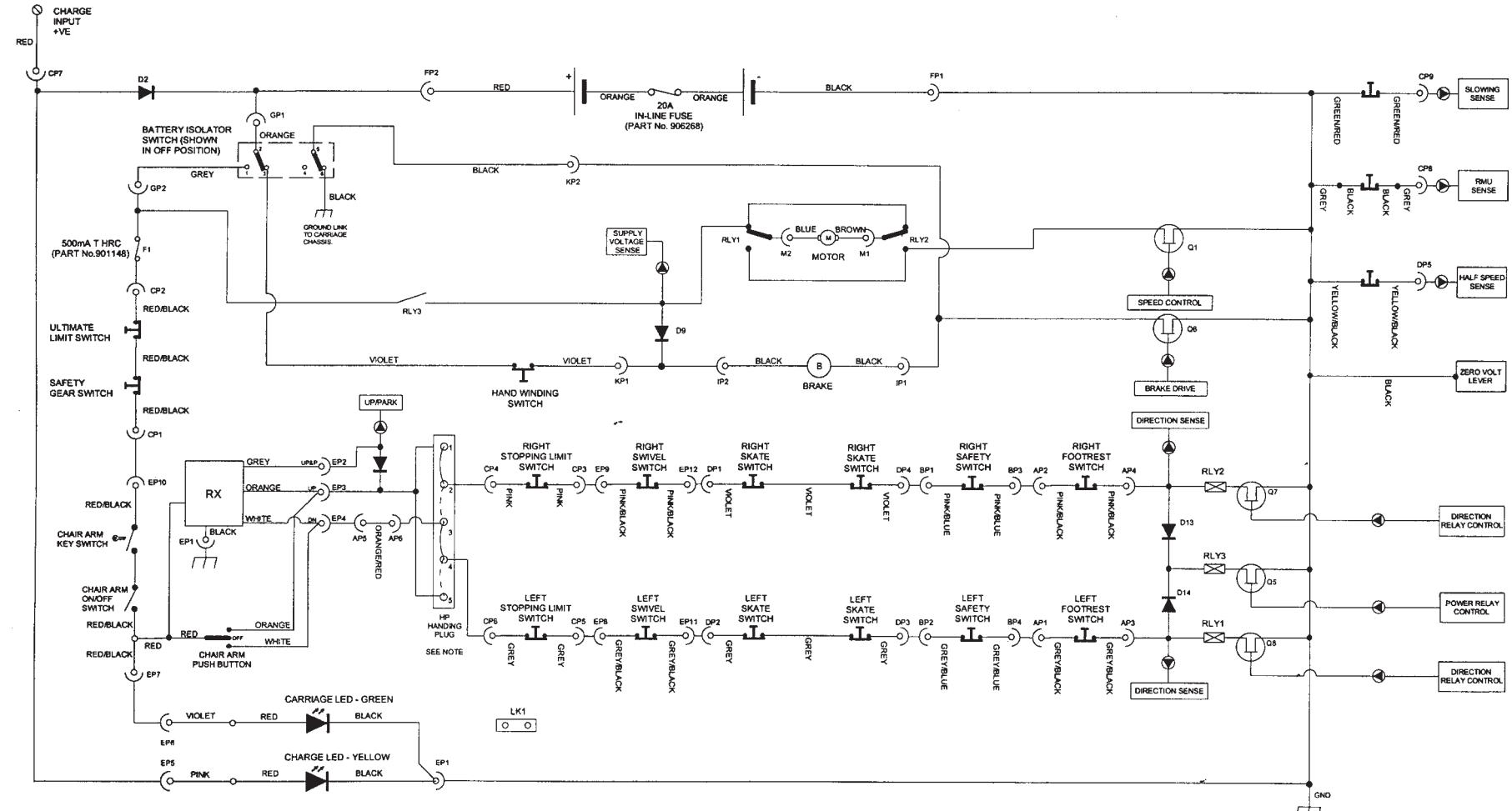
NOTE: LINK 1 AND 2, 3 AND 4
FOR LEFTHAND STAIRLIFT
LINK 2 AND 3, 4 AND 5
FOR RIGHHAND STAIRLIFT.

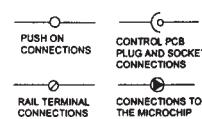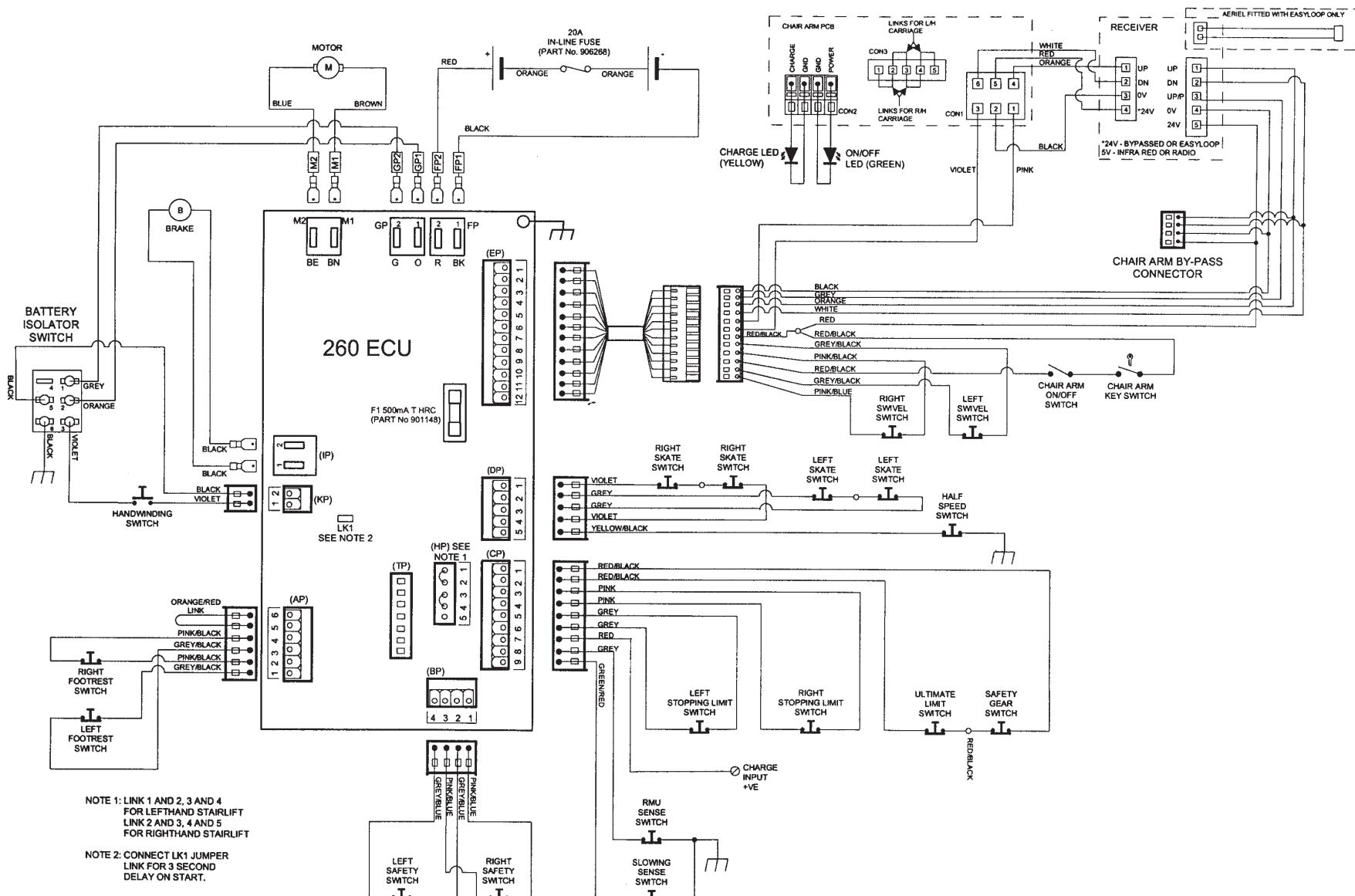NOTE: CONNECT LK1 JUMPER
LINK FOR 3 SECOND
DELAY ON START.

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

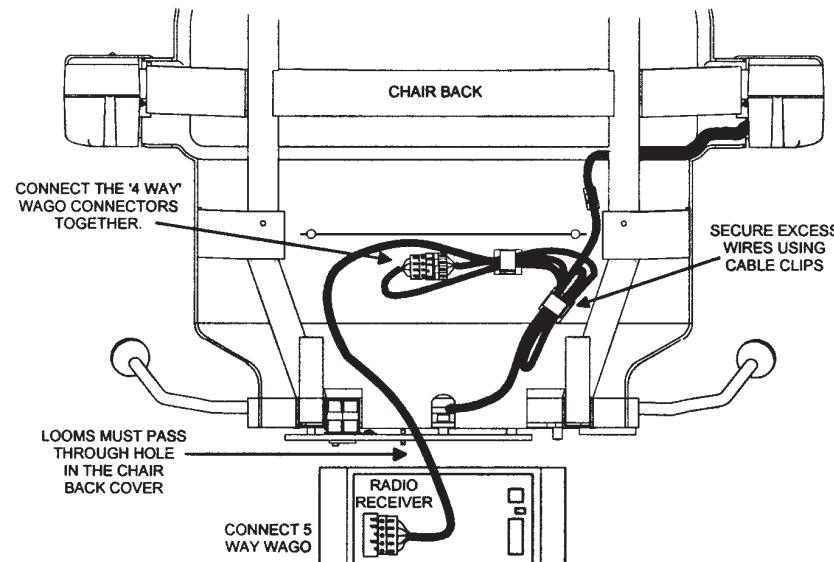
FOR NON HINGED RAILS THE 4 WAY PLUG SHOULD BE CONNECTED TO THE BYPASS PLUG IN THE CHAIR BACK

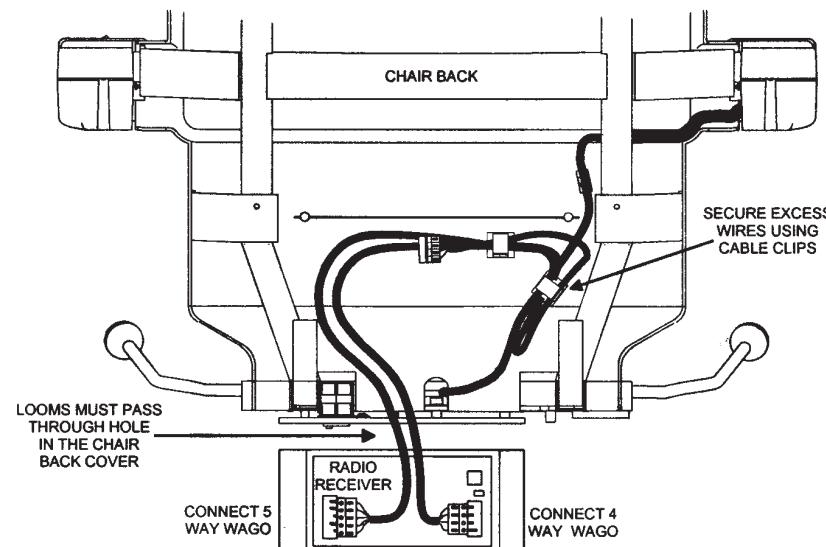IMPORTANT: ON HINGED RAILS THE 4 WAY PLUG MUST BE CONNECTED TO THE RECEIVER AS SHOWN BELOW.

RADIO RECEIVER CONNECTIONS

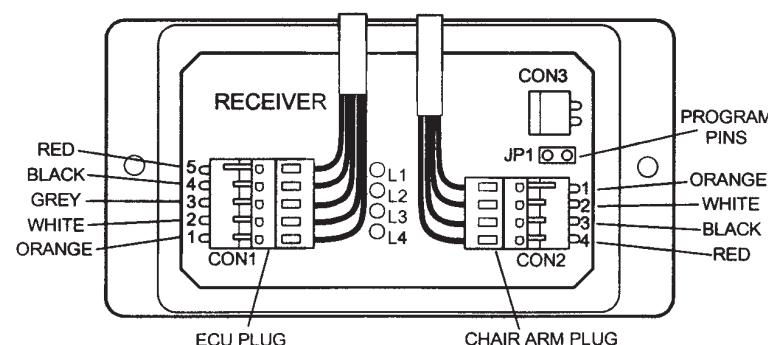NOTES: THE LOOMS FOR THE RADIO RECEIVER ARE TO BE ROUTED THROUGH A HOLE CUT IN THE CHAIR BACK COVER.

THE 30MM HOLE TO BE CUT HAS A RECESS FORMED ON THE INSIDE OF THE REAR COVER NEAR THE LOWER EDGE AS A GUIDE.

THE RECEIVER CAN BE SECURED TO THE OUTSIDE OF THE CHAIR BACK COVER WITH SCREWS USING THE GUIDE HOLES IN THE MOULDING.

RADIO SETS ARE SUPPLIED PROGRAMMED AS A SYSTEM, THEREFORE THE PROGRAM STEPS SHOULD NOT BE REQUIRED. HOWEVER, IF PROGRAMMING IS REQUIRED, THEN REFER TO INSTRUCTIONS.

SEE DIAGRAM 2609334 FOR INFORMATION ON RADIO LED DIAGNOSTICS.

Certificate of Test and Examination After Installation of 260 Model Stairlift to Ensure Compliance with CE Directives:**Machinery Directive SI (92/30/EEC), Low Voltage Directive SI (94/32/EEC), EMC Directive SI (92/23/EEC).**

Name and address of user: Tel:

1. DESCRIPTION

- a) Manufacturer: *Stannah Stairlifts Ltd*
- b) Stairlift reference number: Chair batch No. Carriage batch No.
- c) Safe working load: *120kg*
- d) Rated speed: *0.1m/s*
- e) Motor tested on assembly: YES/NO
- f) Control actuator type: *Constant pressure controls*
- g) Model type: *260*
- h) Type of motor: *DC*
- i) Motor protection (type): *Electronic*
- j) Type of drive: *Rack and pinion*

2. ELECTRICAL TESTS

- a) Electrical supply voltage at time of test: V AC + ve - ve
- b) Battery voltage: V DC
- c) Stall current: Amps
- d) Tripping time: Seconds
- e) Control voltage: *24V DC*
- f) Mains fuse rating: Amps
- g) Polarity check: L-N = L-E = N-E =
- h) Confirm battery polarity check:
- i) AC supply frequency: Hz
- j) Phase: *Single*
- k) Motor reversal delay: Seconds
- l) Motor operation with rated load:
- | | | |
|------|-----------------------|---------------------------|
| UP | Running current: Amps | Running speed: RPM or m/s |
| DOWN | | |

3. BATTERY CHECKS

- a) Battery charge voltage: V DC Confirm batteries are fully charged: YES / NO
- b) Confirm battery enclosure is vented: YES / NO Confirm no leak when battery is fitted: YES / NO
- c) Confirm battery is securely fitted: YES / NO Confirm off charge indicator is functional: YES / NO

4. MECHANICAL / OPERATION TEST**SWITCH OPERATION**

- a) Confirm all on/off stop switches operate correctly: YES / NO
- b) Confirm all control limits operate correctly: upper YES / NO lower YES / NO
- c) Confirm final limits operate correctly: upper YES / NO lower YES / NO
- d) Confirm all sensitive edges/surfaces operate correctly YES / NO
- e) Confirm safety gear switch operates correctly YES / NO
- f) Confirm that all interlock switches operate correctly * YES / NO
- g) Confirm that hinge safety interlock switches operate correctly YES/NO
- COMPONENT FUNCTIONING
- h) State the stopping distance of the carriage with rated load upon application of the brake: Up mm Down mm YES / NO
- i) Confirm hand winding operates correctly: YES / NO
- j) Confirm overspeed governor has been subjected to factory test for correct tripping speed: YES / NO
- k) If an alarm is fitted, confirm mode of operation and that the system operates correctly: YES / NO
- l) State distance of footrest above floor level: Upper mm Lower mm Intermediate mm

5. CONFIRMATION

Confirm operating instructions have been handed to the user: YES / NO

6. DECLARATION

We certify that on this stairlift was thoroughly examined and found to be free from obvious defects and that the foregoing is a correct report of the results.

Signed: Position/Qualifications: Date:

Address: * e.g. on rotating seat, hinged rail etc. ISSUE 2

GB

Introduction

This work method is used for guidance when commissioning a stairlift and completing the Certificate of Test and Examination after the stairlift is installed.

Section 1
Description

- b) Enter details:
 - The contract or order reference number. e.g. 99390
 - The chair serial no.
 - The carriage serial no.
- e) Check that the motor tested label is present on the motor.

Section 2
Electrical Tests

- a) Measure AC supply across the live and neutral terminals at the charger supply input.
- b) Measure battery voltage.
- c) Measure the current draw from the motor during stalling.
- d) Electronic trip time (Maximum 3 seconds).
- f) Fuse rating (Max 2 Amps).
- g) Check polarity at spur or the incoming mains supply.
- h) Check polarity of battery connections

- i) Enter mains AC supply frequency (Hz) for the appropriate country.
- k) Motor reversal delay (Min 3 seconds).
- l) Measure the current draw from the motor during operation in both directions.

Section 3
Battery Tests

- a) Measure battery charge voltage.
- b) Delete as necessary.
- c) Delete as necessary.
- d) Test battery charge condition using the four volt drop principle.
- e) Visually check for any leaks or corrosion.

Section 4
Mechanical / Operation Test**Switch Operation**

- a) Delete as necessary.
- b) Delete as necessary.
- c) Delete as necessary.
- d) Delete as necessary.
- e) Confirm safety gear mechanism has been subjected to factory test (Test label present).
- f) Delete as necessary.
- g) Confirm that hinge safety interlock switches operate correctly.

Component functioning

- h) Record the stopping distance (must not exceed 20mm).
- i) Check the handwinding operation.
- j) Confirm the tested label is present on the overspeed governor assembly.
- k) State option fitted.
- l) Record footrest heights.

Section 5
Confirmation

The stairlift user must be fully instructed on the use of the stairlift including the emergency handwinding procedures. All operating instructions should be handed over to the user on completion.

Section 6
Declaration

This section must be completed in full using the distributor's address and signed.

 **On completion of the final test and examination and completion of the test certificate affix the warning labels to the carriage assembly referring to the installation manual (See page 80). This is important to ensure compliance with CE directives.**

GB

- Demonstration

- Explanation

- User Safety

- User booklet

- Paperwork completion

1. Ask user to stand clear of the stairlift making sure that they have a clear view of the lift.
2. Demonstrate the operation of the seat and footrest; advise the user to stand clear of the footrest to avoid the footrest coming into contact with the users legs.
3. Explain the L.E.D. functions on the chair arm clearly.
4. Explain the operations and positions of all the on/off switches. Explain that the rocker switch should always be turned off before getting in/out of the chair.
5. Explain the operation and position of the directional controls on the chair.
6. Explain that if constant pressure is released from directional controls the lift will stop, and there is approximately a three second delay before the stairlift will restart.
7. Explain the swivel function (manual or powered, if fitted) and also that the lift will stop automatically at the top and bottom of the rail.

8. Ask the user to sit comfortably on the seat and explain the use of the seatbelt (the seatbelt should always be used). Ensure the user does not extend their arm beyond the seat back.
9. Ask the user to operate the stairlift to the top of the stairs. Go up first so as not to block your own access.
10. Ensure that a swivel operation is carried out, and the customer alights from the lift, this will ensure that your footrest height is correct. Ensure the user has turned off the chair arm rocker switch.
11. Ask the user to complete a downward journey.
12. If the stairlift has a hinge, then this option will have to be fully explained at this point, all controls, interlocks, safety features, and general operation.
13. Once the user is clear on the operation of the lift show the operation and position of all safety edges.
14. Ensure that all of your instructions have been clearly understood.
15. Invite the user to sit down comfortably while you explain the user handbook. Ensure that the Service/Distributor details are pointed out.
16. Give the user the top copy of the test sheet, and ask him/her to read and sign the handover sheet. Explain that the handover sheet is also a receipt for any payment collected.

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GB Installation Tools

- 1) 2 ft (500mm) spirit level (approx)
- 2) Extension Bar $\frac{3}{8}$ " or $\frac{1}{2}$ " drive)
- 3) Tape Measure - 5 metres
- 4) 6mm twist drill bit (15/16")
- 5) 8mm masonry drill bits
- 6) Hexagon keys one set - Sizes: 2 - 10mm ('T handles or long series)
- 7) Combination spanners, one set - Sizes: M8 to M24
- 8) Crimping tool
- 9) Cross-head screwdrivers, one set (variable sizes)
- 10) Dust sheet
- 11) Electric drill (Variable speed / Hammer action / Chuck size 13mm minimum)
- 12) Electrical or combination pliers
- 13) Electrical side cutters
- 14) Extension lead (heavy duty)
- 15) External circlip pliers
- 16) Flat blade screwdrivers, one set (variable sizes)
- 17) Hammer 20oz (claw or ball pen)
- 18) Long nose pliers
- 19) Magnetic angle finder
(Starrett: RS 492-005)
- 20) Digital Multi meter/Clamp meter
(must be capable of measuring DC current at 30A for at least 5 seconds)

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