



Precision Rehab Manufacturing  
5325 Kuhl Rd Erie, PA 16510  
Orders@prmrehab.com  
814-899-8731

## Precision-Fit Owner's Manual

### Cleaning and Mounting Instructions

#### *Cleaning Instructions:*

Precision-Fit cushions are designed for ease of maintenance and durability. To ensure their continued performance and hygiene, please follow these cleaning instructions:

1. **Removable Covers:** Precision-Fit cushions feature one sewn, removable incontinent cover and a second of your choice. These covers can be machine washed in cold water, using the delicate cycle for optimal care.
2. **Hang to Dry:** After washing, kindly hang the covers to dry. Avoid using high heat or tumble drying to preserve the quality of the fabric.

*Note: For any inquiries or concerns about your cushion, we encourage you to reach out to the vendor who provided it.*

#### *Mounting Instructions:*

PRM Inc. does not attach the seating system to the wheelchair during the installation process. To ensure the secure attachment of your seating system to your wheelchair, please observe the following guidelines:

1. **Vendor Responsibility:** It is the sole responsibility of the vendor who supplied your seating system to attach it correctly and securely to your wheelchair.
2. **Daily Hardware Inspection:** As an end user, it is imperative to perform a daily inspection of the mounting hardware. Verify that all hardware is securely attached and in good condition.
3. **Immediate Action:** If you observe any loosening or improper attachment of hardware, it is essential to discontinue the use of the seating system immediately.

*Note: If you require further guidance or have questions about your cushion, do not hesitate to contact the vendor who provided it.*