

Maintenance Plan NEURO HITRONIC 20mm

patient commission/configuration ID:

NEURO HITRONIC

functional unit serial number:*

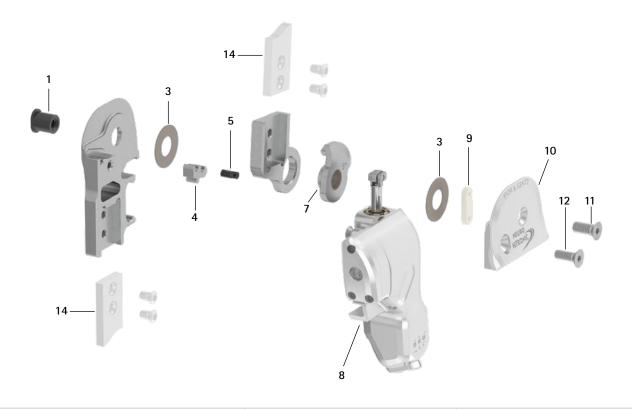


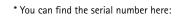







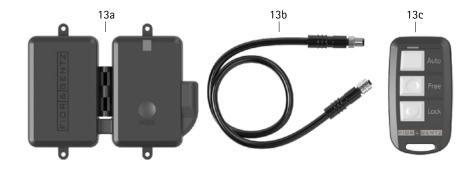

Maintenance Plan NEURO HITRONIC 20mm

| patient commission/configuration ID: | |
|--------------------------------------|--|
| functional unit serial number:* | |

| Item | 20mm | Description | Potential Problem | Measure | Inspection/ Replacement, If Necessary | Latest Replace- ment after |
|-------|------------------|--|--|------------------------|---|-------------------------------|
| Syste | m Joint | | | | | |
| 1 | SB1069-L1110 | bearing nut | wear | replacing | every 6 months | 36 months |
| 3 | GS2411-040 | sliding washer, Ø = 24mm | wear | | every 6 months | 18 months |
| 3 | GS2411-045 | | | | | |
| 3 | GS2411-050 | | | replacing | | |
| 3 | GS2411-055 | | | | | |
| 3 | GS2411-060 | | | | | |
| 4 | SL9005-E005 | 5° extension stop | wear | replacing | every 6 months | - |
| 5 | PN1000-L05/5 | extension stop damper | wear | replacing | every 6 months | 18 months |
| 7 | SL0365-2L | 5° flexion stop disc with sliding bushing | wear | uania sin n | every 6 months | 12 months |
| 7 | SL0365-2R | | | replacing | | |
| 7a | BP1211-L077 | sliding bushing | wear | replacing | every 6 months | 18 months |
| 8 | SL3955-L | functional unit, left lateral | lass of function was | | average Companies | 36 months |
| 8 | SL3955-R | functional unit, right lateral | loss of function, wear | replacing | every 6 months | |
| 8a | SL0355-11 | roll unit | wear | replacing | every 6 months | 6 months |
| 8b | SL0355-12 | air filter | soiling | replacing | every 6 months | 6 months |
| 9 | SL0355-01 | guide piece | wear | replacing | every 6 months | 6 months |
| 10 | SL0355-L/AL | cover plate, left lateral | wear | replacing | every 6 months | 36 months |
| 10 | SL0355-R/AL | cover plate, right lateral | | | | |
| 11 | SC1406-L14 | countersunk flat head screw with hexalobular socket (axle screw) | wear | replacing | every 6 months | 36 months |
| 12 | SC1405-L14 | countersunk flat head screw with hexalobular socket | | | | |
| Contr | oller | | | | | |
| 13a | ET3850 | controller with Bluetooth with lithium-polymer battery | checking the software version battery health (low) | replacing | every 6 months | 36 months |
| 13b | ET0711-01 | connection cable for functional unit, 300mm | damage | replacing | every 6 months | - |
| Remo | te Control | | | | | |
| 13c | ET3840-P | remote control with Bluetooth for the patient | outdated software | updating soft- ware | every 6 months | 36 months |
| Syste | m Anchors and Sy | ystem Side Bars | | | | |
| 14 | see catalogue | system anchor and system side bar | wear or breakage | replacing | every 6 months | - |