

Instructions

The Kind Battery is compatible with 510-thread vaporiser cartridges and is produced by industry leading manufacturer, CCELL®. Featuring variable heating modes and a 5-click unlocking/locking feature to prevent unwanted usage, this battery is suited for patients looking for a straightforward way to administer cannabis medicines through inhalation.

The Kind Battery is button activated to allow patients to preheat their 510 cartridge prior to inhalation, enabling a more consistent draw and prevent blockages.

Using the Battery

1. Screw your 510-threaded cartridge to the top of the battery
2. To turn the battery on, press the button 5 times quickly until the button is illuminated
3. To use, press the battery button and inhale from the cartridge mouthpiece
4. To lock your device, press the button 5 times quickly



Charging the Battery


1. Screw the USB charger to the bottom of the battery and then connect the charger to a USB-compatible socket
2. A green flashing light will indicate that the battery is charging and a solid green light will indicate it is fully charged

Note: It takes approximately 1 hour to fully charge your battery, during which time the button will illuminate red to demonstrate low battery, yellow for moderate battery and green for full battery.


Changing Heating Modes

You may have a preference for a higher or lower heat setting depending on the cartridge you attach to your device. The Kind Battery comes with three heat settings to allow you to choose what works best for you. Below is an overview of the battery's heat settings:


Heat Setting	Button Colour	Voltage
Low	Blue	2.8v
Medium	Yellow	3.2v
High	Green	3.6v

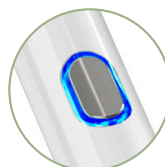

Low

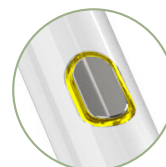

Medium

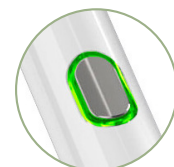

High

How to Change Heating Modes:

1. Ensure your battery is unlocked, but not in use.
2. Press the button 3 times in quick succession to change the heating mode.

Please note: The battery will remember the most recent voltage setting when being turned off and on again.

Troubleshooting

If your battery is not working:

1. Make sure it is fully charged (showing a solid green LED light when plugged in)
2. Ensure your 510 cartridge is screwed into the battery completely
3. Make sure you are pressing the battery button when inhaling
4. If needed, press the button and wait 1-2 seconds before inhaling to allow the battery to preheat the cartridge coil

Instructions

Warnings

- Do not expose device to water and keep away from any flammable gases/liquids
- Do not use in wet environments or extreme temperatures
- Do not leave the device exposed to direct sunlight for prolonged periods of time
- Do not allow other metal objects to contact electrical components in the battery
- Keep device out of reach of children or pets
- Do not drop, throw, or abuse device
- Do not attempt to disassemble device as damage and personal injury may occur
- The Kind Medical 510-thread battery contains a Lithium-ion battery and should be recycled or disposed of in accordance with local requirements to avoid potential injury and/or environmental harm
- Do not dispose of the battery or any component of M3b in a fire as it could cause an explosion or other serious injury

Specifications

- Battery capacity: 350mAh
- Variable heat setting with 3 settings
- Stainless steel housing
- Child-resistant lock

Need Help?

We are here to help you get the most out of your Kind Battery.

Scan for contact details to get in touch.



Scan for additional product information.

