



Pee Wee Valve and Pee Wee Valve Fork PA0001 & PA0009



Instructions for use

NOTE: The Pee Wee Valve Fork, part number PA0009 is only for use when installing them original style Pee Wee Valve, part number PA0001. It is **NOT** for use with the Pee Wee Valve SI, part number PA-PWSI.

Installation into Thermoplastic Sockets:

NOTE: For proper installation, of the Pee Wee Valve (part # PA0001) into thermoplastic sockets, the “Fork” should be used.

1. Drill an 1/8-inch hole in cast at desired location
2. Recess cast model slightly to permit flush mounting of the “Fork”
3. Place valve onto “Fork.” Prongs on the Fork fit snugly into slots under valve
4. Lightly coat valve housing with 732 Dow Corning Silicone adhesive
5. Place protective cap bushing over valve stem
6. Blister-mold thermoplastic socket
7. Using heavy thread, tighten around valve body UNDER black “o”-ring
8. Tighten string around upper housing slot, just under the stem/cap
9. DO NOT OVERTIGHTEN as threads may cut through plastic.
10. Remove thread, cool plastic, and sand top of valve stem/cap
11. Pry out cap and cut excess plastic away with sharp knife.

NOTE: NOT RECOMMENDED FOR LAMINATED SOCKETS. For Laminated Sockets, Use the Pee Wee SI Valve, Part Number: PA-PWSI



ENGINEERED SILICONE
PRODUCTS, LLC
407 Route 94, Columbia
New Jersey 07832, USA
sales@wearesp.com
+1-973-300-5120
www.WEARESP.com



+1-888-932-7377
+1-973-300-5125 (fax)



EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague
The Netherlands