



FlexiSport™, FlexiSleeve™, FlexiKids™



Instructions for use

Features:

1. **Pre-flexed** at 43 degrees to maximize comfort and mobility for all activity levels
2. Constructed of **latex free Neoprene** specially coated to resist cracking or crumbling
3. **Heavy Duty Construction** for durability with excellent stretch for ease of donning/doffing
4. Provides **Suction Seal** for use with transtibial suction valve

Indications:

For use with transtibial prostheses as a suspension aid

Contraindications:

Discontinue use and consult your physician if rash, blisters or skin irritations occur. DO NOT wear suspension sleeve while sleeping or if you have circulatory problems.

Washing and Care:

Wash with soap and cool water and rinse thoroughly as soapy residue can lead to dermatitis. Air dry or lay flat on a towel. DO NOT use excessive heat. DO NOT use solvents or chlorine bleach.

Warranty:

Defects in manufacturing or materials used will be covered for a period of 90 days from the date of purchase. Items covered by warranty will either be replaced or repaired by the manufacturer. Damaged products must be returned clean and dry to the manufacturer. The manufacturer/ distributor will NOT be liable for any damages or injuries resulting from defective materials, workmanship, or problems covered under Indications and Contraindications.

Part Numbers (FlexiSport™):

KS-ESP-S-0-B
KS-ESP-S-1-B
KS-ESP-S-1.5-B
KS-ESP-S-2-B
KS-ESP-S-2.5-B
KS-ESP-S-3-B
KS-ESP-S-4-B
KS-ESP-S-5-B

Part Numbers (FlexiSleeve™):

KS-ESP-0-B, C, T
KS-ESP-1-B, C, T
KS-ESP-1.5-B, C, T
KS-ESP-2-B, C, T
KS-ESP-2.5-B, C, T
KS-ESP-3-B, C, T
KS-ESP-4-B, C, T
KS-ESP-5-B, C, T

Part Numbers (FlexiKids™):

KS-ESP-XS-B, C, T, G, P
KS-ESP-S-B, C, T, G, P
KS-ESP-M-B, C, T, G, P
KS-ESP-L-B, C, T, G, P
KS-ESP-XL-B, C, T, G, P
KS-ESP-XXL-B, C, T, G, P



ENGINEERED SILICONE
PRODUCTS, LLC
407 Route 94, Columbia
New Jersey 07832, USA
sales@wearesp.com
+1-973-300-5120
www.WEARESP.com



+1-888-932-7377
+1-973-300-5125 (fax)



EMERGO EUROPE
Prinsessegracht 20
2514 AP, The Hague
The Netherlands