


CALF CUFF APPLICATION

Wrap the cuff around the calf and secure the Velcro to hold it in place. Make sure the wrap is snug, but not too tight.

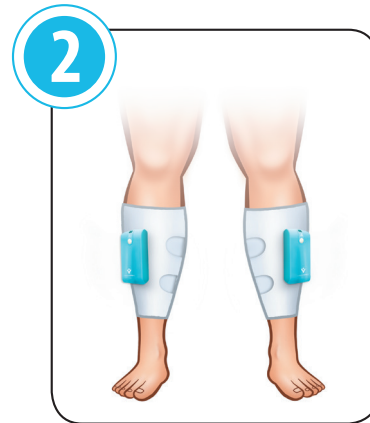


When both wraps are secured on your legs, they should look like the picture above.



TURNING THE DEVICE ON

When the wraps are secured on your legs, PRESS and HOLD the WHITE power button for about a second until the light is illuminated on each unit.

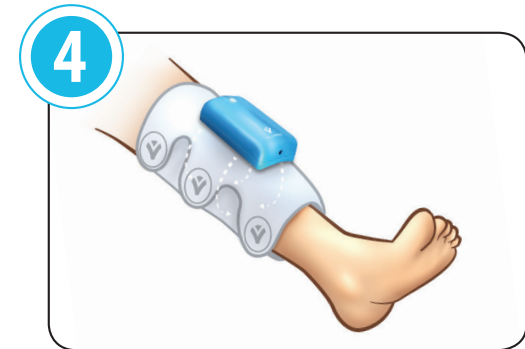

USING THE DEVICE

Don't be startled. The device will make a "humming" sound when inflating and squeezing your leg. **THIS IS NORMAL.** The wraps will inflate once a minute. If you FEEL air releasing around the legs, this is normal. This is a function of the device to keep your legs cool.

PUMP ALARMS

What to do if the Pump Alarms

Don't worry, the alarms are there to insure the units are operating correctly.



BATTERY CRITICAL

If battery voltage drops below a critical level, cycling stops, audible alarm will sound, and UNIT WILL FLASH RED. The Alarm will continue for 30 seconds and automatically turn the unit OFF.



LOW PRESSURE

LOW PRESSURE OR LEAK

If the ALARM sounds and the BLUE LED and RED light is flashing, the pressure is too LOW.

Make sure wrap is attached snugly to the leg. Turn the Unit OFF and then back ON to reset the alarm.

If the unit continues to ALARM after this step, DO NOT try to fix the device.

Call Customer Service for a replacement unit at (800) 604-2487.

POWER



If the power button is SOLID GREEN the units are fully charged and operational.



If both units are plugged in the power button will FLASH GREEN and the batteries WILL charge at the same time.



If the power button is YELLOW plug in the power cord immediately into an electrical outlet.



If battery voltage drops below a critical level, cycling stops, audible alarm will sound, and UNIT WILL FLASH RED. The Alarm will continue for 30 seconds and automatically turn the unit OFF.



PLEASE DO NOT CONTACT YOUR PHYSICIAN OR SURGICAL FACILITY FOR ANY QUESTIONS REGARDING THIS DEVICE.

CALL Customer Service: (800) 604-2487

PLEASE VISIT: www.Ortho8.com for the use of this device.