



AquaGuard®

Shower Protection

Glove



With Water-Seal Band®



3

Gloves

Keep wounds and dressings
dry during showers.

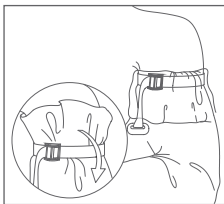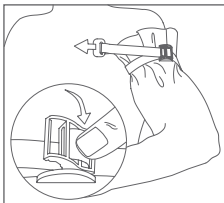

Directions for Use

1. With arm slightly bent slide glove above site to be protected, leaving slack to allow for movement.
2. Press and hold lever down to completely open buckle as shown.
3. Slide Water-Seal Band® over glove and above site to be protected with loose end towards body.
4. Pull loose end of the band until buckle is snug against arm. Confirm effective seal by attempting to insert a finger underneath band.
5. Fold excess glove material over band to form cuff.
6. After shower gently pat cuff and surrounding area dry before removal.
7. Remove buckle by pressing and holding lever while drawing buckle away from arm.

NOTE:

- Some dampness may occur inside glove due to perspiration.
- Glove may be trimmed to size.
- Loose end of the band may be wrapped around arm to secure.

CAUTION:

- Glove can be slippery when wet. Please use caution.
- Suffocation hazard. Keep out of reach of children.
- Do not use if punctured or worn.



TIDI Products, LLC
570 Enterprise Drive
Neenah, WI 54956 USA
Tel +1 920 751 4300
www.tidiproducts.com

