



Made for DSI LTD  
59 haAvoda str.  
Ashdod, 7706300 ISRAEL  
+972-8931-7235  
[www.dsisrael.com](http://www.dsisrael.com)

---

**Filling material for antiseptic treatment of infected canals**

# **DSI Iodoform**



## **INSTRUCTIONS FOR USE**

### **PURPOSE**

The product must be used in medical institution. Iodoform is used for root canal treatment. It is one of the components of the most root canal filling paste materials. Iodoform it acts slightly anesthetic, dries exudates, have a hemostatic effect, do not irritate the periapical tissues. Provides prolonged anti-microbial action on the microflora in the root canal. Iodoform itself doesn't have antiseptic properties, but in contact with tissues, especially with serous or purulent secretion it undergoes gradual decomposition and release free iodine. In generation with the protein substances it acts as an active germicide.

### **APPLICATION**

- Periodontal infections prevention.
- Disinfection dressing (especially at post extraction sockets).
- Treatment of the maxillary cavities infections.
- 

### **DIRECTION FOR USE**

Mix iodoform powder with saline or distilled water. Apply at sterile cotton gauze, cotton swab or paper point (for the root canal treatment). Gauze may be cut to desired size to fit the extraction socket or wound. As bandage does not resorb, the material may remain up to 7 days prior removal.

## **CONTRAINDICATIONS**

Avoid prolonged or repeated contact with the skin and eyes. Avoid prolonged inhalation. Do not take internally. Contact with the skin may cause irritation or even burns in case of the hypersensitivity. If the product comes into contact with eyes, rinse thoroughly with fresh water and consult a physician.

## **PACKAGING**

Powder in jar, 15g

## **STORAGE**

Store in a dry place (humidity < 50%) in tightly closed containers (+5°C...+30°C). Always replace the bottle cap immediately after use.

Avoid contact with moisture and direct sunlight!

Keep out of reach of children.

**Ref: IODOFORM**

## **SYMBOLS**



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

---

*Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.*

*The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.*