



Made for DSI LTD
59 haAvoda str.
Ashdod, 7706300 ISRAEL
+972-8931-7235
www.dsisrael.com

Bandage iodoform gauze for dentistry

DSI Strip Iodoform



INSTRUCTIONS FOR USE

PURPOSE

The scope of use is dentistry. Professional use in the healthcare facility. Strip iodoform gauze for dentistry (hereinafter – Strip) is used for treatment post-extraction wells and for disinfection of upper jaw sinuses.

FEATURES

As a basis of Strip tampon bandage made from 100% cotton with woven edges is used. 100 g Strip contains 5 g of iodoform. Strip has a mild anesthetic, antiseptic, and an effective adsorption. Interact with wound exudate, iodoform allocates free iodine, which provides a pronounced bactericidal effect.

DIRECTION FOR USE

It is necessary the stored or transported at low temperatures material to stand at room temperature for at least 1 hour before applying.

For tamponing the Strip should be cut the desired length and carefully placed in the cavity. Tamponade starting from the bottom of the hole,

pressing down firmly and adding turundas hole is gradually filled to the brim. Strip does not resorb and can remain in the cavity for 2-8 days before removal. Turundas removed from the wound after the granulation wall socket.

PACKAGING

Strip (in jar) 250 cm x 1 cm **Ref: DS-STR10**

Strip (in jar) 250 cm x 2 cm **Ref: DS-STR20**

STORAGE

Store in a dry and dark place at temperature (+5°C...+25°C).

Transport using all types of vehicles at temperatures from -30°C to +30°C. Keep out of reach of children.

UTILIZATION

Residue of bandage and packaging are safe for the environment and can be specified as household waste and utilized according to the rules of medical institution on the basis of current legislation.

SYMBOLS



Consult instructions for use



Manufacturer



Caution, consult accompanying documents



Catalogue number



Temperature limit



Batch code



Keep away from sunlight



Use by



Keep dry



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.