



Made for DSI Ltd.
59 haAvoda str.
Ashdod, 7706300 Israel
+972-8931-7235
www.dsisrael.com

Absorbable dental haemostatic sponge with Iodoform

DSI Sponge Plus



INSTRUCTIONS FOR USE

PURPOSE

Hemostatic antiseptic collagen sponge **DSI Sponge Plus with Iodoform** is used for the treatment and prevention of inflammatory complications in surgical dentistry and periodontics: after tooth extraction, in particular the treatment of alveolitis and periodontal abscesses, as well as the filling of periodontal pockets after a local anti-inflammatory therapy or curettage.

FEATURES

DSI Sponge Plus consists of lyophilized collagen, in which introduced hemostatic components affecting the separate stages of blood coagulation, to stop capillary bleeding, and anesthetic components (lidocaine).

As an antiseptic sponge **DSI Sponge Plus** contains iodoform, that in contact with living tissues allocates iodine, providing antimicrobial action, enabling the formation of granulation tissue, and also has astringent and anti-inflammatory properties.

DSI Sponge Plus does not have a local irritating and bio-toxic action, stimulates the regeneration of tissues in the healing stage. It does not require the doctor's intervention to remove it, full absorption happens after a few days.

DIRECTION FOR USE

By teeth extraction: After the removal of the tooth or the root of a tooth it is necessary to hole sanitation, with a view to the removal of granulation or infected tissue, and bone fragments. Take a pair of tweezers one cube of sponge **DSI Sponge Plus** and place it in

the hole. On the top of the sponge lay a dry tampon. In the case of traumatic removal of the tooth, the top of the sponge can be sutured.

In periodontics: In the area of periodontal defects, **DSI Sponge Plus** promotes the formation of blood clot formation and rapid healing of wounds. After mechanical process of the periodontal pockets sponge or its part placed in the prepared cavity, which helps to eliminate inflammation and prevents penetration of infection in bone.

Only for local use!

Not for use on patients with a sensitivity to iodine drugs!

PACKAGING

Sponge in rectangular shape, 50 pcs in a glass jar.

STORAGE

Store in a dry and dark place, in tightly closed containers (+5°C...+25°C).

Avoid contact with moisture!

Ref: DSPP-50

SYMBOLS



Consult instructions for use



Do not use if package is damaged



Caution, consult accompanying documents



Manufacturer



Temperature limit



Catalogue number



Keep away from sunlight



Batch code



Keep dry



Use by



Non-sterile product



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.