



Made for DSI Ltd.
59 haAvoda str.
Ashdod, 7706300 Israel
+972-8931-7235
www.dsisrael.com

Haemostatic Sponge

DSI SPONGE



INSTRUCTIONS FOR USE

PURPOSE

Haemostatic antiseptic **DSI Sponge** is non-toxic, non-allergenic, non-immunogenic, non-pyrogenic and overall biocompatible material used for the treatment and prevention of inflammatory complications in surgical dentistry and periodontics.

After tooth extraction, in particular the treatment of alveolitis and periodontal abscesses as well as the filling of periodontal pockets after a local anti-inflammatory therapy or curettage.

FEATURES

DSI Sponge absorbs approximately 40-50 times of its weight and adheres easily to the bleeding site. Uniform porosity of the material guarantees a favorable haemostatic by activating the thrombocytes at the moment the blood comes in contact with the sponge matrix.

DSI Sponge does not have a local irritating and bio-toxic action, stimulates the regeneration of tissues in the healing stage. It does not require the doctor's intervention to remove it, full absorption happens within 3-4 weeks.

DIRECTION FOR USE

By removal of teeth:

After the removal of the tooth or the root of a tooth it is necessary to hole sanitation, with a view to the removal of granulation or infected tissue, and bone

fragments. Take a pair of tweezers one cube of sponge DSI Sponge and place it in the hole. On the top of the sponge lay a dry tampon. In the case of traumatic removal of the tooth, the top of the sponge can be sutured.

In periodontics:

In the area of periodontal defects, DSI Sponge promotes the formation of blood clot formation and rapid healing of wounds.

After mechanical process of the periodontal pockets sponge or its part placed in the prepared cavity, which helps to eliminate inflammation and prevents penetration of infection in bone.

Only for local use!

STORAGE

Packed in gamma irradiation sterile blisters with guaranteed sterility.

Once the package is opened, contents are subject to contamination- to be used as soon as the package is opened and unused contents discarded.

Do not reuse/do not re-sterilize.

Store in a dry dark cool place (+5°C...+30°C).

Shelf Life: 3 years from the date of manufacturing.

PACKAGING

Sponge cube in sterile blister, 32 pcs

Ref: DSP-32

SYMBOLS



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.