



Made for DSI Ltd.
59 haAvoda str.
Ashdod, 7706300 Israel
+972-8931-7235
www.dsisrael.com

Absorbable Collagen Plug Sterile Haemostatic

DSI Pure Sponge



INSTRUCTIONS FOR USE

PURPOSE

Post-extraction or oral surgery local hemostasis. The contact of DSI Collagen Pure Sponge with blood leads to aggregation of platelets which precipitate on the collagen matrix, disintegrate and release clotting and plasma factors. It facilitates the formation of new fibrin. The collagen matrix structure provides additional reinforcement to the clot while preventing soft tissue from growing into the extraction alveoli. Using Pure Sponge reduces the alveolar ridge atrophy. Due to its unique structure, Pure Sponge is capable of absorbing large quantities of fluid, removing rejected materials such as bacteria and fibrin clots during this process.

DIRECTIONS OF USE

Apply Pure Sponge directly into the extraction alveola. Trim with scalpel blade if necessary. Pure Sponge moulds into the extraction site, stops bleeding and completely resorbed.

Important Notice: Chlorine-releasing antiseptic agents, tannins and caustics should not be used in combination with collagen.

STERILITY

Sponge is supplied gamma-sterilized. The sterility of undamaged, unopened packs is guaranteed. Damaged packages should be discarded prior to use.

PRECAUTIONS AND WARNINGS

Despite its proven low antigenicity, hypersensitivity reactions may occur due to the xenogenous origin of the collagen. If allergic reaction, dermatitis or rash develops, consult a physician. The Pure Sponge cones/cubes are disposable and intended for single use. After being removed from the protective package must not be re-sterilised and should be discarded. Use only dry instruments and gloves with a Pure Sponge.

STORAGE

Do not store above 25° C. Protect from moisture. Keep out of the reach of children.
Do not refrigerate.

PACKAGING

Products	Size, mm	Pcs	Blister (10 pcs) Pack	Individual Packaging
Short Cone	Ø 4 x 7	10	DS-PS47	DS-PS47-B
Long Cone	Ø 5 x 15	10	DS-PS515	
Short Plug	Ø 8 x 7	10	DS-PS87	DS-PS87-B
Long Plug	Ø 8 x 14	10	DS-PS814	DS-PS814-B
Big Plug	Ø 10 x 20	10	DS-PS1020	DS-PS1020-B
Cube	□ 10 x 10 x 10	10	DS-PS10	DS-PS10-B
Cube	□ 10 x 10 x 10	20	DS-PS20	

SYMBOLS



Consult instructions for use



Do not re-sterilize



Caution, consult accompanying documents



Non-sterile product



Temperature limit



Manufacturer



Keep away from sunlight



Catalogue number



Keep dry



Batch code



Sterilised using irradiation



Use by



Do not use if package is damaged



Date of Manufacture



Do not re-use/ for single use only

ATTENTION! The sterility of the product cannot be guaranteed if the package bears evidence of damage, has been opened previously or wet.

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.