



DSI DENTAL
SOLUTIONS

DENTAL IMPLANT PASSPORT



PATIENT

Dear patient,

Your trusted dentist has just completed an implantology procedure to restore the functionality and aesthetics of your smile. The passport used to record important information about your prosthesis is your guarantee from the implant team about the nature and quality of the components used. This certificate allows you to save the trace of the operation, therefore, in case of unexpected operations, each implantologist can have all the necessary information.

Keeping to work time is largely left to the care you provide, following the advice of your dentist and the instructions contained in this document.

Please fill in your personal information below and bring it to all subsequent visits.

Sincerely yours,
DSI Team

For more detailed information visit DSI website www.dsisrael.com

PATIENT INFORMATION

Last Name:

First Name:

Address:

Phone number:

Date of birth:

E-mail:

Note:

IMPLANT IDENTIFICATION

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

IMPLANT IDENTIFICATION

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

IMPLANT IDENTIFICATION

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....


.....
Date:

Tooth No:

Identification
label (UDI)

Remarks:
.....
.....

POSITION OF THE IMPLANTS

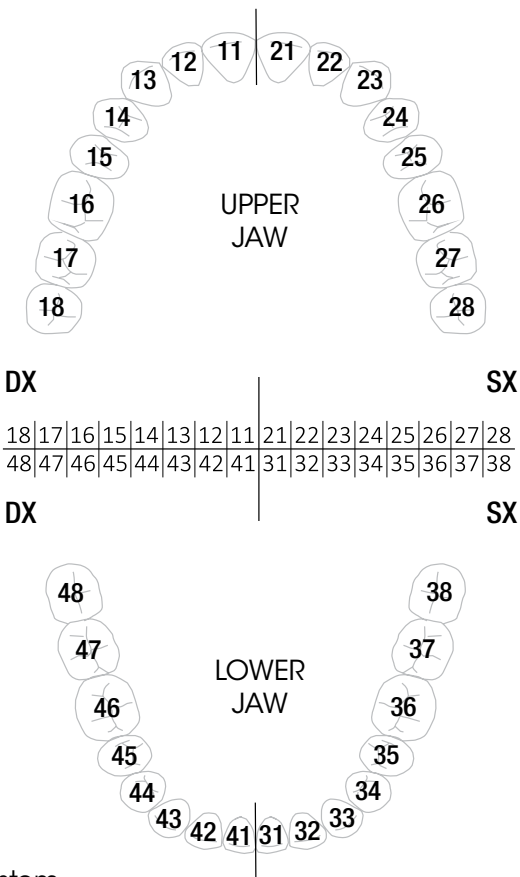

FDI System

IMPLANT POST OPERATIVE INSTRUCTIONS

- A dental implant has been placed in your mouth. The implant may be located above or below the tissue. This type of implant has been selected for your particular situation because of the bone available to place an implant. The implant will usually take a period of one to four months to heal; depending upon your body's healing properties and the type of implant surgery.
- If IV sedation was used for your procedure, you cannot drive a car for the remainder of the day.
- If sutures were placed around your implant, they will be removed and checked by your doctor in one to two weeks.
- Usually, you are able to wear your present partial or flipper, if applicable. Sometimes it is necessary to leave it on the night of the surgery. It is important to keep the appliance as clean as possible during the healing period. Please ask the doctor if you have any questions.
- Every consideration must be given to keep the surgical site clean and free of food particles.

Instructions for mouth care and other helpful information

- **NO SMOKING!** Smoking is to be avoided for the time specified by the doctor. Smoking increases the heat in the surgical site and significantly lowers the body's ability to heal the site.
- **AVOID** these after surgery: alcohol with post-operative medications, commercial mouth rinses, and very hot fluids.
- **GENTLE RINSING** of the mouth should be started the day after surgery. Frequent gentle rinsing with lukewarm salt water will aid the healing process (add one-half teaspoon of salt to a 6oz glass of water). Avoid the use of a water-pick toothbrush.
- **Pain:** A certain amount of pain must be expected with all types of surgery. Appropriate pain medication has been prescribed for you. Please take it according to the directions. It is advised that you do not drive while taking the pain prescription. However, if only Ibuprofen or Tylenol is needed, most people are able to drive without a problem.

-
- **Swelling:** Some swelling and minimal bruising is possible and is to be expected. IT IS NOT UNUSUAL. In most cases, swelling can be prevented/controlled. Apply the ice pack that has been given to you for a period of 20 minutes on and 20 minutes off during the day for the next two days. The application of ice to the outside of the face over the surgical area will minimize swelling. If after five days you still have unusual swelling or pain, please call the office.
 - **Bleeding:** A small amount of bleeding following surgery should not alarm you. If the bleeding is excessive or continuous, please call us at once. Vigorous rinsing of the mouth prolongs bleeding by removing the clotting blood, so when rinsing your mouth, do it gently.
 - **NO drinking** through straws. The use of a straw creates negative pressure in your mouth and will tend to loosen the sutures.
 - **Diet:** Following surgery, it is best to restrict your diet to fluids and soft foods for the first day. Normal diet may then be resumed the following day, but you will want to avoid chewing on the implant site until the tissue is completely healed.

Soft foods such as Jell-O, pudding, mashed potatoes, scrambled eggs, and soups are suggested. If you have difficulty chewing, try blenderized foods or dietary supplements such as Carnation Instant Breakfast and Ensure.

- **Implant Follow-Up:** We will see you for periodic check-ups until the implant has healed. This is usually for a period of one to four months, depending on your implant surgery.

***IF YOU HAVE ANY QUESTIONS AT ANY TIME,
PLEASE CONTACT YOUR DOCTOR***

Your dentist is a DSI partner. Introduced in more than 100 countries around the world, DSI enjoys a worldwide reputation for striving for excellence, which guarantees a high degree of trust with each new development. DSI products comply with all quality standards ISO 13485:2016, CE (Annex VIII of MDR).



DSI DENTAL
SOLUTIONS

DSI Dental Solutions Ltd.
59 haAvoda str.
Light Industrial Zone
Ashdod, 7706300 Israel
Phone: +(972) 8-931-7235
Cell: +(1) 972-200-3265
e-mail: info@dsisrael.com
www.dsisrael.com