



**DESCRIPTION:** DSI ApexSil RC is a radiopaque, non-resorbable, Zinc Oxide-Eugenol based root canal sealer. Its composition is well tolerated by the tissues and provides anti inflammatory action.

**COMPOSITION:** Powder: Zinc Oxide, Barium Sulphate, Accelerators, Preservative. Liquid: Eugenol & Excipients.

**PURPOSE:** It is used as permanent root canal filling material.

**INDICATIONS:** Root canal filling and sealing in association with a gutta-percha point.

**INSTRUCTIONS FOR USE:** Mix carefully for approximately 45-60 sec, one scoop of powder and five drops of liquid to obtain a thick mix, which can easily be introduced into the canal with a suitable root canal filler, or in conjunction with a silver, acrylic, or gutta-percha point. Net setting time is 45-60 min from the start of the mix. The dispensed material shall not be placed back in the container and the dose once applied and used shall not be reused.

**Warning:** Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns, or hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reactions to any of the components.

**PRECAUTIONS:** Do not use ApexSil RC with patients that have a history of allergic reaction to Iodine compounds. Avoids contact with eyes.

**CONTRAINDICATIONS:** Do not use in patients with a known allergy to any of its components.

**STORAGE:** Store at a temperature between 10°C (50°F) to 24°C (75°F). Close the cap carefully after use. Keep away from moisture. Shelf life is not affected after opening if stored in a cool dry place.

**Shelf Life:** 3 years from the date of Manufacturing.

**PACKAGING:** 15g Powder, 10ml Liquid, Dispensing Dropper, Spoon and Mixing Pad.

Ref: ASIL-RC

## SYMBOLS



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Non-sterile product



Do not use if package is damaged



Keep Out of Reach of Children



Medical Device



Use by Prescription only



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material. The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.