



DSI DENTAL
SOLUTIONS

www.dsisrael.com

ZOE CEMENT QUICK SET

Zoer Quick



INSTRUCTIONS FOR USE

DESCRIPTION

DSI Zoer Quick is quick set easy mix zinc oxide-eugenol composition for thermal insulation and temporary restorations.

COMPOSITION: Powder is Zinc Oxide and liquid contains Eugenol and Acetic Acid.

INDICATIONS: Temporary restoration, temporary cementation of crowns, cavity liner under most restorative materials.

INSTRUCTIONS FOR USE

Mix Powder and liquid in a ratio of 0.32 to 0.08 (1scoop powder and 3-4 drop liquid) for about 45-60 seconds to obtains putty consistency. Place the putty into the cavity and remove excess of the product immediately. Allow hardening for 11-12 minutes. Trim the restoration with a tungsten carbide bur.

Warnings: Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns, or hypersensitivity. If contact with skin occurs, immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs, discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult

a physician. It should not be used with patients who have a history of severe allergic reactions to any of the components.

CONTRAINDICATIONS: It is contraindicated for use with patient who have a history of severe allergic reaction to any of the components.

PRECAUTIONS: **Zoer Quick** is a Eugenol based composition and therefore it should not be used as a liner under resin based filling materials. Avoid contact with eyes. Do not invert the bottle upside down while opening to avoid excess dispensing of the powder and avoid excessive force while dispensing the liquid from the dropper.

STORAGE: Store at a temperature between 10°C (50°F) to 24°C (75°F). Close the cap carefully after use. Keep away from moisture. Shelf life is not affected after opening if stored in a cool dry place.

Shelf Life: 3 years from the date of Manufacturing.

PACKAGING:

20g Powder, 10ml Liquid, Spoon, Mixing Pad & Dispensing Dropper.

Ref: ZOER-QS

SYMBOLS



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Non-sterile product



Do not use if package is damaged



Keep Out of Reach of Children



Medical Device



Use by Prescription only



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.