



**DESCRIPTION:** DSI Prophylaxis Paste is a cleaning and polishing paste with a unique combination of abrasive materials zirconium silicate and pumice powder for cleaning and polishing during dental procedures. It contains sodium fluoride at a concentration of 1% fluoride ion for post treatment comfort to the patient. **Prophylaxis Paste** is produced in a variety of fruit flavors for pleasant feeling during the treatment procedure.

**COMPOSITION:** Zirconium Silicate, Pumice Powder, Glycerine, Fumed Silica, Sodium Fluoride, Pigments and flavoring matter.

**PURPOSE:** It is intended to be used as cleaning and polishing paste.

**INDICATIONS:** Prophylaxis Paste, is indicated for use to effectively clean and polish tooth structure with minimal abrasion.

#### INSTRUCTIONS FOR USE

1. Dispense Paste into dappen dish or another suitable container. A saliva ejector may be helpful during the actual prophylaxis.
2. Fill the rubber prophylaxis cup with paste. Before starting the handpiece, carry the paste to the field of operation and deposit a small amount on each tooth surface. It is necessary that fresh paste be provided for each surface in order to obtain maximum cleaning effectiveness.
3. Run the rubber cup at SLOW SPEED over each tooth surface for approximately 10 sec. Make sure that the edge of the cup goes below the free margin of the gingival into the gingival sulcus. Continue the procedure until all exposed surfaces of the teeth have been thoroughly cleaned and polished.
4. A rubber cup or soft bristle brush may be used on occlusal surfaces.
5. The dispensed material shall not be placed back in the container and the dose once applied and used shall not be reused.

**Warnings:** Prophylaxis Paste may cause eye irritation upon contact. If contact with eyes occurs, immediately flush eyes with a copious amount of water for 15 min. Seek medical attention if irritation or discomfort persists.

#### PRECAUTIONS

1. All tooth surfaces should be thoroughly scaled to remove calculus.
2. Necessary examination and radiographs should be taken prior to prophylaxis with Prophylaxis Paste.
3. During use, wear protective glasses and gloves.

**CONTRAINDICATION:** Prophylaxis Paste is contraindicated in patients with a known allergy to any of the components.

**STORAGE:** Store at a temperature between 10°C (50°F) to 24°C (75°F). Close the cap carefully after use. Keep away from moisture. Shelf life is not affected after opening if stored in a cool dry place. **Shelf Life:** 3 years from the date of Manufacturing.

**PACKAGING:** 75g Plastic Jar, Fresh Mint.

Ref: PPASTE-FM

#### SYMBOLS



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Non-sterile product



Do not use if package is damaged



Keep Out of Reach of Children



Medical Device



Use by Prescription only



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material. The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.