Absorbable Hemostatic Compressed Sponge

DSI Sponge HD

INSTRUCTIONS FOR USE

APPLICATION FORM

The sterile absorbable DSI Sponge HD plugs are used as haemostatic agents after tooth extractions and oral surgery. The application of DSI Sponge HD into the defect cavity supports the stabilization of the produced blood clot whilst helping to control bleeding at the application site.

COMPOSITION

1 DSI Sponge HD 10x20 plug contains 90mg of hydrolyzed collagen.

INTENDED USE

The DSI Sponge HD plugs are used as haemostatic agents after tooth extractions and oral surgery. DSI Sponge HD can be combined with DSI Repio Glue in patients with an increased risk of bleeding.

MODE OF APPLICATION

- After removal from the sterile packaging DSI Sponge HD can be cut to the desired shape and size with sterile instruments. Please read instructions carefully prior to use.
- The DSI Sponge HD plugs are used in a dry state with light pressure to achieve better adhesion and haemostasis.
- The DSI Sponge HD is inserted into the extraction socket or the cavity as required.
- Once inserted into the desired cavity DSI Sponge HD can be sutured over and/or held in place with fibrin glue as required.
- Due to the affinity of collagen with blood-covered surfaces, dry instruments and gloves should be used when administering the DSI Sponge HD.
- Due to the absorbability of the DSI Sponge HD plugs, they do not have to be removed. Plugs will be absorbed within 3 – 4 weeks;
- Absorption time may differ as it is dependent upon multiple patient-specific factors including but not limited to the site of implantation, comorbidities and blood perfusion.
- DSI Sponge HD may only be used by qualified medical professionals.

Medical treatment of severe bleeding disorders is mandatory prior to the beginning of the procedure.

Unless otherwise prescribed, the DSI Sponge HD can be applied directly into the extraction alveola.

The DSI Sponge HD plugs mould themselves into the site of extraction or excision wound respectively, stop bleeding and will be absorbed.

CONTRAINDICATIONS

- The DSI Sponge HD must not be used in patients with a known hypersensitivity to collagen products
- The DSI Sponge HD must not be used in infected areas.
- The DSI Sponge HD must not be used during pregnancy and lactation.

WARNINGS

- Chlorine-releasing antiseptic agents, tannins and caustics should not be used in combination with collagen.
- DSI Sponge HD plugs are not intended to be sutured through; doing so may compromise the performance of the DSI Sponge HD.

SIDE EFFECTS

There is a possible risk of intolerance reactions, despite the confirmed low antigenicity.

FURTHER NOTES

Due to the affinity of collagen to bleeding surfaces, dry instruments and gloves have to be used when applying DSI Sponge HD.

The product is intended for single use only.

DSI Sponge HD must not be reused, there is a risk of infection for the patient and/or user and the functionality of the product cannot be guaranteed.

Remaining material, left over due to cutting for example, must be disposed of according to local procedures and guidelines.

NOTES FOR POST-OPERATIVE CARE FOLLOWING DENTAL SURGERY

Actions to be avoided:

- Do not rinse your mouth out on the day of surgery (may dislodge clot).
- Do not drink alcohol or hot drinks on the day of surgery (may increase blood pressure or induce hyperaemia).
- Do not smoke for at least 24 hours (causes vasoconstriction).
- Carry out physical activity on the day of surgery (increasing blood pressure).

Recommended:

- Sip lukewarm drinks carefully (helps avoid clot removal by washing out the socket).
- Bite on gauze or clean, damp handkerchief for 10 minutes if the area bleeds, preferably sitting upright in a chair.
- Rinse mouth gently with hot salt mouthwash or Chlorhexidine (0.2%) mouthwash after 24 hours. Continue regular mouthwash use over the following week, especially after meals.
- Clean teeth as normal, avoiding the area of surgery for 24 hours.

STERILITY

DSI Sponge HD may not be re-sterilized and opened single packs must be discarded. Do not re-use DSI Sponge HD in whole or in part, as it may compromise sterility and/or the performance of the product.

STORAGE

- Store between 0-25°C.
- Store protected from moisture and direct sunlight.
- Do not refrigerate.
- Store out of the sight and reach of children.

PRODUCT CONFORMS TO THE ESSENTIAL REQUIREMENTS OF THE COUNCIL DIRECTIVE 93/42/EEC CONCERNING MEDICAL DEVICES.

SYMBOLS

	Catalogue number
	Batch code
	Use by Date
	Date of Manufacture
	Consult instructions for use
	Caution, consult accompanying documents
	Do not re-use
	Do not sterilize
	Do not use if the package is damaged
	Sterile product
	Temperature limit
	Keep away from sunlight
	Keep dry
	Keep away from children
	Medical Device
	Use by Prescription only
	Manufacturer
	CE Marking and identification number of the notified body

2460

Visit www.dsrael.com or email to info@dsrael.com for more information

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