



For partial or total mummification of the milk tooth root pulp

## DSI Formocresol



### INSTRUCTIONS FOR USE

**PURPOSE:** The product must be used in medical institution.

Used for root canals disinfection, highly efficient in vital pulpotomy due to its antibacterial activity. Being more mild than arsenic paste, it's often used in pediatric dentistry.

### COMPOSITION:

- 19% Formaldehyde, 35% Cresol, Glycerine and Isopropyl alcohol 46%.
- 48.5% Formaldehyde, 48.5% Cresol, 3% Glycerine – Classic formula.

### DIRECTION FOR USE:

- **Immediate treatment:**  
After the coronal pulp extraction, cotton/wool pellet with Formocresol is placed directly on the wound. After 5 minutes, the tampon is removed and cavity is covered with ZOE material (DSI Zoer) and permanently filled.
- **Prolonged treatment:**  
Cotton pellet/turunda with Formocresol is placed on pulp extraction site and covered with the provisional cavity filler (DSI Temp) for one week. After that, tampon is removed, cavity filled with ZOE material (DSI Zoer) and permanently filled.

**Warning:** Contains formaldehyde which can be toxic by inhalation or ingestion. Contact with the skin may cause irritation, burns or hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the

product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician.

### **CONTRAINDICATIONS:**

It should not be used with patients who have a history of severe allergic reactions to any of the components.

**PRECAUTIONS:** Avoid contact with skin and eyes.

### **STORAGE:**

Store in a dry place (humidity < 50%) in tightly closed containers (+5°C...+25°C). Always replace the bottle cap immediately after use. Avoid contact with moisture and direct sunlight! Keep out of reach of children.

### **PACKAGING:**

**Ref: CRESOL** – Bottle, Liquid 15ml

**Ref: CRESOL30** – Bottle, Liquid 30ml

**Ref: CRESOL30F** – Bottle, Liquid 30ml

### **SYMBOLS:**



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



The product is not sterile



Keep Out of Reach of Children



Medical Device



Use by Prescription only



Manufacturer



Catalogue number



Batch code



Use by Date



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material. The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.

Visit [www.dsrael.com](http://www.dsrael.com) or email to [info@dsrael.com](mailto:info@dsrael.com) for more information