



Made for DSI Ltd
59 haAvoda str.
Ashdod, 7706300 Israel
www.dsisrael.com
+972-8931-7235

DSI DiaPaste



Diamond Polishing Paste

DIRECTIONS FOR USE

DESCRIPTION:

DSI Diamond Polishing Paste is used to remove stains and plaque that a regular toothbrush and toothpaste cannot completely remove. The mint-flavored **DSI Diamond Polishing Paste** with minute particles of diamond is available in 2 grits – coarse for tough stains removal and fine for that silky smooth finish and a perfect smile.

Even on the “fine” grit, **diamond paste** is coarser than regular toothpaste, making it more effective in removing stains and smoothing the surfaces of teeth. **DSI Diamond paste** is made with Applied Quartz Technology and is a one-step or two-step universal polishing paste. Smoothing out the surfaces of your teeth limits the adherence of plaque and bacteria and the subsequent development of gingivitis and periodontal disease.

We recommend coming in for a professional clinician check a minimum of two times in a calendar year to make sure we are buffing away stains, and no decay has developed.

INSTRUCTIONS:

DSI Diamond White Paste (coarse):

1. For initial polishing and tooth surface preparation.
 2. Apply a small amount of paste on a dry tooth surface.
-

3. Process with medium speed using electric toothbrush or rubber prophylaxis cap for 60 seconds.
4. Wash the residues from the tooth surface and the instrument head.
5. Rinse your mouth.

DSI Diamond Pink Paste (fine):

1. For the final polishing and to achieve the “dry gloss” appearance.
2. Apply a small amount of paste to the electric toothbrush head / soft polishing wheel.
3. Slowly, using overlapping technique and continuous circular motions, work the surface of the teeth, dry initially, while increasing the speed and pressure from light to moderate.
4. To increase surface gloss, add speed and pressure or use a special buff polishing pad, 30-60 seconds for final high-lustre gloss.
5. Rinse your mouth.
6. Complete the treatment by applying the **DSI Fluoride Gel**.

PRECAUTIONS AND WARNINGS:

1. Carefully read and understand all instructions before using.
2. Do not use with known allergies to one of the components (polyacrylic acid, glycerin, citric acid).
3. Don't exceed one application per 4-week interval to avoid tooth and gingiva irritation.
4. **DSI Diamond Paste** may cause eye irritation upon contact. Protective eyewear is suggested when using this product. If eye contact occurs, immediately flush the eye with copious amounts of water for 15 minutes. Seek medical attention if irritation persists.



Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.