



Made for DSI LTD
59 haAvoda str.
Ashdod, 7706300 ISRAEL
+972-8931-7235
www.dsisrael.com

Light curing flowable temporary filling material

DSI Sil-Flow



INSTRUCTIONS FOR USE

INTRODUCTION

DSI Sil-Flow is a light curing flowable temporary filling material. It is made for a temporary filling of the implant abutments, but can be used as a sealing material for any small pit and cavity.

APPLICATION RANGE

1. Implant abutments temporary sealing.
2. Temporary filling of cavities.
3. Useful for inlay / onlay cases.

APPLICATION METHOD

- **Before the Application:**
 1. Check the expiration date and avoid package contamination.
 2. Read the instructions before first use of the product.
 3. Open the syringe cap and place a disposable tip on the syringe.
- **Application:**
 1. Prepare a cavity.
 2. Extrude the filling material by pressing the plunger of the syringe.
 3. Apply the filling material to the fully fill the cavity.
 4. Light cure for 30 seconds.
 5. The sealing plug may be removed with an explorer when necessary.

- **After the Application:**

1. Remove and dispose the tips.
2. Close the safety cap immediately after use.
3. Store Once-Fil in a cool and dry place.
4. Avoid direct sun-rays and strong ambient light.

PRECAUTIONS AND WARNINGS

1. The material must be handled by dentists and dental specialists according to the instructions before using it.
2. Use of DSI Dental Dam is recommended for all the patients to minimize the tissue contact.
3. Do not use to patients who have rash on their skin, dermatitis or history of hypersensitivity to acrylates, silicones, or other resins.
4. If allergic reaction, dermatitis or rash develops, consult a physician.
5. Might be sensitizing. Avoid repeated contact of uncured dental resin with skin.
6. If accidental eyes contact occurs, wash immediately with a plenty of water and seek medical attention.
7. To avoid cross-contamination, do not re-use tips.
8. Protective device for the respiratory tract is highly recommended before a surgical operation.
9. Keep refrigerated for long-term storage to maintain shelf life.

STORAGE

Store in dark and cool place 1~30 °C. Avoid exposure to direct light.
Keep out of reach of children.

PACKAGING

5 Syringe x 2 g / 10 Mixing Tips.

Ref: SILFLO5



Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material. The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.