



Eugenol dental temporary cement

DSI UniTemp EU



INSTRUCTIONS FOR USE

UniTemp EU Cement Zinc Oxide/Eugenol temporary cement for cementation of provisional restoration (Crown/Bridges/Inlays/Onlays).



Made for DSI LTD
59 haAvoda str.
Ashdod, 7706300 ISRAEL
+972-8931-7235
www.dsisrael.com

PURPOSE

DSI UniTemp EU are Zinc Oxide / Eugenol temporary cement for cementation of provisional restoration (Crown / Bridges / Inlays / Onlays). It is used for cementation or protection of the dental pulp at the surgical operation of restoration or orthodontics.

APPLICATION

- Zinc Oxide / Eugenol Temporary Cement.
- Temporary Cementation of Crown / Bridge / Inlays / Onlays.
- Temporary Cementation of Provisional / Permanent Restoration.

APPLICATION METHOD

Before the Application:

1. Check the expiration date and avoid package contamination.
2. Follow the instructions for use before using it.
3. Open the cap and place a mixing lip on the dual syringe.

Application:

4. Extrude by pressing the syringe piston onto the mixing pad.

CAUTIONS

1. It must be handled by dentists and dental specialists.
2. Recommend the rubber dam to reduce side effects of using for patients who have a history of allergic reaction to this material.
3. Do not use to patients who have an experience skin rash, dermatitis or histories of hypersensitivity or during using.
4. This material is used only to the oral tissue. Avoid contact with eyes.
5. If accidental contact occurs with eyes or skin. Do not use this material for other purposes.
6. Specialists must only use this material. Do not use this material for other purpose.
7. A protective device for the respiratory tract is highly recommended before a surgical operation.
8. Do not use after the expiration date (2 years).
9. Eugenol cement could interrupt polymerization of composite filling materials.

COMPONENT

- Base: Zinc oxide $\geq 60\text{wt}\%$.
- Catalyst: Dimer acid $\geq 60\text{wt}\%$.

CHARACTERISTIC

| Item | Setting time (sec.) | Compressive strength (MPa) | Film Thickness (µm) |
|-------------|---------------------|----------------------------|---------------------|
| Test result | 240~300 | ≤ 35 | ≤ 25 |

STORAGE

- Waste a mixing tip after using.
- Close the caps immediately after use.
- Store this material in a shady, cool and dry place (2-27°C / 35-80 °F).

PACKAGING

DSI UniTemp EU Cement 8g x 1 Syringe

Universal Tip + Lid

Ref:

UTEMP-EU

SYMBOLS



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

*Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.
The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.*