



Bleaching Material for Non-vital Tooth Whitening

DSI UniLite Endo

PURPOSE

DSI UniLite Endo is 60% carbamide peroxide specially formulated gel.

INDICATIONS

Internal teeth discolouration, caused by disease, injury or previous medical treatments.

DIRECTION FOR USE

For temporary filling of cavities, consistency of the mixture should be made so as it doesn't stick to the glass container or to the spatula. For root canal filling received mixture must have a creamy consistency.

INSTRUCTIONS FOR USE

Examine the X-ray to analyze the height of the alveolar crestal bone and establish the proper apical endodontic seal.

Make sure that the tooth is pulpless and endodontic treatment is successfully finished. Remove all filling material from the access opening (determined by the radiograph examination). Any restorative or endodontic filling material must be removed to achieve full contact of the bleaching agent with the internal tooth structure. Make sure that the canals are sealed.

Mix **Glass-FX** glass-ionomer powder at the ratio that creates a firm putty, place it into the cavity bottom to assure a seal. Attach tip to the syringe and express **UniLite Endo** into the pulp chamber while avoiding the soft tissues.

Cover with the cotton pellet and seal a cavity with non-eugenol provisional restorative material. We recommend **DSI Temp**.

Repeat the procedure every 3-5 days until the desired whitening effect is achieved.

Wait 7-10 days before the final tooth restoration. Remove provisional restorative and bleaching material residues. Rinse and clean the access opening. Restore using standard dentistry procedures.

PRECAUTIONS AND WARNINGS

Explain the process and the limitations to the patient and understand the patient's expectations before starting the treatment. Slight over-bleaching is recommended, keeping in mind the possible whitening regression that may occur.

Keep away from soft tissues. Should contact occur, rinse immediately with lots of water. Do not use for injections. Keep out of reach of children.

Do not use UniLite Endo on the patients with known allergic history or sensitivity to peroxides or glycerine.

STORAGE

Keep refrigerated (+5°C...+10°C) in a dry place (humidity < 50%).

PACKAGING

1 x Syringe, 1.5g

Ref: UL-ENDO

2 x Syringe, 1.5g

Ref: UL-ENDO2



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.