



Dental material for tooth socket

DSI Alvo-G

A horizontal row of 14 squares, each a different shade of gray, ranging from very light to medium gray.

INSTRUCTIONS FOR USE

DESCRIPTION

DSI Alvo-G is used for dry socket treatment and post extraction dressing. It provides a soothing effect throughout the healing process. Its consistency allows for easy filling of the socket. DSI Alvo-G acts as a physical barrier after teeth extraction offering protection, once it acts as a moldable alveolar drain plug, forming a solid framework for the tissue regeneration, avoiding contamination of the bone walls. The application is very simple and its paste consistency allows the material to remain in contact with alveolar walls for more time. After placement it requires no special attention other than observation of the healing process. The Eugenol free formulation causes non-irritation in contact with alveolar tissue.

DIRECTION FOR USE

ALVO-G ANTISEPTIC PASTE (WITH PROPOLIS& IODOFORM):

Barrier material, made with the natural components (propolis & beeswax), prepares contamination of alveolar socket tissue. Recommended for the prevention and treatment of alveolitis. Presence of the propolis provides a multiple effect: a healing process stimulation, antiinflammatory and slight analgesic action.

Iodoform was incorporated to the product to assist and complement the action of the filling by the enhancing of the growth of cells, and the formation of granulation tissue, besides having a light antiseptic and astringent effects. It shortens the postoperative period and greatly improve patient tolerance.

After tooth extraction, fill the socket with Alvo-G. It's easily applied and stays in the socket, forming a plastic bandage, quickly reducing inflammation and pain. In a period of 30 days, Alvo-G will be gradually eliminated from alveolar cavity and a normal healing process will take place.

In case of alveolitis:

1. Anesthetize by regional blockade.
2. Perform the alveolar curettage removing all necrotic tissues.
3. Rinse well with saline.
4. Provoke bleeding and wait until the socket will be full of blood.
5. Fill the alveolus with Alvo-G and proceed the suture if necessary.

ALVO-G HEMOSTATIC PASTE (IODOFORM-FREE):

Self-resolving paste that covers the bleeding surface. Upon contact with the blood it swells and presses the capillaries. This, together with the incorporated aminocaproic acid, provides quick and strong hemostatic effect. Additional ingredients provide antiseptic and soothing anesthetic actions.

Take a small amount of paste. Roll a ball with a fingers. Insert into the socket, cover with the small cotton pellet or pad and instruct the patient to bite for 10 minutes. After this, remove the pad. The material resolves completely in 6 hours (or turns into the jelly-like plug- the result depends on the quantity of the material use and individual factors, such as the blood impregnation).

Warning:

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take it internally. Contact with the skin may cause irritation, burns or hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then

wash well with soap and water. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention. If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reactions to any of the components.

CONTRAINDICATIONS

Do not use the product for patients with allergic sensitivity to the product's components.

In case of patient's hypersensitivity to iodine, use Alvo-G iodoform-free paste.

Alvo-G iodoform-free should not be used with antibiotics based on penicillin.

STORAGE

Store at a temperature between 5°C to 30°C. Close the cap carefully after use. Shelf life not affected after opening if stored in a cool dry place.

Avoid contact with moisture and direct sunlight!

Keep out of reach of children.

PACKAGING

Alvo-G with Propolis and Iodoform Paste (jar), 10 g

Ref: ALVOGPRO

Alvo-G Hemostatic Iodoform-free Paste (jar), 20 g

Ref: ALVOG20

SYMBOLS



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Flammable



Keep away from children



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

Visit **www.dsisrael.com** or email to **info@dsisrael.net** for more information

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.