



Light-cured Resin Barrier

DSI UniLite Dam

PURPOSE

DSI UniLite Dam is viscous, light-cured, passively sealing resin in the syringe. Used to isolate and protect soft tissues. Easy to apply and remove, yet strong enough to maintain a barrier. The dark blue colour of Unilite Dam contrasts perfectly with gingiva and teeth, making it an ideal solution for in-office whitening.

DIRECTION FOR USE

Isolation:

Remove the cap and attach the tip to the syringe. Place the plastic cheek retractors. Rinse and dry teeth and upper gingiva. When isolating the mandibular jaw arch, use a saliva ejector and cotton rolls to prevent saliva from flowing through anterior embrasures. Apply the UniLite Dam through the embrasures onto the cingulum. Light cure for 20 sec.

Slowly apply the UniLite Dam onto the gingiva to build a 5mm wide and 2mm thick strip. Lap 0.5mm onto the enamel. Extend the strip beyond the last tooth to be bleached. Fill the open embrasures completely, where exist. Resin "buttons" extending through the gingival embrasures improve the retention of the barrier. Light-cure for 20 sec. using a scanning motion (where less than a full arch needs to be protected, scan the barrier for 3-4 sec. per tooth). Check the resin for a full cure using a probe.

Clean Up:

After cleaning the whitening gel residues, rinse teeth with an air/water syringe and vacuum. Using the explorer or another suitable instrument, lift the UniLite Dam from the gingival surface. The material usually lifts in one piece. Remove interproximal leftovers using explorer and floss. Rinse with firm air/water spray.

PRECAUTIONS AND WARNING

Contains methacrylate resins. Do not use on patients with known sensitivity to resins, especially methacrylates. If allergic reaction, dermatitis or rash develops, consult a physician. Confirm the patient's oral health before using the product.

The tips are disposable and intended for single use.

To avoid cross-contamination, do not re-use tips.

STORAGE

Store in a dry place (humidity < 50%) in tightly closed containers (+5°C...+27°C).

Do not refrigerate. Re-cap syringes between uses.

Keep out of reach of children.

PACKAGING

5 x Syringe 2.0g, 10 tips

Ref: UL-DAM

Dental Solutions Israel Ltd.

59 haAvoda str., Ashdod,

7706300 Israel

Tel: +(972) 8-931-7235

www.dsisrael.com



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.