



Made for DSI Ltd.
59 haAvoda str.
Ashdod, 7706300 Israel
+972-8931-7235
www.dsisrael.com

Caries indicator dye

DSI Cindicador



INSTRUCTIONS FOR USE

DESCRIPTION

DSI Cindicador (Caries indicator) is a green FD&C dye in a propylene glycol base. Caries indicator assists the clinician in visual differentiation between the infected carious dentine and sound and re-mineralizable dentine by its ability to selectively dye the carious dentine.

COMPOSITION

Green FD&C dye, Propylene Glycol.

INDICATIONS

Caries indication by visual differentiation between infected carious dentine and sound, remineralizable dentine.

DIRECTION FOR USE

Make conservative cavity access. Wash and dry carious dentine. Dispense 1 or 2 drops of **DSI Cindicador** in to a dappen dish and apply to the cavity with a brush or cotton pellet. Wait for 5 to 10 sec and rinse with water spray. Remove green stained infected dentine. Repeat the application until carious dentine is removed and no further dentine staining occurs.

Warning:

Avoid prolonged or repeated contact with skin (allergic contact dermatitis), and eyes. Avoid prolonged inhalation. Do not take internally contact with the skin may cause irritation, burns of hypersensitivity. If contact with skin occurs immediately wipe off thoroughly with cotton and alcohol and then wash well with soap and waler. If skin rash and sensitization or other allergic reaction occurs discontinue use and seek medical attention.



If the product comes into contact with the eyes, rinse thoroughly with fresh water and consult a physician. It should not be used with patients who have a history of severe allergic reaction to any of the components.

CAUTION

A low speed rotary instrument should be used for removing the carious dentine since frictional heat generated by the high speed tool can cause pain before all the stained outer carious dentine is removed.

CONTRAINDICATIONS

Cindicator does not contain any hazardous ingredients in sufficient concentration to be harmful to human beings and causing irritation to oral tissues when not used as directed.

STORAGE

Store at temperature between 10 °C to 24 °C. Close the cap carefully after use. Keep away from moisture. Shelf life not affected after opening if stored in a cool dry place.

Shelf life: 3 Years from the date of manufacturing.

PACKAGING

Plastic dropper bottle, liquid 5ml

Ref: CINDICATOR

SYMBOLS



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Flammable



Keep away from children



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.

