



**INSTRUCTIONS FOR USE**  
**UniLite Temp**  
**Temporary Flowable Filling Material**  
**Light-curing**

**PURPOSE**

**UniLite Temp** is a flowable, light-curing material for all types of temporary fillings, sealing and other treatment solutions such as covering the gingival margins

**DIRECTIONS OF USE**

Rinse and dry the cavity before application of **UniLite Temp**.

Apply the **UniLite Temp** directly to the cavity using the provided tip. Don't pull back the plunger during or after use.

Spread with the probe or spatula if necessary.

Light-cure (polymerization light with a minimum of 500mW/cm<sup>2</sup>): **10s** if the film thickness is up to 1mm (gingival margins covering), **20s** for the thickness of up to 5mm, and **40s** for thickness over 5mm.

Hold the lamp as close as possible to the cavity, otherwise, the curing depth may be reduced.

After curing, excess materials are removed with a round bur or scalpel. The temporary restoration might be finished or polished using the silicon point.

**PRECAUTIONS AND WARNINGS**

Do not use this product on patients with a known sensitivity to methacrylates.

If allergic reaction, dermatitis or rash develops, consult a physician.

The tips and delivery syringes are disposable and intended for single use.

To avoid cross-contamination, do not re-use tips.

Phenolic substances, especially preparations containing eugenol, inhibit the curing process.

Use of zinc oxide-eugenol cement or other eugenol-containing substances in combination with **UniLite Temp** is not suggested.

**PACKAGING**

UniLite Temp 2g syringe - 5 pcs

Mixing tips - 10pcs

**STORAGE**

Store in a dry place (humidity < 50%) **in tightly closed containers** (+5°C...+27°C).

Do not refrigerate.

Re-cap syringes, disinfect with an intermediate level disinfectant between uses.

*Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.*

*The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product.*

*Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on user.*

Made for DSI LTD  
59, haAvoda str.  
Ashdod, 7706300  
ISRAEL  
+972-8931-7235  
[www.dsisrael.com](http://www.dsisrael.com)