



Made for DSI LTD
59 haAvoda str.
Ashdod, 7706300 ISRAEL
+972-8931-7235
www.dsisrael.com

Extra-pure 100% Eugenol oil

DSI Eugenol Oil



INSTRUCTIONS FOR USE

PURPOSE

DSI Eugenol is used for the paste preparation, for root canal and temporary cavities filling, or as sedative lining/dressing under non-composite dental restorations. Mainly used with zinc oxide.

DIRECTION FOR USE

For temporary filling of cavities consistency of the mixture should be made so as it doesn't stick to the glass container or to the spatula. For root canal filling received mixture must have creamy consistency.

CONTRAINDICATIONS

Do not use the product for patients with hypersensitivity to the product's components. Do not use under composites since they may disturb polymerization process. Do not use the product for pregnant women. May cause an allergic skin reaction. May cause irritation to the mucous membrane of the oral cavity and eyes. In case of contact rinse immediately with water and seek medical advice. In case of consumption drink plenty of water, don't extract emesis. Seek medical advice. Use rubber-dam while working with the product. Use gloves, protective glasses and clothing.

PACKAGING

Liquid (bottle), 50 ml

STORAGE

Store in a dry place (humidity < 50%) in tightly closed containers (+5°C...+30°C).

Always replace the bottle cap immediately after use.

Avoid contact with moisture and direct sunlight!

Keep out of reach of children.

Ref: EUGENOL

SYMBOLS



Consult instructions for use



Caution, consult accompanying documents



Temperature limit



Keep away from sunlight



Keep dry



Manufacturer



Catalogue number



Batch code



Use by



Date of Manufacture

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of the material.

The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer falls on the user.
