

Required Media

Whether you are (a) sending in your case to get started, (b) fitting your device for the first time, or you are (c) seeking help to solve fit & function concerns; your media provides OrthoPets the data required to provide you with the service and support your pet deserves.

Camera Positioning Is Critical

To capture the required pictures of your pet, it is critical to:

- Hold your camera at the same elevation as your pet's affected anatomical joint
- Hold your camera stationary while your pet walks through your field of view

FOR PROSTHETIC ONLY

OrthoPets has provided a short instructional video demonstrating video techniques for you:

<https://vimeo.com/261067863>

3:2 Media Process

Using our simple 3-2 media process, you can quickly and effortlessly provide OrthoPets with the required media we need to assess your pet's conformation, angulation, and gait pattern.

3 Still Photos of your pet's affected limb while your pet is standing:

- Outside (lateral) surface of limb
- Front (Cranial) surface of limb
- Back (caudal) surface of limb

2 Simple Ways to send media:

- OrthoPets utilizes "Filemail" transfer program enabling you to send large files from your mobile device, tablet, or desktop computer
<https://orthopets.com/file-mail> or you can email directly to info@orthopets.com
- DropBox Link:

<https://www.dropbox.com/scl/fo/ms7i5hcidfbg2pj46bjio/h?dl=0&rlkey=oggmzfjowh3z5y6a0zi6r5k9>

