

# Quick cleaning guide

for Channel-less ENT Endoscopes




This Quick Cleaning Guide only applies to channel-less ENT scopes that are disinfected in UV Smart's D60 after being cleaned. This Quick Cleaning Guide is not a substitute for the manufacturer's official instructions for cleaning and disinfecting the channel-less ENT endoscope. Always be sure to also consult the official instructions for cleaning and disinfecting the channel-less ENT endoscope!

## Supplies

- » non-sterile gloves
- » hydrophilic, lint-free gauze pads (min. 4x4 in)
- » clean, disposable tray
- » bottle of sterile water
- » leak tester (+ 70% rubbing alcohol)


1

Immediately after the examination, use a clean, moistened gauze pad to remove contaminants from the endoscope's input section, distal end, and lens.

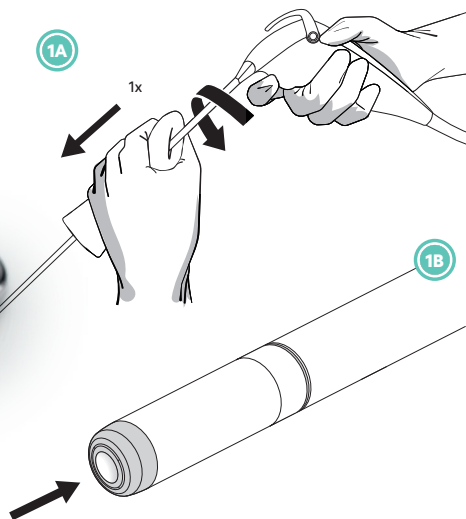

2

As soon as possible after step 1, use a gauze pad moistened with sterile water to clean the control housing and all related parts for at least 20 seconds.

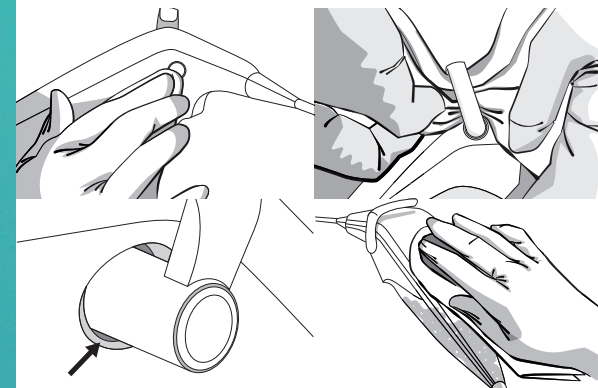

Clean the input section from the control housing to the distal end, the distal end itself, and the lens (see 1a, 1b) with a gauze pad moistened with sterile water.

**Repeat this step two more times.**

3

In step 2, perform a leak test according to the manufacturer's instructions.

4

Inspect the endoscope again for any contaminants. Double-check that the endoscope is at least drip-dry.



Only an adequately cleaned and, at a minimum, drip-dry channel-less ENT endoscope may be disinfected in UV Smart's D60!

 UVsmart