

SUPERFORM TRAY SYSTEM



SUPERFORM TRAY SYSTEM - Reusable impression trays in polycarbonate for edentulous and partially edentulous mouths.

System specifically designed for taking high precision impressions in the production of mobile dentures.

Applying the traditional systems of elaboration of the dentures, a perfect result can be achieved using Superform anatomical impression trays, equipped with tubercles behind the upper and lower molars to facilitate correct pressure on the gums, which is very important for a perfect fit of the dentures.

Only for professional Dental use.

TECHNICAL FEATURES:

Material: Reusable polycarbonate that can be sterilised in a steam autoclave at 140°C.

IMPORTANT: The result of the process of sterilisation is the duty and responsibility of the professional, who should control and verify this process in conformity with

the harmonised European Standard EN 554 (UNI EN ISO 554) Sterilization of medical devices. Method for convalidation and systematic control of steam sterilisation.

The use of adhesives is unnecessary thanks to the presence of three systems of mechanical retention on the surface of the impression holder:

Dotted surface

Stabilising projections

Perforations

The careful design takes into consideration all the anatomical details of the gum arch.

Each product is hand finished and individually checked.

ADVANTAGES

The anatomical shape of the dental arch combined with the provision of tubercles behind the molars ensures a technically perfect impression for the 13 different jaw sizes, which as regards jaw impression holders, these reproduce the irregularities, sliding delicately around the spaces at the rear of the gums without irritating the palate.

With Superform Tray System is possible to achieve the accuracy of individual/anatomical impression holders without the cost of the individual impression holder.


Impression holders of the Superform line can be used with all types of materials for making impressions, but for best results we recommend the following materials: Cromatic alginate, Kromalgin plus, Protesil alginates and the Protesil/Prestige line of silicones.

A compass is provided for making correct measurement of the space between the corners of the mouth of the patient or between the corners and any existing teeth. This makes it possible to select the most suitable impression holder for the patient more accurately and rapidly.

13 lower moulds for edentulous mouths
7 upper moulds for edentulous mouths
4 moulds for partially edentulous mouths
1 steel compass for making measurements

The lot number is indicated on all packages. In communicating with the supplier, it is important to mention this information so as to make it possible to identify the lot involved promptly and correctly.

101024 - 24 polycarbonate trays + 1 steel calliper

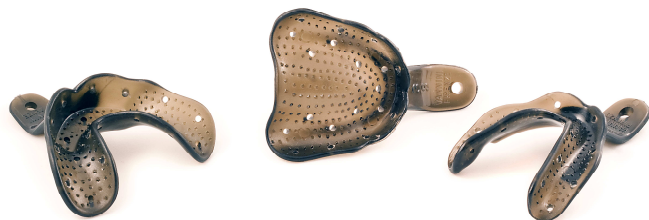

Via di Campigliano 55/A 50012 GRASSINA (FI) – ITALY Phone +39 055 644698 Fax +39 055 644697
E-mail : info@vanninidental.com www.vanninidental.com