

## GENERAL CARE

- Clean Rechargeable Sound Amplifier daily with the provided brush to avoid unwanted earwax build up.
- Brush the device back-and-forth across the ear bud (using the supplied brush) to remove wax and dirt deposited within the ear bud.
- Make sure the ear bud is clear of all debris and dirt.
- Wipe the Rechargeable Sound Amplifier with a clean, dry cloth only.

## STORAGE

When you are not using your Rechargeable Sound Amplifier clean it and store in a dry cool place.


## TROUBLESHOOTING

### Important Information

- It is recommended a certified physician be seen before deciding to wear any sound amplifier and to follow up with constant checkups to make sure none of the following apply.
- The following are conditions under which you should NOT wear the Rechargeable Sound Amplifier. Do not wear the Rechargeable Sound Amplifier if:
  - You have pain and discomfort in your ear.
  - You have traumatic or visible congenital deformities on your ear.
  - Have acute or chronic dizziness.
  - Have history of sudden or rapidly progressive hearing loss that has occurred within the last 90 days.
  - Have unilateral hearing loss or sudden/recent onset within the last 90 days.
  - Bone conduction thresholds are better than your air conduction thresholds by 15dB or greater.
  - Significant build up of wax or other materials in your ear canal.
- Rechargeable Sound Amplifier IS A SOUND AMPLIFIER AND CAN BE USED AS A HEARING AID ALTERNATIVE. PLEASE KEEP THESE INSTRUCTIONS IN A SAFE FOR FUTURE REFERENCE.

DUE TO CONTINUOUS PRODUCT IMPROVEMENT, THE PICTURE ON THE PACKAGING MAY DIFFER SLIGHTLY FROM THE ACTUAL PRODUCT.  
Designed in the USA Assembled China

## DIAGRAM I

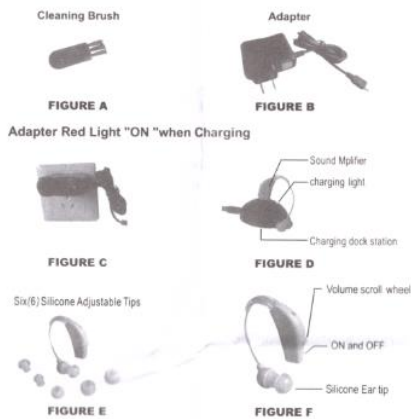

## CAUTION

 <b>Note</b>	Rechargeable Sound Amplifier is NOT for use by children under 15.	 <b>Note</b>	<b>DO NOT WET</b>
 <b>Note</b>	Keep out of reach of children and pets.	 <b>Note</b>	Discontinue use of product in the event of irritation. It is recommended that you consult a doctor before continued use of this product.
 <b>Note</b>	The supply terminals are not to be short-circuited.	 <b>Note</b>	DO NOT remove the Rechargeable Sound Amplifier from the docking station until the light turns green (again) – signaling fully charged.

## Rechargeable Sound Amplifier Instructions

- Please retain instructions for future reference.

- Sound amplifier helps to amplify sound up to 30x with low background noise.
- Inbuilt battery convenient to recharge. There is no battery's required to replace again.
- Fully charged replaceable battery can stand up to 8-10 hours continues use.
- Best hearing keep the ear tips clean and replace.
- Interchangeable ear tips to seal out the noise.

## Set includes

- Rechargeable Sound Amplifier
  - Adapter
  - Charging Docking Station
  - Cleaning Brush
  - Six (6) Silicone Adjustable Tips

## SPECIFICATION

- Battery: 1.2V
- Working Current  $\leq 3\text{mA}$
- Signal to Noise Ratio  $\leq 28\text{dB}$
- Frequency: 100-4000Hz
- Gain: 30dB
- Distortion:  $\leq 6\%$

## GETTING STARTED

- It is important to follow the instructions (1-3) below for the first two times charging the Rechargeable Sound Amplifier

1. Slide the switch down located on the back of the earpiece to turn the Rechargeable Sound Amplifier OFF.
2. Connect the adapter's cord into the rechargeable docking station plug the opposite end of the adapter's cord into an outlet. Upon plugging, the red light on the adapter (Diagram I SEE: Figure C) will illuminate and the green light on the docking station (Diagram I SEE: Figure D) will light simultaneously.
3. Place the Rechargeable Sound Amplifier into the recharging docking station. The green light on the docking station will turn red if connected properly.

- Note: DO NOT remove the Rechargeable Sound Amplifier from the docking station until the light turns green (again) – signaling fully charged.
- Important: Follow the steps below to ensure longer time.

## DIRECTIONS

### How to wear the Rechargeable Sound Amplifier

- The Rechargeable Sound Amplifier comes with additional 6 different silicone ear tips. Select
- the silicone tip that best fits your ear and place onto the Rechargeable Sound Amplifier
- Rechargeable Sound Amplifier can be worn in either the right or left ear.
- Below the volume scroll wheel is the on/off switch. This must be in the "on" position or in the "up" position for use, and slide down "OFF" turn the device OFF.
- Take the ear piece and place it over the top of the desired ear. The ear piece will curve around and fit behind the ear.
- Then take the ear bud of the Rechargeable Sound Amplifier which is at the end of the small wire and carefully insert into the ear canal.
- Adjust the volume to the desired level.