

Attention: Your delivery contains more than one device. Follow these master instructions to complete your blood collection:

- 1) Follow the instructions on the right to schedule your return shipment before beginning your blood collection
- 2) Follow the Prepare and Collect steps below for collection kit #1
- 3) Then, follow the Prepare and Collect steps below for collection kit #2
- 4) Return both samples and corresponding lab forms together using the pre-labeled device box

Scheduling your return shipment:



Scan the QR code on the left or visit wwwapps.ups.com/pickup/schedule to arrange a pick-up at your home by entering the tracking number on your pre-paid shipping label. To schedule by phone, call 1-800-742-5877. You can also drop off your sample at a UPS facility immediately after collection.

Required Items:

timer
pen
razor

1. Prepare



Identify collection site - for best results remove any hair.




Remove items from box and device from



Remove tube and twist off



With fill lines facing out, press tube into device until snug.

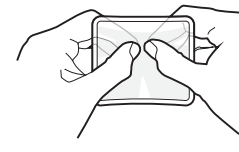

Follow heat pack instructions to activate. If pack does not heat up, discard and use hand instead.



Increase blood flow at collection site



Rub heat pack or hand up and down your shoulder for 2 minutes to warm and increase blood flow.



Clean area with alcohol pad and allow to dry.




Remove clear plastic cover over red button.


2. Collect




You may not feel the device. Leave on until full.

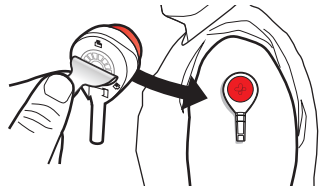


Peel white tab away from device and stick device to shoulder. Hang arm straight down at side.

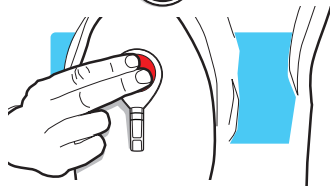

Press button all the way down. Wait 2 seconds and release. 1-2 minutes may be needed for blood to appear.

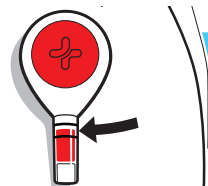


Allow tube to fill to the top line. Usually this happens within 10 minutes, but could take up to 30 minutes.

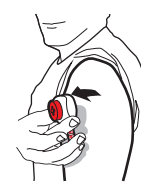

Remove device by peeling from one side. Stop bleeding with supplied bandage.



Remove tube with a slight twist and pull downward. Snap cap fully onto

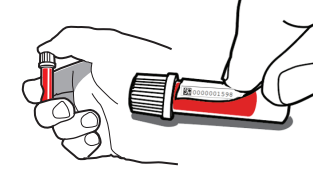


Apply color coded barcode label to tube. This label can be found on the collection date form.



Invert tube to mix the blood sample. A tap may be needed to get sample moving. Repeat 10 times. Gel will not mix.


3. Ship

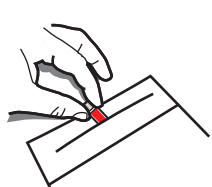

Place tube in clear return bag. Do not remove pad.




Peel off liner to expose adhesive.

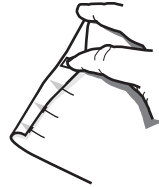

Follow bag instructions to seal bag.



Fill out the collection date form included with your kit. Each device will have a corresponding form.

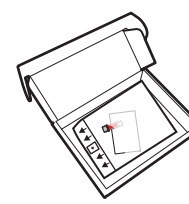

Remove all other items from the pre-labeled kit box, then place both sample return bags and collection date forms in the box.



Peel adhesive backer off box and seal.



Return samples on the same day of collection per instructions above.

Warnings

Prescription use only.

Do not use after expiration date.

Single use only.

Keep out of reach of children.

Minor bruising or residual marks may occur at the sample collection site.

Do not resterilize.

Do not use if pouch has been opened or damaged.

For external use only.

Intended for self-use or use by a healthcare provider.

Use while seated as fainting may occur with any blood sampling procedure.

Refer to manufacturer's instructions for individual components.

Not for use on infant heels.

Users must validate use of product for their specific diagnostic assay.

Intended Use

Allows self-collection of a capillary blood sample in conjunction with a compatible sample collection reservoir that is shipped to an accredited laboratory for analysis.

Disposal

Dispose of used device according to your local or state guidelines for disposal of used sharps.



Tasso, Inc.
1631 15th Ave. W., Suite 105
Seattle, WA 98119 USA
+1 206 822 4186

Kit Contents

Tasso Blood Collection Device

Test Tube

Heat Pack

Bandage

Alcohol Swab

Return Bag with Absorbent Pad

Instructions for Use




Symbols


REF Catalog number

LOT Lot number

IVD For in vitro diagnostic use


 Use by date

 Consult instructions for use

 This device contains sharps

 Do not use if package is damaged

 Single use only

 Do not resterilize

 Store at 18-25°C

Rx ONLY Prescription use only

STERILE **R** Sterilized by irradiation

Tasso+

Instructions For Use



First time using Tasso?

Watch our instructional video

www.tassoinc.com/tassoplus-video

or scan the QR code to access.

Contact us for help: 1-408-708-8451