


# ACM SURGICAL

## ADVANCED COLLAGEN MATRIX

MPM  
MEDICAL

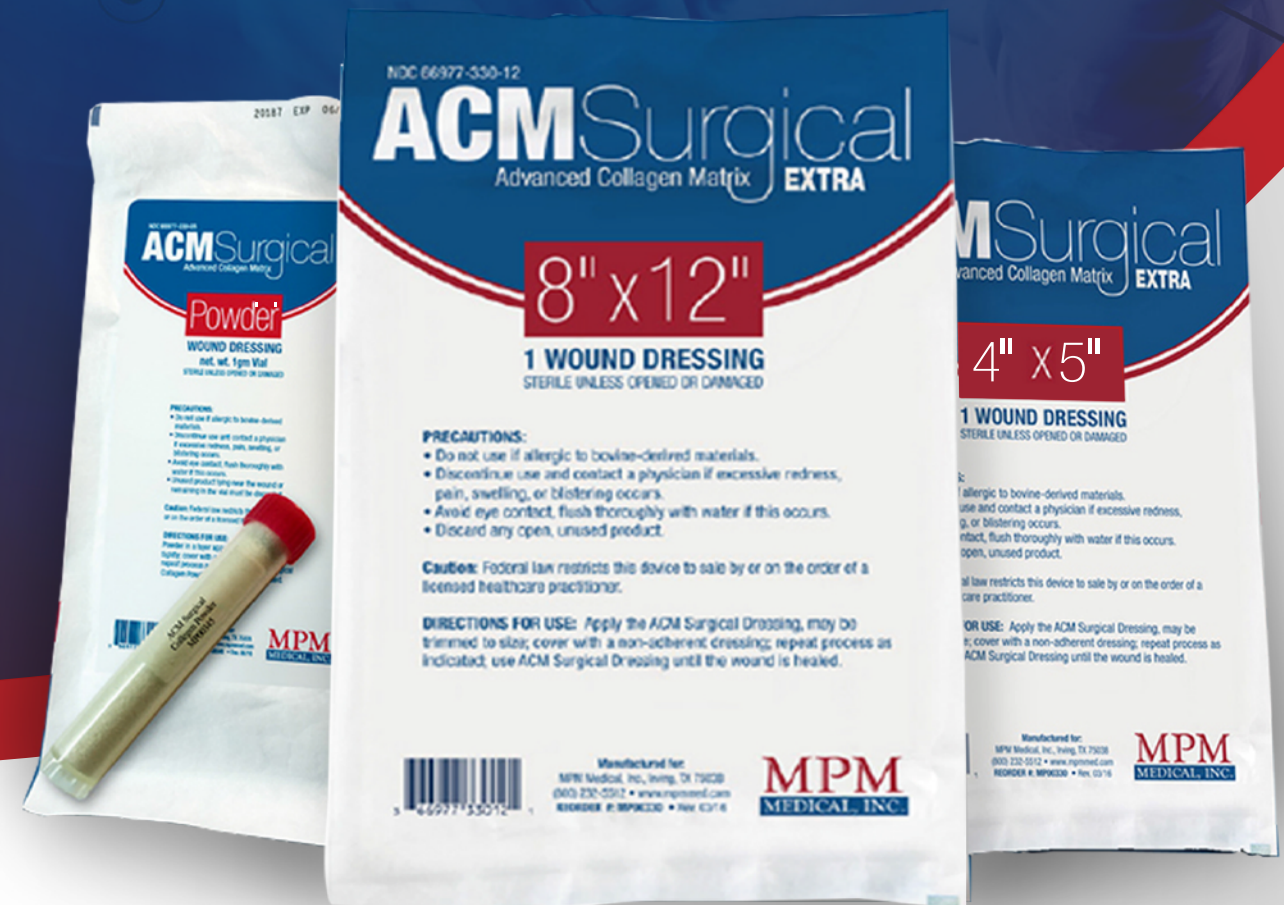

**ACMSurgical**  
EXTRA Advanced Collagen Matrix

(972) 893-4090  
info@mpmmmed.com  
www.acmsurgical.com  
1801 Big Town Blvd. Suite 300  
Mesquite, TX 75149

MPM  
MEDICAL

www.acmsurgical.com

## ABOUT US

MPM Medical specializes in Advanced Wound, Burn and Surgical products, that help facilitate wound healing.

## ACM SURGICAL EXTRA COLLAGEN MATRIX FEATURES

- 100% Type 1 Bovine
- Absorbs up to 30x its own weight
- Conforms to any wound site, forming an occlusive barrier
- Biocompatible, biodegradable and synthetic free
- Effective in all wound phases
- No special storage requirements
- Sterile
- Promotes autolytic debridement
- Minimizes potential for scarring

	PRODUCT#	SIZE	PKG.
ACM Surgical Collagen	MP00329	8" x 12" Sheet	1 matrix/Pch., 5 Pchs. per Box, 10 Boxes/Cs.
ACM Surgical Extra Collagen	MP00330	8" x 12" Sheet	1 matrix/Pch., 5 Pchs. per Box, 10 Boxes/Cs.
ACM Surgical Extra Collagen	MP00332	2" x 2" Sheet	1 matrix/Pch., 5 Pchs. per Box, 10 Boxes/Cs.
ACM Surgical Extra Collagen	MP00333	3" x 3" Sheet	1 matrix/Pch., 5 Pchs. per Box, 10 Boxes/Cs.
ACM Surgical Extra Collagen	MP00334	4" x 5" Sheet	1 matrix/Pch., 5 Pchs. per Box, 10 Boxes/Cs.
ACM Surgical Extra Collagen	MP00337	7" x 7" Sheet	1 matrix/Pch., 5 Pchs. per Box, 10 Boxes/Cs.
ACM Surgical Collagen	MP00345	1 gm Vial	1 matrix/Pch., 5Pchs. per Box, 10 Boxes/Cs.

## DIRECTIONS FOR USE OF COLLAGEN SHEET:

- Cleanse wound with MPM Wound and Skin Cleanser.
- Debride necrotic tissue, biofilm or other non-viable tissue to prepare the wound site.
- Hydrate the collagen matrix sheet before application to the wound surface.
- Apply the porous membrane side of the collagen sheet to the wound, extending beyond the wound surface.
- Once the collagen sheet matrix has been positioned on the wound, remove the non-adherent backing.
- Secured the matrix in place, using the clinician's choice of fixation.
- Cover the collagen matrix with a non-adherent dressing.
- Repeat applications may be required, based on the wound response and clinician judgment.



## DIRECTIONS FOR USE OF COLLAGEN POWDER:

- Cleanse wound with MPM Wound and Skin Cleanser.
- Debride necrotic tissue, biofilm or other non-viable tissue to prepare the wound site.
- Apply ACM Surgical Extra Advanced Collagen Powder on the wound surface approximately 1/4" thick.
- Do not pack tightly.
- Cover the collagen powder with a non-adherent dressing.
- Repeat applications may be required, based on the wound response and the clinician judgment.



(972) 893-4090



[www.acmsurgical.com](http://www.acmsurgical.com)