
# Welcome to Caption AI - Version 1.2.2


Congratulations on upgrading Caption Al! Caption Health is constantly working to build new capabilities and algorithms, refine features, and make Caption Al easier to use and integrate into your workflow.

#### Here's what's new in this version:

# **Modality Worklist & Barcode Scanner Support**

Select orders from a Modality Worklist or use a 1D barcode scanner to populate patient details.

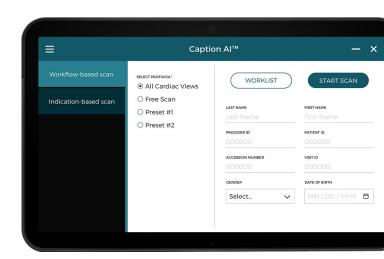

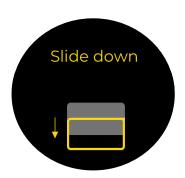


## **Updated Prescriptive Guidance**

The Quality Meter and Prescriptive Guidance have been updated for a more responsive scanning experience.<sup>1</sup>

## **Additional Imaging Presets**

Optimize image quality for non-cardiac views with support for additional presets in Free Scan and Color Doppler modes.<sup>2</sup>

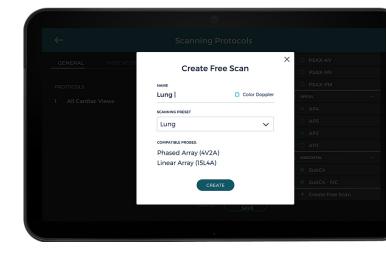

- <sup>1</sup> Note: Prescriptive Guidance updated for the IVC, SubC4, and AP4 views. Quality Meter updated for the IVC view.
- <sup>2</sup> Note: Free Scan and Color Doppler modes do not include Al guidance.

# **Caption Health**

#### **Enhanced Custom Protocols**

Add free scans to a custom protocol to seamlessly transition between guided cardiac views and unguided lung, DVT, or other scan types.

#### **Metadata Export**

Export metadata to identify insights and trends in utilization, image quality, and more. Contact Caption Health for help with analysis and insights.

#### **Linear Probe Support**

Use the linear transducer for vascular and musculoskeletal scanning or ultrasound-guided procedures.

#### **Enhanced Performance**

Faster load times, depth changes, and view transitions create a smoother scanning experience.

### Other Tips

- Use filters on Modality Worklist queries for faster load times.
- Training videos and other helpful resources can be found on the device's desktop.
- Note that stopping an ongoing DICOM export will pause the software for 15-20 seconds.
- Refer to the Quick Reference Guide and other documents provided for more information.

#### **Version History**

Version	Description	Date
1.1.0	Initial release	9/15/2020
1.1.1	Minor bug fixes and enhancements	10/20/2020
1.2.0	Feature release, performance and guidance enhancements	04/2021
1.2.1	Minor bug fixes	05/2021
1.2.2	AutoEF hotfix	05/2021

#### Questions? We're here to help.

