moveUP

# moveUP

Overview

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English Label:

# moveUP

About the  
moveUP app,  
tablets and activity trackers

## moveUP app & devices

The moveUP app runs on:

- iPhones/iPads iOS 10.0 or newer
- Android systems (Android 7.0 and newer)
- moveUP needs an internet connection when in use
- moveUP is compatible with the following browsers:
  - Chrome/Safari/Firefox

### Tablets

In specific cases the following tablets are provided and are supported:

- Lenovo Tab 3 - Android 6.01 or Higher
- Huawei MediaPad T3 - Android 7.00 or higher

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### Download size

- iOS - 78.4MB - version 1.33.1
- Android - 39.6MB - version 1.33.1

For your data safety please enable the 2 factor authentication (2FA) and use a safe network.

Download the latest app version

## Choose region

Europe >

US >

## Important information - Europe

Download the user manual  
**For patients**

Download the user manual  
**For healthcare providers**

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- moveUP COMPANION = monitoring and information, no active follow-up
  - moveUP companion offers targeted information and evolution reports to patients. Their care team takes the evolution reports into account in their further recovery when appropriate.
- moveUP COACH = active follow-up by healthcare team
  - moveUP coach offers targeted information and evolution reports to patients. Their care team is more active involved and takes the evolution reports into account and can provide advice and exercise suggestions through the digital platform.
- moveUP THERAPY = active follow-up by healthcare team, with data driven validated care protocol
  - moveUP therapy offers a data-driven validated care protocol with certain category and level of exercises and activities, specifically targeted to the individual patient. Their care team can manually adapt the data driven validated protocol when needed. Patients can fully rehabilitation with moveUP without leaving their home environment.

Optional functionality that can be enabled: interoperability with Class IIa continuous passive motion (CPM) medical device.

For knee and hip patients who are using a Class IIa CPM medical device, moveUP can interoperate with the Class IIa CPM software. moveUP acts as a facilitator to easily assign a designated CPM exercise protocol chosen by a physician to a patient and to display the performed CPM exercises in the medical dashboard.

# moveUP

- moveUP is not an emergency tool. In case of emergency please contact your doctor or call 112.
- moveUP is not intended to monitor/treat vital parameters of critical illness.

- moveUP companion is not a replacement for any treatment you need to follow, but is used as an addition.
- Patients need to be aware that if questionnaires are not completed on a frequent basis or not answered truthfully, the care team has more difficulty understanding your health situation & progress.

- The moveUP app offers personalised healthcare based on the patient's profile and therefore can not be shared, borrowed or traded between users. The information and advice of the moveUP application do not apply to any other patient other than the one whose profile is stored in the application. The patient cannot grant access to the moveUP app to third parties. Other people could accidentally send wrong information or change the profile of the patient, with the result that the moveUP- guided advice is no longer applicable to the needs of the patient.

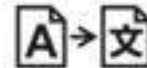
moveUP NV/SA  
Cantersteen 47,  
1000 Brussels, Belgium

### UK legal representative

Abnovo Ltd  
Suite 115, Wey House, 15 Church Street, Weybridge,  
Surrey KT13 8NA, United Kingdom

UDI-PI

Version number: V1.33.1

++G166MOVEUP1  
3303E

The moveUP App adheres to European health, safety and environmental protection standards.

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## Exclusion criteria:

- Patients who are mentally incompetent or having troubles to express what they are feeling (for instance, mentally diseased people, people staying in elderly care centers, ...) are excluded.
- Patients who are not capable of operating a tablet/smartphone and activity tracker.
- Patients who can't understand one of the available languages of the app (Dutch, French, German, English)

## Claims and advertisings:

- moveUP is not an emergency tool. In case of emergency please contact your doctor or call 911.
- moveUP is not intended to monitor/treat vital parameters of critical illness.

- moveUP companion is not a replacement for any treatment you need to follow, but is used as an addition.
- Patients need to be aware that if questionnaires are not completed on a frequent basis or not answered truthfully, the care team has more difficulty understanding your health situation & progress.

- The moveUP app offers personalized healthcare based on the patient's profile and therefore can not be shared, borrowed or traded between users. The information and advice of the moveUP application do not apply to any other patient other than the one whose profile is stored in the application. The patient cannot grant access to the moveUP app to third parties. Other people could accidentally send wrong information or change the profile of the patient, with the result that the moveUP- guided advice is no longer applicable to the needs of the patient.

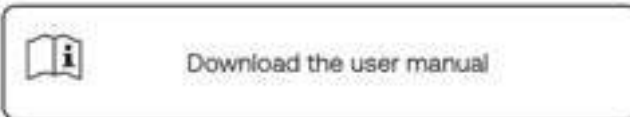
## Important information - US

Download the user manual  
**For patients**

Download the user manual  
**For healthcare providers**

## Intended use

### For healthcare providers

For healthcare providers, moveUP provides efficient clinical management of pathologies and treatments. With use of moveUP, valuable insights are given to the HCP/care team of the status of the patient and its evolution.

### For patients

For patients, moveUP provides personalized information and instructions to help them managing their symptoms and progress in their rehabilitation. The intensity of follow-up is adapted based on patients' needs and timepoint in their patient pathway:

- moveUP COMPANION = monitoring and information, no active follow-up
  - moveUP companion offers targeted information and evolution reports to patients. Their care team takes evolution reports into account in their further recovery when appropriate.
- moveUP COACH = active follow-up by healthcare team
  - moveUP coach offers targeted information and evolution reports to patients. Their care team is more actively involved and takes the evolution reports into account and can provide advice and exercise suggestions through the digital platform.
- moveUP THERAPY = active follow-up by healthcare team, with data driven validated care protocol
  - moveUP therapy offers a data-driven validated care protocol with certain category and level of exercise activities, specifically targeted to the individual patient. Their care team can manually adapt the data of the validated protocol when needed. Patients can fully rehabilitate with moveUP without leaving their home environment.

Optional functionality that can be enabled: interoperability with Class IIa continuous passive motion (CPM) medical device.

For knee and hip patients who are using a Class IIa CPM medical device, moveUP can interoperate with the CPM software. moveUP acts as a facilitator to easily assign a designated CPM exercise protocol chosen by a physician to a patient and to display the performed CPM exercises in the medical dashboard.

# moveUP

### Tradename

moveUP  
Made in Belgium  
The product is software

### US Agent

Claran McCourt,  
Incentown Inc 7421 W Cypresshead Dr Parkland  
FL 33067 US agent 954.682.5053

### Manufacturer

moveUP NV/SA  
Castersteen 47,  
1000 Brussels, Belgium

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# moveUP

## Manual for patients

moveUP – A smart and personalized way of informing, follow-up and treatment for patients.

*App version: 1.31 and later*

*Manual version: 16*

*Date of release manual: 11/07/2022*

*Link to Knowledge Base: <https://support.orthopedics.moveup.care/>*

# Table of content

- Intended use moveUP
- Overview of the main moveUP windows
- Messages
- Survey & info modules
- Steps
- Exercises
- Daily questionnaire
- Activity tracker
- My active follow-up period with moveUP has ended: how does the moveUP app change?

# Support orthopedics – moveUP

## Intended use moveUP

-  Carefully read all information and precautions in this manual. Make sure you understand the use, the screens and the limitations of the moveUP App before using the service. If you have any questions please contact moveUP.

For security reasons, moveUP recommends that the user carefully reads and understands the manual of the devices used in conjunction with the moveUP App.

During the full use of the service, the necessary technical assistance and therapeutical assistance will be provided. In case of a defect or problem with the App, you will be able to contact the moveUP team via the message functionality of the App, by phone or by e-mail.

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### For patients

For patients, moveUP provides personalized information and instructions to help them managing their symptoms and progress in their rehabilitation. The intensity of follow-up is adapted based on patients' needs and timepoint in the patient pathway:

- moveUP **COMPANION** = monitoring and information, no active follow-up
  - moveUP companion offers targeted information and evolution reports to patients. Their care team takes the evolution reports into account in their further recovery when appropriate.
- moveUP **COACH** = active follow-up by the healthcare team
  - moveUP coach offers targeted information and evolution reports to patients. Their care team is more actively involved and takes the evolution reports into account and can provide advice and exercise suggestions through the digital platform.
- moveUP **THERAPY** = active follow-up by the healthcare team, with data-driven validated care protocol
  - moveUP therapy offers a data-driven validated care protocol with certain categories and levels of exercises and activities, specifically targeted to the individual patient. Their care team can manually adapt the data-driven validated protocol when needed. Patients can fully rehabilitate with moveUP without leaving their home environment.

*Optional functionality that can be enabled: interoperability with Class IIa continues passive motion (CPM) medical device.*

*For knee and hip patients who are using a Class IIa CPM medical device, moveUP can interoperate with the Class IIa CPM software. moveUP acts as a facilitator to easily assign a designated CPM exercise protocol chosen by a physician to a patient and to display the performed CPM exercises in the medical dashboard.*

## Intended users

moveUP is intended to be used by patients and healthcare providers.

## Patient

The main user of the mobile app and patient website is the patient. Patient users should be alert, mentally competent and capable of operating a tablet/smartphone and activity tracker.

### Inclusion:

- Age:
  - minimum: 18 years
  - maximum: no limit
- Weight: NA
- Health & condition: capable of performing basic activities of daily living
- Language: understanding one of the available languages of the app (Dutch, French, German, English)

### Exclusion:

- Patients who are mentally incompetent or having trouble expressing what they are feeling (for instance, mentally diseased people, people staying in elderly care centres, ...) are excluded.
- Patients who are not capable of operating a tablet/smartphone and activity tracker.
- Patients who can't understand one of the available languages of the app (Dutch, French, German, English)

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## Target population / indication for use

moveUP companion & coach is used by Musculoskeletal, oncologic, respiratory, gastrointestinal, cardiovascular and neurologic patients, such as patients:

- who underwent or planning a gastric bypass or gastric sleeve operation
- who have or had a stroke, multiple myeloma, covid, familial hypercholesterolemia
- who have or had back or joint problems or operations

moveUP therapy is used by knee & hip arthroplasty patients.

## moveUP app & devices

### Supported devices moveUP app

The moveUP app runs on:

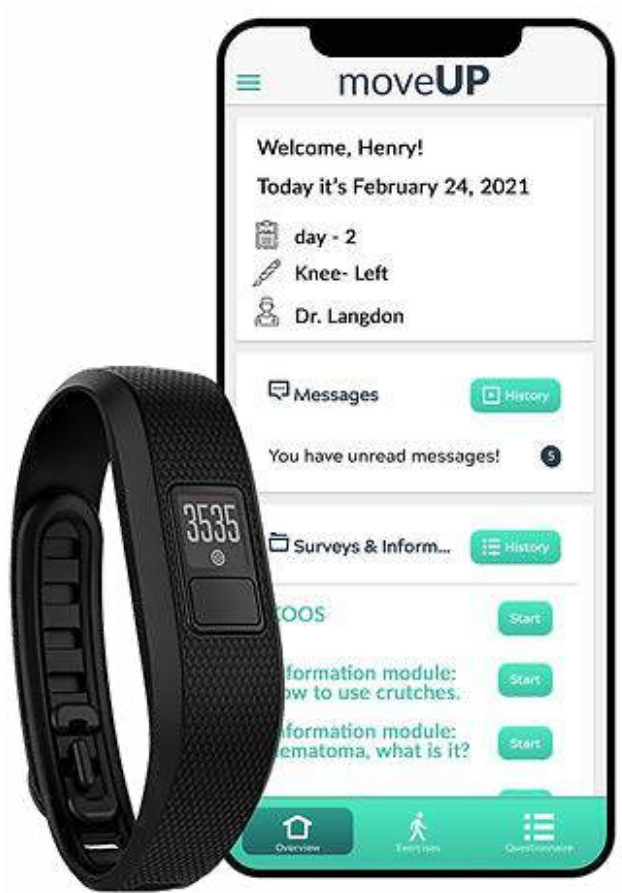
- iPhones/iPads (iOS 11.0 or newer)
- Android systems (Android 7.0.0 and newer).
- moveUP needs an internet connection when in use
- moveUP is compatible with the following browsers:
  - Chrome
  - Safari
  - Edge
  - Firefox
- In specific cases the following tablets are provided and are supported:
  - Lenovo Tab 3 – Android 6.0.1 or Higher
  - Huawei MediaPad T3 – Android 7.0.0 or higher

For your data safety please enable the 2-factor authentication(2FA) and use a safe network

### Activity trackers

The following activity trackers are currently supported:

- Garmin Vivofit 3 and Vivofit 4
- Nokia Withings Go

## Warnings and precautions

**⚠** moveUP is not an emergency tool. In case of emergency please contact your doctor or call 112.  
moveUP is not intended to monitor/treat vital parameters of critical diseases.

**⚠** moveUP companion is not a replacement for any treatment you need to follow but is used as an addition.  
Patients need to be aware if questionnaires are not filled out on a frequent basis or not filled out trustworthy, the care team has more difficulty knowing their health situation & evolution.

**⚠** The moveUP App offers personalized healthcare based on the patient's profile and therefore can not be shared, borrowed or traded between users. The information and advice of the moveUP application do not apply to a patient other than the one whose profile is stored in the application. The patient may not grant access to the moveUP App to third parties. Any other person can, even accidentally, send wrong information or change the profile of the patient, with the result that the moveUP-guided advice no longer corresponds to the needs of the patient.

**⚠** With software, there is always a residual risk for bugs. If you have the feeling something is wrong or you don't get a response in the app, please reach out to us via [support@moveup.care](mailto:support@moveup.care) or call 0800 88 008.

## **Contra-indications and potential side effects**

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## Notification of serious incidents

**A serious incident** means any incident that directly or indirectly led, might have led or might lead to any of the following:

- (a) the death of a patient, user or another person,
- (b) the temporary or permanent serious deterioration of a patient's, user's or other person's state of health that resulted in any of the following:
  - life-threatening illness or injury,
  - permanent impairment of a body structure or a body function,
  - hospitalisation or prolongation of patient hospitalisation,
  - medical or surgical intervention to prevent life-threatening illness or injury or permanent impairment to a body structure or a body function,
  - chronic disease,
- (c) a serious public health threat;

If you detect any of these cases please send an email to [info@moveUP.care](mailto:info@moveUP.care) and;

- Netherlands: [meldpunt@igj.nl](mailto:meldpunt@igj.nl)
- Belgium: [meddev@fagg.be](mailto:meddev@fagg.be)
- France: [dedim.ugsv@ansm.sante.fr](mailto:dedim.ugsv@ansm.sante.fr)
- Germany: [zlg@zlg.nrw.de](mailto:zlg@zlg.nrw.de)

## **Human body contact**

No human body contact with the patient or user, due to the nature of the product (software).

## **Accessories/products used in combination**

There are no accessories.

If the patient has no compatible tablet/smartphone or activity tracker, moveUP can lease these devices to the patient. The leased devices are CE-marked devices that meet the compatibility criteria outlined in the IFU for the app.

## Device lifetime

2-years. Depending on the willingness of the user to update the app. We sent out a notification with the recommendation to update the app, to make sure the app will keep the performance & recently included features.

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- **Enhancement**— moveUP enhances the clinical management of the patient because early detection & management of complications are possible via the symptom & QoL monitoring tool
- **Efficiency**—More efficient clinical management, such as the number of consultations can be reduced
- **Compliance**—Enforces therapy compliance/adherence
- **Information**—The correct information is provided at the right time

### moveUP therapy

- **Rehabilitation from home**—With the use of moveUP therapy knee & hip arthroplasty patients can fully rehabilitate via the in-app care team without leaving their home environment.

## Company details

moveUP NV

Head office Brussel: Cantersteen 47, B-1000 Brussels, Belgium Office Ghent: Oktrooiplein 1, B-9000 Ghent, Belgium

Tel: 0800 88 008

Email: [info@moveUP.care](mailto:info@moveUP.care)

[www.moveUP.care](http://www.moveUP.care)

Identification of the device:

Basic UDI-DI code:

++G166MOVEUPV1310WH

If you have any questions about the App, the tablet or the activity tracker, you can contact moveUP via the following channels – in order of priority: Via the message system of the App. Via email to [support@moveup.care](mailto:support@moveup.care) – Via phone during office hours: +32 800 88 008

Updated on July 11, 2022

# Overview of the main moveUP windows

## Support orthopedics – moveUP

**I read this email on a smart device**

## **Step 1: Download the app on your smart device**

Click on the below links to download the moveUP app from your Appstore or Google Play (depending on your device)

I have an iPhone or tablet from Apple (iOS)

I have another smartphone or tablet (android)

## Step 2: Open the moveUP app and register

- Open the moveUP app (if the app shows the moveUP logo, It is correctly installed)
- Register with the email address and the code you received in the “installation email” from moveUP
- Check your personal data and the operation performed before the first use. In case of any error please contact moveUP using the chat function.

### **Step 3: Update the moveUP app**

Each time the moveUP app is updated you'll receive a mail with the release notes. Please, update it as soon as possible to enjoy the last features.

To upload the app please use the link above to be redirected to your app store and click on the button "update".

These updates can include safety or data integrity upgrades, please read carefully the Release Notes and the Instructions for Use.

## Security measures

Because we process personal and health data, we will instruct you how to set a complex password at the moveUP login page.

Please use a safe network when you are connected to the app.

Updated on May 4, 2022

# Support orthopedics – moveUP

## Quick start guide

There are 3 main tabs in the app. You can switch between them by tapping the respective symbols at the bottom of your screen.

1. Overview
2. Exercises
3. Questionnaire

## Tab 1: Overview

When you open the moveUP app you will automatically be taken to the overview tap. This tab has 4 sections:

1. **Welcome message**
2. **Messages** – here you can communicate with your health provider
3. **Survey & Information** – here you can find your questionnaires that still need answering. Also, your video-recording requests will appear here.
4. **Steps** – here you can see the steps you did over the past week

## Tab 2: Exercises

In the second tab “Exercises” you can find your daily exercises.

## Tab 3: Questionnaire

The screenshot shows a mobile application interface titled "Questionnaire". At the top, the time is 17:26. Below the title, it says "Your questionnaire for today". The first section asks "How are you feeling today?" and "Indicate on the scale." with "Good" on the left and "Bad" on the right. Below this is a row of six emoji icons representing a scale from happy to angry. The second section asks "How was your night?" and "Indicate on the scale." with "Good" on the left and "Bad" on the right, also followed by the same row of six emoji icons. The third section asks "Did you experience any morning stiffness in your affected joint?" and "Indicate on the slider." with "None" on the left and "Very stiff" on the right. A red arrow points from the "Very stiff" label down to a red box around the "Questionnaire" icon in the bottom navigation bar. The bottom bar also contains icons for "Home" and "My Care".

In the 3rd tab "Questionnaire" there will appear a daily questionnaire after 4pm each day. The questionnaire is available until midnight. It should be filled out every day, starting 14 days before surgery.

Read more on the daily questionnaire

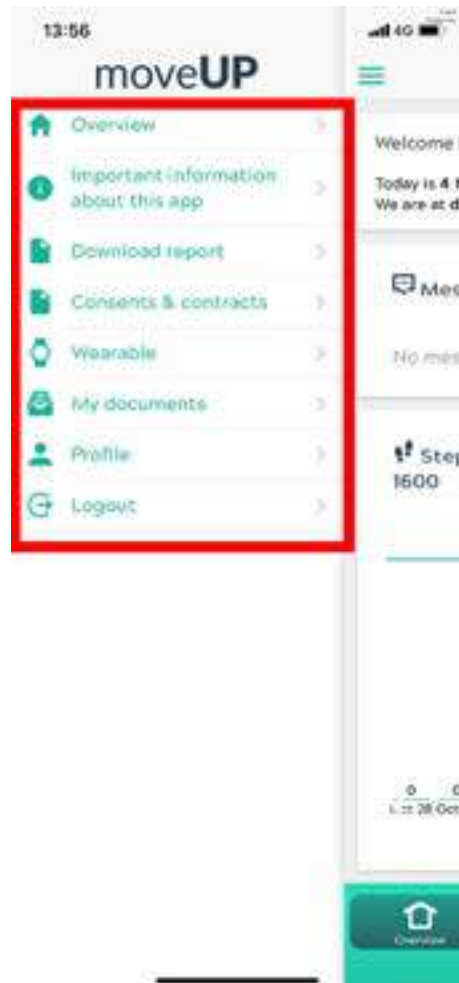
It is important you start answering the daily questionnaire starting 14 days before your operation. This way we can better help you after the operation.

## Menu

At the top left of the moveUP you will find the menu.

When you click on this icon the menu appears.

This menu contains important sections that are not directly related to your rehabilitation.

- **Overview:** Exits the menu and returns to the overview screen
- **Important information about this app:** some info and warnings on the app
- **Download report:** This link is not working at the moment. Work in progress!
- **Consents & contracts:** Here you can read the general terms and conditions and privacy statement.
- **Wearable:** Here you can link your smart bracelet to the moveUP App or see which application is linked.
- **Profile:** your name and national registration number
- **Log out:** Here you can log out of your account and return to the registration and login page.

# Messages

# Support orthopedics – moveUP

## Messages

In the overview tab, below the greeting, you find the “Messages” section. This is the section where you can communicate with your physiotherapist.

You can send your physiotherapist a message each moment of the day. You will receive an answer within 24 hours.

If the question is a medical urgency, don't use the messaging system as it is too slow. Call your hospital or medical practitioner instead.

## Open the message functionality

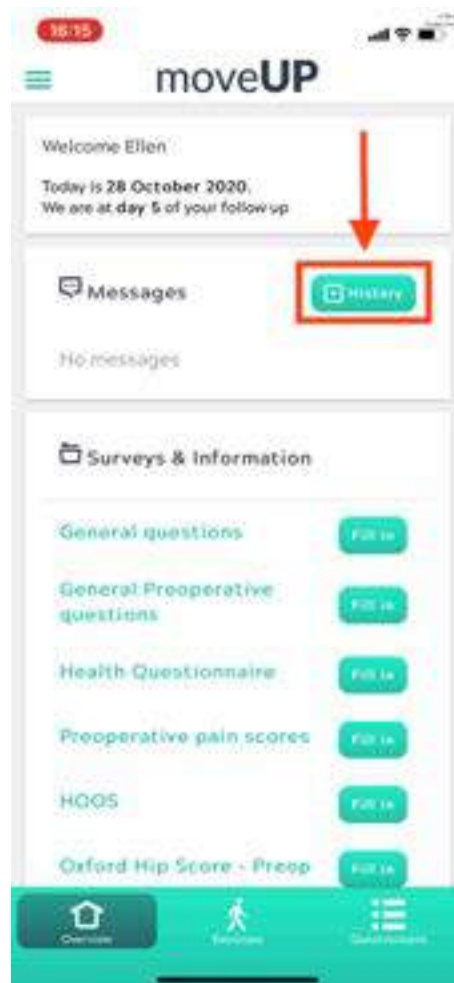
Click on the history button to open the message functionality.

## Sending messages

You can send us messages every moment of the day. We will respond within 24 hours


To send a message click in the “type a message” box. The keyboard will appear. After typing your message click on the send symbol (airplane).


## Sending a picture

You can send us a picture (of the wound for example) by clicking on the picture symbol in the left lower corner.
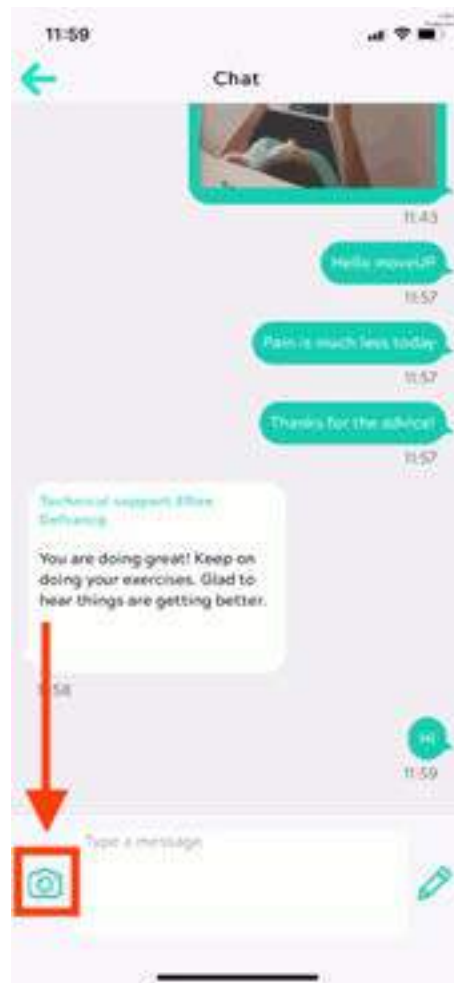
## Go back to the overview screen

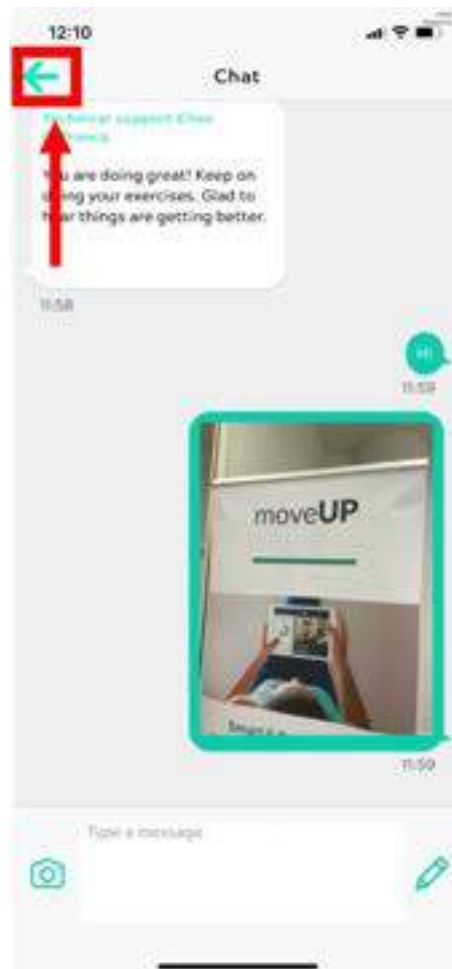By clicking on the arrow in the left upper corner you can go back to the overview screen.

# **Survey & info modules**

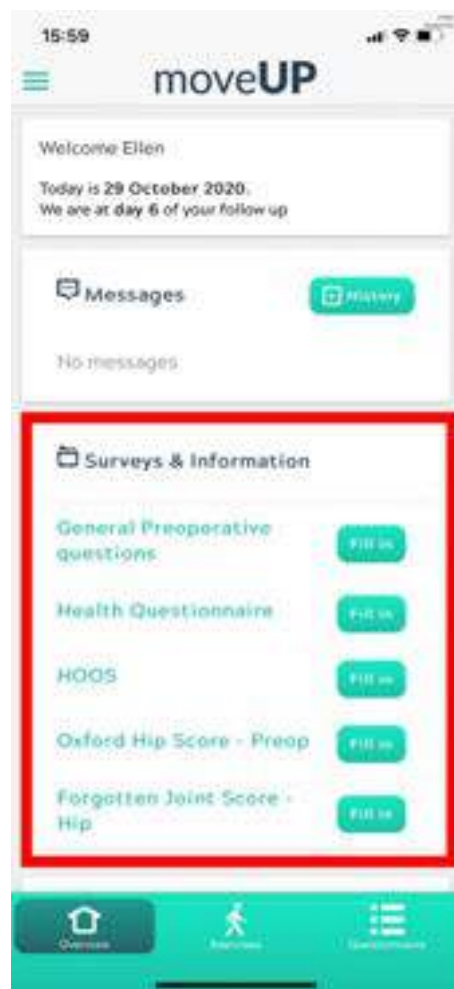
# Support orthopedics – moveUP

## Surveys & Information

Medical questionnaires are available at various moments before and after the surgery.


These questionnaires are of critical importance to evaluate the outcome of your surgery and to guide your physiotherapist in his decision making.

From the moment a questionnaire becomes available, we ask you to start filling it out as soon as possible.

## Submitting a survey

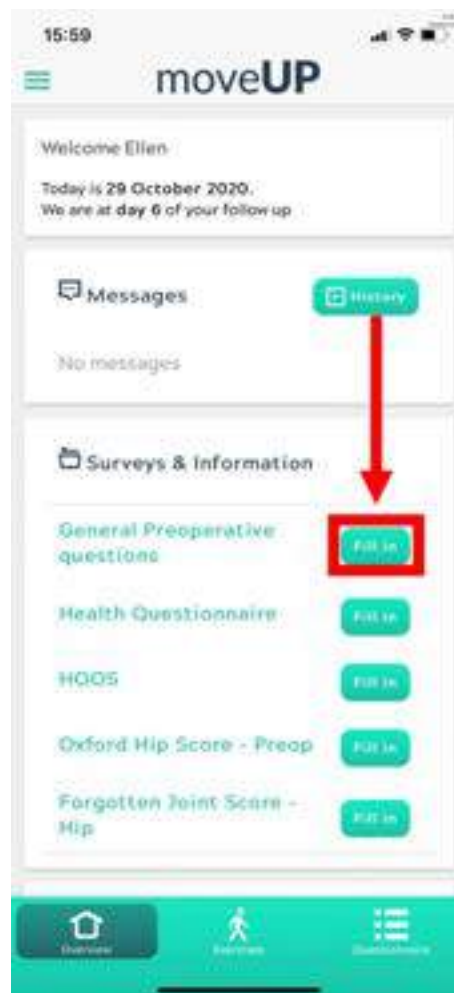To fill out a survey you click the “Fill in” button.

After filling out the question click on the “send” button.

16:09

General Preoperative questions

Never

Sometimes, during walks

Always

Do you experience problems with your balance?

Select from the list.

Never

Sometimes

Often

Always

Weight

Your weight (kg)

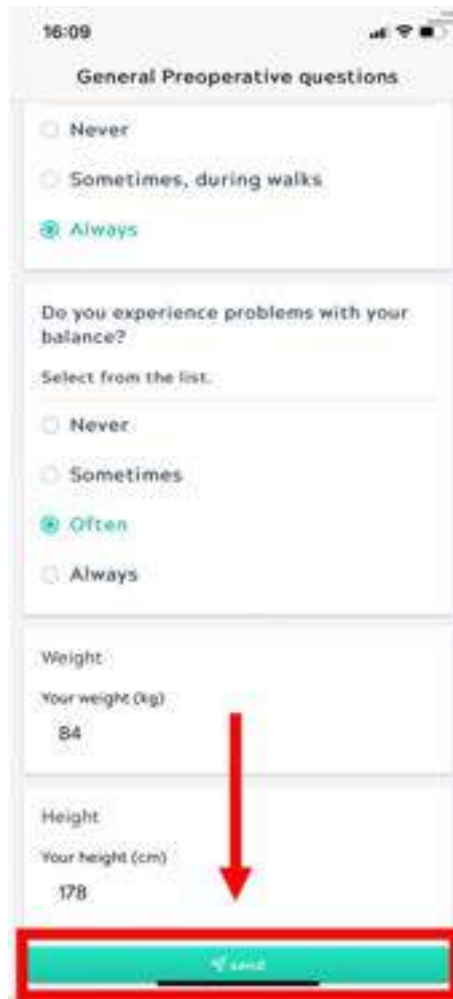
84

Height

Your height (cm)

178

send

You will see the following screen if the answers are sent to us correctly.

17:55

Your answers are sent to the  
medical team!

[Go back to the overview](#) >

# Support orthopedics – moveUP

## How to reread an info module?

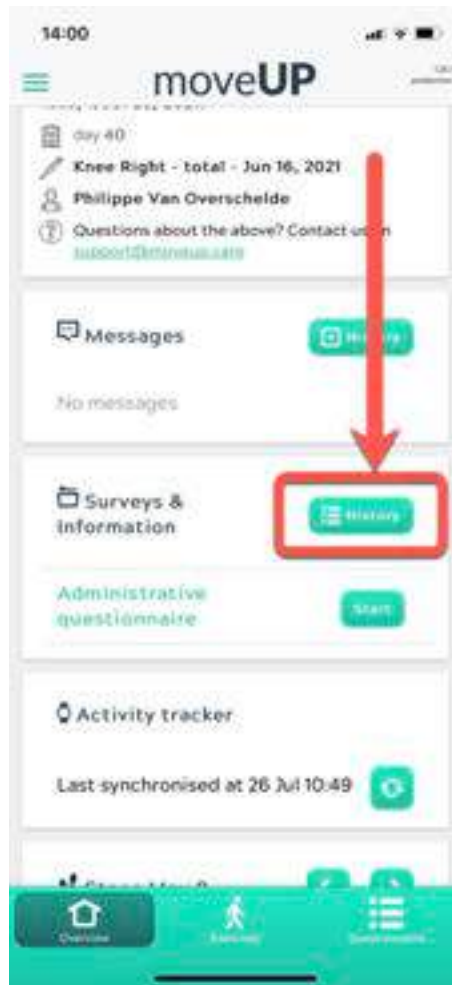
To reread an info module that you already read:

- On the overview tab (1) go to the “Surveys & Information” section (2).

- Tap the “history” button.

- Here you will find the complete list of all the info modules you have read so far.
- Select the one you want to read again.

14:30

## Articles

100%

READ: Anatomy, osteoarthritis and total hip replacement surgery

READ: Driving after surgery

READ: Wound care: from incision to scar

READ: Safe use of crutches

READ: Sleep positions after joint replacement

READ: Haematoma

READ: Pain

READ: Improve your sleep after surgery

# Steps

# Support orthopedics – moveUP

## Steps area explained

Your steps are automatically counted by your activity tracker, the activity tracker does not need to be activated, just be worn around the wrist.

To see your steps you have to have paired your activity tracker with the moveUP app. This pairing needs to be done only once. After you have to sync your activity tracker daily with the moveUP app.

## Steps graph

- Activity data from the past week is visualized in a graph.
- The different colors only indicate the different days and have no further meaning
- Activity is represented in number of steps.
- You can use the arrows left and right to see previous or next days in the graph.

Every activity with arm movement will contribute to the count. This means if you walk by pushing a stroller or a walker the steps may be undercounted. If you are ironing (arm movement) they may be overcounted.

## Personalised step max

Every day, a personalized step maximum is displayed by a number and a red line. Try not to exceed the step maximum with > 20%.

The step maximum is not a goal! It is a limit to avoid overloading the operated joint.

# Exercises

# Support orthopedics – moveUP

## Exercises

The second tab contains your exercise program for the day. Day by day you will only see prescribed exercises specific for that day.

If you feel strong pain while doing a specific exercise, stop that exercise and mention its number in the messaging system.

## Number and name of the exercise

On top of each exercise you can find it's number and the name of the exercise.

## Video

There is also a short video per exercise that indicates how the exercise should be performed.

To start the video, click on the “Play” icon (triangle) on the top of the video.

## Frequency and intensity

You will find the frequency and intensity per exercise (eg: 3x a day, 10 repetitions).

You report the number of sessions via the + and – symbol. It is important to enter this accurately so that your moveUP physiotherapist is kept informed of your progress.

We recommend that you do **not** exceed the number of prescribed sessions and repetitions.

# **Daily questionnaire**

# Support orthopedics – moveUP

## Daily questionnaire

The daily questionnaire is our compass to guide you through your revalidation. It is very important you fill it out daily after 4 pm.

By filling out the daily questionnaire you give your doctor and healthcare team more insight into the progress of your treatment: your symptoms, your complaints, your evolution. It consists of questions about your general well-being, complaints, activities, sleep quality etc... These questions are adapted to the type of treatment you are having. These daily questionnaires take approximately 45-60 seconds to complete. Your doctor and healthcare team will also see how you experience the impact of your condition on your quality of life.

The combination of the information retrieved from questionnaires, photo's, video's and your activity profile via the smart bracelet allows your doctor and care team to follow you up and adapt your treatment.

For optimal results, you need to use the App and follow the given instructions within the App on a daily basis.

It is important we have a good baseline of your situation before the operation. That is why we ask you to start filling it out every day, ***starting 14 days before your operation.***

The questionnaire can be found in the 3rd tab of your app every day after 4 pm.

The screenshot shows the 'Questionnaire' app interface. At the top, the time is 17:26 and the title is 'Questionnaire'. Below the title, it says 'Your questionnaire for today'. The first question is 'How are you feeling today?' with the instruction 'Indicate on the scale.' and a scale from 'Good' to 'Bad' with six corresponding emoji icons (happy, neutral, sad, angry, etc.). The second question is 'How was your night?' with the instruction 'Indicate on the scale.' and a scale from 'Good' to 'Bad' with the same six emoji icons. A red arrow points to the 'Bad' emoji in this section. The third question is 'Did you experience any morning stiffness in your affected joint?' with the instruction 'Indicate on the slider.' and a scale from 'None' to 'Very stiff'.

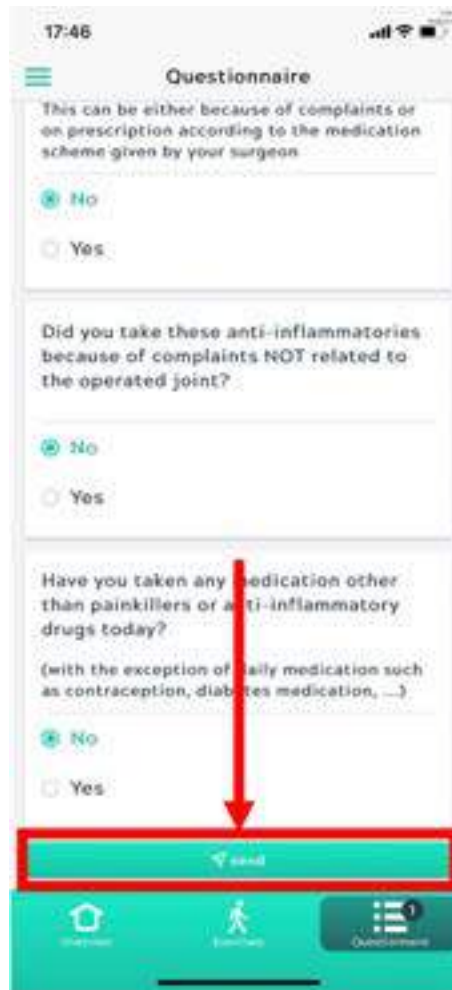
To indicate your pain level you can tap on the line above the faces and/or slide to the appropriate position.

---

---

When finished click on the “send” button to share your answers with us.

The following screen appears if answers are sent to us correctly.

17:55

Your answers are sent to the  
medical team!

[Go back to the overview](#) >

# Activity tracker

## Si ddcfhcfhcded]cgâ a cj eUP

Hck hc da]f ñci f ach]j ]hmfac\_ef (Gafa ]b V]j còh4) k ]h the a cj eUP add?

To pair your activity tracker click the green button on your homescreen.

Choose "Garmin Vivofit 4" and click continue

Updated on January 26, 2022

# Support orthopedics – moveUP

## How to synchronise my steps daily?

We advise you to wear your activity tracker around your wrist day and night. But the day is the most important for us to register your activity. The activity tracker is always active and captures how active you are, hence it does not need to be activated.

You do need to manually sync your activity tracker data **daily** in the moveUP app. This way your physiotherapist can monitor how active you are and decide if any adaptations to your exercise scheme are needed.

**We advise syncing your steps daily in the evening after filling out the daily questionnaire.** This way your physiotherapist will see the next day how active you were.

The activity tracker has to be paired with the moveUP app. This pairing only has to happen once at the beginning of your rehabilitation. You can find the pairing instructions [here](#).

The Bluetooth should be activated on your device to be able to sync your activity tracker.

- Press the synchronising button marked in red.

- You will see the following screen. Click on “How to synchronise”

- Follow the instructions on the screen. Basically you have to press the silver button on your activity tracker twice for 2 seconds.
- At the end you will automatically be taken back to the homescreen and your steps will be visible in the graph.

- You will see the synced steps data on your home screen.

If you have problems synchronising please send an email to [support@moveup.care](mailto:support@moveup.care) and try to describe exactly in which step you encounter problems and what messages you see or get on your screen.

**My active follow-up period with moveUP has ended: how does the moveUP app change?**

## Support orthopedics – moveUP

### My active follow-up period with moveUP has ended: how does the moveUP app change?

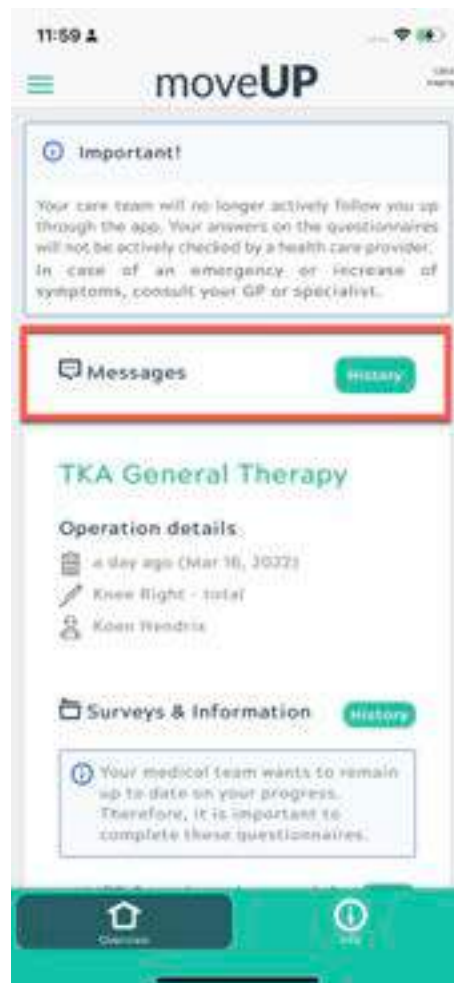
When the active follow-up period has ended your moveUP app will be different. You are no longer actively followed up by a physiotherapist. What changes in the app?

The app will consist of 2 tabs now: “Overview” and “Info” (instead of “Overview”, “Exercises” and “Questions”).

## Overview tab


### Messages

The message function will still be available for seven days. After one week the message function will be disabled. Your message history will remain available in the app.

### moveUP journey card

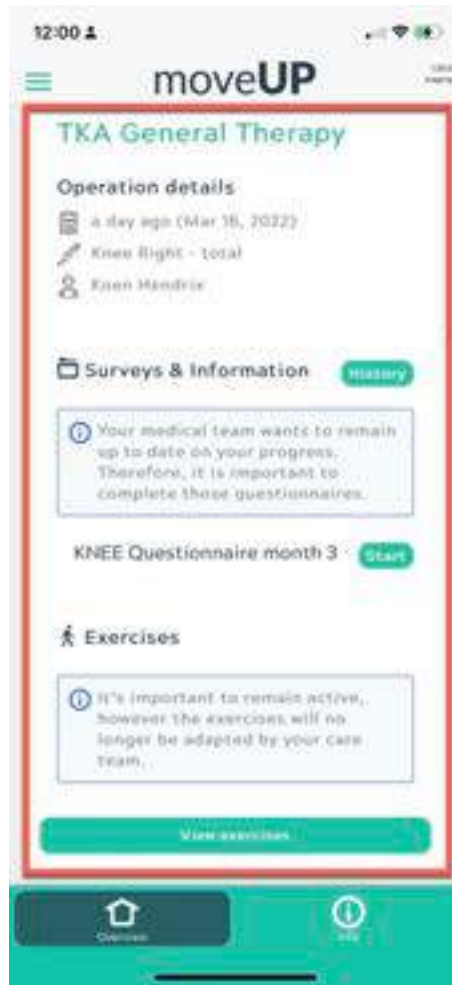The next block is your “moveUP journey card”. It contains your operation details, surveys that still need to be filled out and an option to review the exercises you have done with the app during the course of your rehabilitation


If you had more than one journey with moveUP, you can scroll down the page and find them all in separate cards.

## Info tab

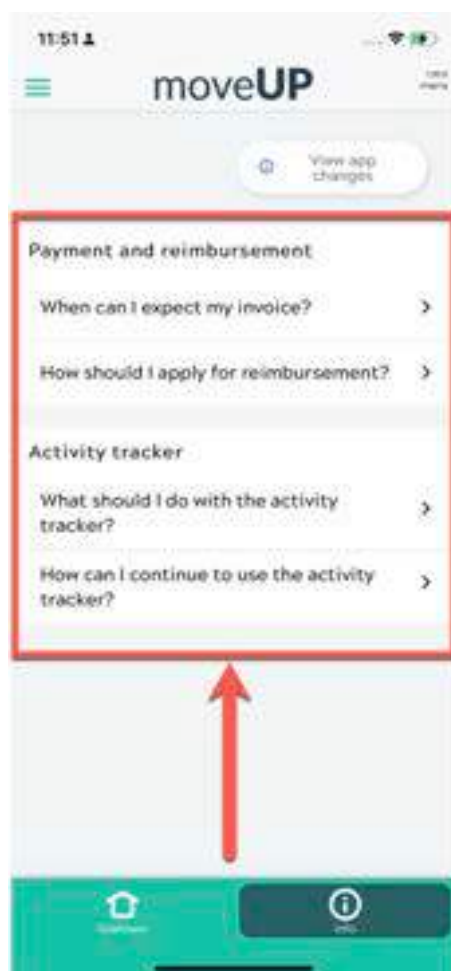
## FAQ

If you click on the info tab, you will find a page that is dedicated to frequently asked questions.

## View app change

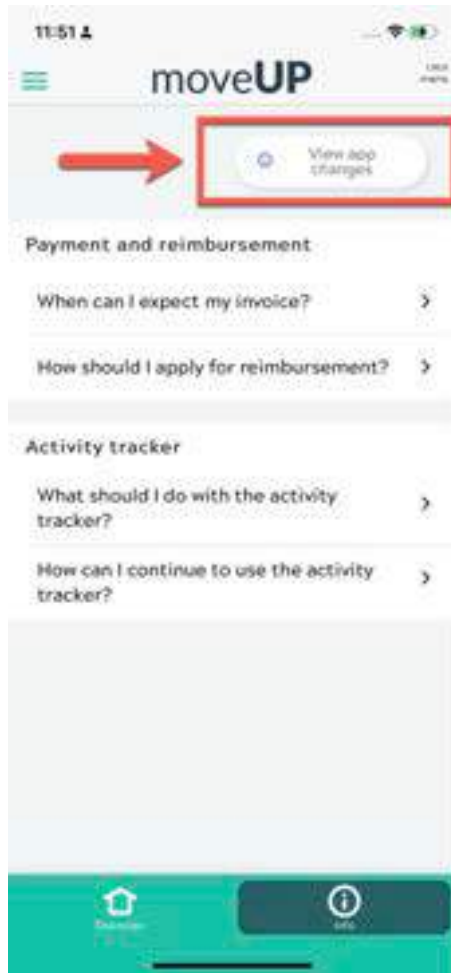
Here you can find an overview of what has changed as discussed on this page.


## Do you have another active follow-up activated already?

In this case you want find your previous information in the overview tab. But you can still access your previous “moveUP journey cards” via the menu.

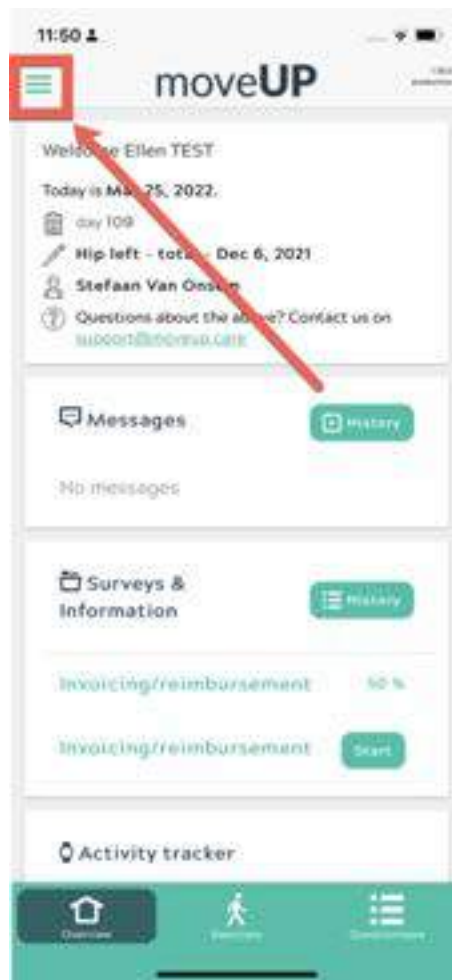Go to the menu by clicking on the top left menu icon.

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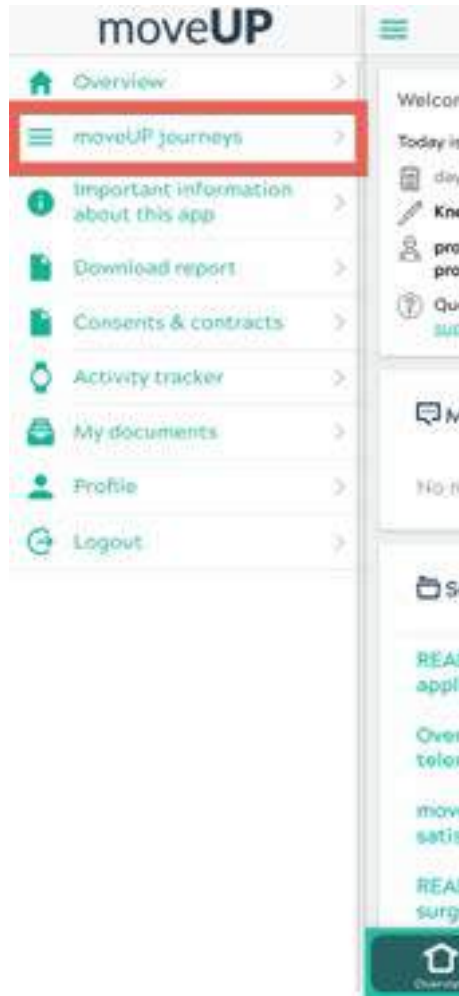The screenshot displays the 'Täglicher Fragebogen' (Daily Questionnaire) interface. It consists of three sections, each with a question and a slider or selection option. The first section asks about pain at night, with a slider set to 'Keine Schmerzen' (No pain). The second section asks about pain during the day, with a slider set to 'Viele Schmerzen' (A lot of pain). The third section asks if pain was felt elsewhere, with a radio button selected for 'Ja' (Yes). A red arrow points to the 'Täglicher Fragebogen' tab in the bottom navigation bar.

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The screenshot shows the 'Täglicher Fragebogen' app interface. It contains three sections for pain assessment:

- Section 1:** 'Geben Sie an, wie viel Schmerz Sie nachts in Ihrem betroffenen Gelenk hatten' (Indicate how much pain you had at night in your affected joint). It includes a scale from 'Keine Schmerzen' (No pain) to 'Starke Schmerzen' (Severe pain) with a slider and smiley faces.
- Section 2:** 'Wie viel Schmerz hatten Sie tagsüber in und um das betroffene Gelenk?' (How much pain did you have during the day in and around the affected joint?). It includes a scale from 'Keine Schmerzen' (No pain) to 'Viele Schmerzen' (A lot of pain) with a slider and smiley faces. A red arrow points to this section.
- Section 3:** 'Hatten Sie letzte Nacht oder heute Schmerzen an einer anderen Stelle als an dem betroffenen Gelenk?' (Did you have pain last night or today in a different location than the affected joint?).

The bottom navigation bar has three tabs: 'Übersicht', 'Anfragen', and 'Täglicher Fragebogen' (highlighted with a red box and a red arrow).

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The screenshot shows a mobile application interface titled 'Questionnaire'. At the top, the time is 17:26. Below the title, it says 'Your questionnaire for today'. The first question is 'How are you feeling today?' with the instruction 'Indicate on the scale.' and a scale from 'Good' to 'Bad' with six corresponding emoji icons. The second question is 'How was your night?' with the instruction 'Indicate on the scale.' and a scale from 'Good' to 'Bad' with six corresponding emoji icons. The third question is 'Did you experience any morning stiffness in your affected joint?' with the instruction 'Indicate on the slider.' and a scale from 'None' to 'Very stiff'. A red arrow points from the 'Very stiff' label to the 'Questionnaire' icon in the bottom navigation bar, which is highlighted with a red box.

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The screenshot shows the 'Vragen' (Questions) tab in a mobile application. The interface is in Dutch and contains three questions:

- Question 1:** "Is het betrokken gewricht gezwollen?" (Is the affected joint swollen?). Below it, it says "Duid aan op de schaal" (Indicate on the scale). The scale has two endpoints: "Niet gezwollen" (Not swollen) on the left and "Zeer gezwollen" (Very swollen) on the right.
- Question 2:** "Geef de pijn weer die u ervaart in het betrokken gewricht tijdens de nacht" (Indicate the pain you experience in the affected joint during the night). Below it, it says "Duid aan op de schaal" (Indicate on the scale). The scale has two endpoints: "Geen pijn" (No pain) on the left and "Veel pijn" (A lot of pain) on the right. Below the scale is a row of six smiley face icons representing different levels of pain, from happy (no pain) to sad (a lot of pain).
- Question 3:** "Hoeveel pijn ervaart u in en rond het betrokken gewricht tijdens de dag?" (How much pain do you experience in and around the affected joint during the day?). Below it, it says "Duid op de schaal aan" (Indicate on the scale). The scale has two endpoints: "Geen pijn" (No pain) on the left and "Veel pijn" (A lot of pain) on the right.

A red arrow points from the top of the second question's scale down to the 'Vragen' tab icon in the bottom navigation bar. The bottom navigation bar has three icons: a house (Home), a person (Profile), and a list (Questions).

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The diagram illustrates the timeline of medical questionnaires. It starts with 'Preop' at Day 0, followed by 'Sortie' at Day 7. 'Questionnaire de décharge' is administered at Day 7. 'Satisfaction de la rééducation' is administered at Day 23. 'Postop' is administered at Day 4. The timeline also shows the use of various assessment tools: HOUS/NDOS, CHS, ED, SD, FRS, HRS, KSS, and ICS, which are used at different points in the timeline (Preop, Day 7, Day 23, Day 4, and Day 4 Postop).

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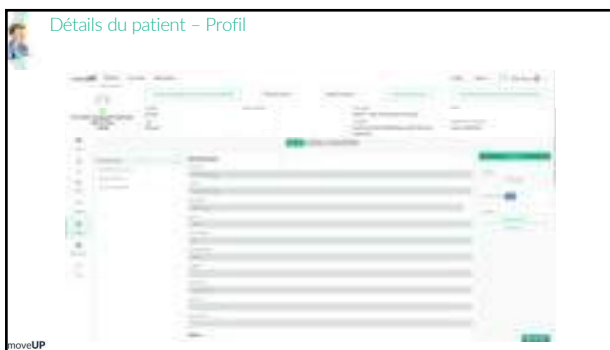
The screenshot shows the 'Patient Onboarding Dashboard' for a user named Ellen. At the top, there are navigation links for 'Dashboard' and 'Patients', and utility links for 'Support' and 'Account'. A security warning banner is present, stating: 'We're working with sensitive medical data. That's why you should enable two factor authentication. Two factor authentication secures your account better. Enable it in your account settings.' Below the banner, there are five data cards: '227 moveUP candidates' with a '+ New patient' button (highlighted by a red arrow), '3706 patients active', '13 patients reminded', '0 candidates not interested', and '0 filled in anamnesis questionnaire'.

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## Managing Organization \*

AZ Alma Ecklo

Organization that is the custodian of the patient record

## Name \*

Names associated with the patient

### Given names \*

Ellen

Given names (first names). First's included middle names

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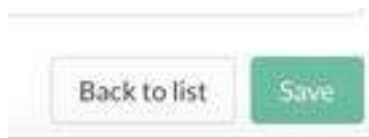
## Edit care plans for Ellen Test

### Patient's care plans

Start a care plan to add intervention and treatment for this patient

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## Edit care plans for Ellen Test

### Patient's care plan

Add intervention, treatment or study. Click or drag 'n drop items to add.

Activated care plan (move care plan to active list in medical dashboard)

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## Edit care plans for Ellen Test

### Patient's care plan

▼ Knee Right - total, TKA General Therapy

Intervention: Knee Right - total

Date  1

Doctor  2

Treatment: TKA General Therapy

Start Date

Code

HCP

Activated care plan (move care plan to active list in medical dashboard)

+ Start new care plan

3

### AZ Alma Eeklo Interventions

Bilateral total hip arthroplasty    Bilateral total knee arthroplasty    Hip left - resurfacing

Hip left - revision    Hip left - total    Hip right - resurfacing    Hip right - revision    Hip right - total

Knee Right - total    Knee left - revision    Knee left - total    Knee left - uni    Knee right - revision

Knee right - uni

### moveUP Treatments

THA Anterior Therapy    THA Bilateral Therapy    THA General Therapy    TKA Bilateral Therapy

TKA General Therapy    Unknown procedure date

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The screenshot displays the 'Patient Onboarding Dashboard' for user 'Elien'. At the top, there is a navigation bar with 'Dashboard' and 'Patients' on the left, and 'Support' and 'Account' on the right. The main heading is 'Patient Onboarding Dashboard' followed by 'Welcome, Elien'. Below this is an orange notification banner with a warning icon and text: 'We're working on providing medical data. You'll be able to view and manage your medical data when it's available. Check it in your account settings.' The dashboard features five data cards: a blue card for '227 new MUP candidates' with a '+ New patient' button (highlighted by a red arrow), a green card for '3706 patients active', an orange card for '13 patients reminded', a pink card for '0 candidates not interested', and a light green card for '0 filed in anamnesis questionnaire'.

Metric	Value
New MUP candidates	227
Patients active	3706
Patients reminded	13
Candidates not interested	0
Filed in anamnesis questionnaire	0

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20

# moveUP

## User manual – Health care platform & Patient app for HCPs

Get insight in the rehabilitation of your patients and provide close follow up

App & Healthcare platform version: 1.30 till current

Manual version: 9

Date of Publication: 17/03/2022

Link to e-learning: <https://moveup.talentlms.com/index>

# Table of contents

1. Instructions
2. Warnings and precautions
3. Intended use and intended users
4. Indication for use
5. Contact details
6. Patient app
  - a) Patient app functionalities
  - b) How to pair a wearable
7. Healthcare platform
  - a) Healthcare platform functionalities
  - b) Clinical guidelines for physiotherapists
  - c) Registration of a moveUP patient
  - d) Stop the treatment of a patient in moveUP

# INSTRUCTIONS

Read carefully this instruction for use. Make sure that you know how to use moveUP, but also understand the limitations of moveUP before you start using our service. If you do have any questions on the use, contact moveUP.

In the final chapter of this document you find our contacts, which you can use during the use of our service, in case you would have technical problems with the application (moveUP dashboard or patient app) or when expertise is required about a care path.

The Healthcare practitioner can't give access to third parties to access the moveUP dashboard. Every other person can, even by accident, send out wrong information or adapt the patient profile, with the result that the advice sent by moveUP no longer corresponds with the needs of the patient.

In case the application is use while moveUP & b.clinic is not asked to do the follow-up of patients, the individual healthcare provider or the health care facility is responsible for the follow-up of the patient and the action that are carried out.

In compliance with the medical device legislation, you are required to report any complaints towards the application, any alteration regarding characteristics / specifications of the application and incidents / complications potentially linked to the use of the application to [support@moveup.care](mailto:support@moveup.care). The AFMPS can also be contacted in case of incident via [https://www.famhp.be/en/human\\_use/health\\_products/medical\\_devices\\_accessories/materiovigilance/how\\_notify](https://www.famhp.be/en/human_use/health_products/medical_devices_accessories/materiovigilance/how_notify)

In compliance with the GDPR legislation, please report eventual data leaks linked to the use of moveUP. Please, read the entire privacy policy : <https://www.moveup.care/privacy>

# Warnings and precautions

- moveUP is not an emergency tool. In case of emergency please contact your doctor or call 112.
- moveUP is not intended to monitor/treat vital parameters of critical diseases.

- moveUP companion is not a replacement of any treatment you need to follow, but is used as an addition.
- Patients need to be aware if questionnaires are not filled out on frequent basis or not filled out trustworthy, the care team has more difficulty to know their health situation & evolution.

- The moveUP App offers personalized healthcare based on the patient's profile and therefore can not be shared, borrowed or traded between users. The information and advice of the moveUP application do not apply to a patient other than the one whose profile is stored in the application. The patient may not grant access to the moveUP App to third parties. Any other person can, even accidentally, send wrong information or change the profile of the patient, with the result that the moveUP-guided advice no longer corresponds to the needs of the patient.

*With software, there is always a residual risk for bugs. If you have the feeling something is wrong or you don't get a response in the app, please reach out to us via support@ or call 0800 88 008.*

The moveUP IFU are only supplied in electronic format. If you need a full printed version please contact us at the e-mail address info@moveup.care. This mail is available 24/7

# INTENDED USE

For healthcare providers, moveUP provides efficient clinical management of pathologies and treatments. With the use of moveUP, valuable insights are given to the HCP/care team of the status of the patient and its evolution.

For patients, moveUP provides personalised information and instructions to help them managing their symptoms and progress in their rehabilitation. The intensity of follow-up is adapted based on patients' needs and timepoint in the patient pathway:

## **moveUP companion = monitoring and information, no active follow-up**

moveUP companion offers targeted information and evolution reports to patients. Their care team takes the evolution reports into account in their further recovery when appropriate.

## **moveUP coach = active follow-up by healthcare team**

moveUP coach offers targeted information and evolution reports to patients. Their care team is more actively involved and takes the evolution reports into account and can provide advice and exercise suggestions through the digital platform.

## **moveUP therapy = active follow-up by healthcare team, with data driven validated care protocol**

moveUP therapy offers a data-driven validated care protocol with certain category and level of exercises and activities, specifically targeted to the individual patient. Their care team can manually adapt the data driven validated protocol when needed. Patients can fully rehabilitation with moveUP without leaving their home environment.

*Optional functionality that can be enabled: interoperability with Class IIa continuous passive motion (CPM) medical device. For knee and hip patients who are using a Class IIa CPM medical device, moveUP can interoperate with the Class IIa CPM software. moveUP acts as a facilitator to easy assign a designated CPM exercise protocol chosen by a physician to a patient and to display the performed CPM exercises in the medical dashboard.*

# Intended users

moveUP is intended to be used by patients and healthcare providers. The main user of the mobile app and patient website is the patient.

## Inclusion:

- Age: minimum 18 years / maximum no limit
- Health & condition: capable of performing basic activities of daily living
- Language: understanding one of the available languages of the app (Dutch, French, German, English)

## Exclusion:

- Patients who are mentally incompetent or having troubles to express what they are feeling (for instance, mentally diseased people, people staying in elderly care centres, ...) are excluded.
- Patients who are not capable of operating a tablet/smartphone and activity tracker.
- Patients who can't understand one of the available languages of the app (Dutch, French, German, English)

The main user of the medical web interface is a healthcare practitioner (group) or clinical researcher (group), named the care team. The care team is able to operate a web interface via web browser on PC/tablet/smartphone. The healthcare practitioner needs to understand one of the available languages of the web interface (today only available in English).

## Contra-indications and potential side effects

There are no contra-indications or known side effects.

## Human body contact

No human body contact with patient or user, due to nature of the product (software).

## Accessories/products used in combination

There are no accessories. If the patient has no compatible tablet/smartphone or activity tracker, moveUP can lease these devices to the patient. The leased devices are CE marked devices that meets the compatibility criteria outlined in the IFU for the app.

## Device lifetime

2-years. Depending of the willingness of the user to update the app. We sent out a notification to recommend to update the app, to make sure the app will keep the performance & recent included features.

## Claims

### *moveUP companion/coach/therapy*

The intensity of follow-up is adapted based on the needs of the patient, via the moveUP Symptom & QoL monitoring tool.

moveUP enhances the clinical management of the patients, because early detection & management of complications is possible via the symptom & QoL monitoring tool

More efficient clinical management, such as the number of consultations can be reduced

Enforces therapy compliance / adherence

The correct information is provided at the right time

### *moveUP therapy*

With the use of moveUP therapy knee & hip arthroplasty patients can fully rehabilitate via the in-app care team without leaving their home environment.

# COMPANY DETAILS

Head office Brussel: Cantersteen 47,  
B-1000 Brussels, Belgium Office Ghent:  
Oktooipllein 1, B-9000 Ghent, Belgium  
Tel: 0800 88 008  
Email: [info@moveUP.care](mailto:info@moveUP.care) [www.moveUP.care](http://www.moveUP.care)

Basic UDI-DI: ++G166MOVEUPV130WE0P

In case of questions about the App, the tablet or the smart bracelet, you can contact moveUP via the following channels - in order of priority:

- Via the message system of the App. Please preface the message with **“Technical question:”**
- Via email to [support@moveup.care](mailto:support@moveup.care)
- Via phone during office hours: **0800 88 008**

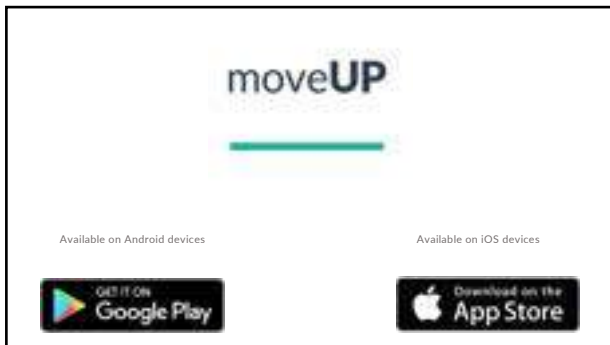
# moveUP


Patient app


CE


1

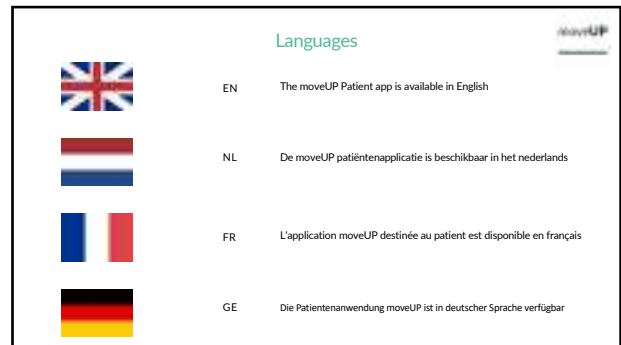
2

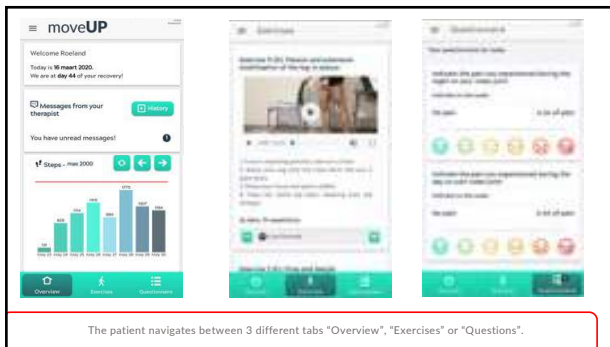
3

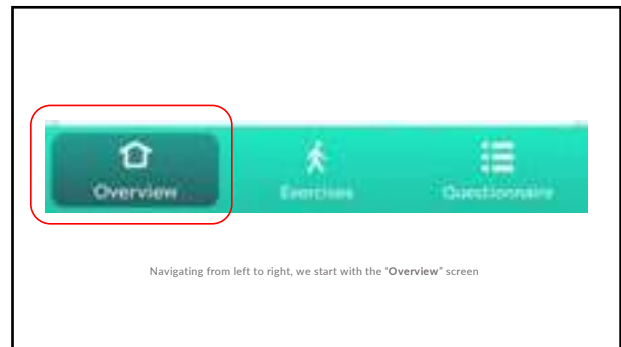
4

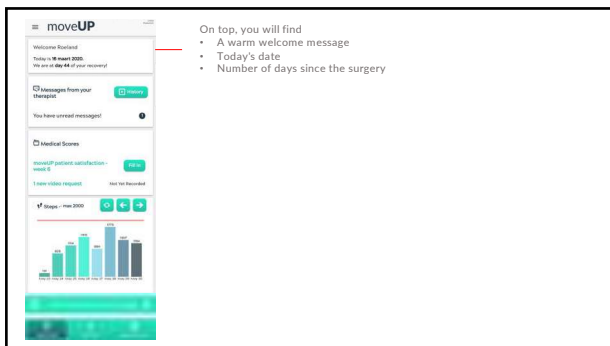5

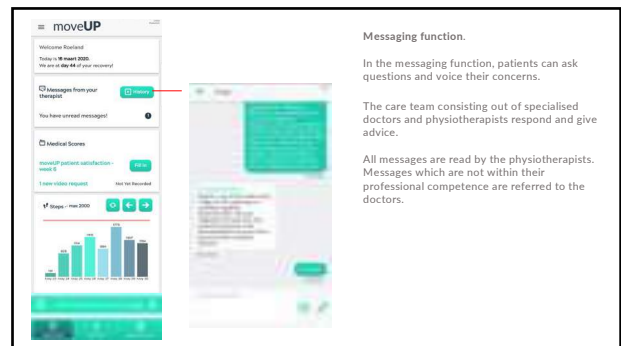
6

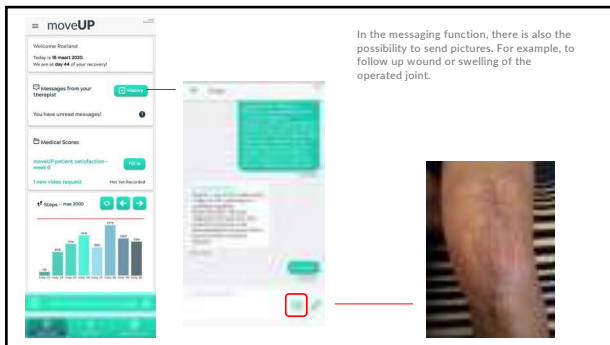
In the messaging function, there is also the possibility to send pictures. For example, to follow up wound or swelling of the operated joint.

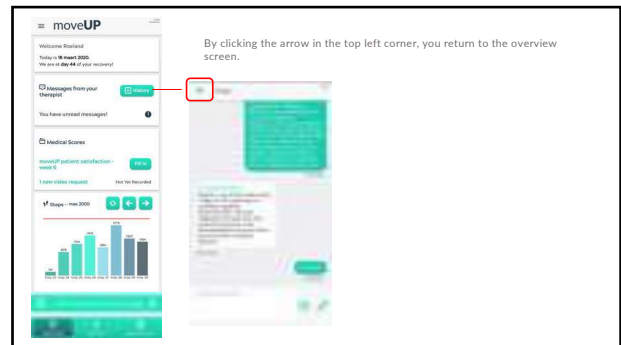7

By clicking the arrow in the top left corner, you return to the overview screen.

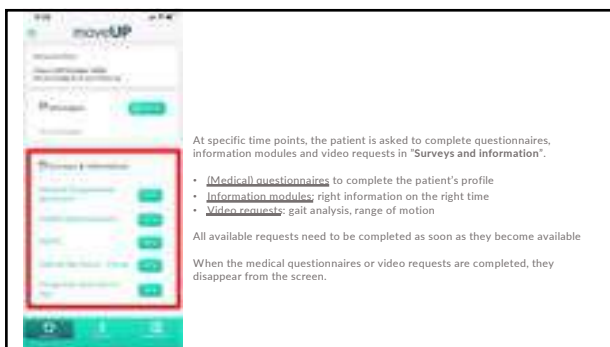8


At specific time points, the patient is asked to complete questionnaires, information modules and video requests in "Surveys and Information".

- Medical questionnaires to complete the patient's profile
- Information modules: right information on the right time
- Video requests: gait analysis, range of motion

All available requests need to be completed as soon as they become available

When the medical questionnaires or video requests are completed, they disappear from the screen.

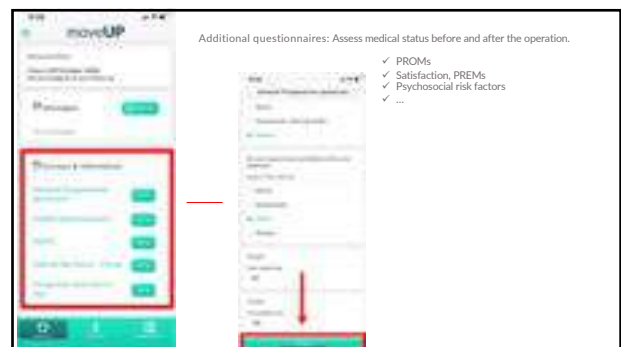
9

Additional questionnaires: Assess medical status before and after the operation.


- ✓ PROMs
- ✓ Satisfaction, PREMs
- ✓ Psychosocial risk factors
- ✓ ...

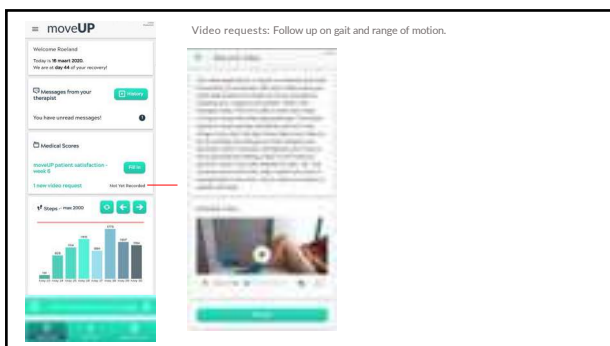10

Video requests: Follow up on gait and range of motion.

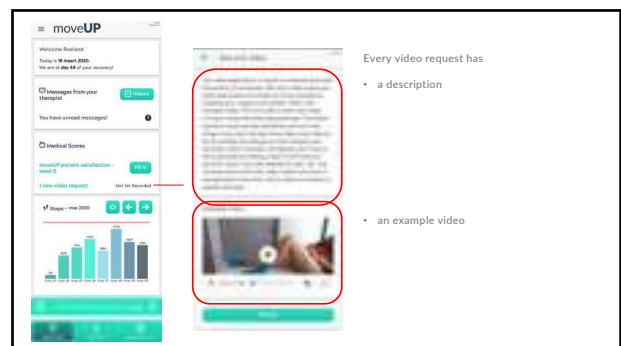
11


Every video request has

- a description
- an example video

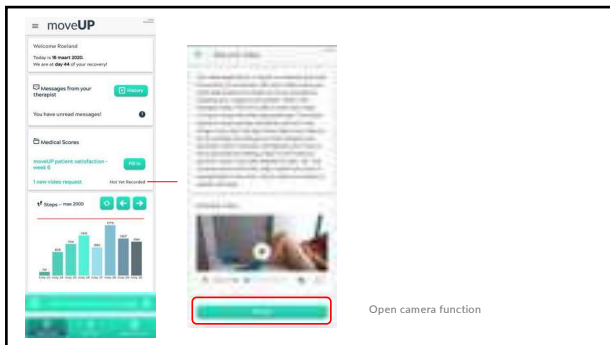
12


13

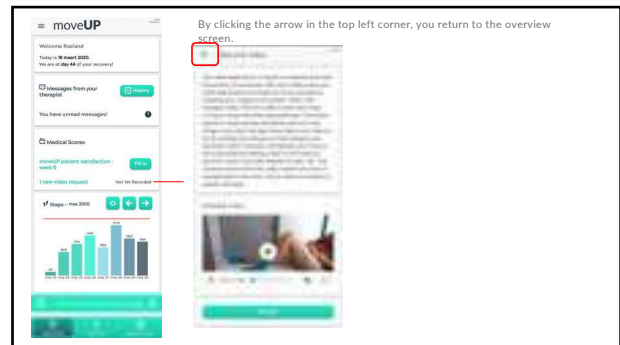
14

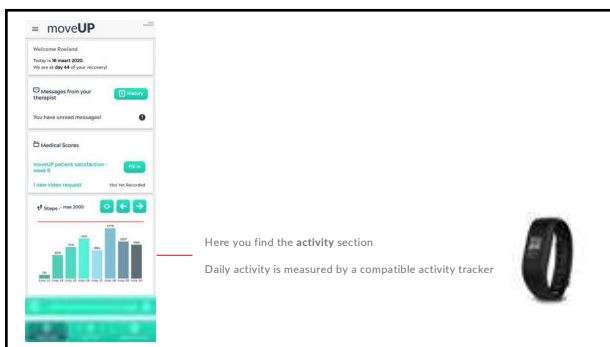
15

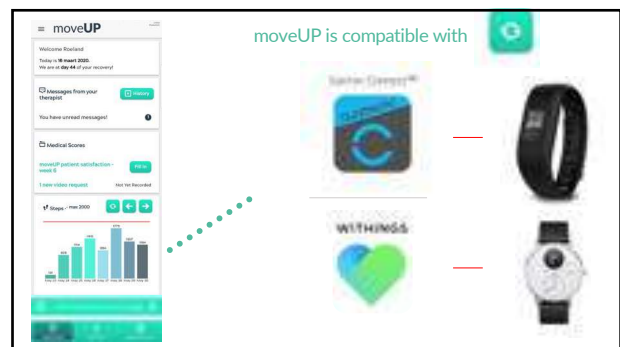
16

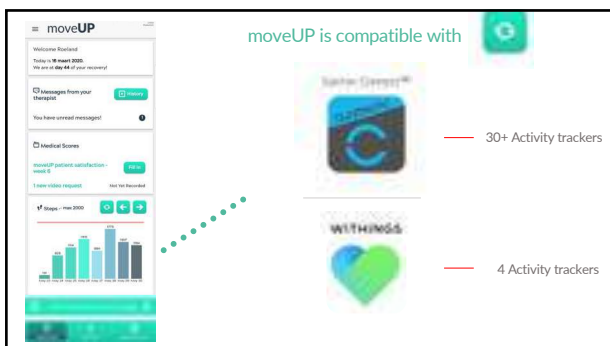17

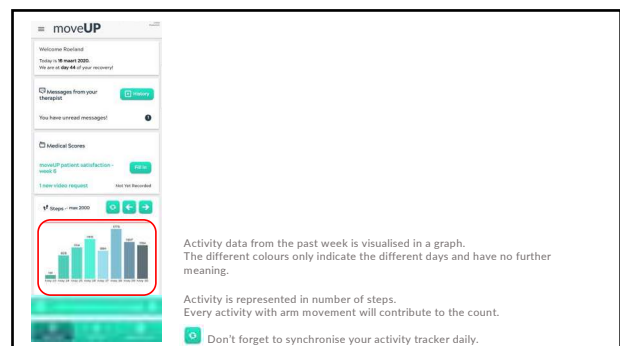
18

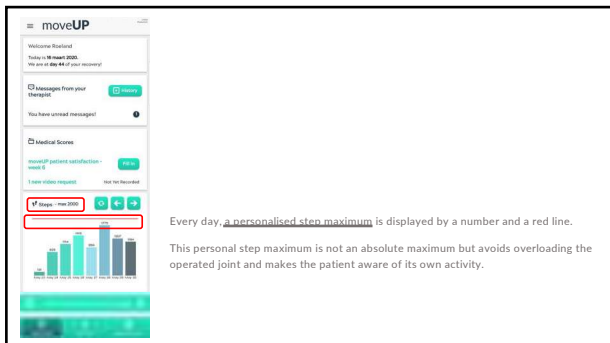
Every day, a personalised step maximum is displayed by a number and a red line. This personal step maximum is not an absolute maximum but avoids overloading the operated joint and makes the patient aware of its own activity.

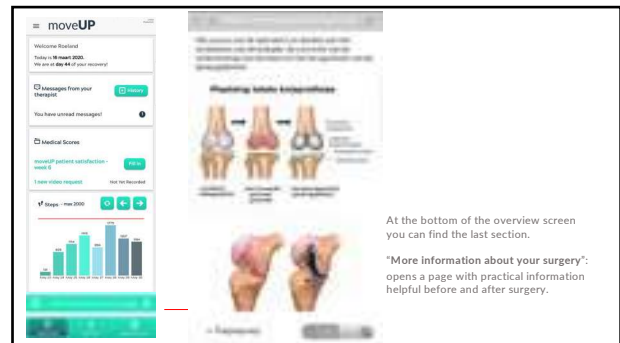19


At the bottom of the overview screen you can find the last section. "More information about your surgery": opens a page with practical information helpful before and after surgery.

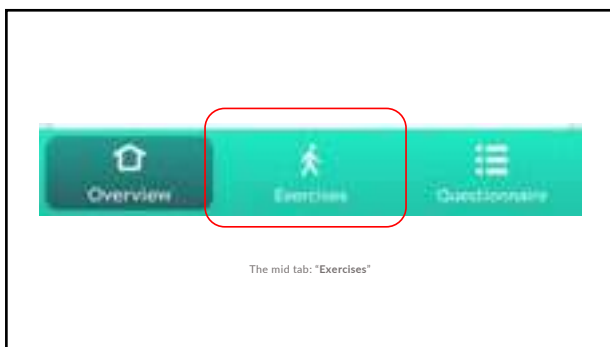20

The mid tab: "Exercises"

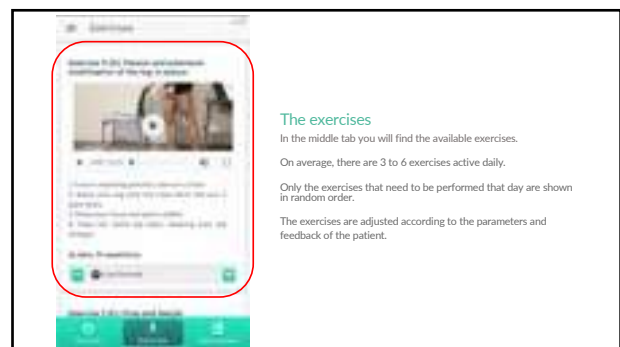21

### The exercises

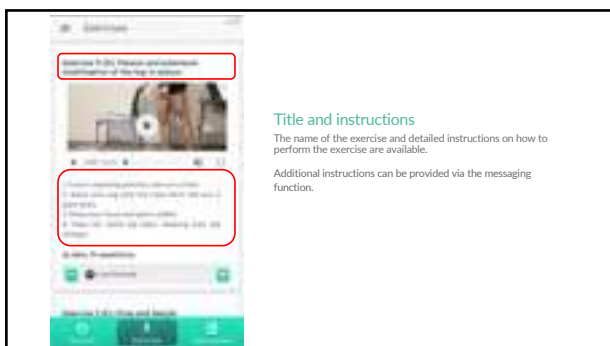
In the middle tab you will find the available exercises.

On average, there are 3 to 6 exercises active daily.

Only the exercises that need to be performed that day are shown in random order.

The exercises are adjusted according to the parameters and feedback of the patient.

22


### Title and instructions

The name of the exercise and detailed instructions on how to perform the exercise are available.

Additional instructions can be provided via the messaging function.

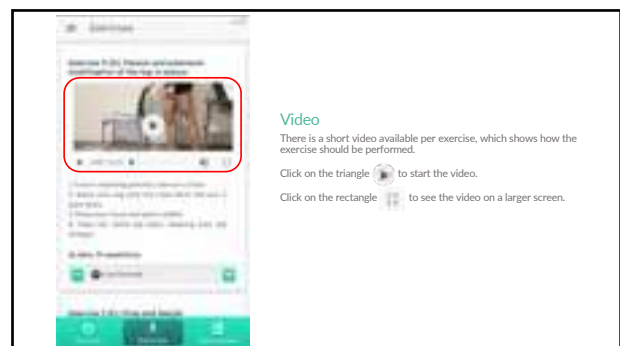23

### Video

There is a short video available per exercise, which shows how the exercise should be performed.

Click on the triangle to start the video.

Click on the rectangle to see the video on a larger screen.

24

**Intensity and frequency**  
 Frequency and intensity are provided per exercise.  
 e.g. 6x a day, 1 minute

We recommend not to exceed the number of prescribed series and repetitions.

25

**Reporting**  
 The patient reports the number of series performed via the + and - symbol.

In this way, the healthcare team is informed of the number of exercises performed.

26

The right tab: "Questionnaire"

27

**Daily questions**  
 By completing the daily questionnaire, the healthcare team is provided with the necessary information to guide and follow up the patient in the best possible way.

The questionnaire is available between 4pm and midnight.

Completing the daily questionnaire takes about 2-3 minutes.

28

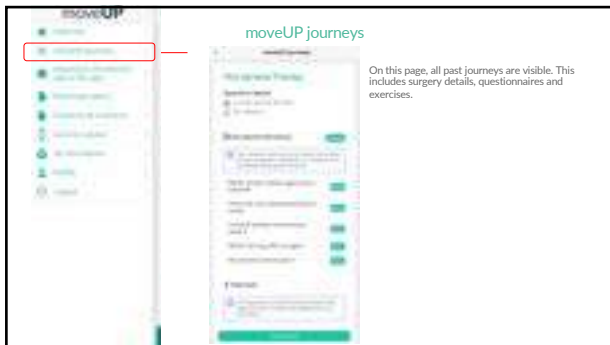
**Send**  
 Patients must complete the questionnaire on a daily basis to receive proper follow-up.

Click on "send" at the bottom of the questionnaire.

29

**App menu**  
 Click on the drop-down button to display the app menu.


30

### moveUP journeys

On this page, all past journeys are visible. This includes surgery details, questionnaires and exercises.

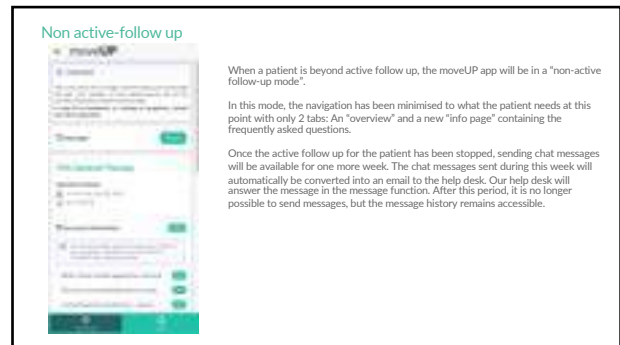31


### Non active-follow up

When a patient is beyond active follow up, the moveUP app will be in a "non-active follow-up mode".


In this mode, the navigation has been minimised to what the patient needs at this point with only 2 tabs: An "overview" and a new "info" containing the frequently asked questions.

Once the active follow up for the patient has been stopped, sending chat messages will be available for one more week. The chat messages sent during this week will automatically be converted into an email to the help desk. Our help desk will answer the message in the message function. After this period, it is no longer possible to send messages, but the message history remains accessible.

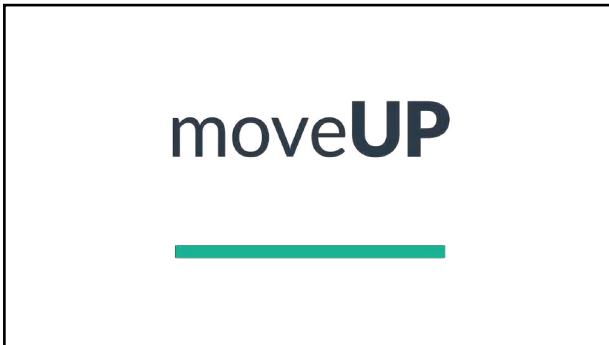
32


1

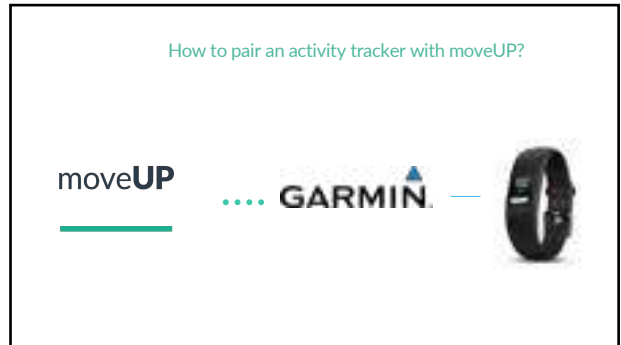
2

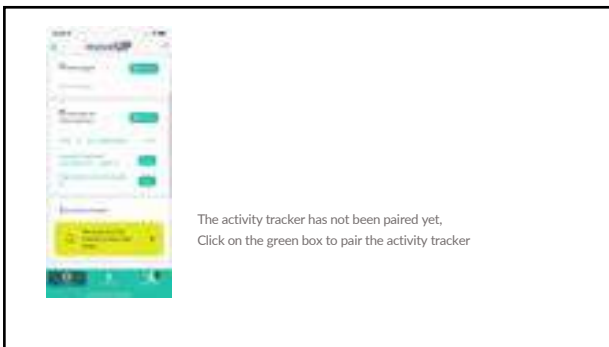
3

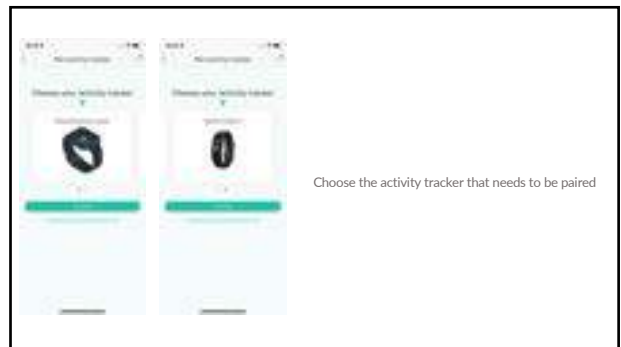4

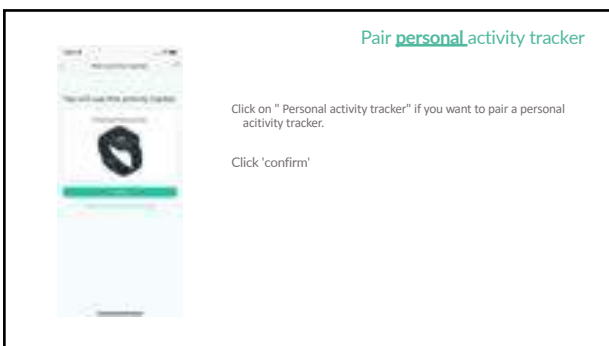5

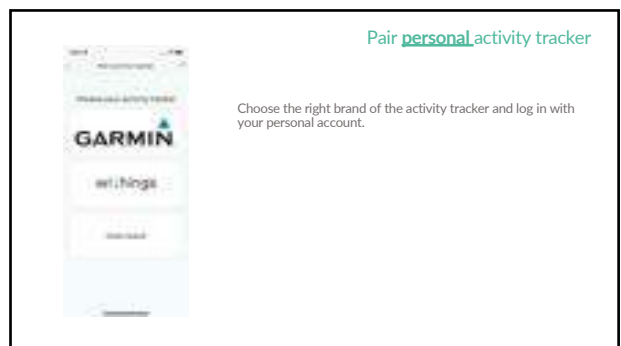
6

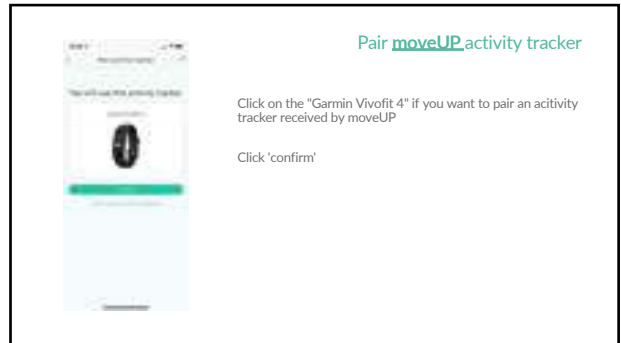
7


8

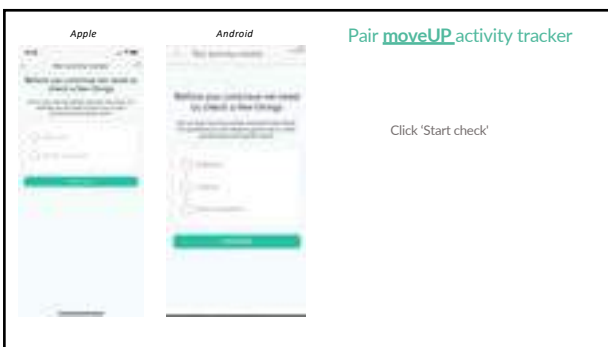
9

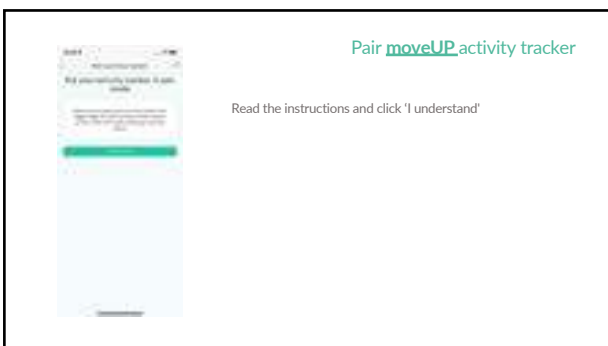10


11

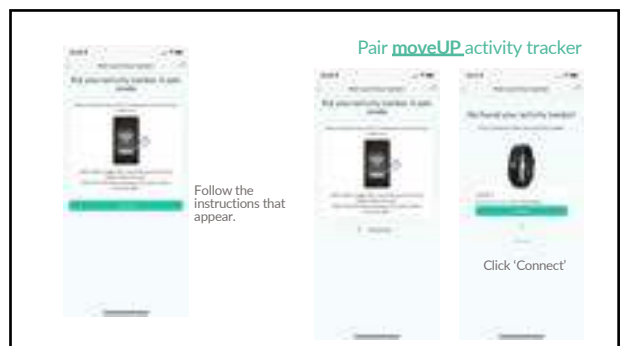
12

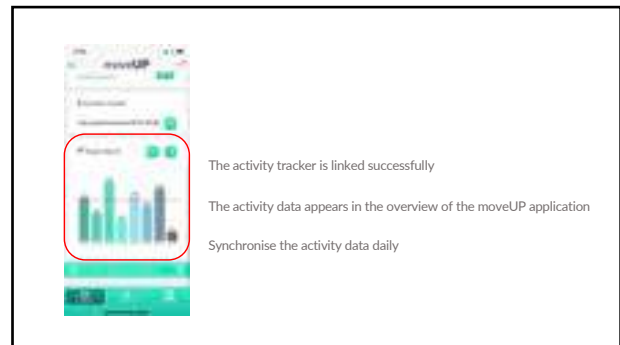
13


14

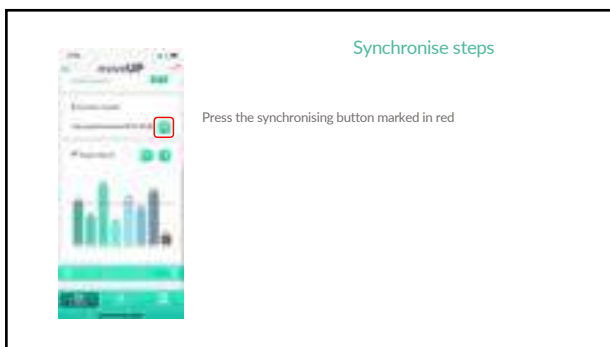
15

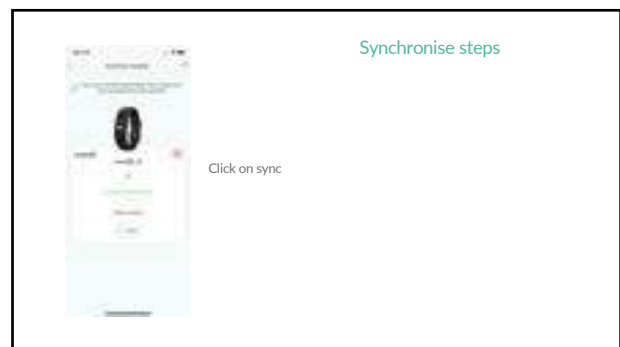
16

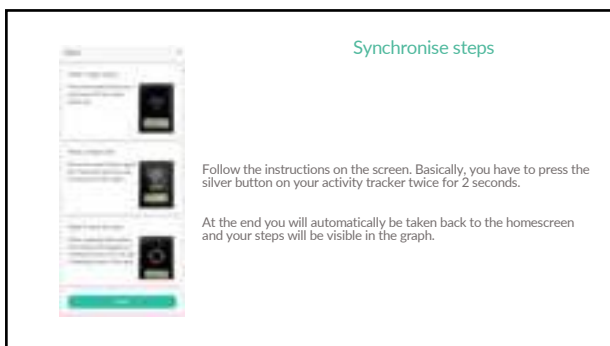
17

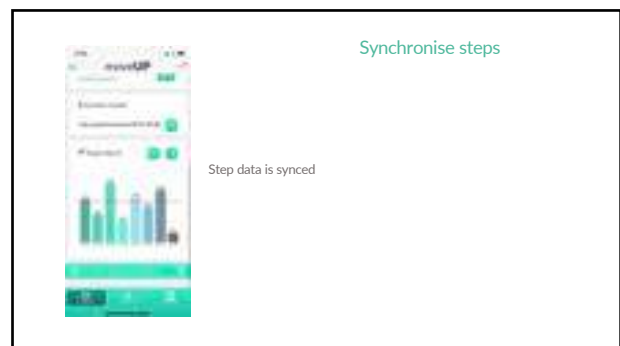18

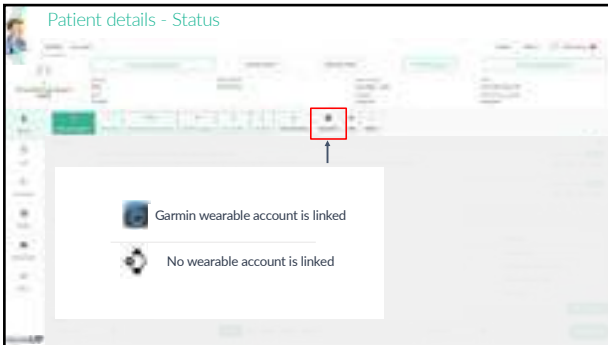19

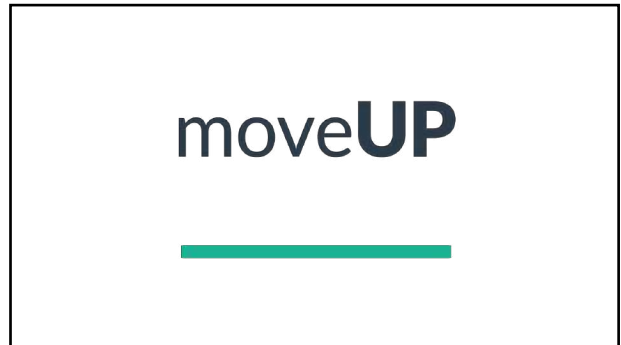
20

# moveUP

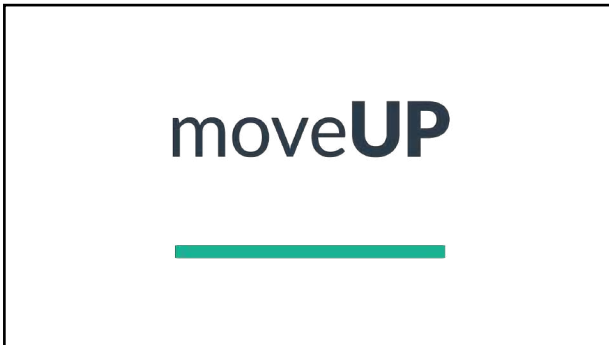Healthcare Platform

CE

1

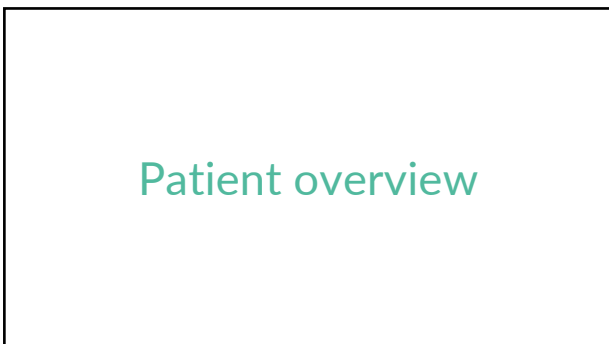
2

3


4

5

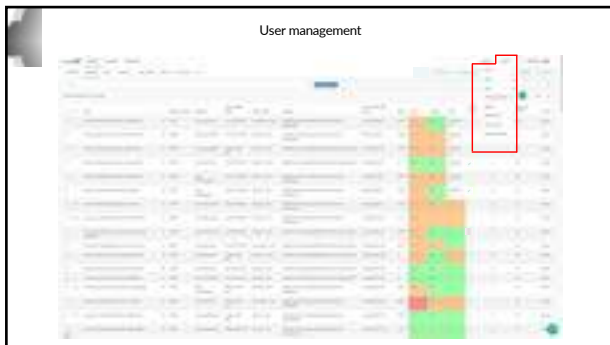6


7

8

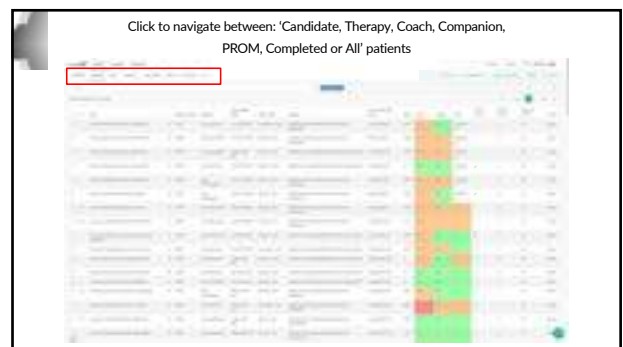9

10

11

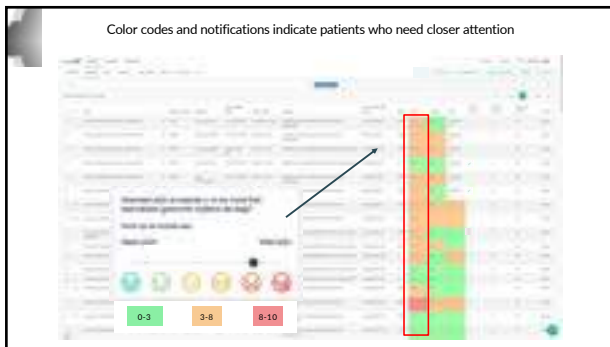
12

13

14

15

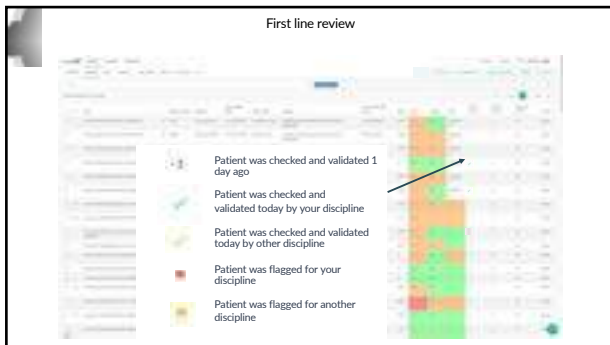
16


17

18

19

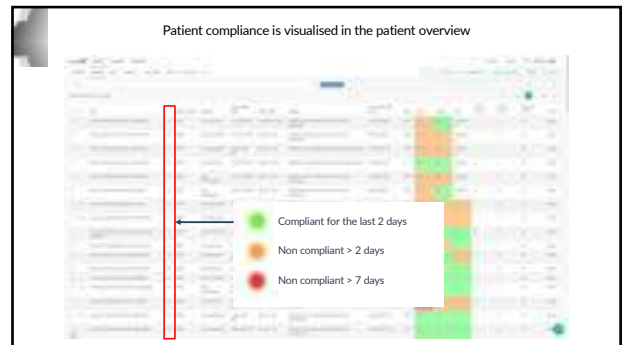
20


21

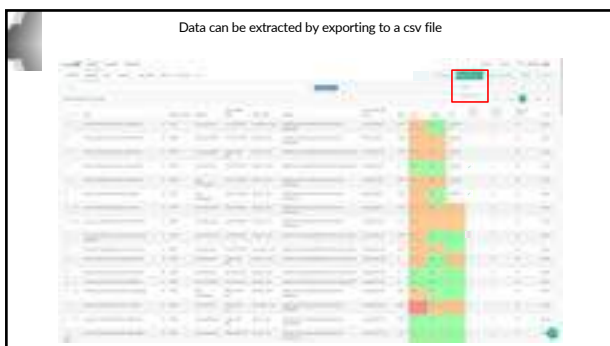22

23

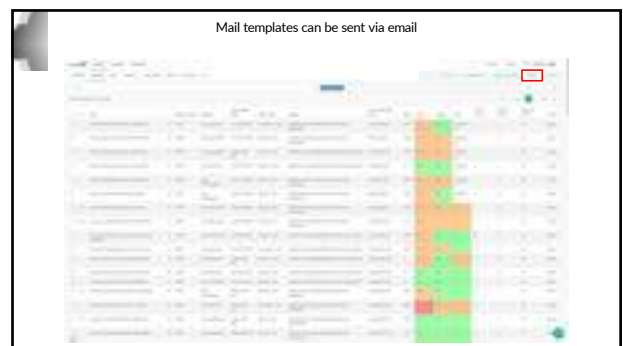24


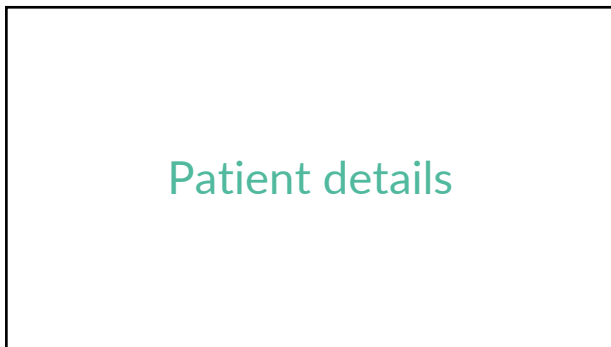
25


26


27

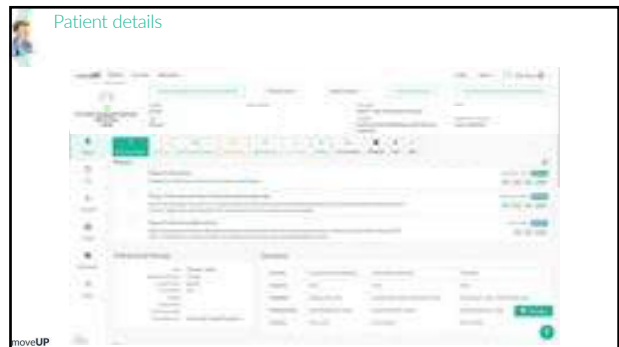
28

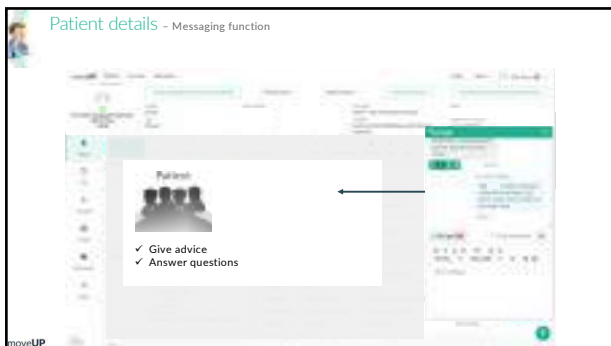
29

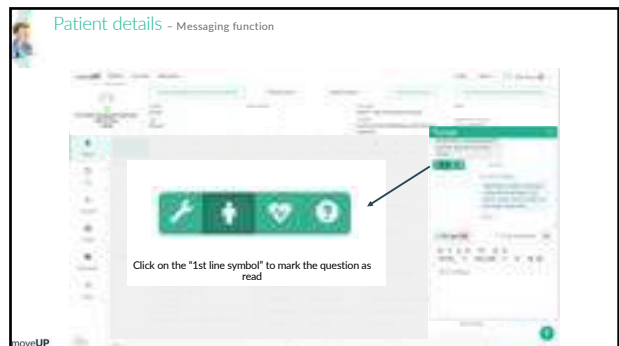
30

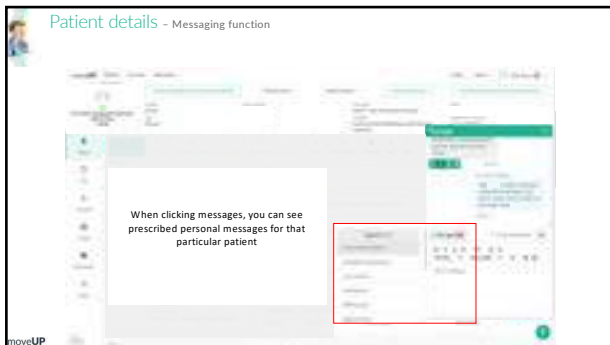
31

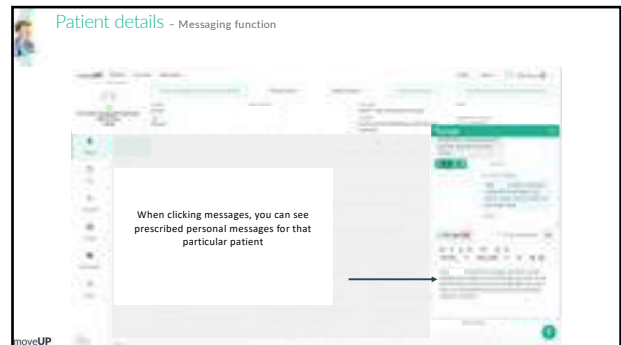
32

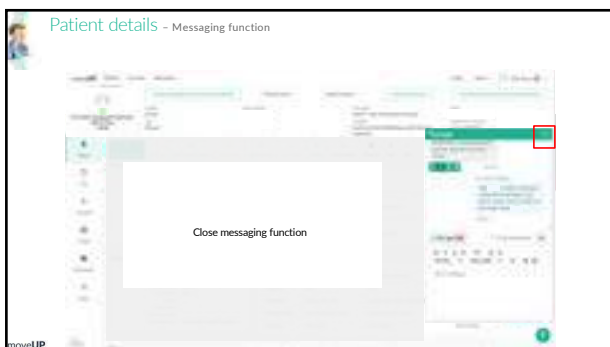
33

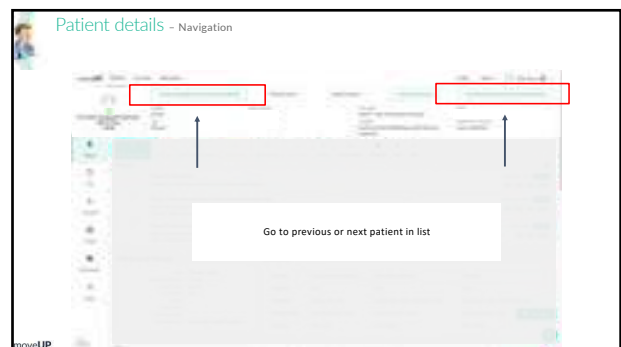34

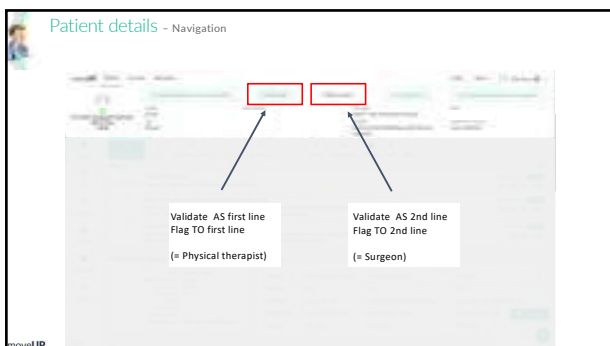35

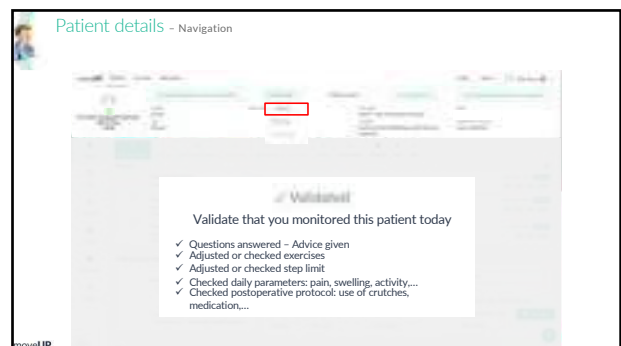36

Patient details - Navigation

**Add Flag**

Flag someone from first line

- ✓ Patient needs to be checked by someone from first line (f.e. a physical therapist)
- ✓ Give a reason for flag
- ✓ Select the discipline to flag
- ✓ Optional: add an e-mail notification to flag. Flagged discipline will receive an email about the flag
- ✓ A flag will add a new line in the patient log with the reason for flagging (see further)

37

Patient details - Navigation

**Add Flag**

Flag someone from second line

- ✓ Patient needs to be checked by someone from second line (f.e. a surgeon)
- ✓ Give a reason for flag
- ✓ Select the discipline to flag
- ✓ Optional: add an e-mail notification to flag. Flagged discipline will receive an email about the flag

38

Patient details - Navigation

**Technical Issue**

Report technical issue

39

Patient details - Status

40

Patient details - Status

See details of the patient and his intervention

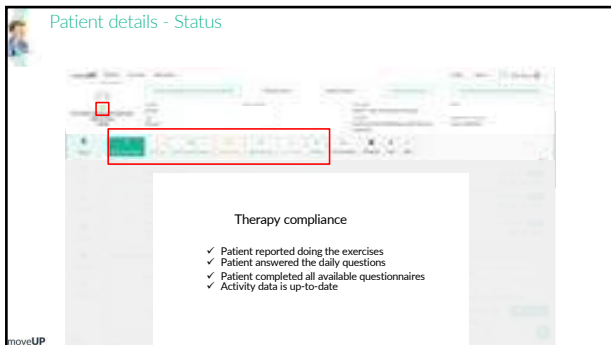
41

Patient details - Status


**Therapy compliance**

- Compliance OK for today or yesterday
- Non compliant > 2 days
- Non compliant > 7 days


42


43

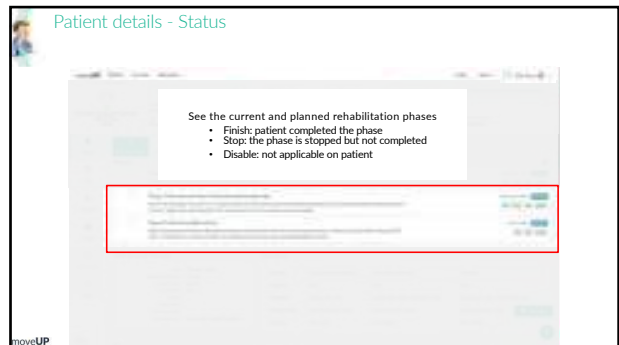
44

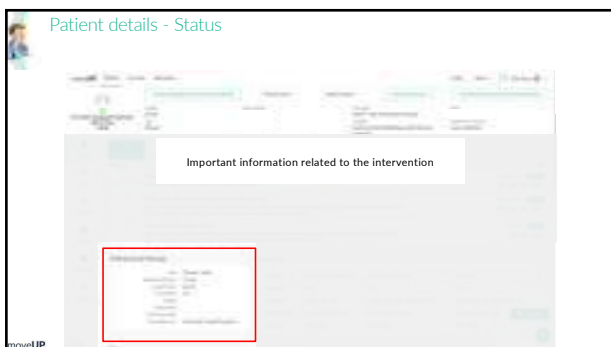
45

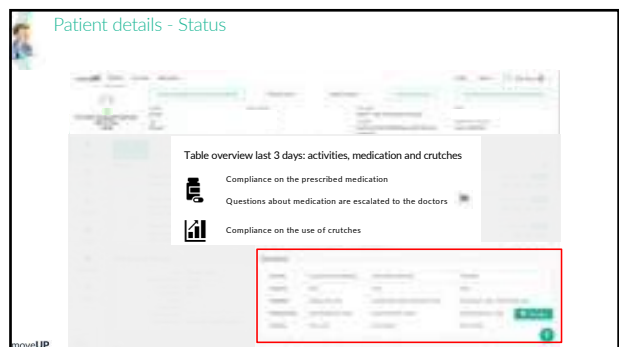
46

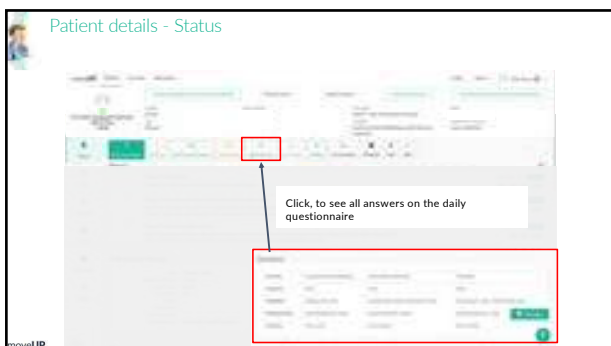
47

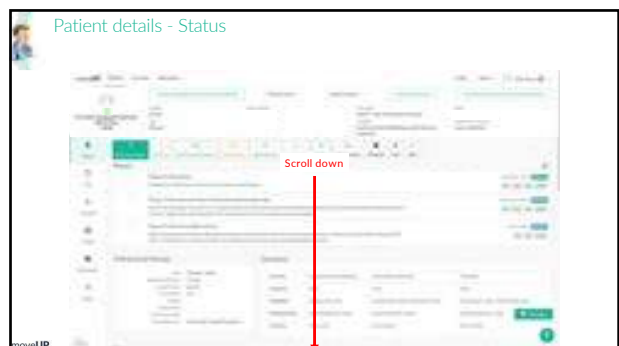
48

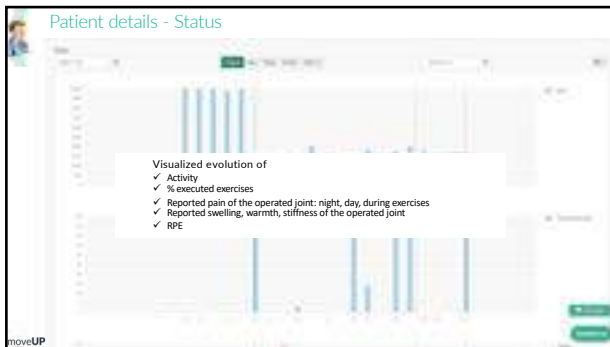
49

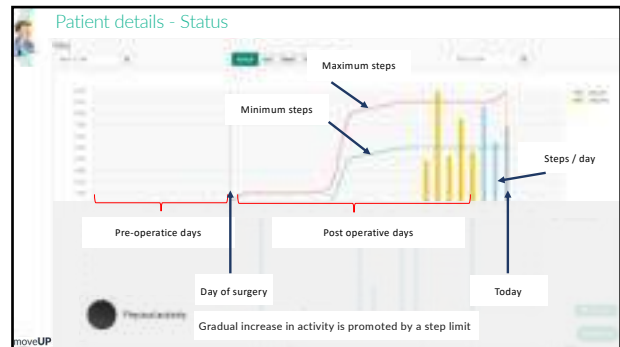
50

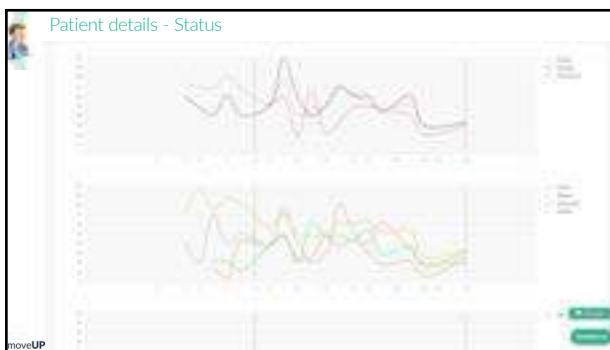
51

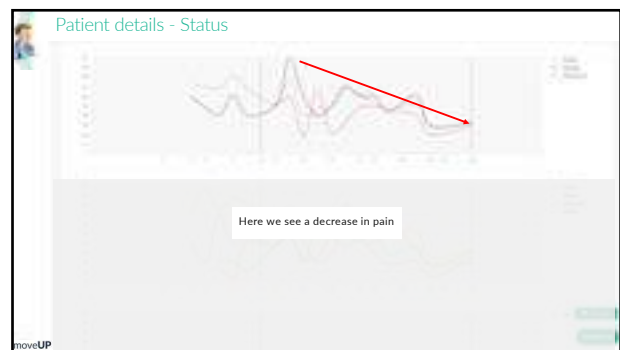
52

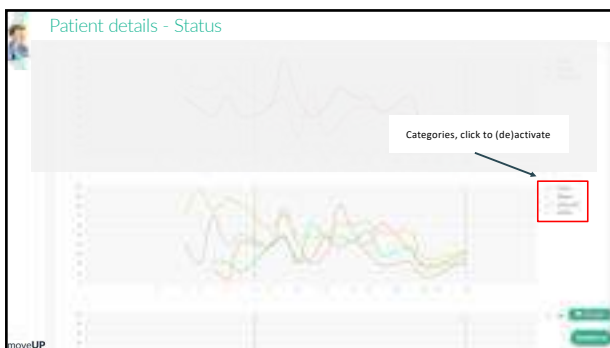
53

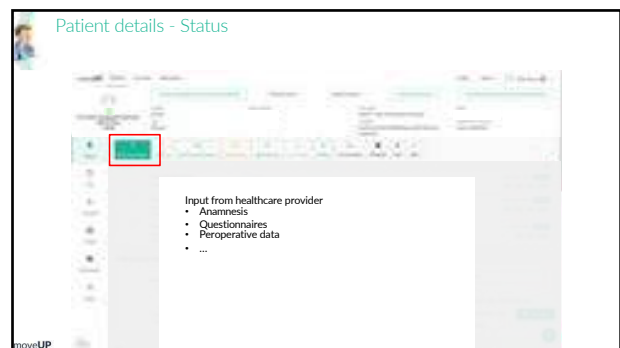
54

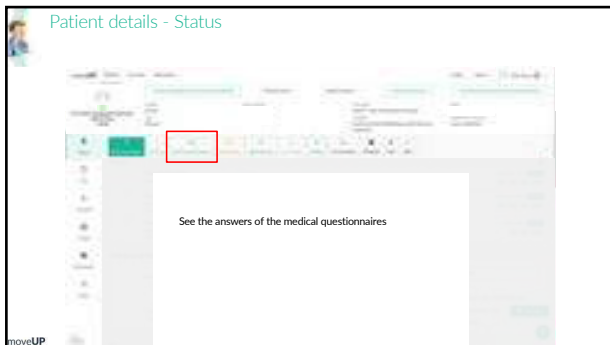
55

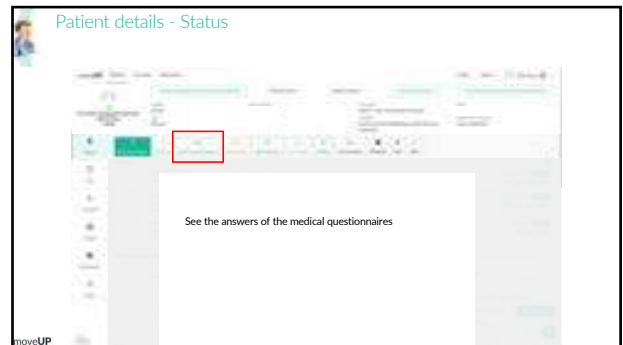
56

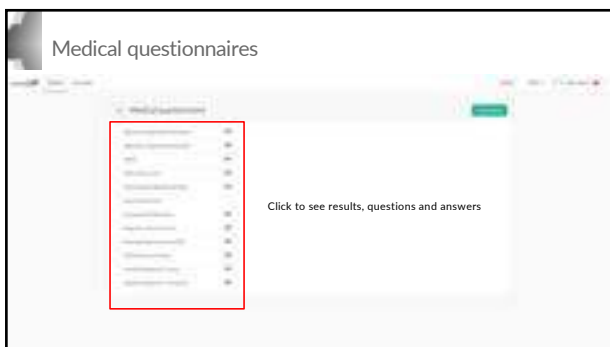
57

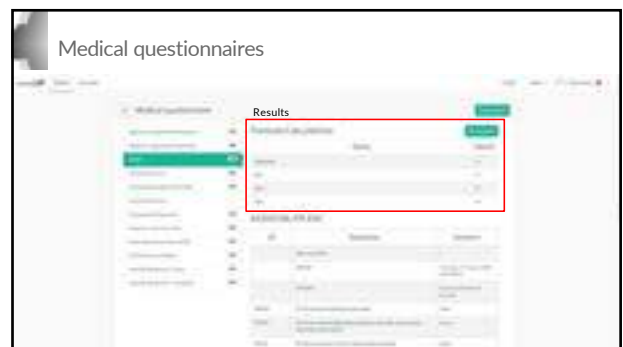
58

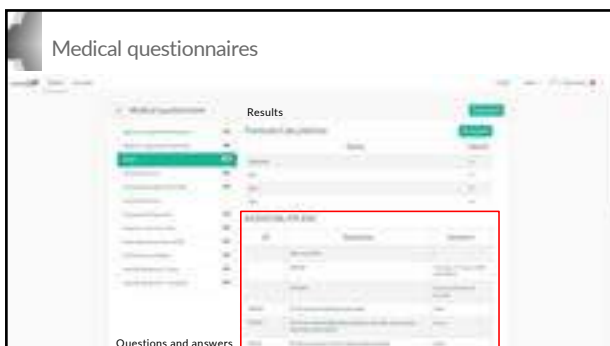
59

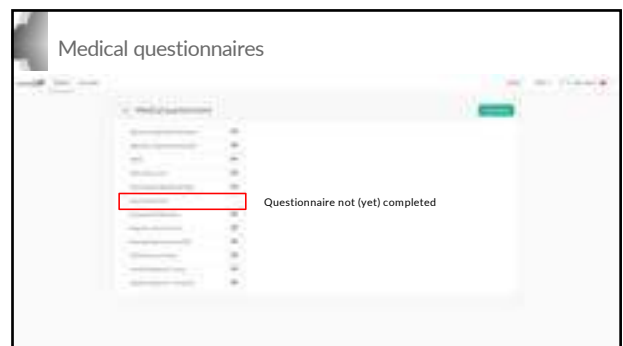
60

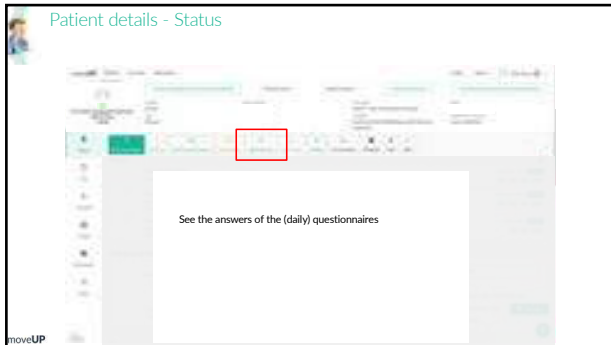
61

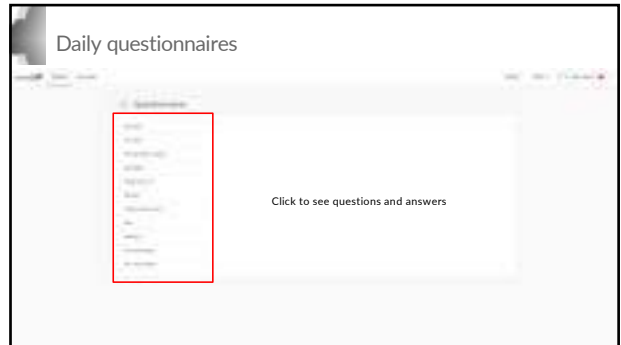
62

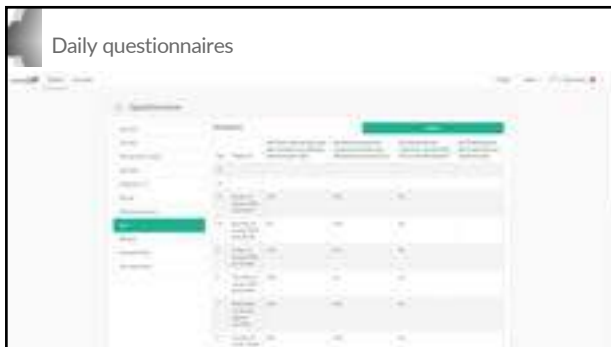
63

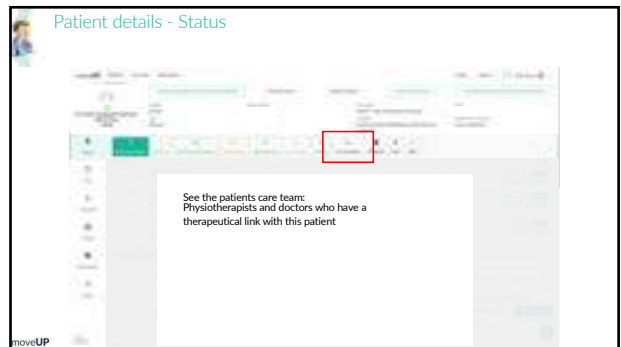64

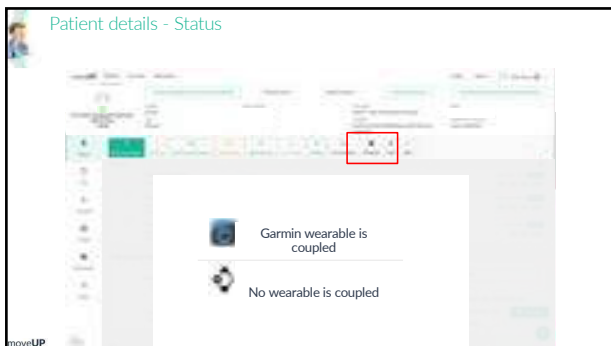65

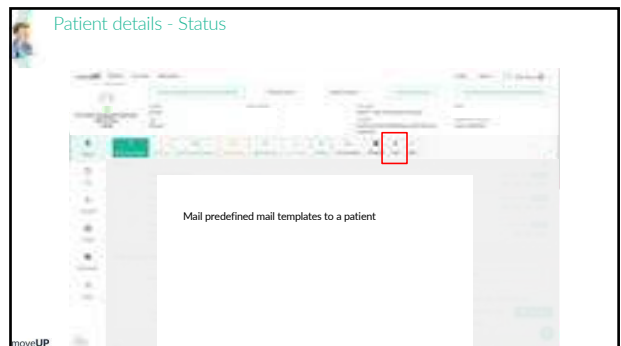
66


# Patient details - Log

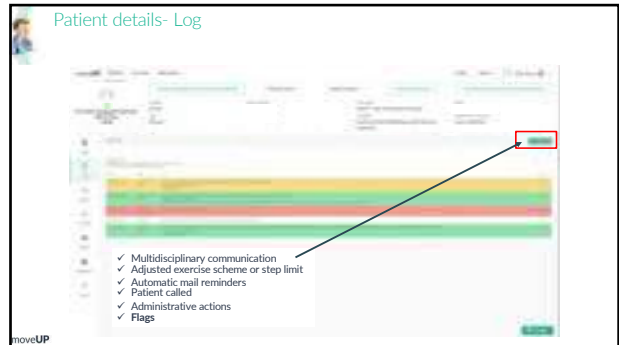67

68

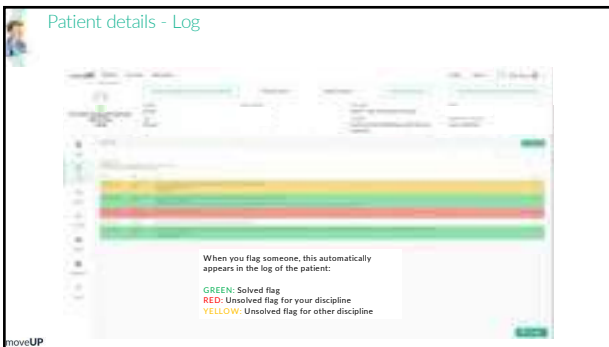
69

# Patient details - Report

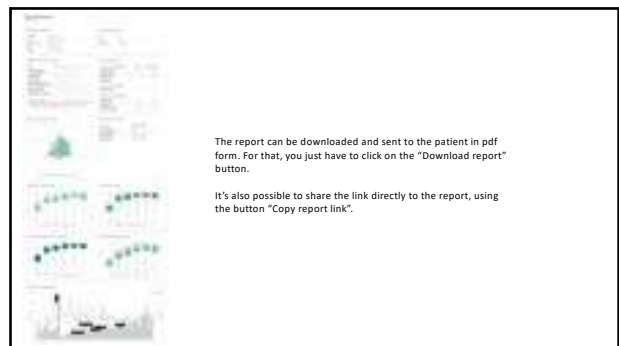70

71


72


# Patient details - Careplan adapt treatment of a patient

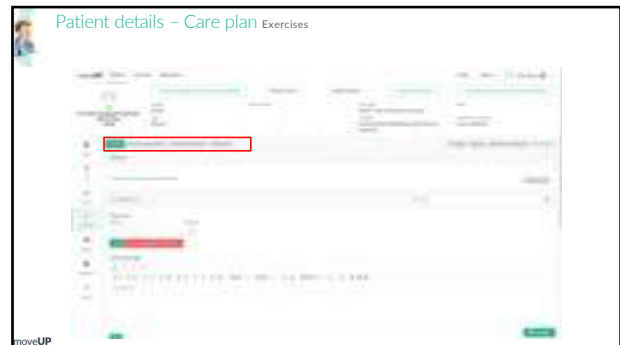
73


74

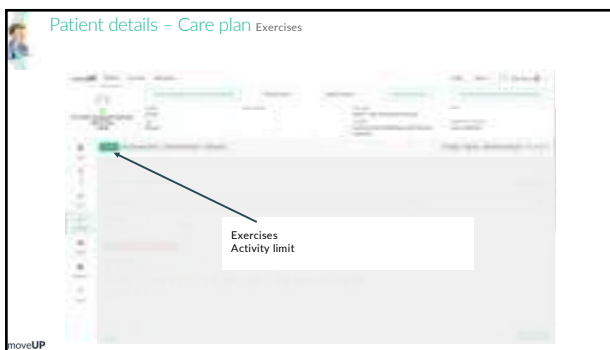
75

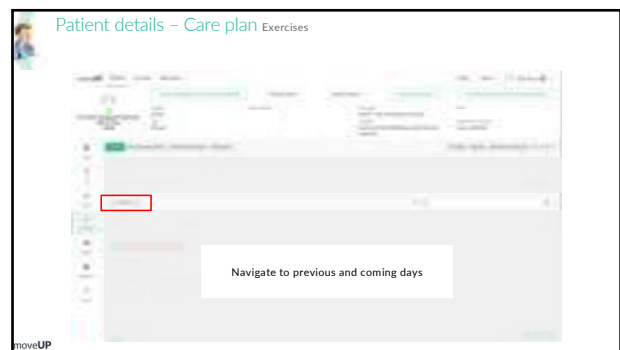76

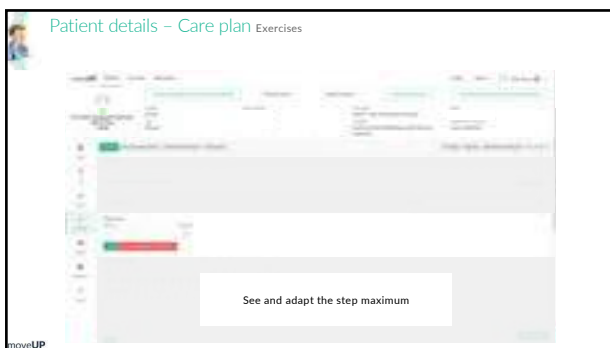77

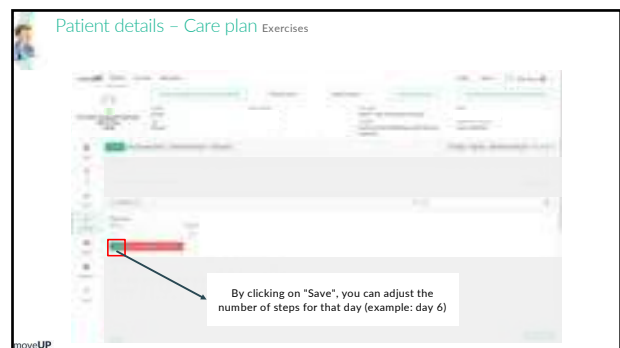78

Patient details - Care plan Exercises

By clicking on "Save & update previous + future days" you can adjust the number of steps for the previous and future days. This way you can change the number of steps for the next 10 days for example.

For example you go 10 days further in this case you navigate to day 16 and set f.e. step maximum to 4000. Then the number of steps evaluate progressively until day 16. (See next slides)

moveUP

79

Patient details - Care plan Exercises

Fill in '4000'

moveUP

80

Patient details - Care plan Exercises

All previous and future days are updated automatically

moveUP

81

Patient details - Care plan Exercises

Adapt the daily message displayed in the exercise scheme

moveUP

82

Patient details - Care plan Exercises

See the exercises for today  
Adapt frequency and intensity

moveUP

83

Patient details - Care plan Exercises

See and adjust the exercise scheme

moveUP

84

**Patient details – Care plan**  
A standardized exercise scheme is personalized based on the patient's need

Blue check marks means that the exercise has been activated  
Gray check marks means that the exercise has not been activated

85

**Patient details – Care plan**  
A standardized exercise scheme is personalized based on the patient's need

There is a planning feature to schedule exercises over a longer period of time (click on *Advanced settings*)

- **Start day/End day:**
  - Start day = the day of the cell you clicked on
  - End day= Fill in the day you want this exercise to end
- **Repeat input field:**
  - Choose how the exercise will repeat till the end day
- **Intensity/ Frequency START:**
  - This is the value where the patient will start on
- **Intensity/Frequency TARGET:**
  - This is the value where the patient will end on.  
E.g. Start day=10, End day=20, Intensity start= 1, Intensity target=10 => On day 15 the intensity will be automatically calculated and in this case will be set to 5.

86

**Patient details – Care plan** Recurring questionnaires

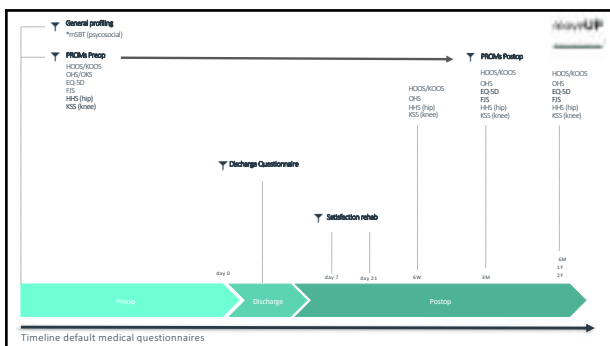
moveUP

87

**Patient details – Care plan** Medical questionnaires

moveUP

88


89


**Patient details – Care plan** Video analysis

moveUP

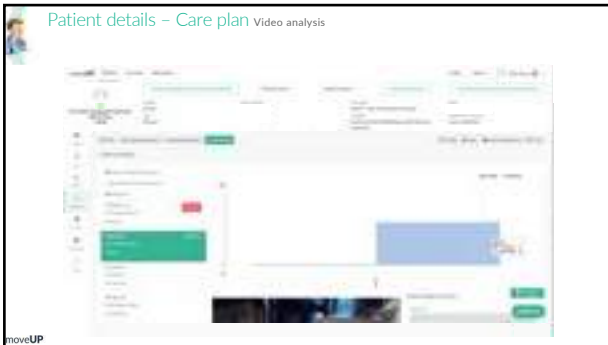
90


91

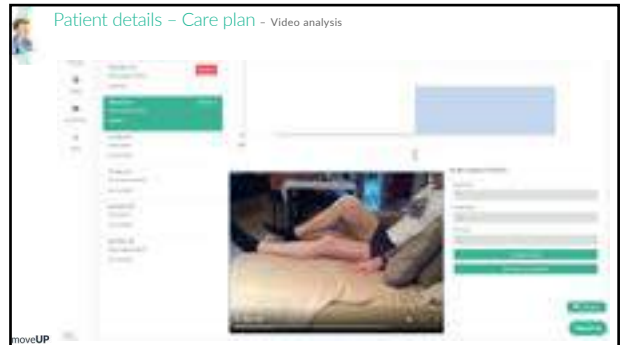
92

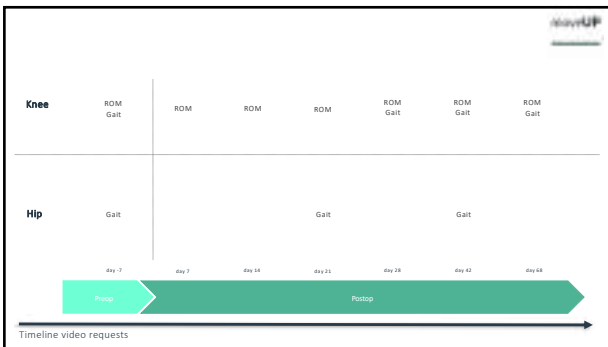
93

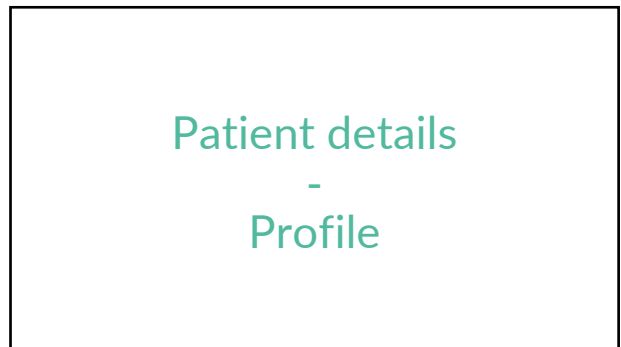94

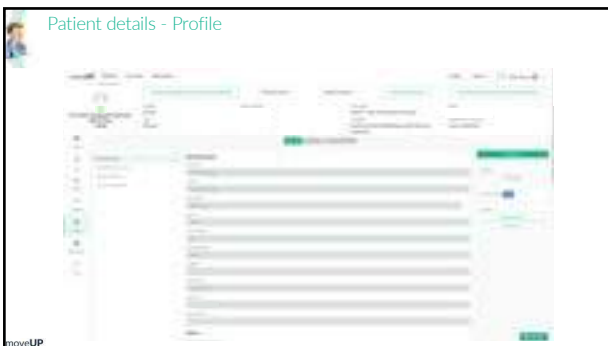95

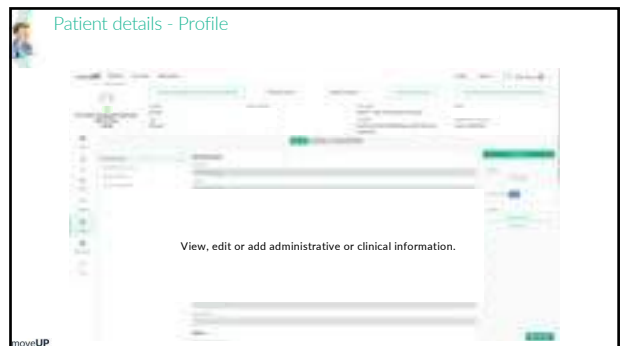
96

## Patient details - Documents

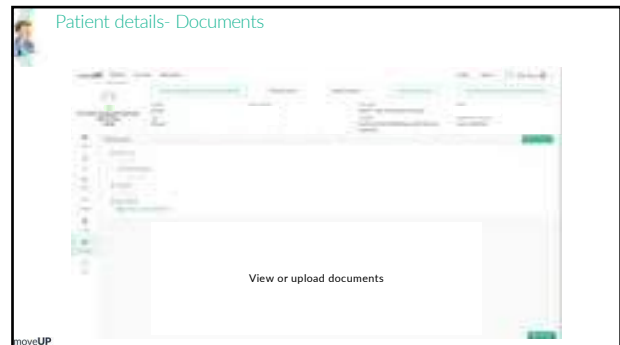
97

98

## Patient details - More

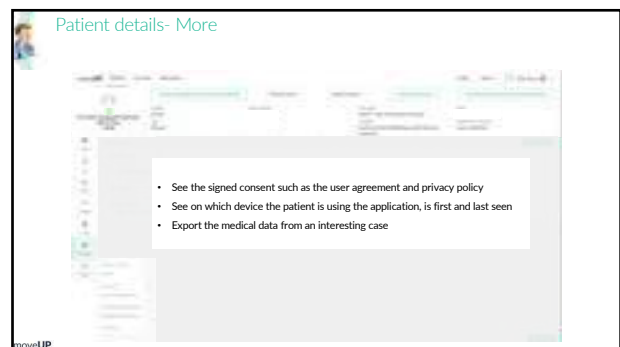99


100


moveUP

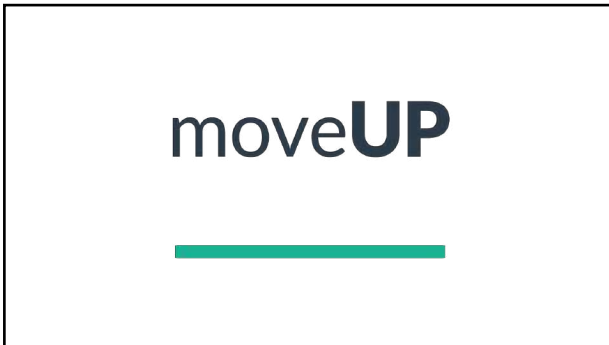101


1

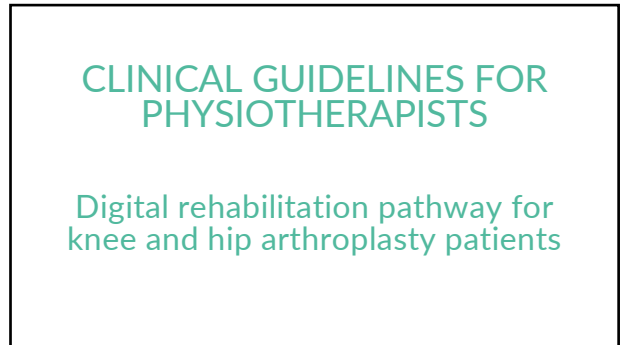
2

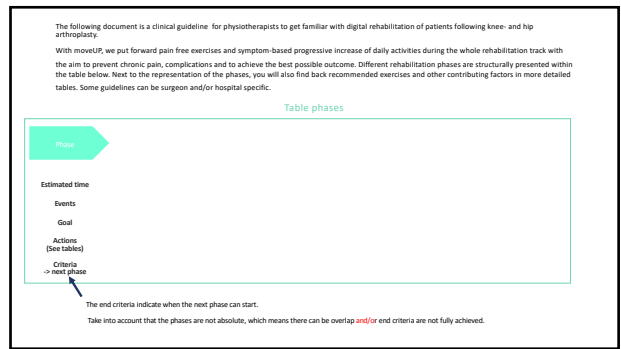3

4

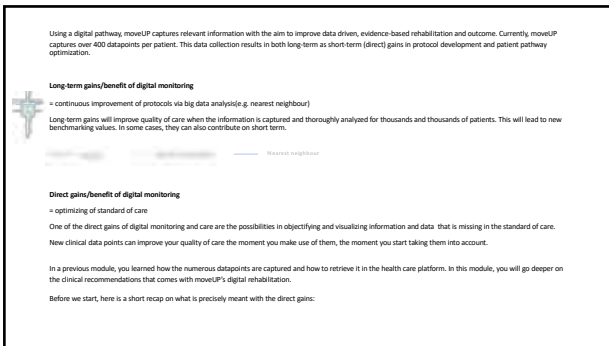5

6

**Direct gain** - e.g. visualization of patient reported parameters: Joint warmth, swelling

- Patient reported warmth and swelling of the operated joint can give you usefull insights on the healing proces.
- See below a decrease in patient reported swelling (green).

7

**Direct gain** - e.g. insight in patient's activity captured by a wrist worn activity tracker

Insight in postoperative activity levels

- Which activity level does the patient reach?
- Does the patient gradually increase activity?
- Does the patient follow the advised step maximum?

Insight in preoperative activity levels

8

**Direct gain** - e.g. consider daily reported activities, medication-intake and crutches

Table overview last 3 days: activities, medication and crutches

- Check compliance on the prescribed medication
- Questions about medication are escalated to the doctors
- Compliance on the use of crutches
- Give advice about transition 2 -> 1 -> 0 crutches
- See which activities the patient has already picked up

9

**Escalation to 2nd line doctors**

- Picture of wound
- Alarming or atypical parameters
- High pain levels and possibility to adapt medication
- Non compliance of medication intake
- Question about medication
- Extra motivation
- ...

Flag is automatically reported in patients log:

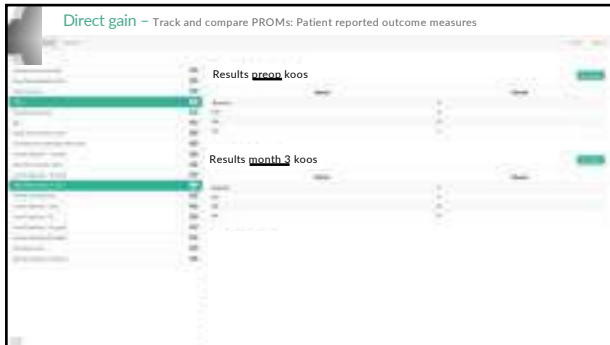
10

**Direct gain** - Track your patient's range of motion over time


11

**Direct gain** - Evaluate specific PROMs answers

12

13

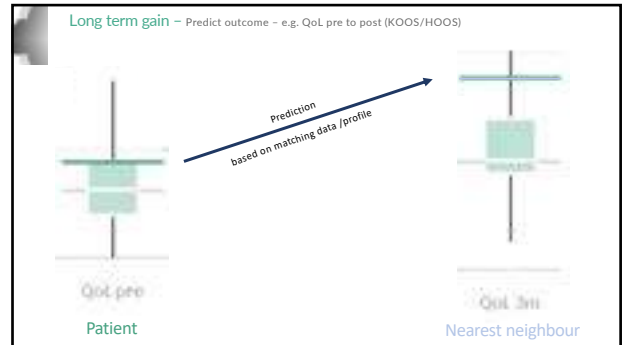14

# Knee arthroplasty

Total knee  
Uni knee

15

Preparation	Hospitalisation Protection post surgical knee	Return to daily activity	Return to active lifestyle
<b>Estimated time</b>	Day 0 - 21	Day 7 - 60	Day 35 - ...
<b>Events</b>	Surgery Hospital stay Discharge First week home Removal of staples	First postoperative consult Week 6 PROMs evaluation Back to work Driving car	Second postoperative consult Month 3 PROMs evaluation Back to work Driving car
<b>Goal</b>	Protection post surgical knee Prevent complications Pain & symptoms control Regain proprioception Knee movement within pain limits	Gradually decrease in pain <2/10 Gradual stop of medication Gain in pain free basic strength ROM 0-90° week 4 Normalised gait pattern Basic balance / coordination	ROM 0 - 110° week 8 Reach full ROM 0 crutches Regain functional strength
<b>Actions (See tables)</b>	Educate & support patient Supervise use of crutches Supervise medication intake Provide pain free exercises Control activity level Check wound Check mobility	Educate & support patient Support transfer 2 crutches -> 1/0 crutches Adapt exercise scheme Support transfer 2 crutches -> 1/0 crutches Add pictures of swelling/around Add non default exercises	Adapt exercise scheme Reassure normal progression Add non default exercises Finalise rehabilitation
<b>Criteria -&gt; next phase</b>	Protection & education Pain & symptom control	A minimum of / no pain A minimum feeling of warmth of the knee Painkilling medication stopped or limited to occasional intake of WHO type 1 medication No more NSAIDs	For some / or exercises surgeon approval is advised e.g. driving, biking, lunges...

16

Preparation	Hospitalisation Protection post surgical knee	Return to daily activity	Return to active lifestyle
<b>Isometric quadriceps setting</b> -> 3 x 10	<ul style="list-style-type: none"> <li>Hip - 1 x 15 min</li> <li>Clavicular - 1 x 15 min</li> <li>Heel sit - 1 x 10</li> </ul>	<ul style="list-style-type: none"> <li>Hip/Knee flexion/extension standing - 3 x 15</li> <li>Half squats/homotrainer - 2 x 5 min</li> <li>Hip/Knee flexion to hip/knee extension - 3 x 10</li> <li>Hip flexion and knee extension mobility exercise on the chair - 3 x 15</li> <li>Mini squat - 3 x 10</li> <li>Standing march - 3 x 1 min</li> </ul>	<ul style="list-style-type: none"> <li>Homotrainer - 3 x 10 min</li> <li>Squat - 3 x 15</li> <li>Step up - 3 x 10</li> <li>Single leg stance - 3 x 10</li> <li>Local step - 3 x 10</li> <li>Forward lunge - 3 x 10</li> <li>Side lunge - 3 x 10</li> <li>Bridging - 3 x 10</li> </ul>

Table knee: Exercises

17

Preparation	Hospitalisation Protection post surgical knee	Return to daily activity	Return to active lifestyle
<b>Activity</b>	<ul style="list-style-type: none"> <li>1. Preop avg &lt; 4000</li> <li>• Max steps 1000/day</li> <li>2. Preop avg 4000 - 8000</li> <li>• Max steps 1500/day</li> <li>3. Preop avg &gt; 8000</li> <li>• Max steps 2000/day</li> </ul> From day 5: + 100 steps/day	Gradual increase until 50-100% preop steps  <b>Positive evaluation</b> +100-200 steps / day  <b>Negative evaluation</b> +100 steps on total b. Decrease in Max steps	Gradual increase until 75% - 100% or > preop steps  <b>Positive evaluation</b> +100-200 steps / day  <b>Negative evaluation</b> a. Max steps on total b. Decrease in Max steps
<b>Medication</b>	PK WHO 1 - 2 NSAID 2/day - 2 weeks Blood thinning medication	PK WHO 1 NSAID 1/day - 2 weeks	Stop PK No NSAID
<b>Mobility</b>	Pain free mobility exercises without forcing	ROM 0° - 90° week 4 Focus on full extension	ROM 0 - 130° week 8 Full ROM
<b>Crutches</b>	2 crutches	2 or 1 crutch	No crutches
<b>Homotrainer</b>	No homotrainer	Homotrainer with resistance	Homotrainer with resistance
<b>Biking</b>		No biking	Biking after surgeon approval
<b>Car</b>		Driving (when feeling safe to do so) *Check the need for surgeon or car insurance approval	

Table knee: Activity - Medication (see addendum) - ROM - ...

18

	Preparation	Hospitalisation Protection post surgical knee	Return to daily activity	Return to active lifestyle
Activity		High activity >50% prep Unsupported unilateral weightbearing Jumping Running Biking Loading activities or sports	High activity >100% prep Jumping Running Biking "High" loading activities or sports	Jumping Running "High" loading activities or sports → Surgeon dependent
Exercises		Pain provoking exercises Exercises against heavy resistance	Pain provoking exercises Exercises inducing pain after execution	Pain provoking exercises Exercises inducing pain after execution
Mobility		Forcing towards flexion / extension	Forcing towards flexion / extension	Forcing towards flexion / extension
Crutches		No use of crutches	No use of crutch(es) for long distances	
Pain		Pain provoking movements or exercises	Pain provoking movements or exercises	Pain provoking movements or exercises
Sleep		Rest/sleep with a flexed knee (e.g. pillow under operated joint)	Rest/sleep with a flexed knee (e.g. pillow under operated joint)	Rest/sleep with a flexed knee (e.g. pillow under operated joint)

Table knee: Don'ts – contra-indications

19

### Hospitalisation & Protection post surgical knee

is a first postoperative phase.

- Focus on protection of the post-surgical knee and education of the patient on how to do so: the patient should take his prescribed medication, use crutches, put ice and forearm rest and elevation of the operated leg.
- Symptom control: Aim to get pain, warmth and swelling under control as soon as possible. Preventing swelling, pain and warmth will prohibit a smooth rehab. Ice, control and prevention of these parameters is key.
- Low load mobility: Motivate the patient to execute low load mobilising exercises frequently. Changing position regularly will reduce stiffness.

**How?**

**Exercises:** See the exercises in the table. The priority are circulatory and light mobilising exercises. Circulatory exercises help drain postoperative blood and swelling and prevent thrombosis. Quadriceps setting stimulates proprioception and measurement of muscle force. These exercises are automatically active in the exercise scheme. Exercises against resistance or those which provoke pain, swelling, stiffness and warmth have to be avoided, including those during a full-to-flow session!

**Activity:** Guidance in activity is provided by a daily, personalized step maximum. (see table for reference)

**The diet:** is used to educate, motivate the patient, address concerns, answer questions and clarify therapeutic actions.

**The use of crutches:** should be explained. Crutches are important to prevent falling and to support the load on the knee so soft tissue get time to heal.

**Goal patients:** Check in heel-to-toe gait pattern and an neutral foot positioning during video analysis.

**Key stage of rehab:** is indicated by video analysis requests: The first mobility check is done at day 7, this first check is a reference point to track improvement. During RCM analysis, also look for movement fluency (e.g. sign of fear of movement), early off knee symptoms and compensation. Reaching full extension without overpressure, has high priority in the beginning of the rehabilitation.

**What if...**

- If a patient doesn't manage to get his pain under control? 2nd line doctors can be flagged to give advice on medication. Most TKA patients are advised to start with WHO step 3 painkillers. Step 2 painkillers can temporarily slow down pain relieving patients.
- If a patient reports a "red flag"? Always flag 2nd line doctors in case of a bleeding wound, picture of wounds, gastric problems, reports of fever, positive urines, visit to general practitioner, medication intake or any other sign of complication.

Flagging the wound care: a picture of the wound after removal of dressings should be saved. Most wounds are healed. The staples are removed starting from 14 days to 21 days postoperative, until staples are removed a patch should cover the wound. In most cases the patch should be changed every 3th day (unless specifically stated to be longer).

20

### Common actions Hospitalisation & Protection post surgical knee

Exercises (see table)	Messages	Check
Default scheme	<p><b>Questions</b></p> <ul style="list-style-type: none"> <li>How did surgery go?</li> <li>Message when home</li> <li>How are the exercises going?</li> </ul>	<p>Therapy Compliance</p>
High activity >no more need for circulatory exercises	<p><b>Feedback</b></p> <ul style="list-style-type: none"> <li>Pain is under control</li> <li>You have difficulties at night</li> <li>Warmth, stiffness, swelling</li> <li>Schedule removal staples</li> </ul> <p><b>Stimulate</b></p> <ul style="list-style-type: none"> <li>To move</li> <li>Adherence of step limit</li> <li>Adherence of use of crutches</li> <li>Long &amp; elevation</li> <li>To not provoke pain</li> <li>To take medication as prescribed</li> </ul>	<p>Heal</p> <p>Wound RCM</p> <p>Early RCM/progression (if not necessary)</p> <p>Swelling</p> <p>Warmth</p>
		<p>Use of crutches</p> <p>Medication intake</p> <p>Daily activity and step max</p>

Actions Hospitalisation & protection post surgical knee

21

### Protection post surgical knee

✓ Protection & education  
✓ Pain & symptom control

Return to daily activity

### CRITERIA

- ✓ The patient is compliant to the postoperative rehabilitation guidelines.
- ✓ The patient found a balance between low load, regular, basic exercises and rest to decrease/control post-operative pain and swelling.
- ✓ Transition to the next phase is advised when the patient has control over the postoperative complaints.

Transition phase protection post surgical knee -> Return to daily activity

22

### Return to daily activity:

is a second postoperative phase.

- Focus on a continuous decrease in symptoms and pain while reducing WHO type 2 medication to a minimum level to increasing load by new exercises and allowing higher activity.
- Support the patient finding the right balance between rest and activity.

**How?**

**Exercises:** End back the exercises in the table. Personalize (advise or delay) them in the exercise scheme based on the daily parameters. A maximum of 4 exercises / day is advised. With movement, we believe in controlled and painfree exercise during the whole rehabilitation. Therefore, when continuing exercises from previous phase and/or adding new exercises, a positive evolution should still be noticeable. Performing any of the exercises should not provoke pain, induce warmth and/or swelling, rising and extension of the operated leg is still advised to get these parameters fully under control.

**Activity:** Guidance in activity is provided by a daily, personalized step maximum.

- During a positive evolution, the patient is allowed to increase activity by 100-200 steps/day.
- During a negative evolution, the patient should remain on its activity level or have more rest (decrease in activity).
- avoid pain activity during the whole rehabilitation (see table for reference).

**The message feedback:** is still continuously used to motivate the patient, address concerns, answer questions and clarify therapeutic actions.

**Goal patients:** Check in heel-to-toe gait pattern and an neutral foot positioning during video analysis. Check on compensation.

**The use of crutches:** Transition from 2 to 1 and 1 to no crutches should be motivated when painfree and safe.

**Key range of motion:** It is very important to see continuous improvement in the active ROM. A minimum of 90° active flexion should be achieved before the end of week 4. If not, contact the treating surgeon and provide him/her with this information. Full knee extension is stimulated daily by knee extension posture added with instructions to also focus on full extension while walking and standing. During knee or passive flexion mobilizations, it is not advised to force reaching up to the painless.

Besides daily data collection and visualization, during this phase, patient's evolution is tracked by postoperative RCM collection. Compare the pre- to postoperative results and give feedback to the patient. In most cases re-assessment of overall progression is sufficient. At the end of this phase, the biggest gains have been made but naturally, patients should still experience improvement. Most patients generate improvement until one year post-op.

23

### Common actions Return to daily activity

Exercises (see table)	Messages	Check
Adapt exercise scheme: advance or delay phase 2 exercises	<p><b>Questions</b></p> <ul style="list-style-type: none"> <li>How was first postoperative consult?</li> </ul>	<ul style="list-style-type: none"> <li>Reaction on adaptations in the exercise scheme</li> <li>wound</li> </ul>
Basic strength Proprioception Mobilization	<p><b>Feedback</b></p> <ul style="list-style-type: none"> <li>Circular increase in exercises and activity</li> <li>Painkilling medication lowered to occasional use of WHO type 1 medication</li> <li>Resistance and supporting when there are ups and downs</li> <li>PRIMA results</li> <li>Timing 2-1-1 To crutches</li> <li>Heading towards excellent result</li> </ul>	<p>PRIMA: prep vs week 6</p>
Delay exercises which increase pain, warmth, swelling	<p><b>Stimulate</b></p> <ul style="list-style-type: none"> <li>Circular built-up basic strength and ACL without pain</li> <li>Motivate RCM in ACL</li> </ul>	<ul style="list-style-type: none"> <li>Knee ROM (90° before week 4)</li> <li>Gait pattern</li> </ul>

Actions Return to daily activity

24

Return to daily activity → **Minimum of pain**  
Painkilling stopped or limited use → Return to active lifestyle

### CRITERIA

- ✓ A minimum of / no pain
- ✓ A minimum feeling of warmth of the knee
- ✓ Painkilling medication stopped or limited to occasional intake of WHO type 1 medication
- ✓ No more NSAIDs
- ✓ for some /a/ or exercises surgeon approval e.g. driving, biking, lunges,..

Transition Return to daily activity -> Return to active lifestyle

25

### Return to active lifestyle

in a 3<sup>rd</sup> postoperative phase.

- Focus on functional strengthening exercises.

**Overload:** First back the exercises in the table gradually and the exercises done by one, while clearly following the patient's parameters in the graph or table. Do not forget to increase exercises from previous phase. A maximum of 6 exercises/ day is advised.

**Before starting the functional strengthening exercises,** pain, warmth and swelling are stable and at a low level. Medication is limited or stopped. During and after the functional strengthening exercises, these parameters should stay stable and/or at a minimum. There should be no need to increase medication intake.

**During this phase,** using can still be useful at the end of a busy day.

**Activity:** Still take a gradual increase in activity. Give recommendations about physical activity in general.

**The evening function:** During this phase, reassurance of normal progression is often needed. Point out the fact that most patients notice a positive evolution up till 1 year after surgery.

**Goal patients:** Check on training goal patients and on overall non-painkilling during video analysis. Check on compensation.

**Range of motion:** While swelling and pain are reduced to a minimum, mobility should have evolved to 0°-130° at around week 8-10. Consider that the achievable ROM greatly depends on the patient's pre-mobility.

**Finalize rehabilitation** by checking if all expectations are met. You can check patient's satisfaction and outcome at the month 3 PROM collection.

26

### Common actions Return to active lifestyle

Exercises	Messages	Check
Adapt exercise scheme: advance or delay certain exercises	<p><b>Questions</b></p> <ul style="list-style-type: none"> <li>How was the 3m consult?</li> <li>How far from your goals/expectations?</li> <li>Any movements/activities which are still difficult/painful?</li> </ul>	Evaluate response on higher load
Functional strength	<p><b>Feedback</b></p> <ul style="list-style-type: none"> <li>Right timing 1- &gt; 0 crutches</li> <li>PROMs 3 month</li> </ul> <p><b>Stimulate</b></p> <ul style="list-style-type: none"> <li>Back to work</li> <li>Full ACL update</li> </ul>	<p>Video ROM</p> <p>Video gait</p> <p>Patient satisfaction</p>
		<ul style="list-style-type: none"> <li>Functional ROM (130° + full extension)</li> <li>Normal gait pattern</li> </ul>

Actions Return to active lifestyle

27

### APPROACH knee arthroplasty

**Prep** → **Hospitalisation** (Protection post surgical knee) → **Return to daily activity** → **Return to active lifestyle**

**Preop day -1 to day -1:** Functional strength & mobility: Abduction, quadriceps setting, isometric glutes, hamstring, knee extension posture. Adapted on demand and patient profile.

**Surgery - hospital stay: day 0 to day 5:** Preload regular gait patterns, Elevation and circulation.

**Symptom control, proprioception and mobility: day 0 to day 21:** Reduction swelling and pain, full extension, 90° flexion.

**ADP: Circulatory, heel and regular (low-load) movement**

**Proprioception: Quadriceps setting**

**Basic strength and mobility: day 7 to day 60:** Prevention of over-jumping/landing and build up of ACL.

**Basic mobility: Prone/side, single flexion, extension posture.**

**Basic strength: Side toe taps, mini squat...**

**Functional strength: from day 25:** Low load to higher load.

**Functional strength: Single leg stance, squats, lunges, bridging.**

28

### Hip arthroplasty

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- DAA
- ALA

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29

### Phases hip

	Preparation	Hospitalisation Protection post surgical hip	Return to daily activity	Return to active lifestyle
<b>Estimated time</b>	Day 0 - 21	Day 10 - 60	Day 35 - ...	Day 55 - ...
<b>Events</b>	Surgery Hospital stay Discharge	First postoperative consult --	Second postoperative consult Back to work Driving car --	Full ROM 6 crutches Functional strength
<b>Goal</b>	Protection post surgical hip Prevent complications Pain & symptoms control Proprioception	Gradually have no more pain c2/10 Gradual stop medication Basic strength Normalized gait pattern Prevent/normalize groin pain Basic balance / coordination		
<b>Actions (See tables)</b>	Educate & support patient Supervise use of crutches Supervise medication intake Provide pain free exercises Control activity level Check wound Gait check	Educate & support patient Advise 2 crutches → 10 crutches Adapt exercise scheme		
<b>Criteria → next phase</b>	Pain & symptom control Protection & education	A minimum of / no pain Painkilling medication stopped or limited to occasional intake of WHO type 1 medication No more NSAIDs	For some /a/ or exercises surgeon approval is advised e.g. driving, biking, lunges...	

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31

### Differences surgical approach

Anterior protocol	Posterior protocol
<p><b>Anterior protocol</b></p> <p>! More prone for hip flexor irritation</p> <ul style="list-style-type: none"> <li>Avoid extension past 20 degrees</li> <li>Avoid external rotation past 50 degrees</li> </ul> <p><b>Anterior approach in daily exercise scheme:</b></p> <p>Less focus on hip flexor exercises in acute phase, e.g.:</p> <ul style="list-style-type: none"> <li>Triple flexion (with assistance) 3x8: d7-d11-d15-d19 ...</li> <li>Hip flexion to extension 3x8: d9-d13-d17-d21 ...</li> </ul> <p>More focus on abductor and glute exercises, e.g.:</p> <ul style="list-style-type: none"> <li>Side toe taps (3x15): d5-d7-d9</li> <li>Abduction (3x15): d11-d13-d15...</li> <li>Mini squat (3x10): d10-d12-d14...</li> </ul>	<p><b>Posterior protocol</b></p> <p>! More prone for abductor weakness and/or irritation</p> <ul style="list-style-type: none"> <li>Avoid hip flexion past 90 degrees</li> <li>Avoid internal rotation past neutral</li> <li>Avoid adduction past neutral</li> </ul> <p><b>Posterior approach in daily exercise scheme:</b></p> <p>More focus on hip flexor exercises, e.g.:</p> <ul style="list-style-type: none"> <li>Triple flexion (without assistance) 3x10: d5-d7-d9-d11 ...</li> <li>Hip flexion to extension 3x10: d6-d8-d10-d12 ...</li> </ul> <p>Less focus on abductor and glute exercises in acute phase, e.g.:</p> <ul style="list-style-type: none"> <li>Side toe taps (3x10): d7-d11-d13</li> <li>Abduction (3x10): d15-d17-d19...</li> <li>Mini squat (3x10): d14-d16-d18...</li> </ul>

32

Preparation	Hospitalisation Protection post surgical Hip	Return to daily activity	Return to active lifestyle
<p><b>Activity</b></p> <ol style="list-style-type: none"> <li>Preop: mg &lt; 4000 Max steps 1000/day</li> <li>Preop: mg 4000 - 9000 + Max steps 1500/day</li> <li>Preop: mg &gt; 9000 + Max steps 2000/day</li> </ol> <p>From day 5: + 100 steps/day</p>	<p>Gradual increase until 50-100% preop steps +100-200 steps / day</p> <p><b>Positive evolution</b> +100-200 steps / day</p> <p><b>Negative evolution</b> a. Max steps ON HOLD b. Decrease in Max steps</p>	<p>Gradual increase until 70%-100% of preop steps +100-200 steps / day</p> <p><b>Positive evolution</b> +100-200 steps / day</p> <p><b>Negative evolution</b> a. Max steps ON HOLD b. Decrease in Max steps</p>	<p>Gradual increase until 70%-100% of preop steps +100-200 steps / day</p>
<p><b>Medication</b></p> <p>PK WHO 1 - 2 NSAID 2/day - 2 weeks Blood thinning medication</p>	<p>PK WHO 1 NSAID 1/day - 2 weeks</p>	<p>PK WHO 1 No NSAID</p>	<p>Stop PK No NSAID</p>
<p><b>Mobility</b></p> <p>Pain free mobility exercises without forcing</p>	<p>Pain free mobility exercises without forcing</p>	<p>Pain free mobility exercises without forcing</p>	<p>Pain free mobility exercises without forcing</p>
<p><b>Crutches</b></p> <p>2 crutches</p>	<p>2 or 1 crutch</p>	<p>No crutches</p>	<p>No crutches</p>
<p><b>Hometrainer</b></p> <p>No hometrainer</p>	<p>Hometrainer with resistance</p>	<p>Hometrainer with resistance</p>	<p>Hometrainer with resistance</p>
<p><b>Biking</b></p> <p>Car</p>	<p>Driving (when feeling safe to do so) *Check the need for surgeon or car insurance approval</p>	<p>Biking after surgeon approval</p>	<p>Biking after surgeon approval</p>

Table hip: Activity - Medication (\* see addendum) - Crutches - ...

33

Preparation	Hospitalisation Protection post surgical hip	Return to daily activity	Return to active lifestyle
<p><b>Activity</b></p> <p>High activity &gt; 50% preop</p> <p>Biking</p>	<p>High activity &gt; 50% preop</p> <p>Biking</p>	<p>High activity &gt; 100% preop</p> <p>Biking</p>	<p>Jumping Running High loading activities or sports -&gt; Surgeon dependent</p>
<p><b>Exercises</b></p> <p>Pain provoking exercises Exercises against heavy resistance Straight leg raises (SLR)</p> <p><b>ANTERIOR don't:</b> Repetitive Hip Flexion</p>	<p>Pain provoking exercises Exercises inducing pain after execution</p> <p><b>POSTERIOR don't:</b> Hip flexion &gt; 90° Endorotation - neutral Adduction &gt; midline</p>	<p>Pain provoking exercises Exercises inducing pain after execution</p> <p><b>POSTERIOR don't:</b> Hip flexion &gt; 90° Endorotation - neutral Adduction &gt; midline</p>	<p>Pain provoking exercises Exercises inducing pain after execution</p>
<p><b>Mobility</b></p> <p>a. Excessive ranges b. POSTERIOR don't: Hip flexion &gt; 90° Endorotation - neutral Adduction &gt; midline</p>	<p>a. Excessive ranges b. POSTERIOR don't: Hip flexion &gt; 90° Endorotation - neutral Adduction &gt; midline</p>	<p>a. Excessive ranges b. POSTERIOR don't: Hip flexion &gt; 90° Endorotation - neutral Adduction &gt; midline</p>	<p>a. Painful excessive ranges</p>
<p><b>Crutches</b></p> <p>No use of crutches</p>	<p>No use of crutches with dysfunctional (limping) gait pattern or for long distances</p>	<p>No use of crutches with dysfunctional (limping) gait pattern</p>	<p>No use of crutches with dysfunctional (limping) gait pattern</p>
<p><b>Pain</b></p> <p>Pain provoking movements or exercises</p>	<p>Pain provoking movements or exercises</p>	<p>Pain provoking movements or exercises</p>	<p>Pain provoking movements or exercises</p>
<p><b>Sleep</b></p> <p>Sleeping on non-op and/or op side (surgeon dependent)</p>	<p>Sleeping on non-op and/or op side (surgeon dependent)</p>	<p>Sleeping on non-op and/or op side (surgeon dependent)</p>	<p>Sleeping on non-op and/or op side (surgeon dependent)</p>

Table hip: Don'ts - Contra-indications

34

### Hospitalisation & Protection post surgical hip

In a first postoperative phase:

- Focus on protection of the post surgical hip and education of the patient on how to do so: the patient should take its prescribed medication, use crutches, put ice and forearm rest and elevation of the operated leg
- Symptom control: Aim to get pain, warmth and swelling under control
- Low load mobility: Low load and light mobility exercises are advised, motivating the patient to move his/her leg regularly without loading and regularly changing position

Attention:

- High Anterior approach: On table don'ts
- High Posterior approach: See table don'ts

**How?**

**Exercises:** See the exercises in the table. Focus is on circulatory and light mobility exercises. Circulatory exercises help drain postoperative blood and swelling and prevent thrombosis, haematoma setting stimulates proprioception and maintenance of muscle tone. These exercises are automatically active in the exercise scheme. Exercises against resistance or that provoke pain, swelling, stiffness and warmth are to be avoided, including during a first to next session. Guidance in exercise is provided by a fully personalized step maximum. (see table for reference).

**The message is used to educate, motivate the patient, address concerns and answer questions.**

**The use of crutches should be supervised.** Crutches are important to prevent falling and lower the load on the hip to soft tissue get time to heal and subsidence is avoided.

**Wound pattern:** Check on heel-to-toe gait pattern and an neutral foot positioning during video analysis.

What if...

- A patient doesn't manage to get his pain under control? 2nd line doctors can be flagged to give advice on medication. Most hip patients are advised to start with 1mg step 1 paracetamol. Step 2 paracetamol can temporarily help high pain perceiving patients.
- A patient reports a 'red flag'? Always flag 2nd line doctors in case of a bleeding wound, picture of wounds, gastric problems, reports of fever, postop nausea, visit to general practitioner, medication intake.

! Regarding the wound care: a picture of the wound after removal of dressings/tapes should be taken. Most wounds are stapled. The staples are removed starting from 10 days to 14 days postoperatively, until staples are removed a patch should cover the wound. In most cases the patch should not be changed until the day (unless specifically stated by surgeon). Some wounds are glued (no patch is needed, infection can show but they not touch the skin).

35

### Common actions Hospitalisation & Protection post surgical hip

Exercises	Messages	Check
<p><b>Default scheme</b></p>	<p><b>Questions</b></p> <ul style="list-style-type: none"> <li>How did surgery go?</li> <li>Message when home</li> <li>Removal staples</li> <li>How are the exercises going?</li> </ul>	<p>Therapy Compliance</p>
<p>High activity =&gt; no more need for circulatory exercises</p>	<p><b>Feedback</b></p> <ul style="list-style-type: none"> <li>Pain is under control</li> <li>You have difficulties at night</li> <li>Warmth, stiffness, swelling...</li> <li>Transfer from 2 &gt; 1 crutch: safe and pain free?</li> </ul> <p><b>Stimulate</b></p> <ul style="list-style-type: none"> <li>To move</li> <li>Achievement of step limit</li> <li>Achievement of use of crutches</li> <li>Long &amp; elevation</li> </ul>	<p>Picture haematoma Swelling Warmth</p> <p>Use of crutches Medication intake Daily activity and step max</p>

Actions Hospitalisation & Protection post surgical hip

36

Protection post surgical hip → **✓ Protection & education**  
**✓ Pain & symptom control** → Return to daily activity

### CRITERIA

- ✓ The patient is compliant to the postoperative rehabilitation guidelines.
- ✓ The patient found a balance between low load, regular, basic exercises and rest to decrease/control post-operative pain and swelling.
- ✓ Transition to the next phase is advised when the patient has control over its postoperative complaints.

Transition protection post surgical hip -> Return to daily activity

37

### Return to daily activity

is a second postoperative phase.

- Focus on exercise tolerance in symptoms and pain while reducing WHO type 2 medication to a minimum level to increasing level by new exercises and slowing higher activity.
- How having a clear overview on the patient's evolution during the first postoperative period, support the patient finding the right balance between rest and activity.
- Attention:
  - Hip anterior approach: See table date's
  - Hip posterior approach: See table date's

How?

**Exercises:** Find back the exercises in the table. Personalize (adjust or delay) them in the exercise scheme based on the daily parameters. With in case, we believe in controlled and positive exercises during the whole rehabilitation. Therefore, using low exercises, a positive exercise should still be suitable and performing the exercises should not provoke pain, induce warmth and/or swelling, tingling and elevation of the operated leg, is still advised to get these parameters fully under control.

**How?**

- Gradual pain is often seen after hip arthroscopy, temporary, avoid exercises which provoke great pain.

**Hip anterior:** Focus on hip abductor, greater anterior region leg.

**Activity:** Guidance in activity is still provided by a daily, personalized step maximum.

- During a positive evolution, the patient is allowed to increase activity by 100-200 steps/day.
- During a negative evolution, the patient should remain on its activity level or have more rest (decrease in activity).

**Avoid peak activity during the whole rehabilitation (see table for reference).**

**The message function:** is still continuously used to evaluate the patient, take concerns and answer questions.

**The use of crutches:** Transition from 2 to 1 and 1 to no crutches should be motivated to pain free and safe.

**Gait pattern:** Check on heel-toe gait pattern and on neutral foot positioning during video analysis. Check on abductor strength or compensations.

Besides daily data collection and visualization, during this phase, patient's evolution is tracked by postoperative PROM collection. Comparison pre to post can be made. Interpret the results and give feedback to the patient. In most cases reassurance of normal progression is sufficient. Of course there is still room for improvement there most patients still perceive evolution until 1 year postop.

38

### Common actions Return to daily activity

Exercises	Messages	Check
<b>Adult exercise scheme:</b> Advance or delay phase 2 exercises  <b>Basic strength:</b> Proprioception Mobilization  Delay exercises which increases pain, stiffness, swelling	<b>Questions</b> <ul style="list-style-type: none"> <li>• How was first postoperative consult?</li> </ul> <b>Feedback</b> <ul style="list-style-type: none"> <li>• Gradual increase in exercises and activity</li> <li>• Painkilling medication lowered to occasional use of WHO type 1 medication before starting next level exercises</li> <li>• Reassurance and supporting when there are ups and downs</li> <li>• PROMs results</li> <li>• Timing 2 -&gt; 3 Occasions</li> <li>• Heading towards excellent result</li> </ul> <b>Stimulate</b> <ul style="list-style-type: none"> <li>• Gradual built up basic strength and ACL without pain</li> <li>• Moderate ROM in ACL</li> </ul>	<ul style="list-style-type: none"> <li>• Reaction on adaptations in the exercise scheme</li> <li>• Wound</li> <li>• Gradual increase in activity/load with decrease of symptoms and pain</li> <li>• PROMs: preop vs week 6</li> <li>• Gait pattern</li> </ul>

Actions Return to daily activity

39

Return to daily activity → **✓ Minimum of pain**  
**✓ Painkilling stopped or limited use** → Return to active lifestyle

### CRITERIA

- ✓ A minimum of / no pain
- ✓ A minimum feeling of stiffness & swelling of the hip
- ✓ Painkilling medication stopped or limited to occasional intake of WHO type 1 medication
- ✓ No more NSAIDs
- ✓ for some /a/ or exercises surgeon approval e.g. driving, biking, lunges,...

Transition Return to daily activity -> Return to active lifestyle

40

### Return to active lifestyle

In a 3rd postoperative phase,

- focus on functional strengthening exercises.

**Exercises:** Gradually add the exercises one by one, while closely following the patient's parameters in the graph or table.

Before starting the functional strengthening exercises, pain, warmth and swelling are stable and at a minimum. Medication is limited or stopped. During and after the functional strengthening, these parameters should remain stable and/or at a minimum. There should be no more need for (extra) medication also.

During this phase, icing can still be useful at the end of a busy day.

**Activity:** Stimulate a gradual increase in activity.

**The messaging function:** During this phase, reassurance of normal progression is often needed. Point out the fact that most patients notice a positive evolution up till 1 year after surgery.

**Gait pattern:** Check on heel-toe gait pattern and on neutral foot positioning during video analysis. Check on abductor strength or compensations.

Finalize rehabilitation by checking if all expectations are met. You can check patient's satisfaction and outcome at the month 3 PROM collection.

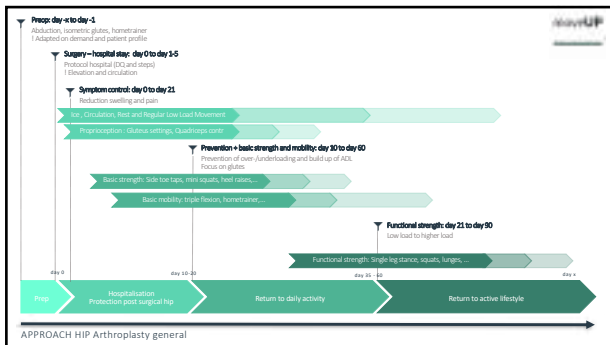
41

### Common actions Return to active lifestyle


Exercises	Messages	Check
Functional strengthening	<b>Questions</b> <ul style="list-style-type: none"> <li>• How was first consult?</li> <li>• How far from preop goals/expectations?</li> <li>• Any movements/activities which are still difficult/painful?</li> </ul> <b>Feedback</b> <ul style="list-style-type: none"> <li>• Right timing 1 -&gt; 2 Occasions</li> <li>• PROMs 3 month</li> </ul> <b>Stimulate</b> <ul style="list-style-type: none"> <li>• Back to work</li> <li>• Full ACL uptake</li> </ul>	<ul style="list-style-type: none"> <li>• Video gait</li> <li>• Patient satisfaction</li> <li>• Functional ROM</li> <li>• Normal gait pattern</li> </ul>

Actions Return to active lifestyle


42


43

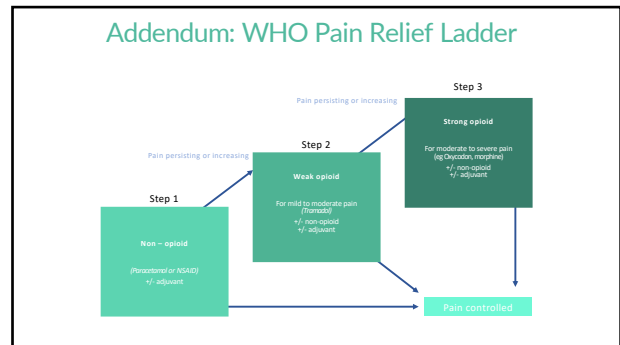
44

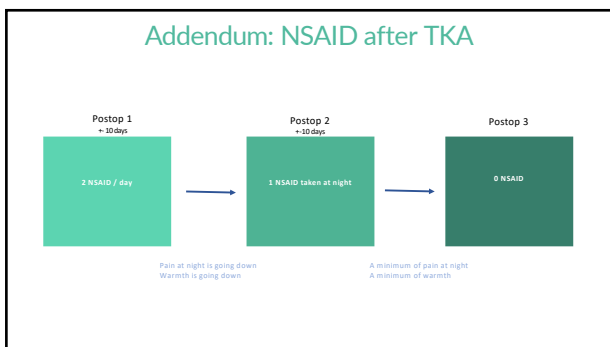45

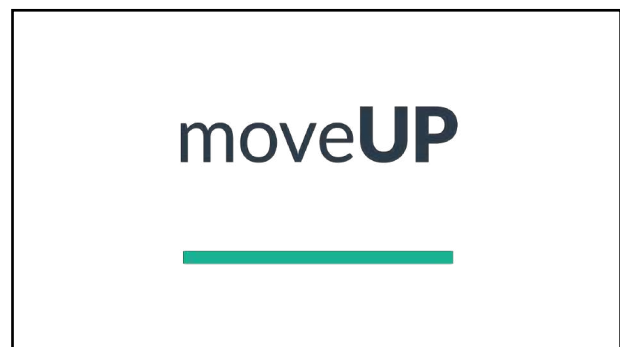
46

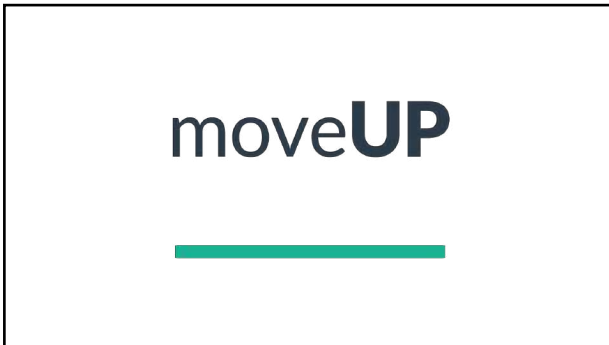1

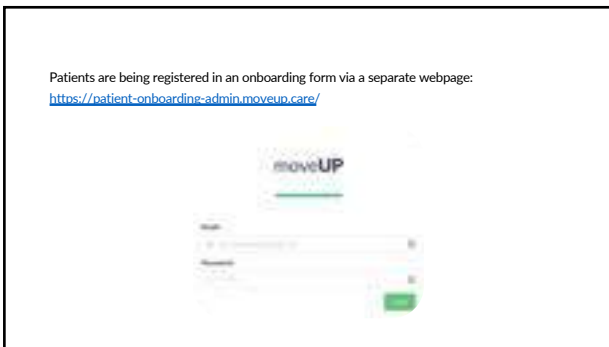2


3

4

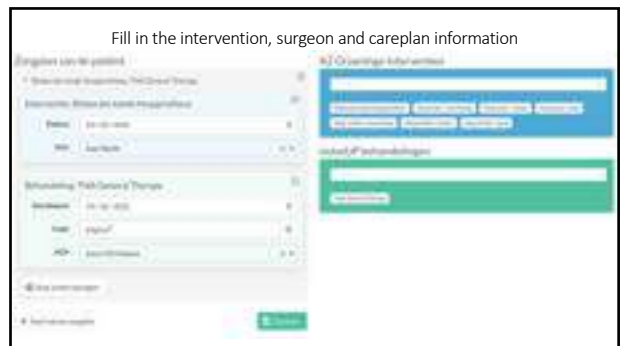5

6

#### What happens next?

Within one working day the patient will receive an email from moveUP containing:

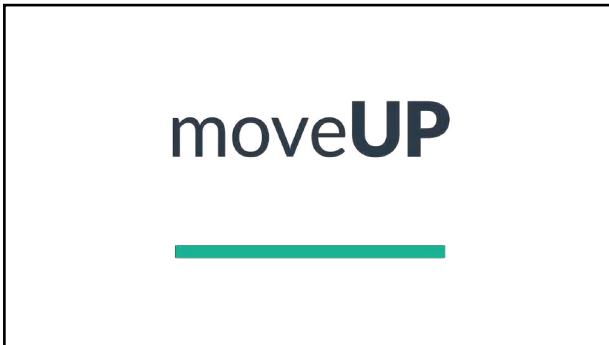
1. **Link to set a password**
  2. **Instructions how to install moveUP** - With a short introduction of moveUP, information link on [pricing](#) and [instructions on how to install moveUP](#) with login details, and a link to the [quick start guide](#)
  3. **Link to order activity tracker** – An online order form for the activity tracker
- > [The patient can access the moveUP app and will appear in the healthcare platform](#)

7


# moveUP


8

1

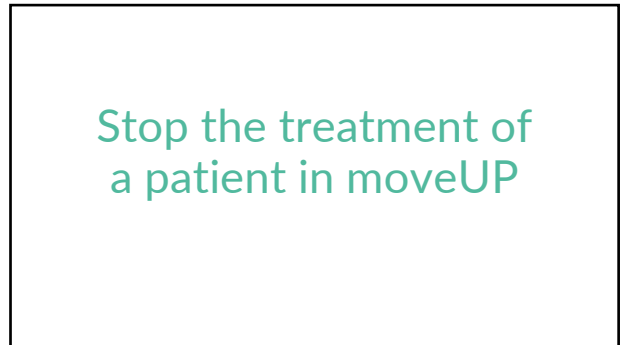
2

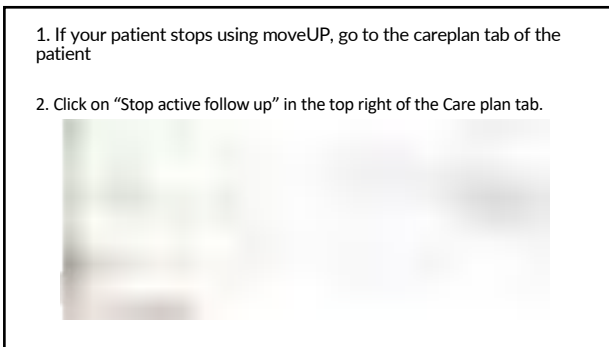3

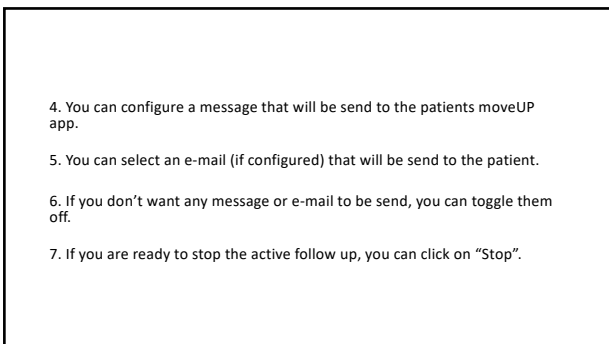4

5

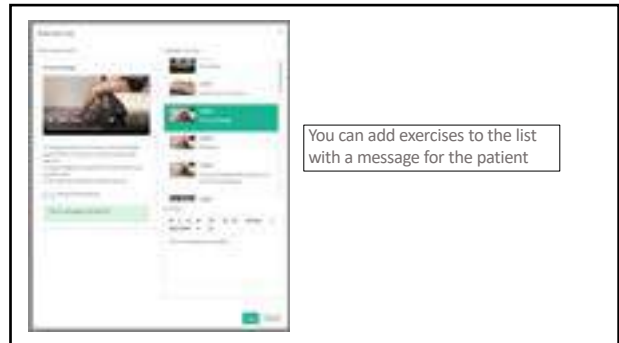
6


7


8

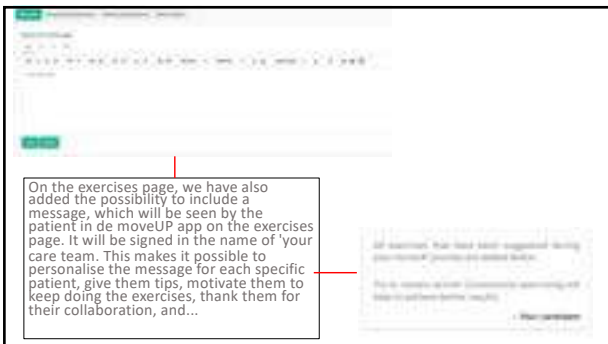9

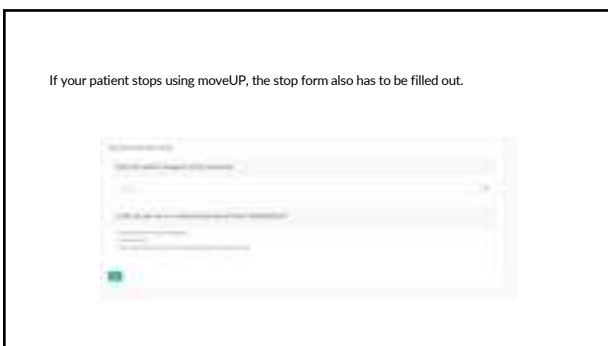10


11

12

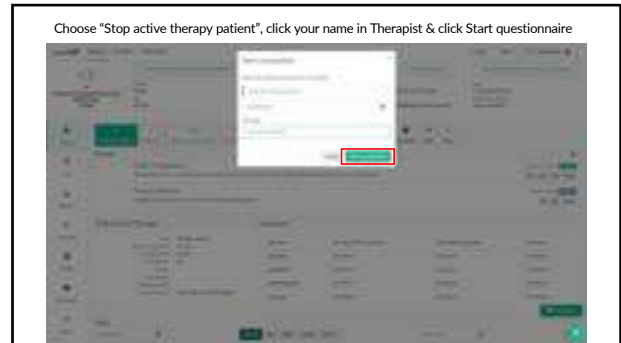13

14

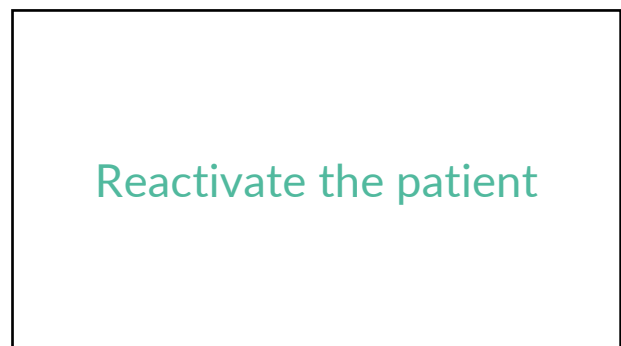
15

16

17

18

If you want to reactivate the patient, go to careplan and click 'back to active follow up'

19

20

move**UP**

21