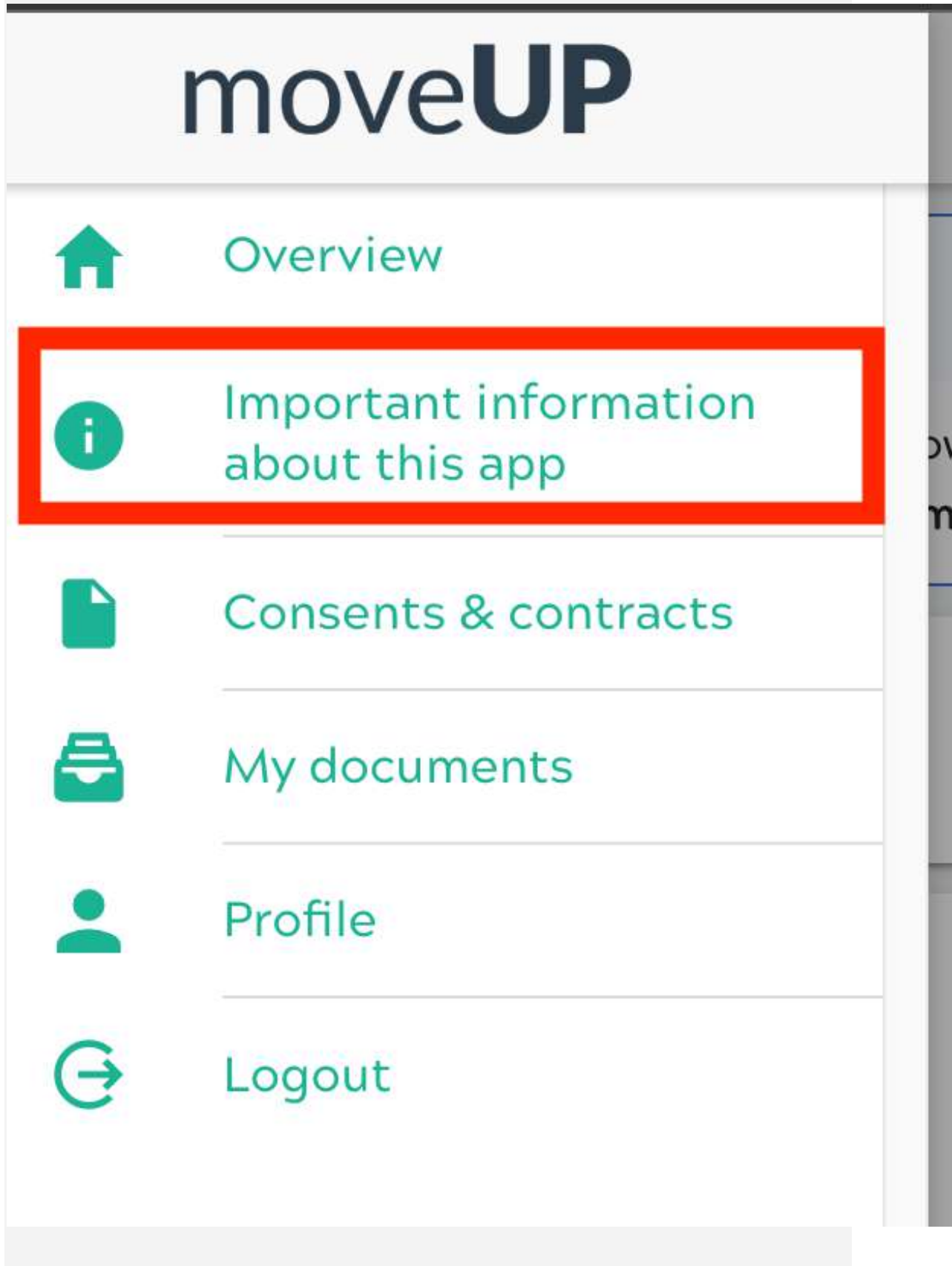
# moveUP

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## moveUP app & devices

The moveUP app runs on:

- iPhones/iPads (iOS 11.0 or newer)
- Android systems (Android 7.0 and newer)
- moveUP needs an internet connection when in use.
- moveUP is compatible with the following browsers:
- Chrome/Safari/Firefox

### Tablets

In specific cases the following tablets are provided and are supported:

- Lenovo Tab 3 - Android 6.0.1 or Higher
- Huawei MediaPad T3 - Android 7.0.0 or higher

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### Download size

- iOS - 82.2MB - version 1.35
- Android - 89MB - version 1.35

For your data safety please enable the 2 factor authentication (2FA) and use a safe network.

Download the latest app version

Choose region

Europe

US

Important information - Europe

# moveUP

Download the user manual for patients

Download the user manual for healthcare providers

If you need the eFU from an older version please contact support  
If you need the eFU as MG patient, [download here](#)

## Intended use

### For healthcare providers

For healthcare providers, moveUP provides efficient clinical management of pathologies and treatments. With the use of moveUP, valuable insights are given to the HCP care team of the status of the patient and its evolution.

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- **moveUP COMPANION** = monitoring and information, no active follow-up  
- moveUP companion offers targeted information and evolution reports to patients. Their care team takes the evolution reports into account in their further recovery when appropriate.
- **moveUP COACH** = active follow-up by healthcare team  
- moveUP coach offers targeted information and evolution reports to patients. Their care team is more actively involved and takes the evolution reports into account and can provide advice and exercise suggestions through the digital platform.
- **moveUP THERAPY** = active follow-up by healthcare team, with data-driven validated care protocol  
- moveUP therapy offers a data-driven validated care protocol with certain category and level of exercises and activities, specifically targeted to the individual patient. Their care team can manually adapt the data-driven validated protocol when needed. Patients can fully rehabilitate with moveUP without leaving their home environment.

Optional functionality that can be enabled: Interoperability with Class IIa continuous passive motion (CPM) medical device. For knee and hip patients who are using a Class IIa CPM medical device, moveUP can interoperate with the Class IIa CPM software. moveUP acts as a facilitator to easily assign a designated CPM exercise protocol chosen by a physician to a patient and to display the performed CPM exercises in the medical dashboard.

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- **moveUP COMPANION** = monitoring and information, no active follow-up  
- moveUP companion offers targeted information and evolution reports to patients. Their care team takes the evolution reports into account in their further recovery when appropriate.
- **moveUP COACH** = active follow-up by healthcare team  
- moveUP coach offers targeted information and evolution reports to patients. Their care team is more actively involved and takes the evolution reports into account and can provide advice and exercise suggestions through the digital platform.
- **moveUP THERAPY** = active follow-up by healthcare team, with data driven validated care protocol  
- moveUP therapy offers a data-driven validated care protocol with certain category and level of exercises and activities, specifically targeted to the individual patient. Their care team can manually adapt the data driven validated protocol when needed. Patients can fully rehabilitation with moveUP without leaving their home environment.

Optional functionality that can be enabled: interoperability with Class IIa continuous passive motion (CPM) medical device. For knee and hip patients who are using a Class IIa CPM medical device, moveUP can interoperate with the Class IIa CPM software, moveUP acts as a facilitator to easily assign a designated CPM exercise protocol chosen by a physician to a patient and to display the performed CPM exercises in the medical dashboard.

## Claims and advertisings

- moveUP is not an emergency tool. In case of emergency please contact your doctor or call 911.
- moveUP is not intended to monitor/treat vital parameters of critical illness.

- moveUP companion is not a replacement for any treatment you need to follow, but is used as an addition.
- Patients need to be aware that if questionnaires are not completed on a frequent basis or not answered truthfully, the care team has more difficulty understanding your health situation & progress.

- The moveUP app offers personalized healthcare based on the patient's profile and therefore can not be shared, borrowed or traded between users. The information and advice of the moveUP application do not apply to any other patient other than the one whose profile is stored in the application. The patient cannot grant access to the moveUP app to third parties. Other people could accidentally send wrong information or change the profile of the patient, with the result that the moveUP-guided advice is no longer applicable to the needs of the patient.

moveUP NV/SA  
Cantersbeen 47,  
1000 Brussels,  
Belgium

### UK legal representative:

Atrivos Ltd  
Suite 215, Wey House, 15 Church Street, Weybridge, Surrey KT13  
6NA, United Kingdom

UDI-DE-1424MOVEUPVLS0001

moveUP

The moveUP App adheres to European health, safety and environmental protection standards.

## Important information - US

[Download the user manual for patients](#)

[Download the user manual for healthcare providers](#)

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- **moveUP COMPANION** = monitoring and information, no active follow-up  
- moveUP companion offers targeted information and evolution reports to patients. Their care team takes the evolution reports into account in their further recovery when appropriate.
- **moveUP COACH** = active follow-up by healthcare team  
- moveUP coach offers targeted information and evolution reports to patients. Their care team is more actively involved and takes the evolution reports into account and can provide advice and exercise suggestions through the digital platform.
- **moveUP THERAPY** = active follow-up by healthcare team, with data-driven validated care protocol  
- moveUP therapy offers a data-driven validated care protocol with certain category and level of exercises and activities, specifically targeted to the individual patient. Their care team can manually adapt the data-driven validated protocol when needed. Patients can fully rehabilitation with moveUP without leaving their home environment.

# moveUP

## Inclusion criteria:

- Age:
  - ↳ minimum: 18 years
  - ↳ maximum: no limit
- Weight: NA
- Health & condition: capable of performing basic activities of daily living
- Language: understanding one of the available languages of the app (Dutch, French, German, English)

## Exclusion criteria:

- Patients who are mentally incompetent or having troubles to express what they are feeling (for instance, mentally diseased people, people staying in elderly care centers, ...) are excluded.
- Patients who are not capable of operating a tablet/smartphone and activity tracker.
- Patients who can't understand one of the available languages of the app (Dutch, French, German, English)

## Claims and advertisings

- moveUP is not an emergency tool. In case of emergency, please contact your doctor or call 911.
- moveUP is not intended to monitor/treat vital parameters of critical illness.

- moveUP companion is not a replacement for any treatment you need to follow, but is used as an addition.
- Patients need to be aware that if questionnaires are not completed on a frequent basis or not answered truthfully, the care team has more difficulty understanding your health situation & progress.

- The moveUP app offers personalized healthcare based on the patient's profile and therefore can not be shared, borrowed or traded between users. The information and advice of the moveUP application do not apply to any other patient other than the one whose profile is stored in the application. The patient cannot grant access to the moveUP app to third parties. Other people could accidentally send wrong information or change the profile of the patient, with the result that the moveUP-guided advice is no longer applicable to the needs of the patient.

## Tradename

moveUP  
Made in Belgium  
The product is software

## US Agent

Claran McCourt  
Incorbowin Inc 7421 W Cypresshead Dr Parkland  
FL 33067 US agent 954.602.5053

## Manufacturer

moveUP NV/SA  
Cartersteen 47,  
1000 Brussels,  
Belgium

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The screenshot displays the 'Täglicher Fragebogen' (Daily Questionnaire) app interface. It consists of three sections, each with a question and a slider or radio button for the answer. The first section asks about pain at night, with a slider set to 'Keine Schmerzen' (No pain). The second section asks about daily pain, with a slider set to 'Viele Schmerzen' (A lot of pain). The third section asks about pain elsewhere, with a radio button selected for 'Ja' (Yes). A red arrow points to the 'Täglicher Fragebogen' icon in the bottom navigation bar.

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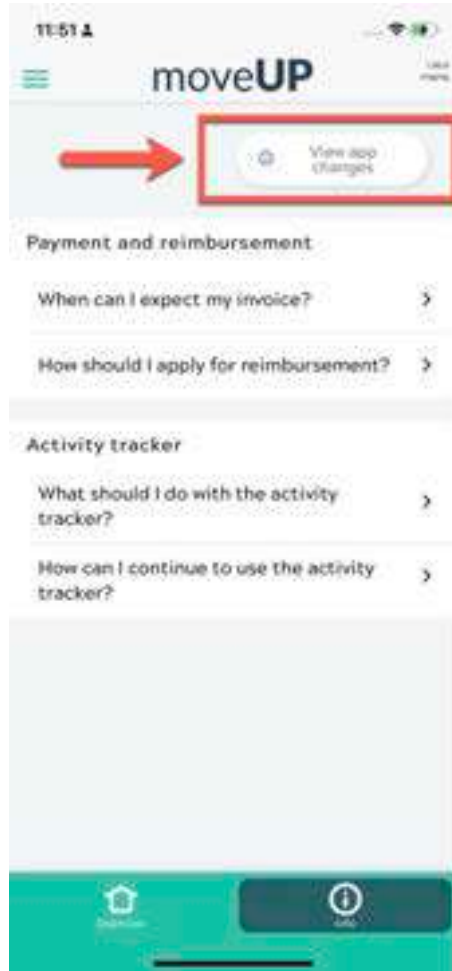
---

## Manual for patients

moveUP – A smart and personalized way of informing, follow-up and treatment for patients.

*App version: 1.31 and later*

*Manual version: 16*

*Date of release manual: 11/07/2022*

*Link to Knowledge Base: <https://support.orthopedics.moveup.care/>*

# Table of content

- Intended use moveUP
- Overview of the main moveUP windows
- Messages
- Survey & info modules
- Steps
- Exercises
- Daily questionnaire
- Activity tracker
- My active follow-up period with moveUP has ended: how does the moveUP app change?

# Support orthopedics – moveUP

## Intended use moveUP

-  Carefully read all information and precautions in this manual. Make sure you understand the use, the screens and the limitations of the moveUP App before using the service. If you have any questions please contact moveUP.

For security reasons, moveUP recommends that the user carefully reads and understands the manual of the devices used in conjunction with the moveUP App.

During the full use of the service, the necessary technical assistance and therapeutical assistance will be provided. In case of a defect or problem with the App, you will be able to contact the moveUP team via the message functionality of the App, by phone or by e-mail.

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### For patients

For patients, moveUP provides personalized information and instructions to help them managing their symptoms and progress in their rehabilitation. The intensity of follow-up is adapted based on patients' needs and timepoint in the patient pathway:

- **moveUP COMPANION** = monitoring and information, no active follow-up
  - moveUP companion offers targeted information and evolution reports to patients. Their care team takes the evolution reports into account in their further recovery when appropriate.
- **moveUP COACH** = active follow-up by the healthcare team
  - moveUP coach offers targeted information and evolution reports to patients. Their care team is more actively involved and takes the evolution reports into account and can provide advice and exercise suggestions through the digital platform.
- **moveUP THERAPY** = active follow-up by the healthcare team, with data-driven validated care protocol
  - moveUP therapy offers a data-driven validated care protocol with certain categories and levels of exercises and activities, specifically targeted to the individual patient. Their care team can manually adapt the data-driven validated protocol when needed. Patients can fully rehabilitate with moveUP without leaving their home environment.

*Optional functionality that can be enabled: interoperability with Class IIa continues passive motion (CPM) medical device.*

*For knee and hip patients who are using a Class IIa CPM medical device, moveUP can interoperate with the Class IIa CPM software. moveUP acts as a facilitator to easily assign a designated CPM exercise protocol chosen by a physician to a patient and to display the performed CPM exercises in the medical dashboard.*

## Intended users

moveUP is intended to be used by patients and healthcare providers.

## Patient

The main user of the mobile app and patient website is the patient. Patient users should be alert, mentally competent and capable of operating a tablet/smartphone and activity tracker.

### Inclusion:

- Age:
  - minimum: 18 years
  - maximum: no limit
- Weight: NA
- Health & condition: capable of performing basic activities of daily living
- Language: understanding one of the available languages of the app (Dutch, French, German, English)

### Exclusion:

- Patients who are mentally incompetent or having trouble expressing what they are feeling (for instance, mentally diseased people, people staying in elderly care centres, ...) are excluded.
- Patients who are not capable of operating a tablet/smartphone and activity tracker.
- Patients who can't understand one of the available languages of the app (Dutch, French, German, English)

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## Target population / indication for use

moveUP companion & coach is used by Musculoskeletal, oncologic, respiratory, gastrointestinal, cardiovascular and neurologic patients, such as patients:

- who underwent or planning a gastric bypass or gastric sleeve operation
- who have or had a stroke, multiple myeloma, covid, familial hypercholesterolemia
- who have or had back or joint problems or operations

moveUP therapy is used by knee & hip arthroplasty patients.

## moveUP app & devices

### Supported devices moveUP app

The moveUP app runs on:

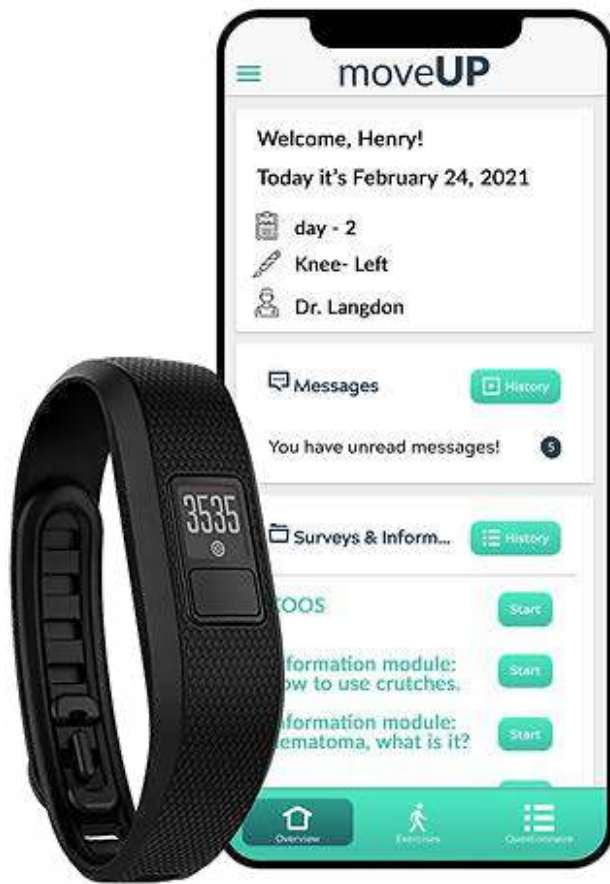
- iPhones/iPads (iOS 11.0 or newer)
- Android systems (Android 7.0.0 and newer).
- moveUP needs an internet connection when in use
- moveUP is compatible with the following browsers:
  - Chrome
  - Safari
  - Edge
  - Firefox
- In specific cases the following tablets are provided and are supported:
  - Lenovo Tab 3 – Android 6.0.1 or Higher
  - Huawei MediaPad T3 – Android 7.0.0 or higher

For your data safety please enable the 2-factor authentication(2FA) and use a safe network

### Activity trackers

The following activity trackers are currently supported:

- Garmin Vivofit 3 and Vivofit 4
- Nokia Withings Go

## Warnings and precautions

**!** moveUP is not an emergency tool. In case of emergency please contact your doctor or call 112.  
moveUP is not intended to monitor/treat vital parameters of critical diseases.

**!** moveUP companion is not a replacement for any treatment you need to follow but is used as an addition.  
Patients need to be aware if questionnaires are not filled out on a frequent basis or not filled out trustworthy, the care team has more difficulty knowing their health situation & evolution.

**!** The moveUP App offers personalized healthcare based on the patient's profile and therefore can not be shared, borrowed or traded between users. The information and advice of the moveUP application do not apply to a patient other than the one whose profile is stored in the application. The patient may not grant access to the moveUP App to third parties. Any other person can, even accidentally, send wrong information or change the profile of the patient, with the result that the moveUP-guided advice no longer corresponds to the needs of the patient.

**!** With software, there is always a residual risk for bugs. If you have the feeling something is wrong or you don't get a response in the app, please reach out to us via [support@moveup.care](mailto:support@moveup.care) or call 0800 88 008.

## **Contra-indications and potential side effects**

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## Notification of serious incidents

**A serious incident** means any incident that directly or indirectly led, might have led or might lead to any of the following:

- (a) the death of a patient, user or another person,
- (b) the temporary or permanent serious deterioration of a patient's, user's or other person's state of health that resulted in any of the following:
  - life-threatening illness or injury,
  - permanent impairment of a body structure or a body function,
  - hospitalisation or prolongation of patient hospitalisation,
  - medical or surgical intervention to prevent life-threatening illness or injury or permanent impairment to a body structure or a body function,
  - chronic disease,
- (c) a serious public health threat;

If you detect any of these cases please send an email to [info@moveUP.care](mailto:info@moveUP.care) and;

- Netherlands: [meldpunt@igj.nl](mailto:meldpunt@igj.nl)
- Belgium: [meddev@fagg.be](mailto:meddev@fagg.be)
- France: [dedim.ugsv@ansm.sante.fr](mailto:dedim.ugsv@ansm.sante.fr)
- Germany: [zlg@zlg.nrw.de](mailto:zlg@zlg.nrw.de)

## **Human body contact**

No human body contact with the patient or user, due to the nature of the product (software).

## **Accessories/products used in combination**

There are no accessories.

If the patient has no compatible tablet/smartphone or activity tracker, moveUP can lease these devices to the patient. The leased devices are CE-marked devices that meet the compatibility criteria outlined in the IFU for the app.

## Device lifetime

2-years. Depending on the willingness of the user to update the app. We sent out a notification with the recommendation to update the app, to make sure the app will keep the performance & recently included features.

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- **Enhancement**— moveUP enhances the clinical management of the patient because early detection & management of complications are possible via the symptom & QoL monitoring tool
- **Efficiency**—More efficient clinical management, such as the number of consultations can be reduced
- **Compliance**—Enforces therapy compliance/adherence
- **Information**—The correct information is provided at the right time

### moveUP therapy

- **Rehabilitation from home**—With the use of moveUP therapy knee & hip arthroplasty patients can fully rehabilitate via the in-app care team without leaving their home environment.

## Company details

moveUP NV

Head office Brussel: Cantersteen 47, B-1000 Brussels, Belgium Office Ghent: Oktrooiplein 1, B-9000 Ghent, Belgium

Tel: 0800 88 008

Email: [info@moveUP.care](mailto:info@moveUP.care)

[www.moveUP.care](http://www.moveUP.care)

Identification of the device:

Basic UDI-DI code:

++G166MOVEUPV1310WH

If you have any questions about the App, the tablet or the activity tracker, you can contact moveUP via the following channels – in order of priority: Via the message system of the App. Via email to [support@moveup.care](mailto:support@moveup.care) – Via phone during office hours: +32 800 88 008

Updated on July 11, 2022

# Overview of the main moveUP windows

## Support orthopedics – moveUP

**I read this email on a smart device**

## **Step 1: Download the app on your smart device**

Click on the below links to download the moveUP app from your Appstore or Google Play (depending on your device)

I have an iPhone or tablet from Apple (iOS)

I have another smartphone or tablet (android)

## Step 2: Open the moveUP app and register

- Open the moveUP app (if the app shows the moveUP logo, It is correctly installed)
- Register with the email address and the code you received in the “installation email” from moveUP
- Check your personal data and the operation performed before the first use. In case of any error please contact moveUP using the chat function.

### **Step 3: Update the moveUP app**

Each time the moveUP app is updated you'll receive a mail with the release notes. Please, update it as soon as possible to enjoy the last features.

To upload the app please use the link above to be redirected to your app store and click on the button "update".

These updates can include safety or data integrity upgrades, please read carefully the Release Notes and the Instructions for Use.

## Security measures

Because we process personal and health data, we will instruct you how to set a complex password at the moveUP login page.

Please use a safe network when you are connected to the app.

Updated on May 4, 2022

# Support orthopedics – moveUP

## Quick start guide

There are 3 main tabs in the app. You can switch between them by tapping the respective symbols at the bottom of your screen.

1. Overview
2. Exercises
3. Questionnaire

## Tab 1: Overview

When you open the moveUP app you will automatically be taken to the overview tap. This tab has 4 sections:

1. **Welcome message**
2. **Messages** – here you can communicate with your health provider
3. **Survey & Information** – here you can find your questionnaires that still need answering. Also, your video-recording requests will appear here.
4. **Steps** – here you can see the steps you did over the past week

## Tab 2: Exercises

In the second tab “Exercises” you can find your daily exercises.

## Tab 3: Questionnaire

The screenshot shows a mobile application interface titled "Questionnaire". At the top, the time is 17:26. Below the title, it says "Your questionnaire for today". The first section asks "How are you feeling today?" and "Indicate on the scale." with "Good" on the left and "Bad" on the right. Below this is a row of six emotion icons: a blue happy face, a green neutral face, a yellow sad face, a yellow angry face, a red angry face, and a red angry face with a red outline. The second section asks "How was your night?" and "Indicate on the scale." with "Good" on the left and "Bad" on the right, followed by the same row of six emotion icons. The third section asks "Did you experience any morning stiffness in your affected joint?" and "Indicate on the slider." with "None" on the left and "Very stiff" on the right. A red arrow points from the "Very stiff" label down to a red box around the "Questionnaire" icon in the bottom navigation bar. The bottom navigation bar has three icons: a house for "Home", a person for "My Profile", and a questionnaire icon for "Questionnaire".

In the 3rd tab "Questionnaire" there will appear a daily questionnaire after 4pm each day. The questionnaire is available until midnight. It should be filled out every day, starting 14 days before surgery.

Read more on the daily questionnaire

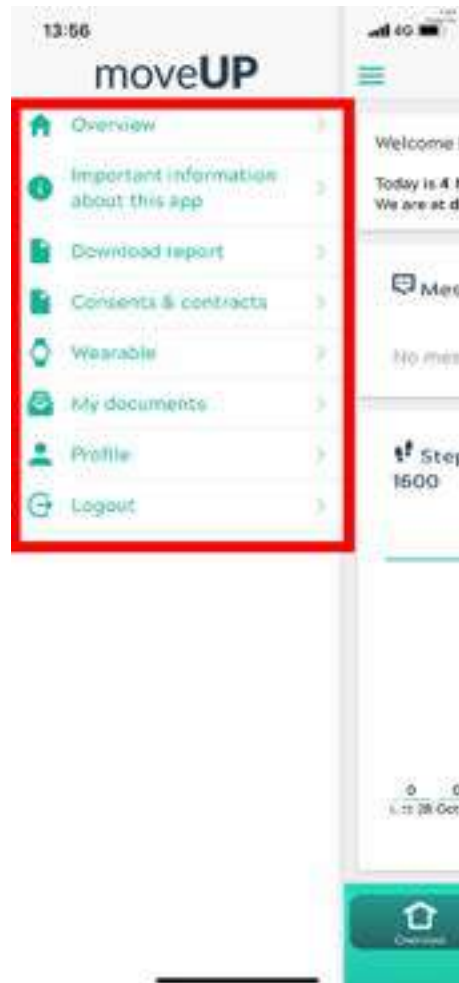
It is important you start answering the daily questionnaire starting 14 days before your operation. This way we can better help you after the operation.

## Menu

At the top left of the moveUP you will find the menu.

When you click on this icon the menu appears.

This menu contains important sections that are not directly related to your rehabilitation.

- **Overview:** Exits the menu and returns to the overview screen
- **Important information about this app:** some info and warnings on the app
- **Download report:** This link is not working at the moment. Work in progress!
- **Consents & contracts:** Here you can read the general terms and conditions and privacy statement.
- **Wearable:** Here you can link your smart bracelet to the moveUP App or see which application is linked.
- **Profile:** your name and national registration number
- **Log out:** Here you can log out of your account and return to the registration and login page.

# Messages

# Support orthopedics – moveUP

## Messages

In the overview tab, below the greeting, you find the “Messages” section. This is the section where you can communicate with your physiotherapist.

You can send your physiotherapist a message each moment of the day. You will receive an answer within 24 hours.

If the question is a medical urgency, don't use the messaging system as it is too slow. Call your hospital or medical practitioner instead.

## Open the message functionality

Click on the history button to open the message functionality.

## Sending messages


You can send us messages every moment of the day. We will respond within 24 hours

To send a message click in the “type a message” box. The keyboard will appear. After typing your message click on the send symbol (airplane).

## Sending a picture


You can send us a picture (of the wound for example) by clicking on the picture symbol in the left lower corner.
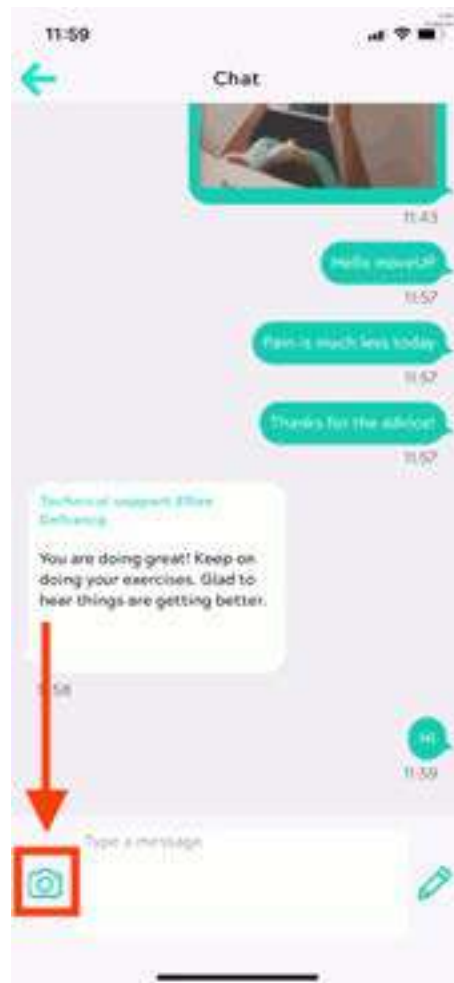
## Go back to the overview screen

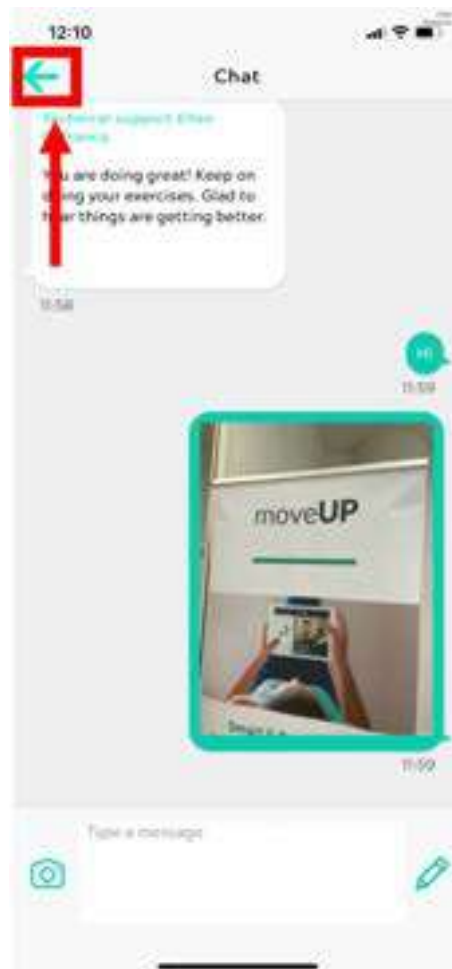By clicking on the arrow in the left upper corner you can go back to the overview screen.

# **Survey & info modules**

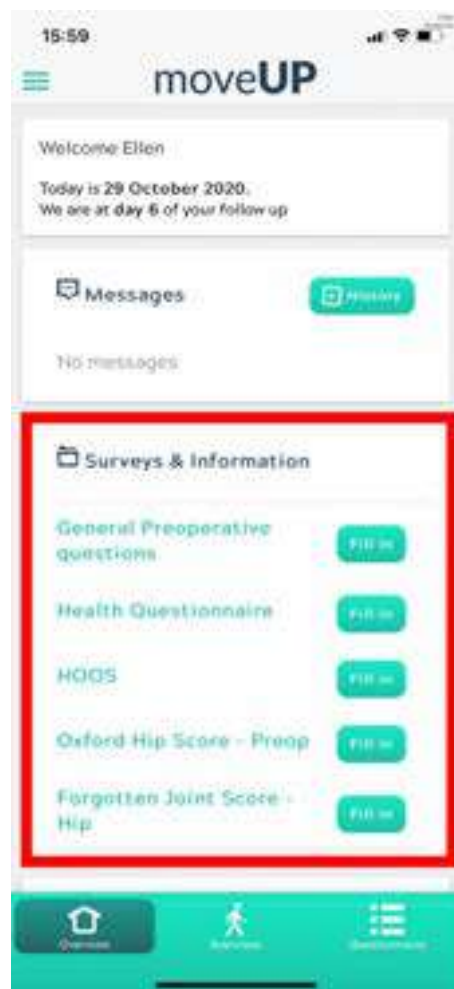
# Support orthopedics – moveUP

## Surveys & Information

Medical questionnaires are available at various moments before and after the surgery.


These questionnaires are of critical importance to evaluate the outcome of your surgery and to guide your physiotherapist in his decision making.

From the moment a questionnaire becomes available, we ask you to start filling it out as soon as possible.

## Submitting a survey

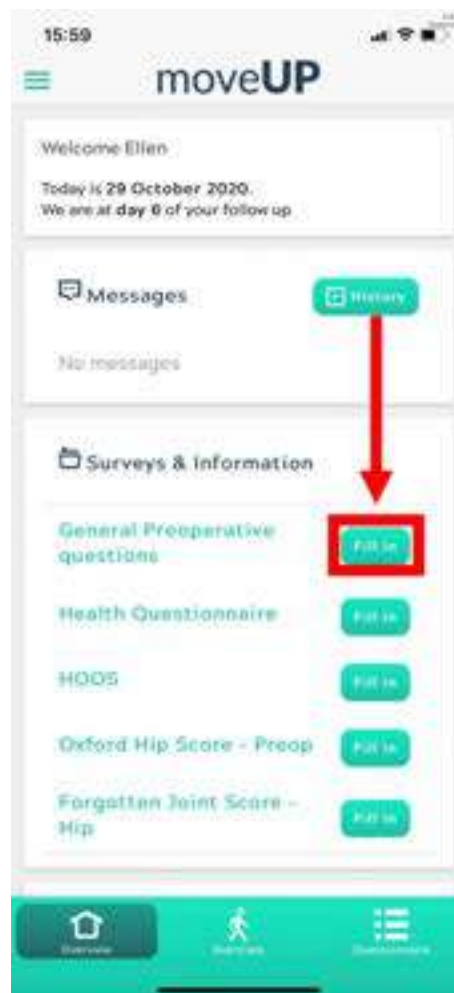To fill out a survey you click the “Fill in” button.

After filling out the question click on the “send” button.

16:09

General Preoperative questions

Never

Sometimes, during walks

Always

Do you experience problems with your balance?

Select from the list:

Never

Sometimes

Often

Always

Weight

Your weight (kg)

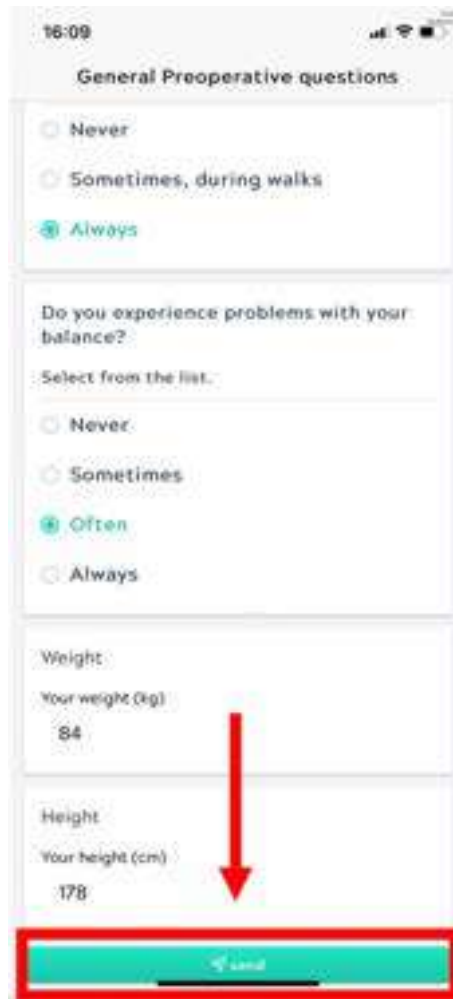
84

Height

Your height (cm)

178

send

You will see the following screen if the answers are sent to us correctly.

17:55

Your answers are sent to the  
medical team!

[Go back to the overview](#) >

# Support orthopedics – moveUP

## How to reread an info module?

To reread an info module that you already read:

- On the overview tab (1) go to the “Surveys & Information” section (2).

- Tap the “history” button.

- Here you will find the complete list of all the info modules you have read so far.
- Select the one you want to read again.

14:10

## Articles

1/20

READ: Anatomy, osteoarthritis and total hip replacement surgery

READ: Driving after surgery

READ: Wound care: from incision to scar

READ: Safe use of crutches

READ: Sleep positions after joint replacement

READ: Haematoma

READ: Pain

READ: Improve your sleep after surgery

# Steps

# Support orthopedics – moveUP

## Steps area explained

Your steps are automatically counted by your activity tracker, the activity tracker does not need to be activated, just be worn around the wrist.

To see your steps you have to have paired your activity tracker with the moveUP app. This pairing needs to be done only once. After you have to sync your activity tracker daily with the moveUP app.

## Steps graph

- Activity data from the past week is visualized in a graph.
- The different colors only indicate the different days and have no further meaning
- Activity is represented in number of steps.
- You can use the arrows left and right to see previous or next days in the graph.

Every activity with arm movement will contribute to the count. This means if you walk by pushing a stroller or a walker the steps may be undercounted. If you are ironing (arm movement) they may be overcounted.

## Personalised step max

Every day, a personalized step maximum is displayed by a number and a red line. Try not to exceed the step maximum with > 20%.

The step maximum is not a goal! It is a limit to avoid overloading the operated joint.

# Exercises

# Support orthopedics – moveUP

## Exercises

The second tab contains your exercise program for the day. Day by day you will only see prescribed exercises specific for that day.

If you feel strong pain while doing a specific exercise, stop that exercise and mention its number in the messaging system.

## Number and name of the exercise

On top of each exercise you can find it's number and the name of the exercise.

## Video

There is also a short video per exercise that indicates how the exercise should be performed.

To start the video, click on the “Play” icon (triangle) on the top of the video.

## Frequency and intensity

You will find the frequency and intensity per exercise (eg: 3x a day, 10 repetitions).

You report the number of sessions via the + and – symbol. It is important to enter this accurately so that your moveUP physiotherapist is kept informed of your progress.

We recommend that you do **not** exceed the number of prescribed sessions and repetitions.

# **Daily questionnaire**

# Support orthopedics – moveUP

## Daily questionnaire

The daily questionnaire is our compass to guide you through your revalidation. It is very important you fill it out daily after 4 pm.

By filling out the daily questionnaire you give your doctor and healthcare team more insight into the progress of your treatment: your symptoms, your complaints, your evolution. It consists of questions about your general well-being, complaints, activities, sleep quality etc... These questions are adapted to the type of treatment you are having. These daily questionnaires take approximately 45-60 seconds to complete. Your doctor and healthcare team will also see how you experience the impact of your condition on your quality of life.

The combination of the information retrieved from questionnaires, photo's, video's and your activity profile via the smart bracelet allows your doctor and care team to follow you up and adapt your treatment.

For optimal results, you need to use the App and follow the given instructions within the App on a daily basis.

It is important we have a good baseline of your situation before the operation. That is why we ask you to start filling it out every day, ***starting 14 days before your operation.***

The questionnaire can be found in the 3rd tab of your app every day after 4 pm.

The screenshot shows the 'Questionnaire' app interface. At the top, the time is 17:26 and the title is 'Questionnaire'. Below the title, it says 'Your questionnaire for today'. The first question is 'How are you feeling today?' with the instruction 'Indicate on the scale.' and a scale from 'Good' to 'Bad' with six corresponding emoji icons (happy, neutral, sad, angry, etc.). The second question is 'How was your night?' with the instruction 'Indicate on the scale.' and a scale from 'Good' to 'Bad' with six corresponding emoji icons. A red arrow points to the 'Bad' emoji in this section. The third question is 'Did you experience any morning stiffness in your affected joint?' with the instruction 'Indicate on the slider.' and a scale from 'None' to 'Very stiff'.

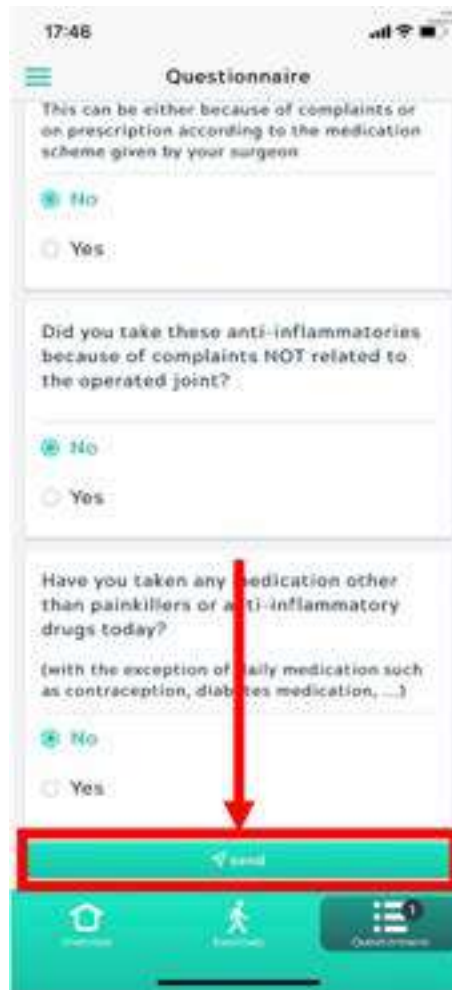
To indicate your pain level you can tap on the line above the faces and/or slide to the appropriate position.

---

---

When finished click on the “send” button to share your answers with us.

The following screen appears if answers are sent to us correctly.

17:55

Your answers are sent to the  
medical team!

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# Support orthopedics – moveUP

## How to synchronise my steps daily?

We advise you to wear your activity tracker around your wrist day and night. But the day is the most important for us to register your activity. The activity tracker is always active and captures how active you are, hence it does not need to be activated.

You do need to manually sync your activity tracker data **daily** in the moveUP app. This way your physiotherapist can monitor how active you are and decide if any adaptations to your exercise scheme are needed.

**We advise syncing your steps daily in the evening after filling out the daily questionnaire.** This way your physiotherapist will see the next day how active you were.

The activity tracker has to be paired with the moveUP app. This pairing only has to happen once at the beginning of your rehabilitation. You can find the pairing instructions [here](#).

The Bluetooth should be activated on your device to be able to sync your activity tracker.

- Press the synchronising button marked in red.

- You will see the following screen. Click on “How to synchronise”

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- You will see the synced steps data on your home screen.

If you have problems synchronising please send an email to [support@moveup.care](mailto:support@moveup.care) and try to describe exactly in which step you encounter problems and what messages you see or get on your screen.

**My active follow-up period with moveUP has ended: how does the moveUP app change?**

## Support orthopedics – moveUP

### **My active follow-up period with moveUP has ended: how does the moveUP app change?**

When the active follow-up period has ended your moveUP app will be different. You are no longer actively followed up by a physiotherapist. What changes in the app?

The app will consist of 2 tabs now: “Overview” and “Info” (instead of “Overview”, “Exercises” and “Questions”).

## Overview tab

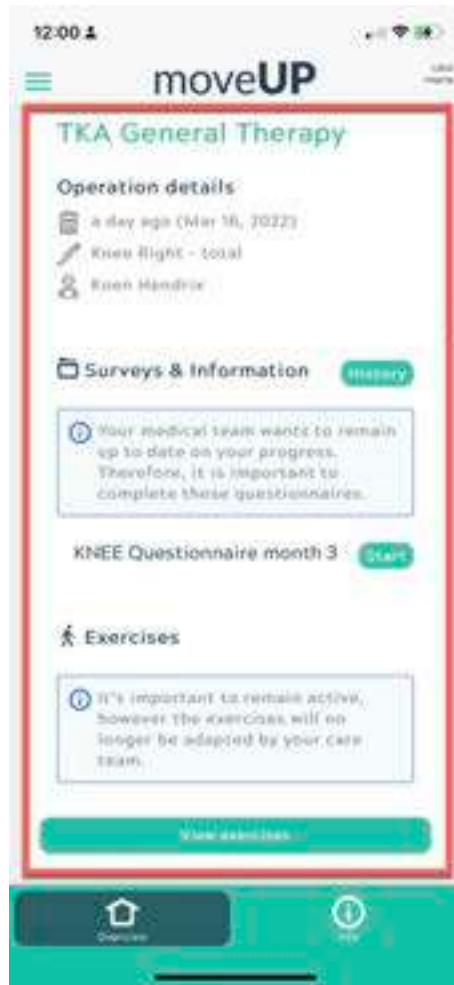
### Messages

The message function will still be available for seven days. After one week the message function will be disabled. Your message history will remain available in the app.

### moveUP journey card

The next block is your “moveUP journey card”. It contains your operation details, surveys that still need to be filled out and an option to review the exercises you have done with the app during the course of your rehabilitation


If you had more than one journey with moveUP, you can scroll down the page and find them all in separate cards.

## Info tab

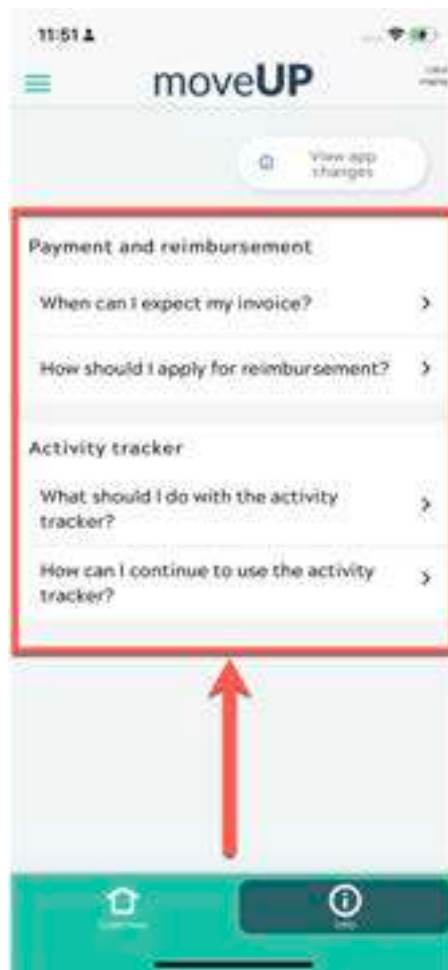
## FAQ

If you click on the info tab, you will find a page that is dedicated to frequently asked questions.

## View app change

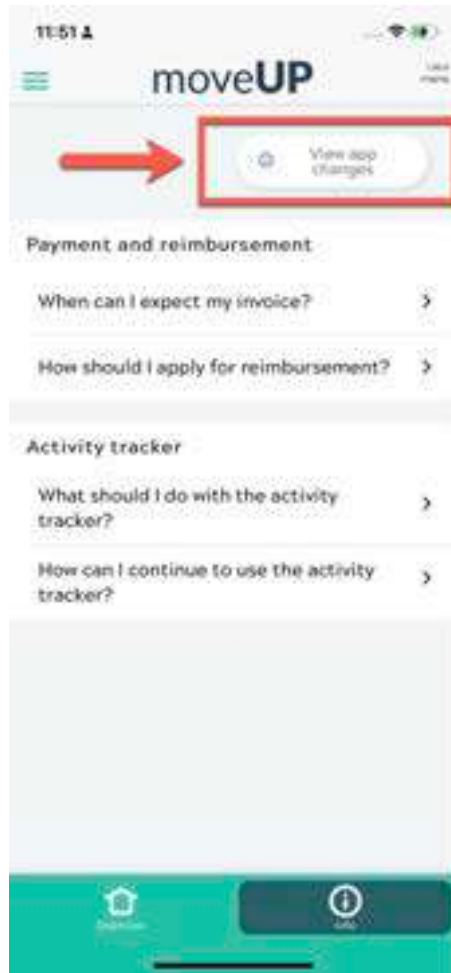
Here you can find an overview of what has changed as discussed on this page.


## Do you have another active follow-up activated already?

In this case you want find your previous information in the overview tab. But you can still access your previous “moveUP journey cards” via the menu.

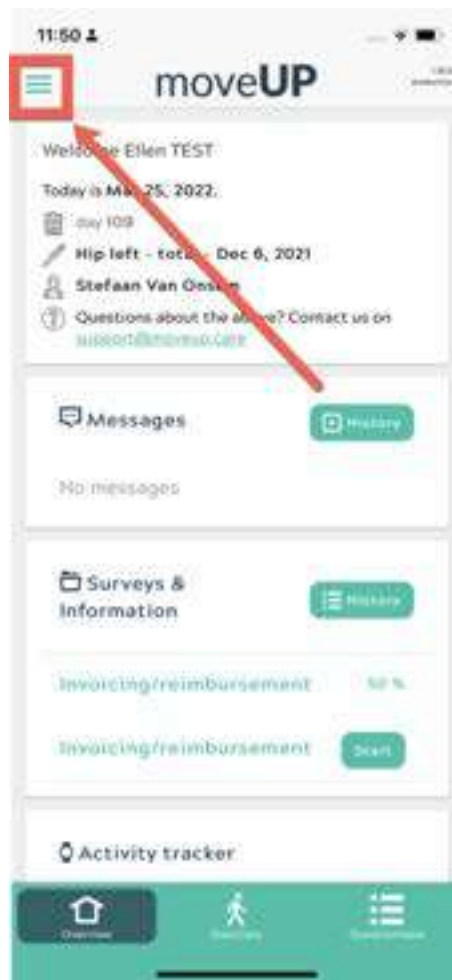Go to the menu by clicking on the top left menu icon.

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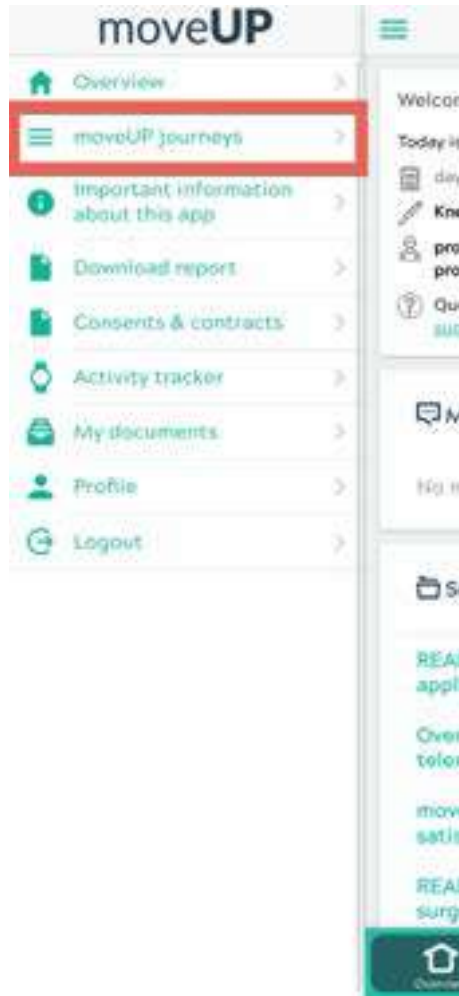The screenshot shows the 'Questionnaire' app interface. It features three sections for reporting symptoms:

- Section 1:** 'Est-ce que l'articulation concernée est gonflée?' (Is the affected joint swollen?). It asks to indicate on a scale from 'Pas de gonflement' (No swelling) to 'Très gonflée' (Very swollen).
- Section 2:** 'Indiquer la douleur ressentie, pendant la nuit, à votre articulation' (Indicate the pain felt during the night at your joint). It asks to indicate on a scale from 'Pas de douleur' (No pain) to 'Beaucoup de douleur' (A lot of pain). Below this is a row of six smiley face icons representing different pain levels.
- Section 3:** 'Indiquer la douleur ressentie, pendant la journée, à votre articulation' (Indicate the pain felt during the day at your joint). It asks to indicate on a scale from 'Pas de douleur' (No pain) to 'Beaucoup de douleur' (A lot of pain). Below this is another row of six smiley face icons.

A red arrow points to the 'Beaucoup de douleur' label in the third section. At the bottom of the screen, there is a navigation bar with three icons: a person, a person with a cane, and a person with a wheelchair. The 'Personne avec un fauteuil roulant' icon is highlighted with a red box.

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The screenshot shows the 'Vragen' (Questions) tab in a mobile application. The interface is in Dutch and contains three questions:

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A red arrow points from the top of the second question's slider down to the bottom of the third question's slider. At the bottom of the screen, there is a navigation bar with three icons: a house, a person, and a list. The list icon is highlighted with a red box.

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The screenshot shows the 'Patient Onboarding Dashboard' for user 'Ellen'. At the top, there are navigation links for 'Dashboard' and 'Patients', and utility links for 'Support' and 'Account'. A warning banner at the top states: 'We're working with sensitive medical data. That's why you should enable two factor authentication. Two factor authentication secures your account better. Enable it in your account settings.' Below the banner, there are five data cards: '227 moveUP candidates' with a '+ New patient' button (indicated by a red arrow), '3706 patients active', '13 patients reminded', '0 candidates not interested', and '0 filed in anamnesis questionnaire'.

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# Edit patient

Basic  Advanced

## Managing Organization \*

AZ Alma Ecklo

Organization that is the custodian of the patient's record

## Name \*

Names associated with the patient

### Given names \*

Ellen

Given names (not always first) includes middle names

### Family name \*

Toet

Family name (last name) (Surname)

+ Add Name

## Contact Point \*

A contact detail for the individual

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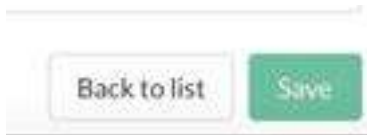
Patient Overview Edit Patient Edit Care Plan Email Overview

## Edit care plans for Ellen Test

### Patient's care plans

Start a care plan to add intervention and treatment for this patient

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## Edit care plans for Ellen Test

### Patient's care plan

Add intervention, treatment or study. Click or drag 'n drop items to add.

Activated care plan (move care plan to active list in medical dashboard)

+ Start new care plan

### AZ Alma Eeklo Interventions

Search

Achilles Left Achilles Right Elateral total hip arthroscopy Bilateral total knee arthroscopy  
Hip left - resurfacing Hip left - revision Hip left - total Hip right - resurfacing Hip right - revision  
Hip right - total Knee right - total Knee left - revision Knee left - total Knee left - use  
Knee right - revision Knee right - use

### moveUP Treatments

Search

THA Anterior Therapy THA Bilateral Therapy THA General Therapy THA Bilateral Therapy  
THA General Therapy Unknown procedure date

### AZ Alma Eeklo Studies

Search

Converion

### AZ Alma Eeklo Care Team Participants

Search

UCLin Belgium Orthopedic care team

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## Edit care plans for Ellen Test

### Patient's care plan

▼ Knee Right - total, TKA General Therapy

Intervention: Knee Right - total

Date  1

Doctor  2

Treatment: TKA General Therapy

Start Date

Code

HCP

Activated care plan (move care plan to active list in medical dashboard)

+ Start new care plan

3

### AZ Alma Eeklo Interventions

Bilateral total knee arthroplasty    Bilateral total knee arthroplasty    Hip left - resurfacing

Hip left - revision    Hip left - total    Hip right - resurfacing    Hip right - revision    Hip right - total

Knee Right - total    Knee left - revision    Knee left - total    Knee left - uln    Knee right - revision

Knee right - uln

### moveUP Treatments

THA Anterior Therapy    THA Bilateral Therapy    THA General Therapy    TKA Bilateral Therapy

TKA General Therapy    Unknown procedure code

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B.Clinic Belgium Orthopedics care team is part of the care team

profile.firstname g47c1meoja profile.lastname jgf6d95u4v  
/ Daphne Piron  
Dieter Kreynen Dieter Kreynen  
Rehabilitation specialist Femke Smeets  
Rehabilitation specialist Gaëtan Haeneccour  
Hugo Ghazi  
Joren Christiaens  
Lize Paridaens  
Kiss Lut Van De Wiele  
profile.firstname ruh33iv83 profile.lastname 6gr0y386m  
Stefaan Van Onsem  
Kris Valerie Timmermans  
Margot Verhaeghe

Stefaan Van Onsem as surgeon is part of the care team

Stefaan Van Onsem

Activated care plan (move care plan to active list in medical dashboard)

+ Start new care plan

Save Next

You saved a care plan. Click [Next] to send emails to the patient.

AZ Alma Eeklo Studies

AZ Alma Eeklo studies

B.Clinic Belgium Orthopedics care team

Dieter profile.firstname ma0hu5r83 profile.lastname 0r6b0yv4b as surgeon

Dieter profile.firstname 0ffarwagh profile.lastname 42f8r5efp as surgeon

Hug & Kris team AZ Alma

Stefaan Van Onsem as surgeon

profile.firstname B8kz9p0rb profile.lastname 47axd0f8k as surgeon

profile.firstname rrrdc33u profile.lastname urv9t0uq3p as surgeon

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moveUP

20

## User manual – Health care platform & Patient app for HCPs

Get insight in the rehabilitation of your patients and provide close follow up

App & Healthcare platform version: 1.34.1 till current

Manual version: 11

Date of Publication: 2023/03/06

Link to e-learning: <https://moveup.talentlms.com/index>

# Table of contents

1. Instructions
2. Warnings and precautions
3. Intended use and intended users
4. Indication for use
5. Contact details
6. Patient app
  - a) Patient app functionalities
  - b) How to pair a wearable
7. Healthcare platform
  - a) Healthcare platform functionalities
  - b) Clinical guidelines for physiotherapists
  - c) Registration of a moveUP patient
  - d) Stop the treatment of a patient in moveUP

# INSTRUCTIONS

Read carefully this instruction for use. Make sure that you know how to use moveUP, but also understand the limitations of moveUP before you start using our service. If you do have any questions on the use, contact moveUP.

In the final chapter of this document you find our contacts, which you can use during the use of our service, in case you would have technical problems with the application (moveUP dashboard or patient app) or when expertise is required about a care path.

The Healthcare practitioner can't give access to third parties to access the moveUP dashboard. Every other person can, even by accident, send out wrong information or adapt the patient profile, with the result that the advice sent by moveUP no longer corresponds with the needs of the patient.

In case the application is use while moveUP & b.clinic is not asked to do the follow-up of patients, the individual healthcare provider or the health care facility is responsible for the follow-up of the patient and the action that are carried out.

In compliance with the medical device legislation, you are required to report any complaints towards the application, any alteration regarding characteristics / specifications of the application and incidents / complications potentially linked to the use of the application to [support@moveup.care](mailto:support@moveup.care). The AFMPS can also be contacted in case of incident via [https://www.famhp.be/en/human\\_use/health\\_products/medical\\_devices\\_accessories/materiovigilance/how\\_notify](https://www.famhp.be/en/human_use/health_products/medical_devices_accessories/materiovigilance/how_notify)

In compliance with the GDPR legislation, please report eventual data leaks linked to the use of moveUP. Please, read the entire privacy policy : <https://www.moveup.care/privacy>

# Warnings and precautions

- moveUP is not an emergency tool. In case of emergency please contact your doctor or call 112.
- moveUP is not intended to monitor/treat vital parameters of critical diseases.

- moveUP companion is not a replacement of any treatment you need to follow, but is used as an addition.
- Patients need to be aware if questionnaires are not filled out on frequent basis or not filled out trustworthy, the care team has more difficulty to know their health situation & evolution.

- The moveUP App offers personalized healthcare based on the patient's profile and therefore can not be shared, borrowed or traded between users. The information and advice of the moveUP application do not apply to a patient other than the one whose profile is stored in the application. The patient may not grant access to the moveUP App to third parties. Any other person can, even accidentally, send wrong information or change the profile of the patient, with the result that the moveUP-guided advice no longer corresponds to the needs of the patient.

*With software, there is always a residual risk for bugs. If you have the feeling something is wrong or you don't get a response in the app, please reach out to us via [support@](mailto:support@) or call 0800 88 008.*

The moveUP IFU are only supplied in electronic format. If you need a full printed version please contact us at the e-mail address [info@moveup.care](mailto:info@moveup.care). This mail is available 24/7

# INTENDED USE

For healthcare providers, moveUP provides efficient clinical management of pathologies and treatments. With the use of moveUP, valuable insights are given to the HCP/care team of the status of the patient and its evolution.

For patients, moveUP provides personalised information and instructions to help them managing their symptoms and progress in their rehabilitation. The intensity of follow-up is adapted based on patients' needs and timepoint in the patient pathway:

## **moveUP companion = monitoring and information, no active follow-up**

moveUP companion offers targeted information and evolution reports to patients. Their care team takes the evolution reports into account in their further recovery when appropriate.

## **moveUP coach = active follow-up by healthcare team**

moveUP coach offers targeted information and evolution reports to patients. Their care team is more actively involved and takes the evolution reports into account and can provide advice and exercise suggestions through the digital platform.

## **moveUP therapy = active follow-up by healthcare team, with data driven validated care protocol**

moveUP therapy offers a data-driven validated care protocol with certain category and level of exercises and activities, specifically targeted to the individual patient. Their care team can manually adapt the data driven validated protocol when needed. Patients can fully rehabilitation with moveUP without leaving their home environment.

*Optional functionality that can be enabled: interoperability with Class IIa continuous passive motion (CPM) medical device. For knee and hip patients who are using a Class IIa CPM medical device, moveUP can interoperate with the Class IIa CPM software. moveUP acts as a facilitator to easy assign a designated CPM exercise protocol chosen by a physician to a patient and to display the performed CPM exercises in the medical dashboard.*

### **Contra-indications and potential side effects**

There are no contra-indications or known side effects.

### **Human body contact**

No human body contact with patient or user, due to nature of the product (software).

### **Accessories/products used in combination**

There are no accessories. If the patient has no compatible tablet/smartphone or activity tracker, moveUP can lease these devices to the patient. The leased devices are CE marked devices that meets the compatibility criteria outlined in the IFU for the app.

### **Device lifetime**

2-years. Depending of the willingness of the user to update the app. We sent out a notification to recommend to update the app, to make sure the app will keep the performance & recent included features.

### **Claims**

*moveUP companion/coach/therapy*

The intensity of follow-up is adapted based on the needs of the patient, via the moveUP Symptom & QoL monitoring tool.

moveUP enhances the clinical management of the patients, because early detection & management of complications is possible via the symptom & QoL monitoring tool

More efficient clinical management, such as the number of consultations can be reduced

Enforces therapy compliance / adherence

The correct information is provided at the right time

*moveUP therapy*

With the use of moveUP therapy knee & hip arthroplasty patients can fully rehabilitate via the in-app care team without leaving their home environment.

# Intended users

moveUP is intended to be used by patients and healthcare providers. The main user of the mobile app and patient website is the patient.

## Inclusion:

- Age: minimum 18 years / maximum no limit
- Health & condition: capable of performing basic activities of daily living
- Language: understanding one of the available languages of the app (Dutch, French, German, English)

## Exclusion:

- Patients who are mentally incompetent or having troubles to express what they are feeling (for instance, mentally diseased people, people staying in elderly care centres, ...) are excluded.
- Patients who are not capable of operating a tablet/smartphone and activity tracker.
- Patients who can't understand one of the available languages of the app (Dutch, French, German, English)

The main user of the medical web interface is a healthcare practitioner (group) or clinical researcher (group), named the care team. The care team is able to operate a web interface via web browser on PC/tablet/smartphone. The healthcare practitioner needs to understand one of the available languages of the web interface (today only available in English).

# COMPANY DETAILS

Head office Brussel: Parvis sainte-gudule 5,  
B-1000 Brussels, Belgium Office Ghent:  
Oktrooiplein 1, B-9000 Ghent, Belgium  
Tel: 0800 88 008  
Email: [info@moveUP.care](mailto:info@moveUP.care) [www.moveUP.care](http://www.moveUP.care)

If you need the eIFU in paper format please contact moveUP and we'll supply you a copy in paper in 7 days.

moveUP supplies the IFU in electronic format due to the topology of the product (software) and the availability of the eIFU at any moment that is needed for the user.

+G166G166MOVEUPV1340WS00+5571341/16D202303100Y

Basic UDI-DI: ++G166MOVEUPV1340WS

In case of questions about the App, the tablet or the smart bracelet, you can contact moveUP via the following channels - in order of priority:

- Via the message system of the App. Please preface the message with **“Technical question:”**
- Via email to **support@moveup.care**
- Via phone during office hours: **0800 88 008**


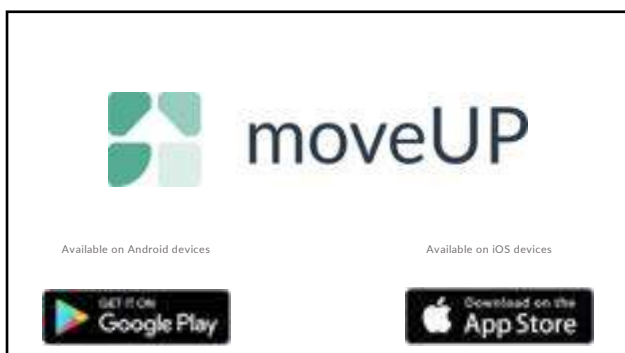
# moveUP

Patient app


CE

1

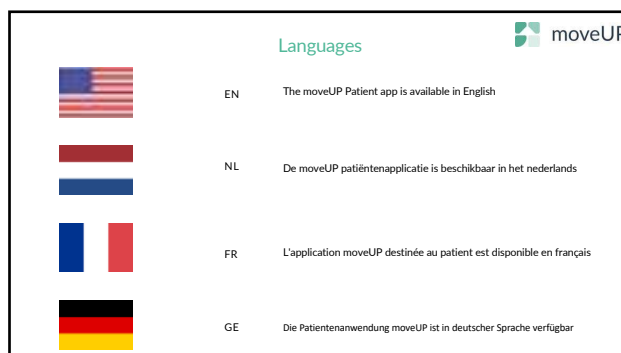
2

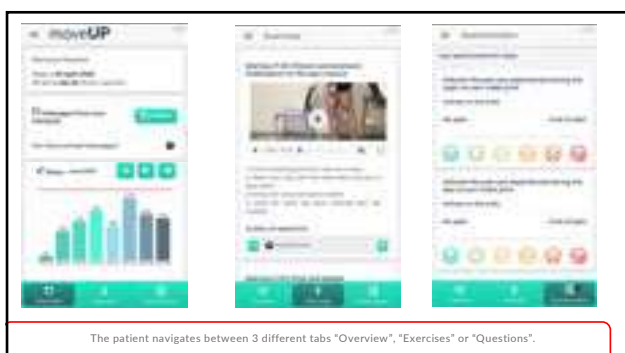
The patient navigates between 3 different tabs "Overview", "Exercises" or "Questions".

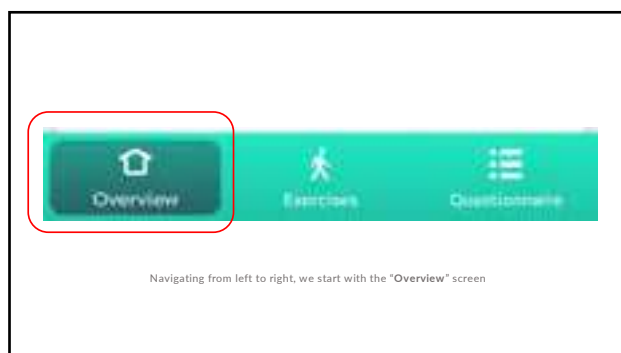3


4

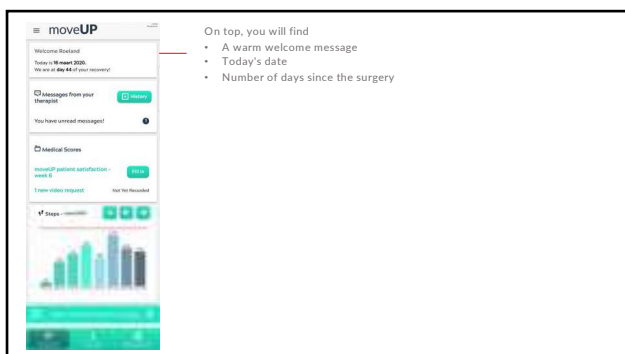On top, you will find

- A warm welcome message
- Today's date
- Number of days since the surgery

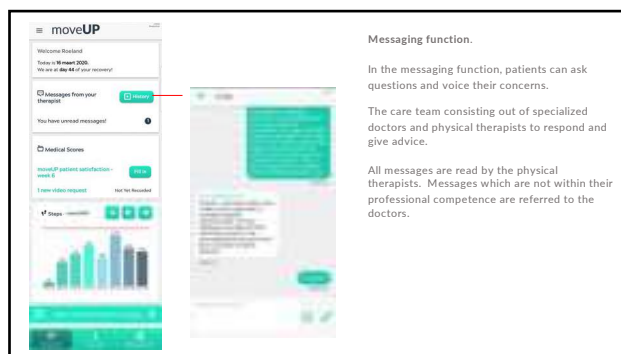5

Messaging function.

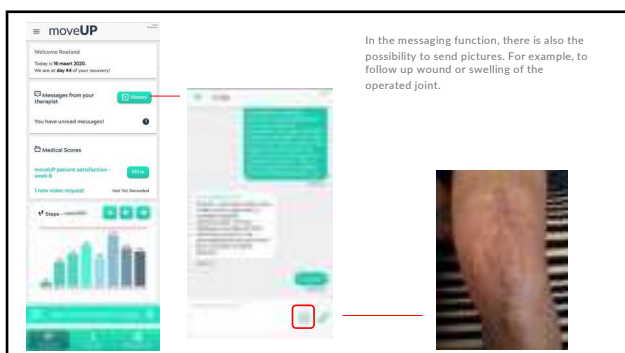
In the messaging function, patients can ask questions and voice their concerns.

The care team consisting out of specialized doctors and physical therapists to respond and give advice.

All messages are read by the physical therapists. Messages which are not within their professional competence are referred to the doctors.


6


In the messaging function, there is also the possibility to send pictures. For example, to follow up wound or swelling of the operated joint.

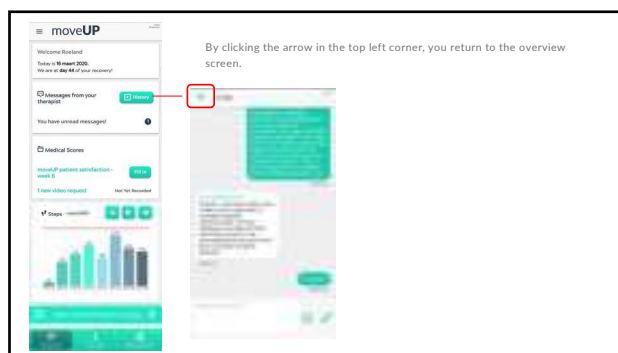7

By clicking the arrow in the top left corner, you return to the overview screen.

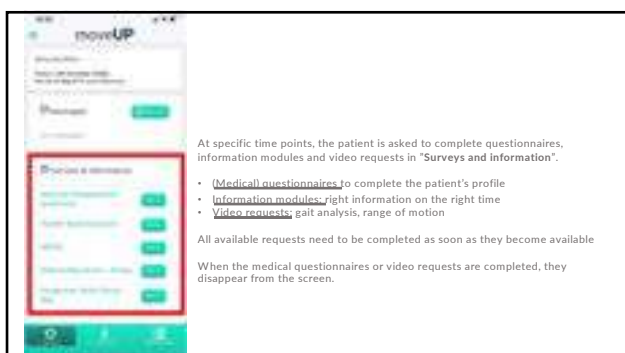8


At specific time points, the patient is asked to complete questionnaires, information modules and video requests in "Surveys and Information".

- **Medical questionnaires** to complete the patient's profile
- **Information modules:** right information on the right time
- **Video requests:** gait analysis, range of motion

All available requests need to be completed as soon as they become available

When the medical questionnaires or video requests are completed, they disappear from the screen.

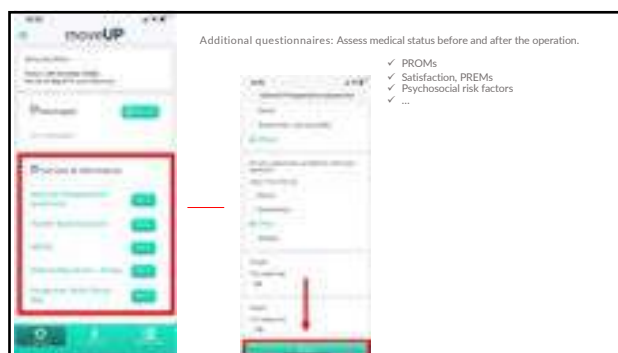
9

Additional questionnaires: Assess medical status before and after the operation.


- ✓ PROMs
- ✓ Satisfaction, PREMs
- ✓ Psychosocial risk factors
- ✓ ...

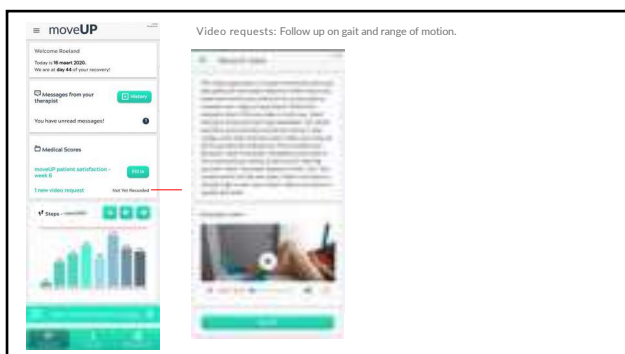10

Video requests: Follow up on gait and range of motion.

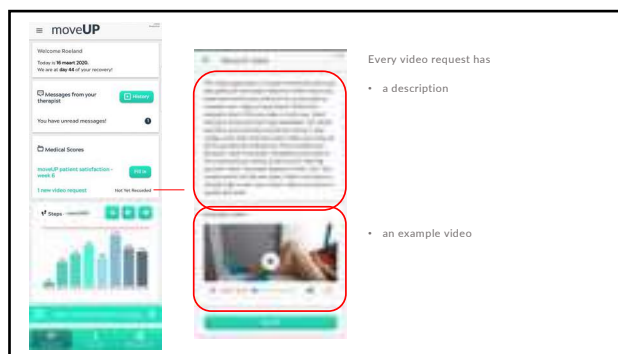11


Every video request has


- a description
- an example video

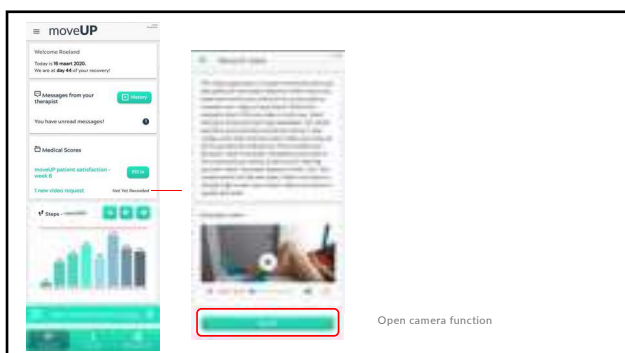12



13

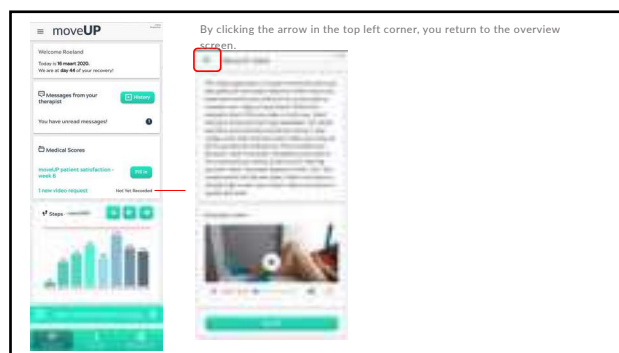
14

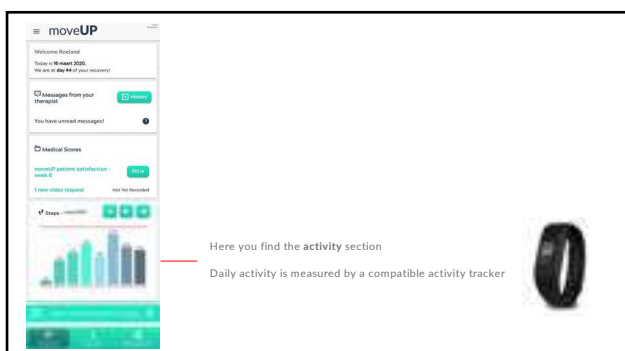
15

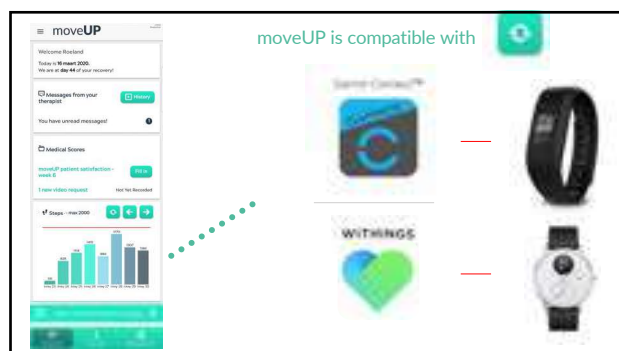16

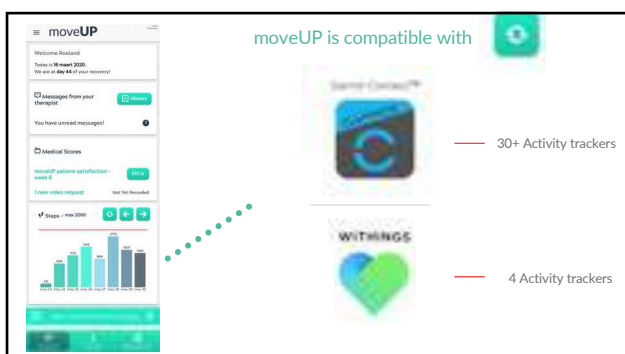
17

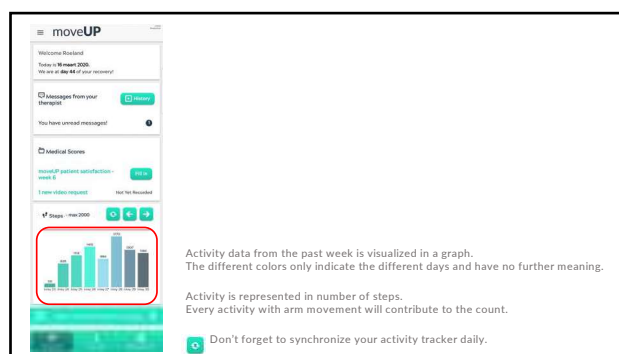18

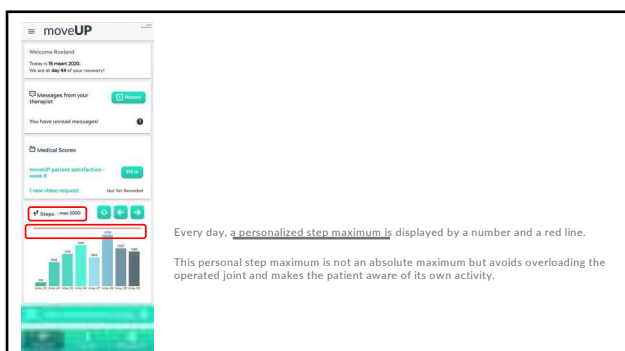Every day, a personalized step maximum is displayed by a number and a red line. This personal step maximum is not an absolute maximum but avoids overloading the operated joint and makes the patient aware of its own activity.

19

At the bottom of the overview screen you can find the last section.

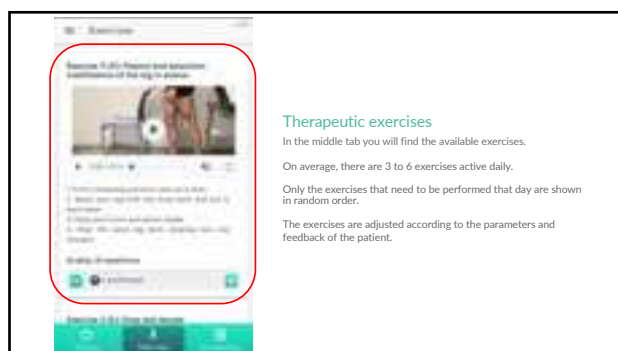
"More information about your surgery": opens a page with practical information helpful before and after surgery.

20

The mid tab: "Exercises"

21

### Therapeutic exercises

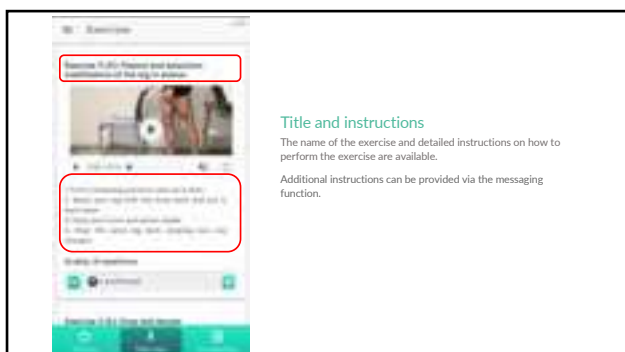
In the middle tab you will find the available exercises.

On average, there are 3 to 6 exercises active daily.

Only the exercises that need to be performed that day are shown in random order.

The exercises are adjusted according to the parameters and feedback of the patient.

22


### Title and instructions

The name of the exercise and detailed instructions on how to perform the exercise are available.

Additional instructions can be provided via the messaging function.

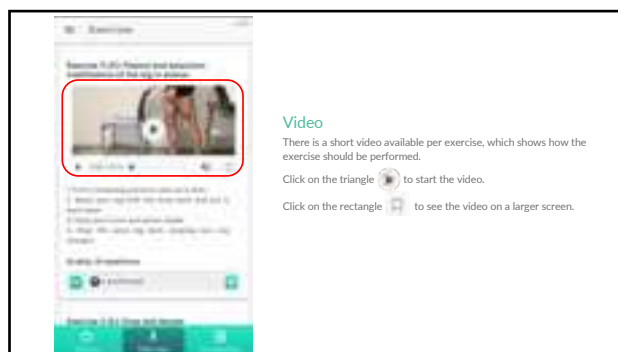23

### Video

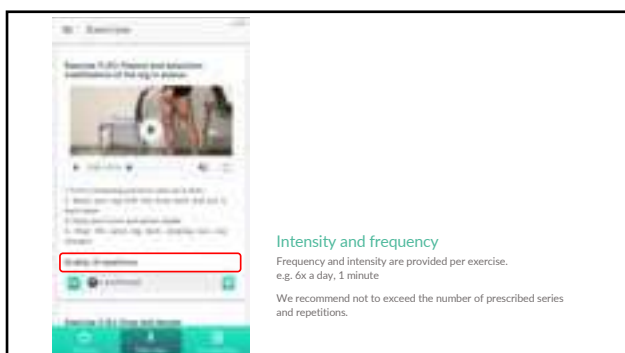
There is a short video available per exercise, which shows how the exercise should be performed.

Click on the triangle  to start the video.

Click on the rectangle  to see the video on a larger screen.

24


### Intensity and frequency

Frequency and intensity are provided per exercise, e.g. 6x a day, 1 minute

We recommend not to exceed the number of prescribed series and repetitions.

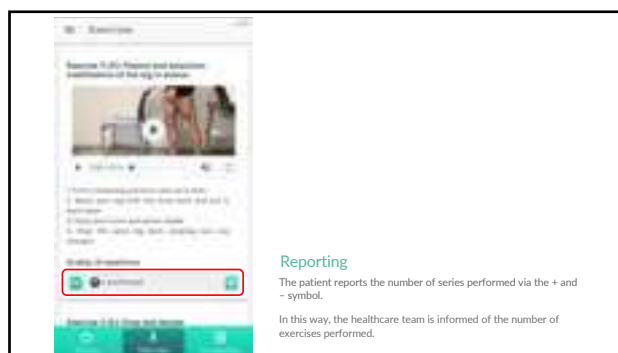25

### Reporting

The patient reports the number of series performed via the + and - symbol.

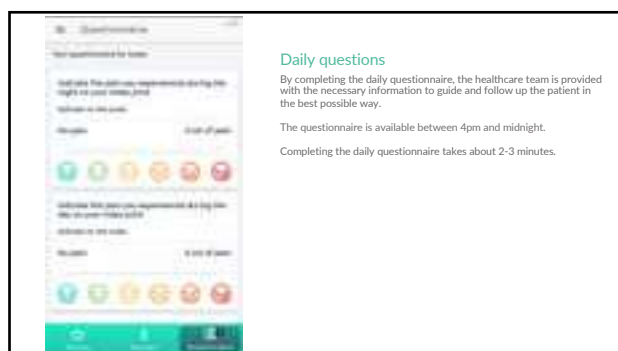
In this way, the healthcare team is informed of the number of exercises performed.

26

The right tab: "Questionnaire"

27


### Daily questions

By completing the daily questionnaire, the healthcare team is provided with the necessary information to guide and follow up the patient in the best possible way.

The questionnaire is available between 4pm and midnight.

Completing the daily questionnaire takes about 2-3 minutes.

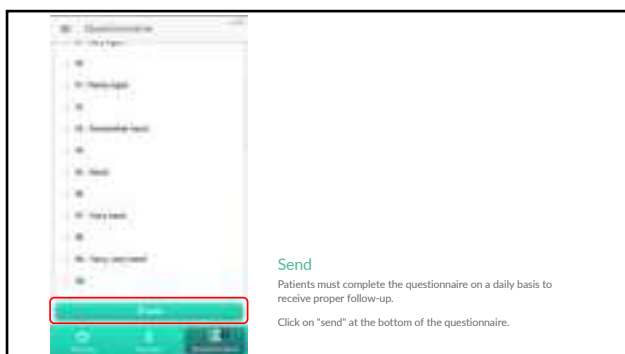28


### Send

Patients must complete the questionnaire on a daily basis to receive proper follow-up.

Click on "send" at the bottom of the questionnaire.

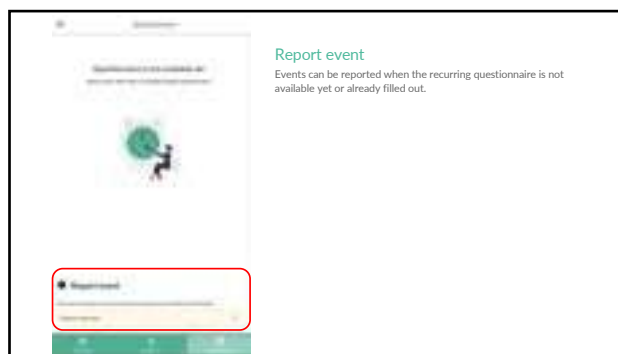29


### Report event

Events can be reported when the recurring questionnaire is not available yet or already filled out.

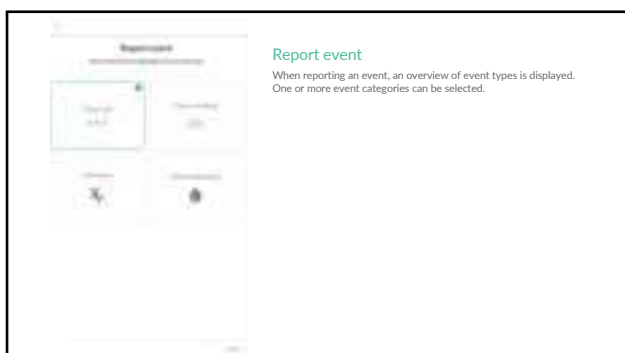30


**Report event**  
When reporting an event, an overview of event types is displayed. One or more event categories can be selected.

31


For each selected category, a set of questions is displayed that need to be filled out. Once all questionnaires have been answered, the given answers are sent to the practitioner.

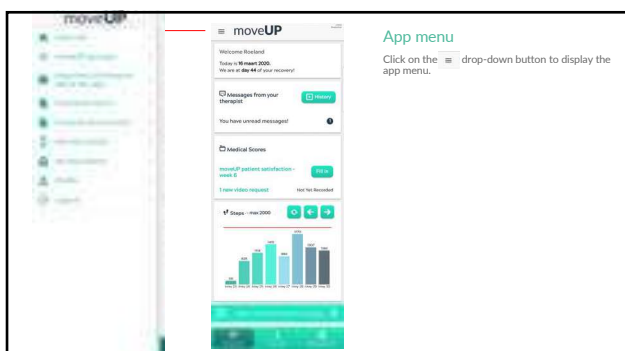32


**App menu**  
Click on the  drop-down button to display the app menu.

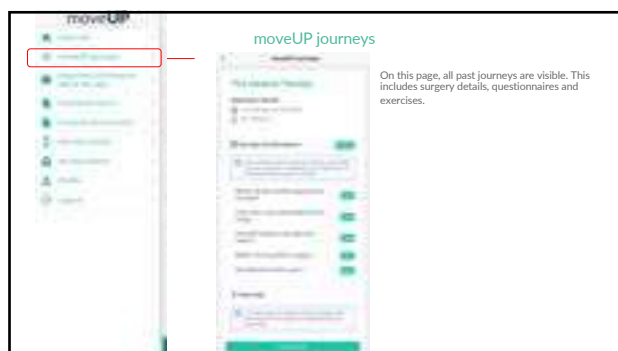33

**moveUP journeys**  
On this page, all past journeys are visible. This includes surgery details, questionnaires and exercises.

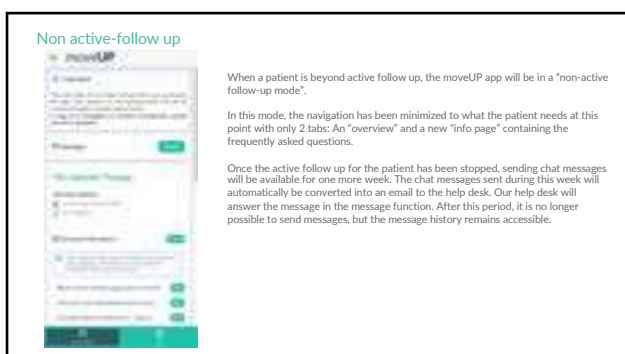34

**Non active-follow up**  
When a patient is beyond active follow up, the moveUP app will be in a 'non-active follow-up mode'.  
In this mode, the navigation has been minimized to what the patient needs at this point with only 2 tabs: An 'overview' and a new 'info page' containing the frequently asked questions.  
Once the active follow up for the patient has been stopped, sending chat messages will be available for one more week. The chat messages sent during this week will automatically be converted into an email to the help desk. Our help desk will answer the message in the message function. After this period, it is no longer possible to send messages, but the message history remains accessible.

35

## How to pair your Garmin Vivofit 4 & Withings go activity tracker with the moveUP app?

**i** The following activity trackers are recommended by moveUP for usability and accuracy reasons. You can use any activity tracker of your choice.

**i** Pairing the activity tracker with the moveUP app is essential to your care trajectory. The pairing only needs to be done once.

To pair your activity tracker click the green button on your homescreen.

Choose "Garmin Vivofit 4" and click continue

Go through all the steps that will appear on your screen.

! Be aware: when you are pairing the activity tracker a 6-digit code will show on the screen of the **activity tracker**.

On some android devices, a pop-up appears on the screen of the phone with a message the code is often 1111 or 1234. Please ignore that message. It is the **6-digit code from the activity tracker** screen you have to enter.

- If everything went well you will end up on this screen.
- You can click on the arrow at the left top to go back to the overview screen.

 If you have problems pairing your activity tracker please send an email to [support@moveup.care](mailto:support@moveup.care)

Withings Go:

To pair your activity tracker click the green button on your homescreen.

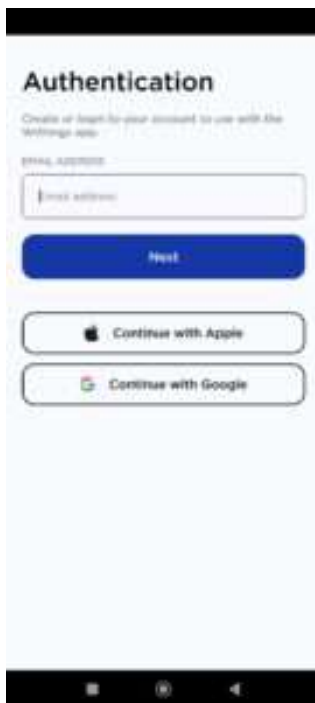
Choose "Personal Activity tracker"


Select the Withings option

Pair the moveUP app with your Withings account. If you have any doubt you can find more info at: <https://support.withings.com/hc/en-us/articles/214484018-Installing-my-Go->

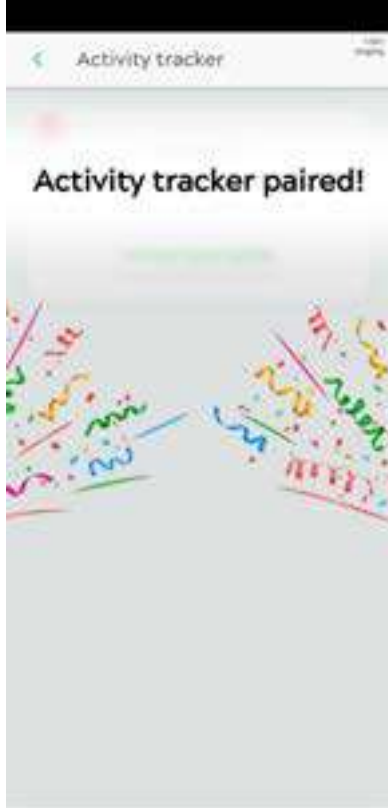Your device is paired with the moveUP app.

## How to synchronize my steps daily?

We advise you to wear your activity tracker around your wrist day and night. However, using during the day, is more important for us to register your activity. The activity tracker is always on and captures how active you are, hence it does not need to be activated.

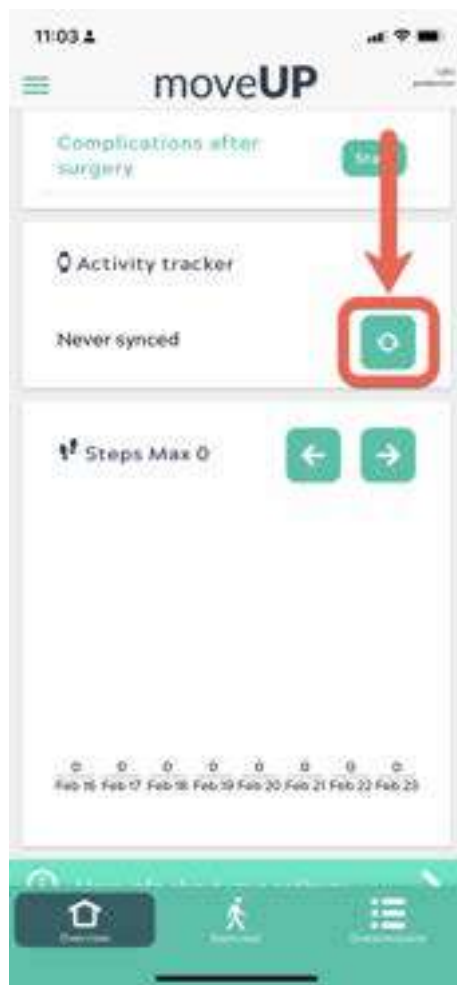
You must need to manually sync your activity tracker data **daily** in the moveUP app. This way your care team can monitor how active you are and decide if any adaptations to your exercise scheme are needed.

 **We advise syncing your steps daily in the evening.**  
This way your care team will see the next day how active you were.

 Make sure you have at least made 100 steps with your activity tracker before you try to synchronize it.  
Don't sync your steps too often during the day. Once a day after filling out the daily questionnaire is more than enough.

 The activity tracker has many options in the menu, but we will not be using them. The instructions below are the only ones you will use during your recovery. At the end of the treatment, if you wish to keep the activity tracker, we will send you a full manual by email.

- Press the synchronizing button marked in red.

- You will see the following screen. Click on “Sync”

- Follow the instructions on the screen. Basically you have to press the silver button on your activity tracker twice for 2 seconds.
- At the end you will automatically be taken back to the homescreen and your steps will be visible in the graph.

- You will see the synced steps data appear on the graph on your home screen.

**i** If you have problems synchronizing please send an email to [support@moveup.care](mailto:support@moveup.care) and try to describe exactly in which step you encounter problems and what messages you see or get on your screen.


# moveUP

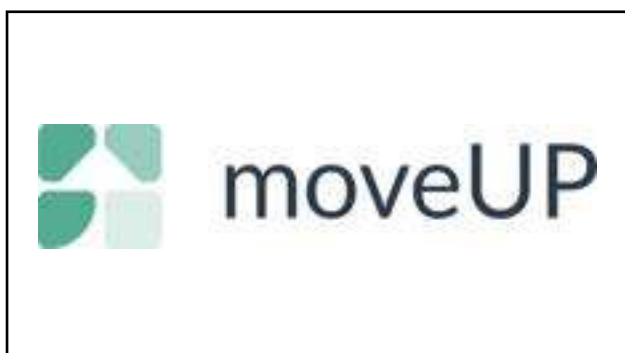Healthcare Platform

CE


1


2

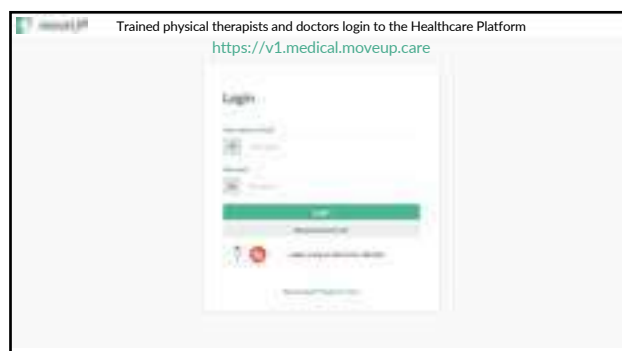3

4

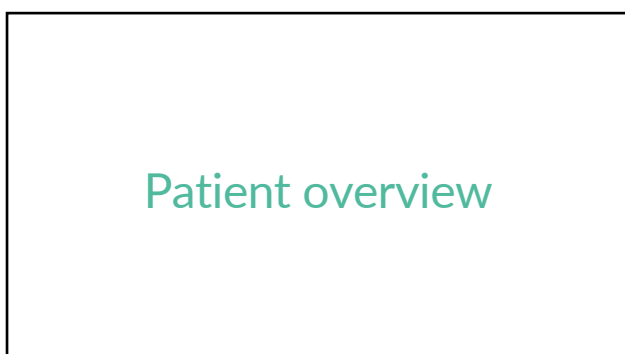
5

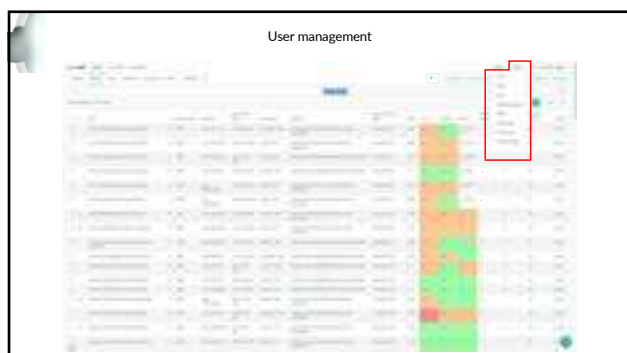
6

7

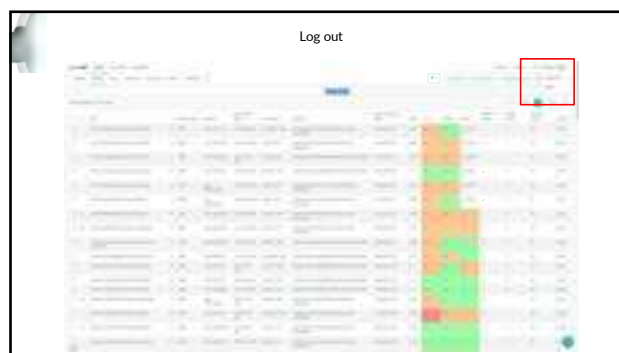8

9

10

**Candidates**  
Patients who are awaiting a care plan. They downloaded the moveUP app and registered or are created by a hcp.

**Therapy**  
Patients with an active Therapy care plan before or after surgery.

**Coach**  
Patients with an active Coach care plan before or after surgery.

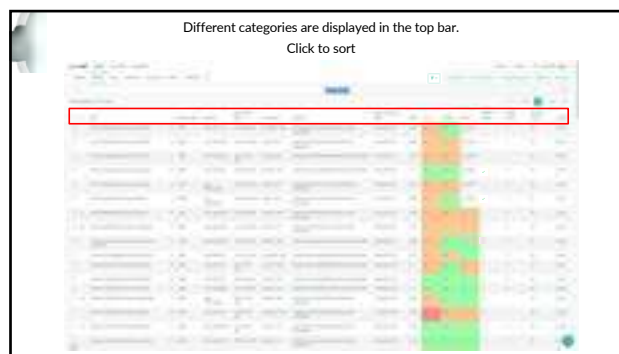
**Companion**  
Patients with an active Companion care plan before or after surgery.

**PROM**  
Patients who will still receive medical questionnaires for long term follow up on specific timepoints (e.g. 3m - 6m, ...).

**Completed**  
Patients who have completed their long term follow up.

**All**  
All patients.

11


12

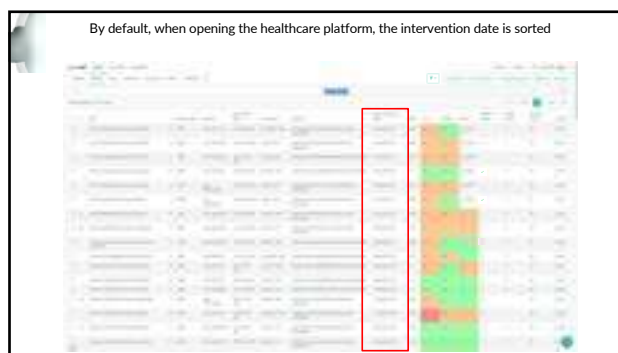
13

14

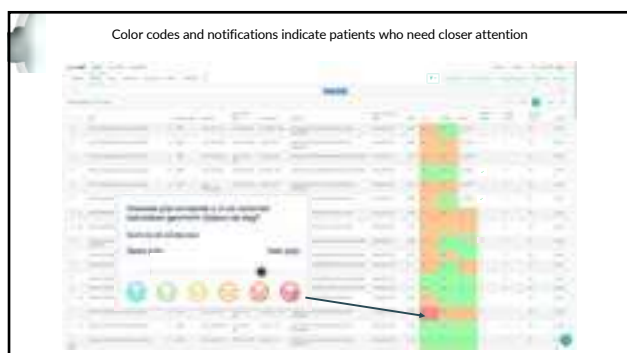15

16

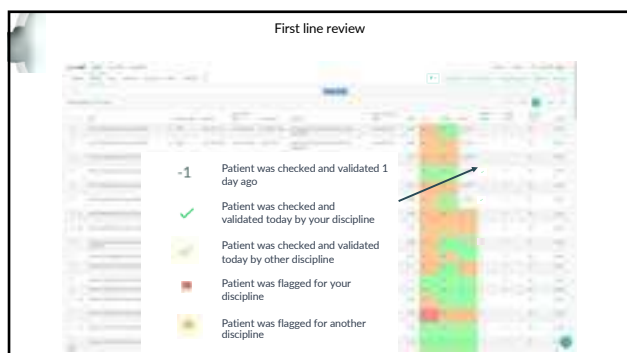
17


18



19

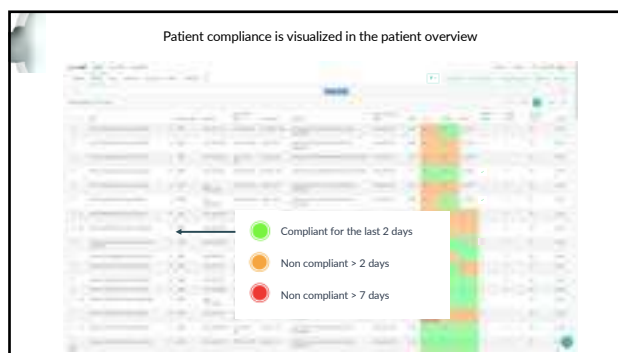
20

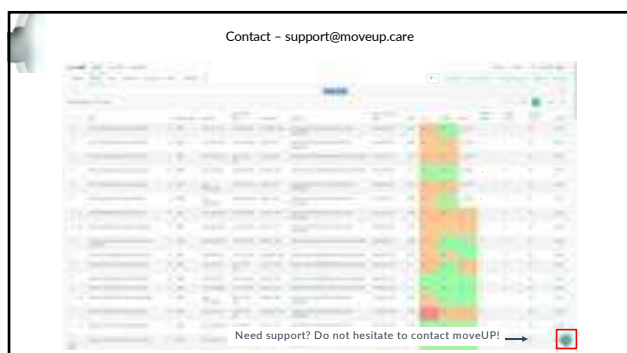
21

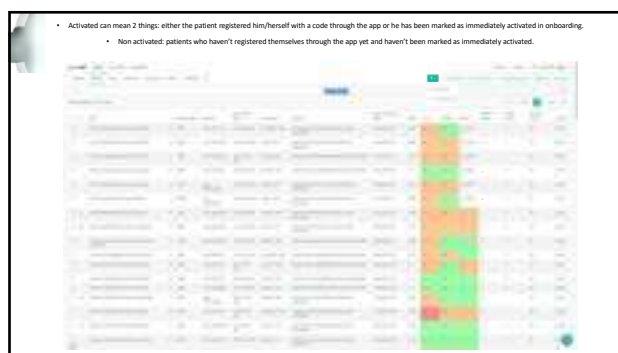22

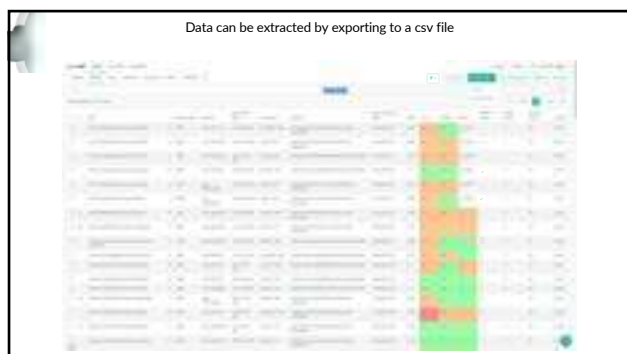23

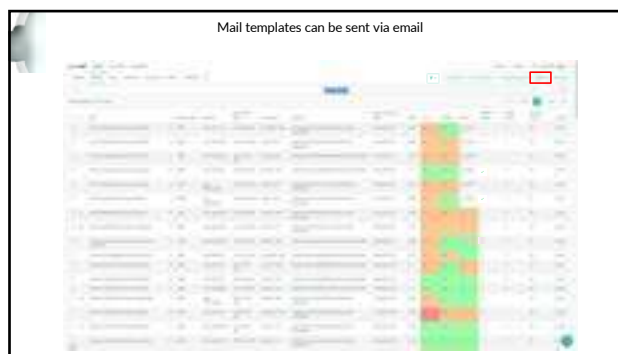24

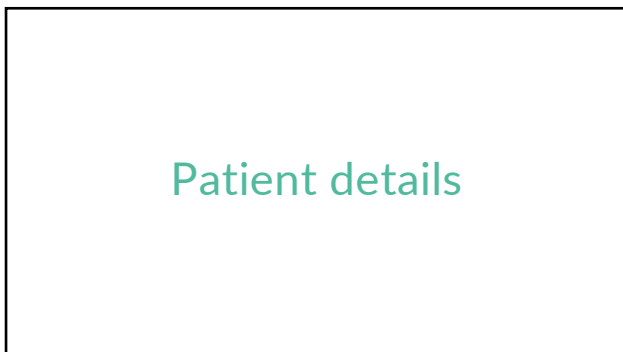
25


26


27

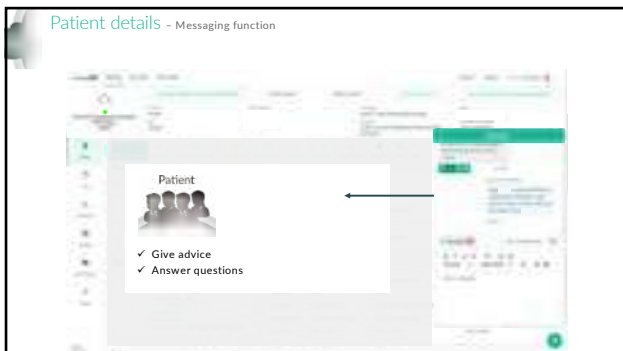28

29

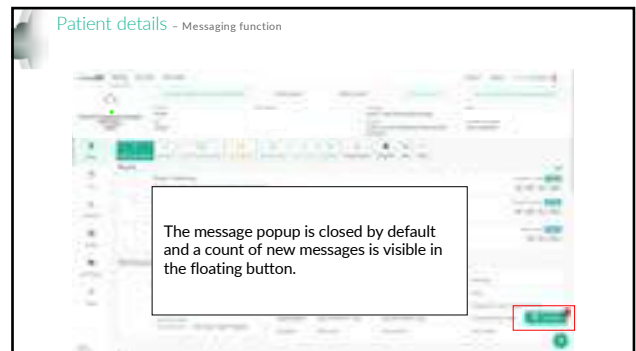
30


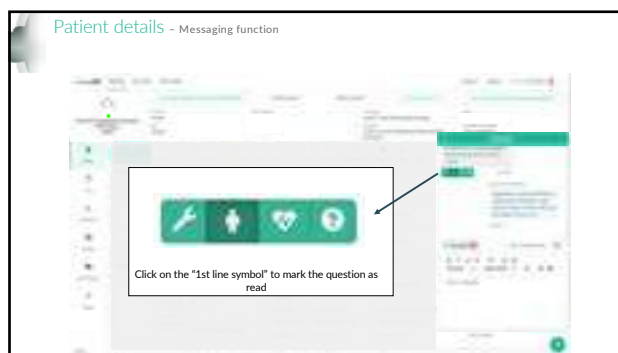
31


32

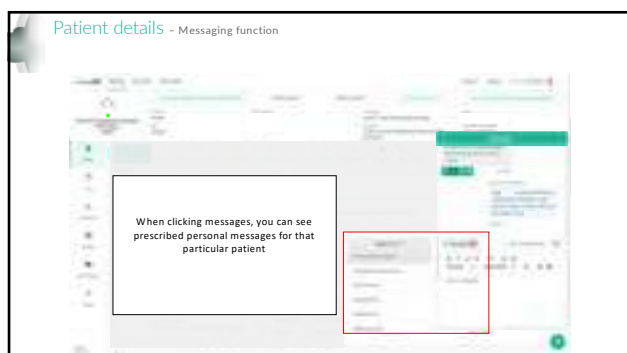33

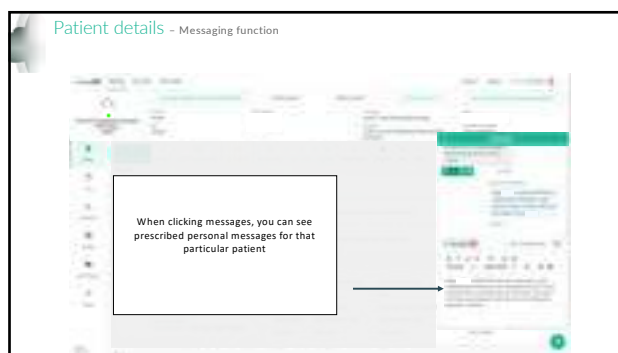34

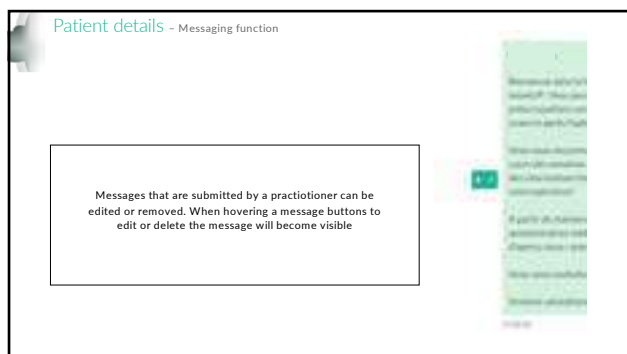
35

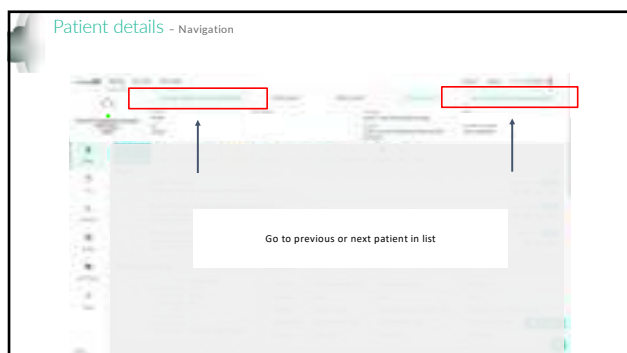
36



37

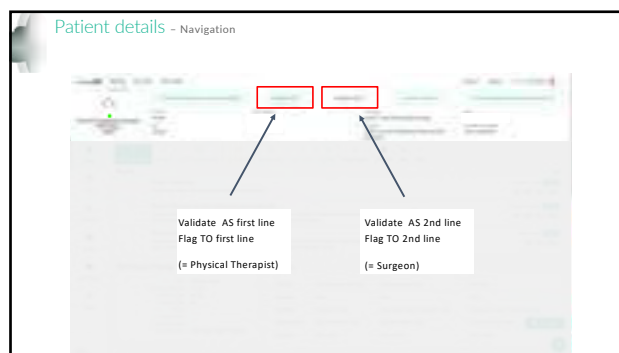
38

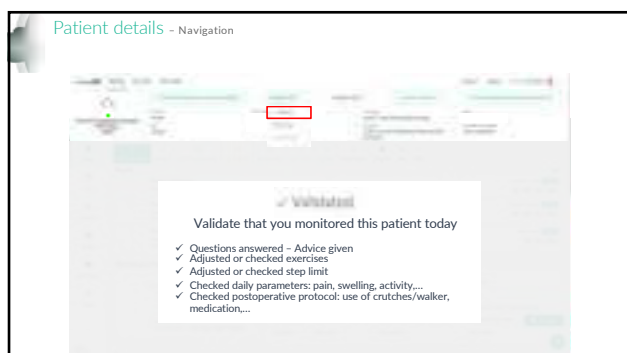
39

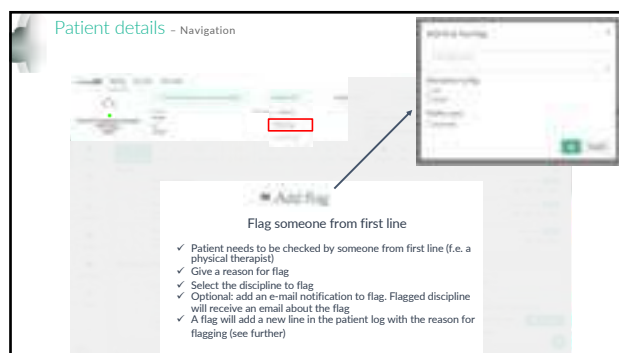40

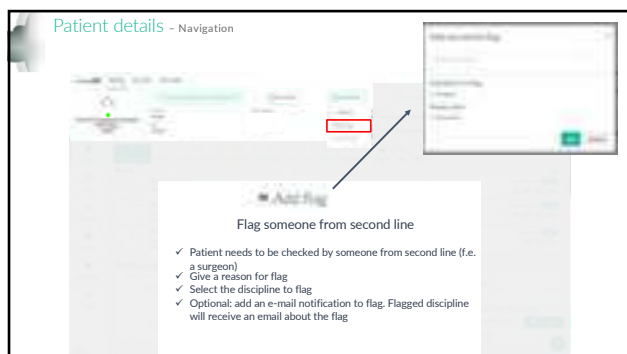41

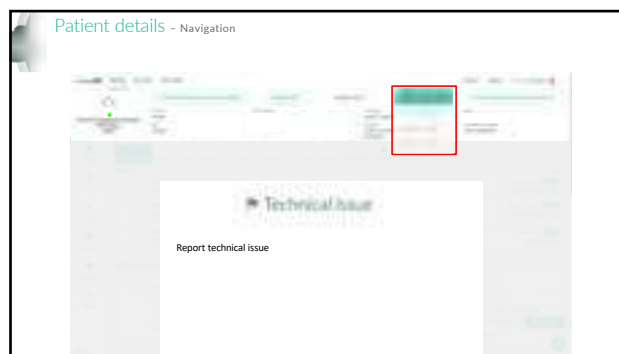
42


# Patient details - Status

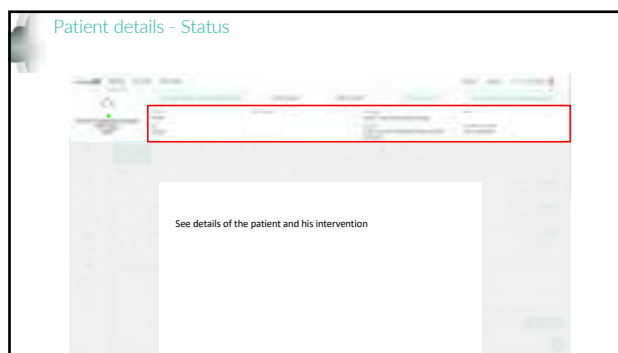
43


44

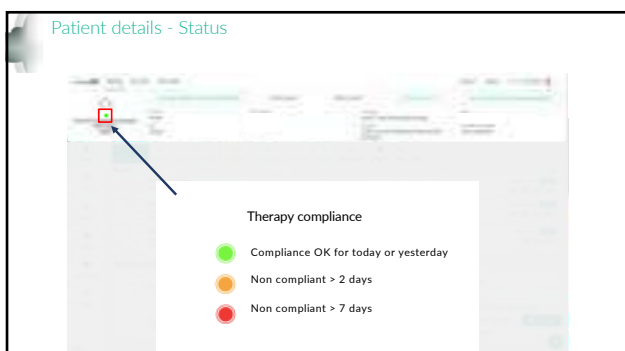
45

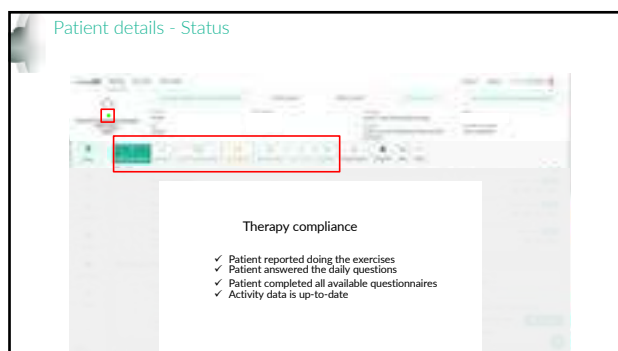46

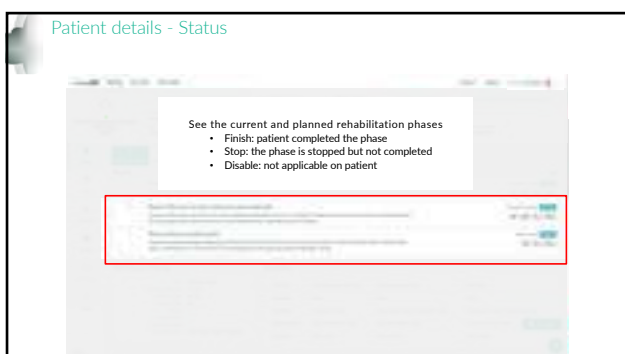
47

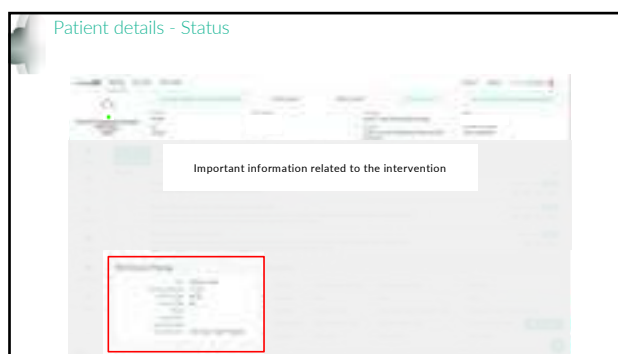
48


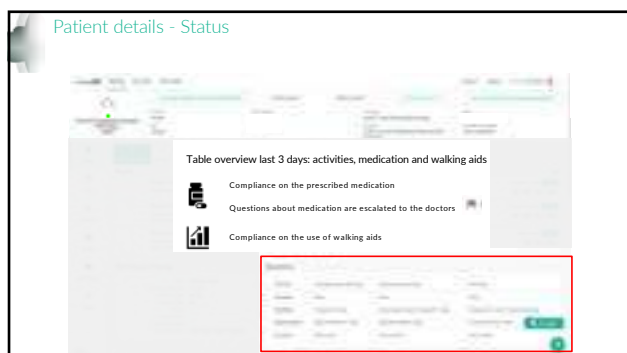
49

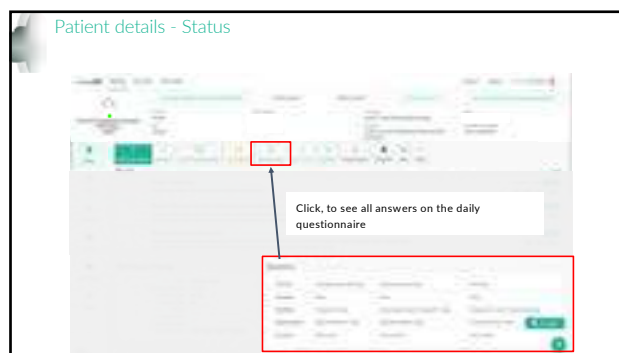
50

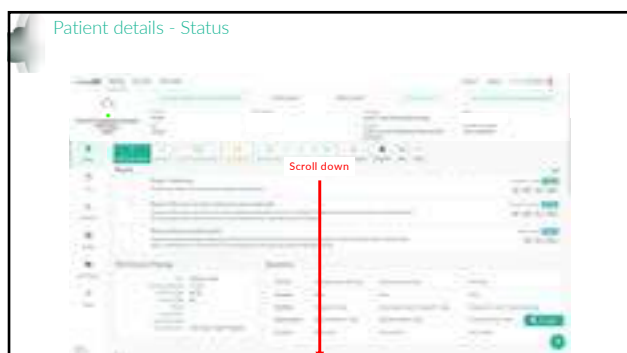
51

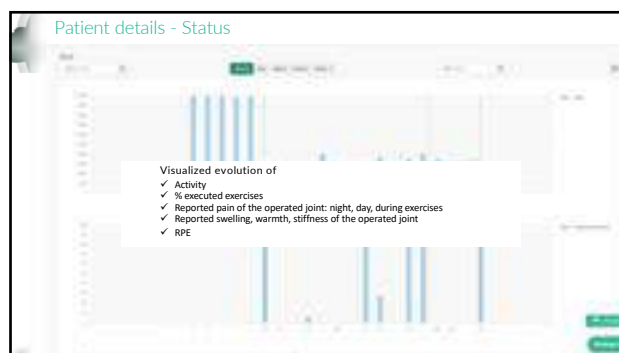52

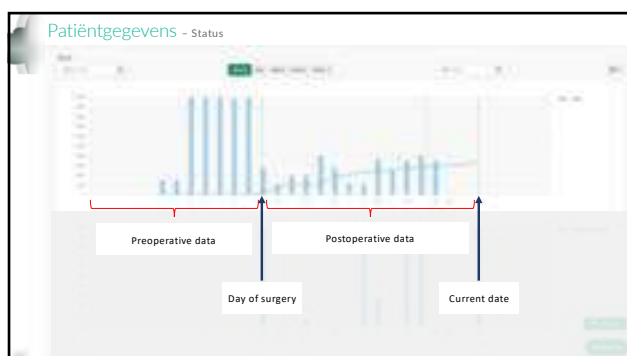
53

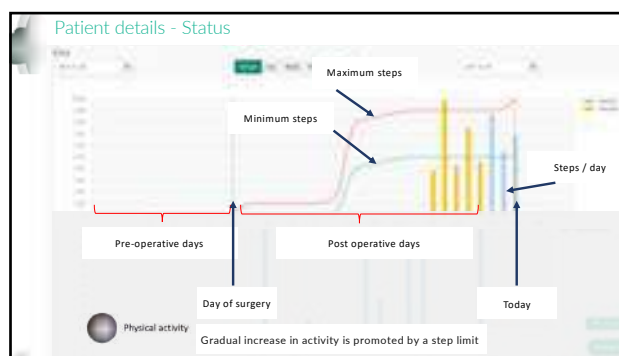
54


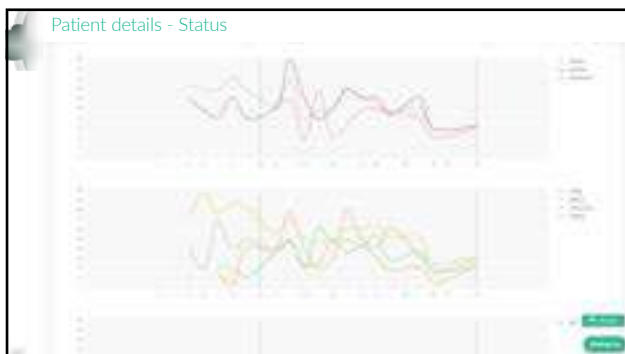
55

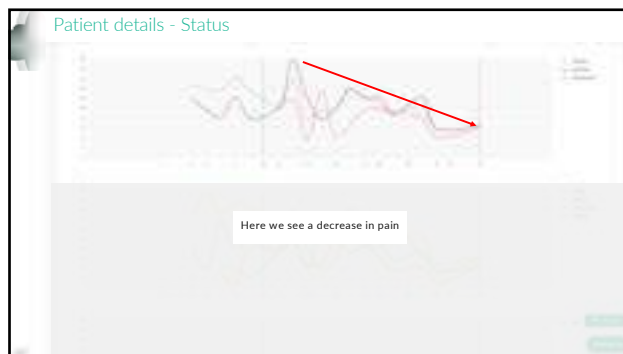
56

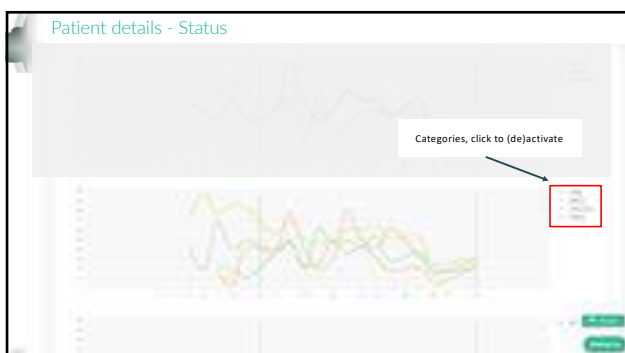
57

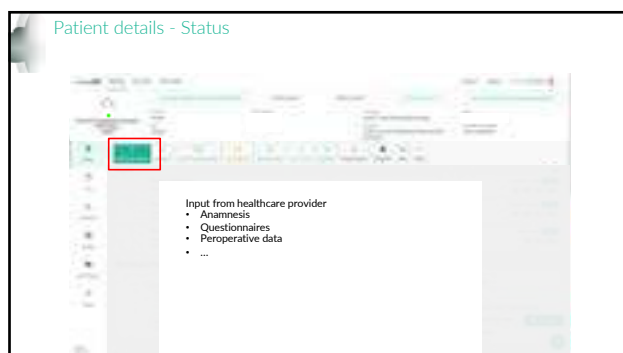58

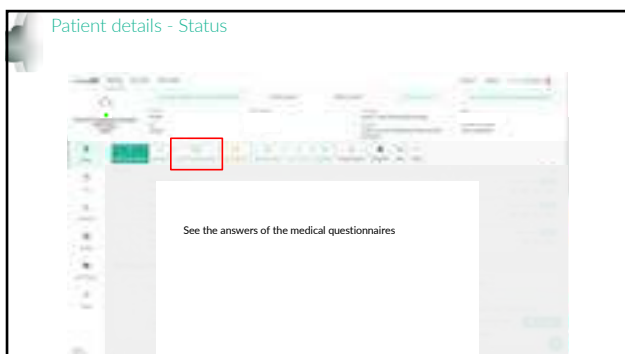
59

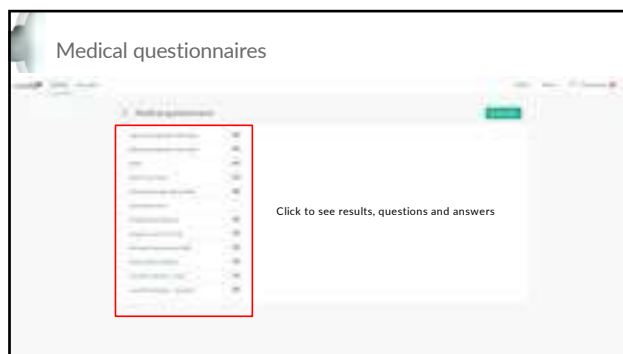
60


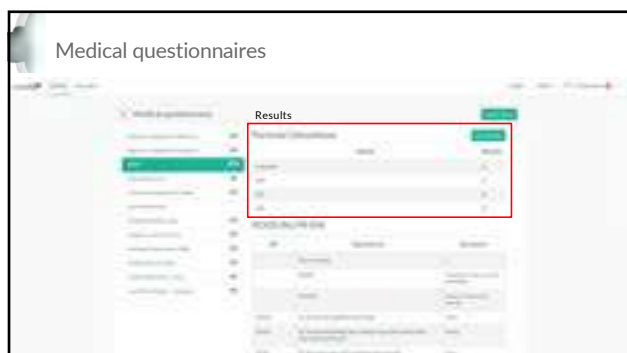
61

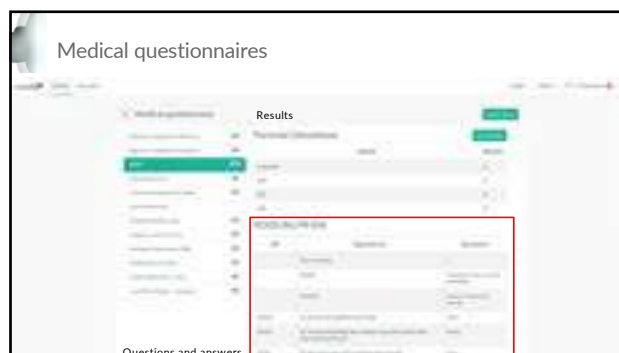
62

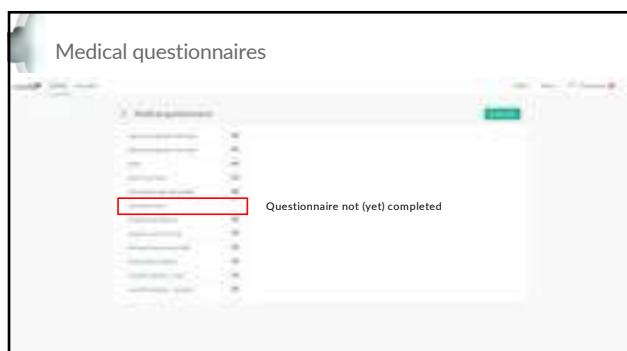
63

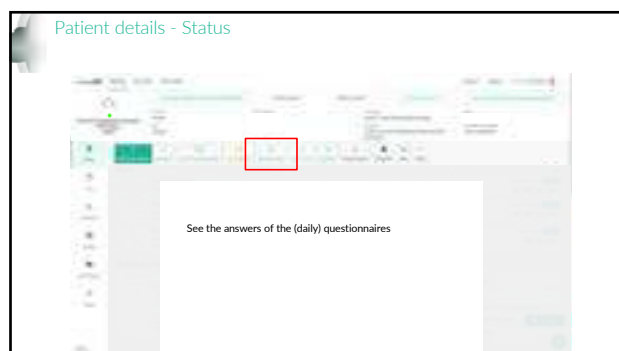64

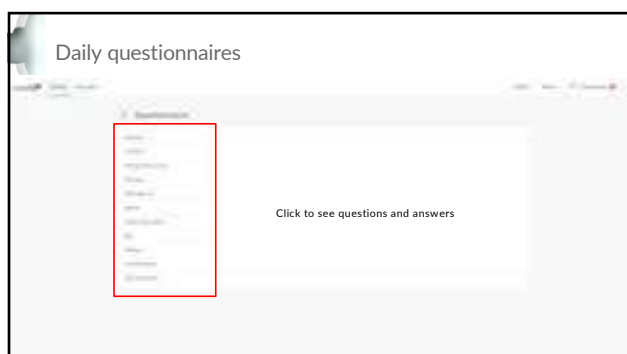
65

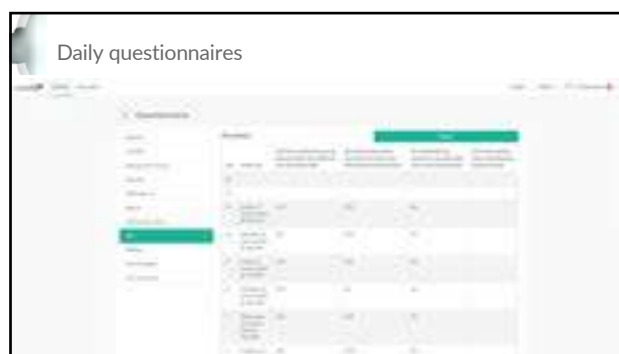
66


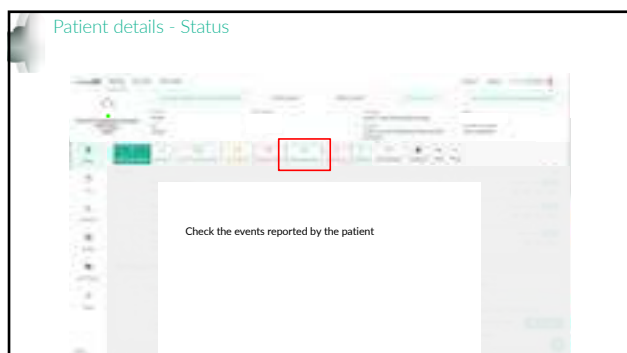
67

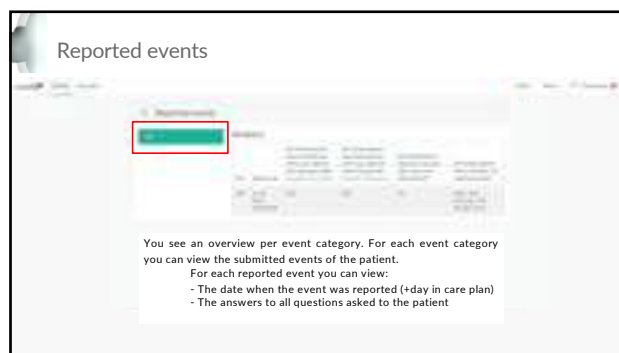
68

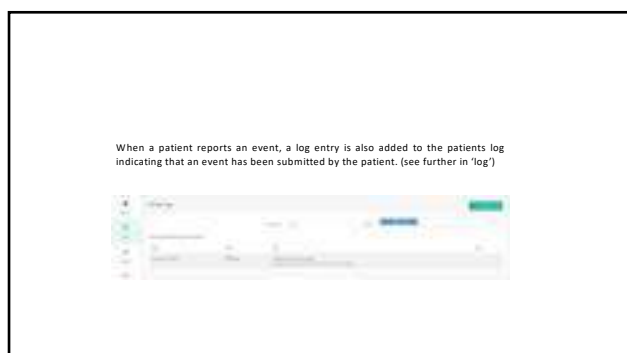
69

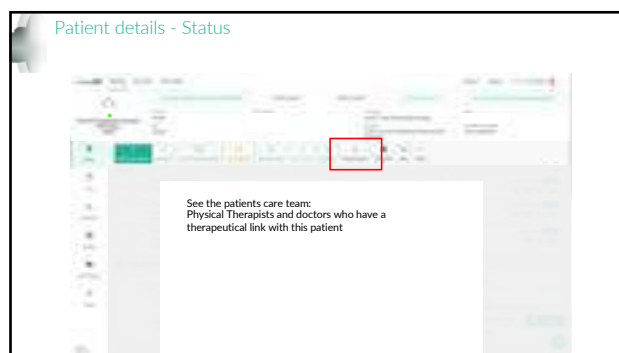70

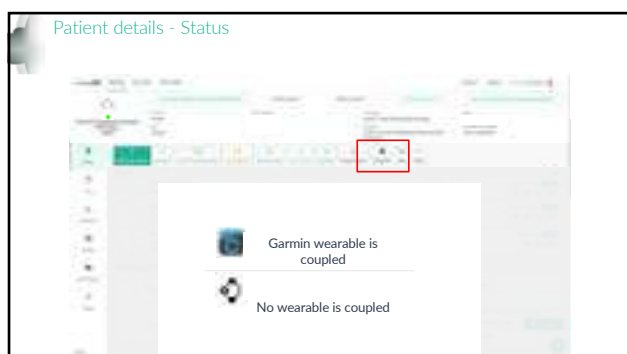71

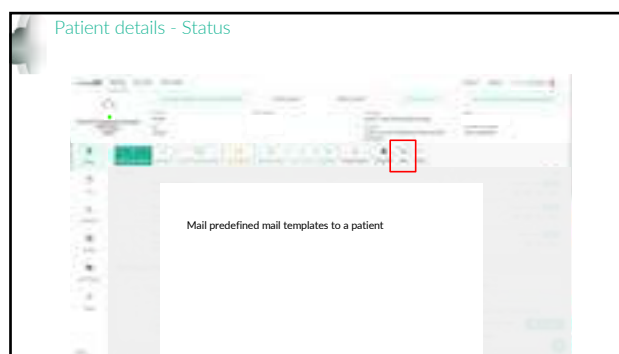72

# Patient details - Log

73

Patient details- Log

- ✓ Multidisciplinary communication
- ✓ Adjusted exercise scheme or step limit
- ✓ Automatic mail reminders
- ✓ Patient called
- ✓ Administrative actions
- ✓ Flags

74

Patient details - Log

When you flag someone, this automatically appears in the log of the patient:

- GREEN: Solved flag
- RED: Unsolved flag for your discipline
- YELLOW: Unsolved flag for other discipline

75

# Patient details - Report

76

Patient details Report

This is a patient/medical professional report used to inform the main milestones and medical scores of the patient, as well as the evolution of the patient during their treatment with moveUP.

77

The report can be downloaded in pdf form by clicking the "Download report" button. This way you can send the report to the patient.

It's also possible to share the link directly to the report, using the button "Copy report link".

78

**Patient details** Report

The proof of Engagement report gathers the dedication of the care team to the patient on the one hand and the adherence of the patient to the system on the other hand.

The purpose of the report is to be able to **prove activity for reimbursement**. Therefore two main variables are taken into account:

1. Time spent by the HCP by being active on the patient specific pages and by (logged) calling to the patient.
2. Actions by the patient in the app subdivided into: answers on daily questionnaires, sent messages and syncing the activity tracker

An export button copies the displayed table to an excel file

79

**Patient details** Proof of engagement report

In this slider you can select the time frame of which you want to see the actions

To find out what each column means, you can click on the question mark next to the name above the table.

80

**Patient details** Proof of engagement report

Below, you can find a weekly/monthly report that summarizes the actions of a given week, for billing purposes.

81

**Patient details**  
-  
**Careplan**  
patient's adapted treatment

82

**Patient details - Care plan** Exercises

83

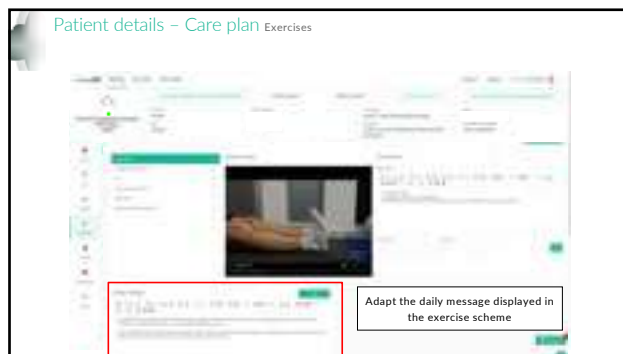
**Patient details - Care plan** Exercises

84


85


86

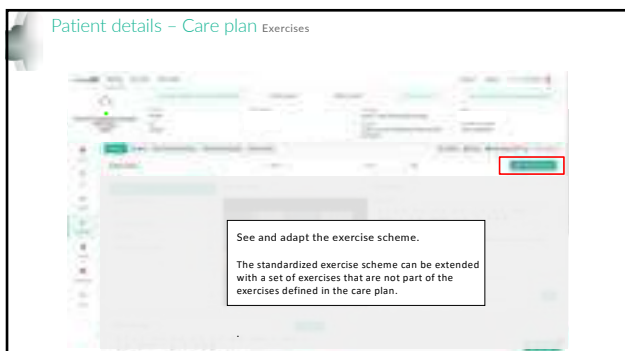
87

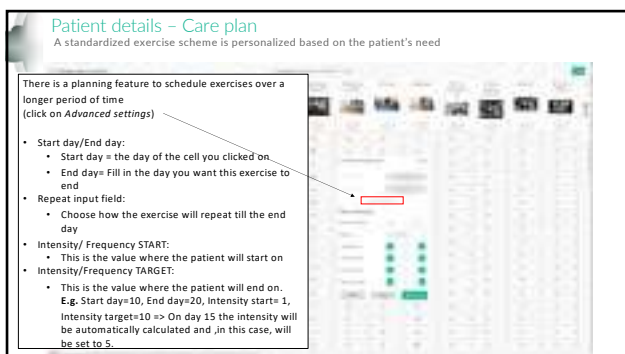88

89

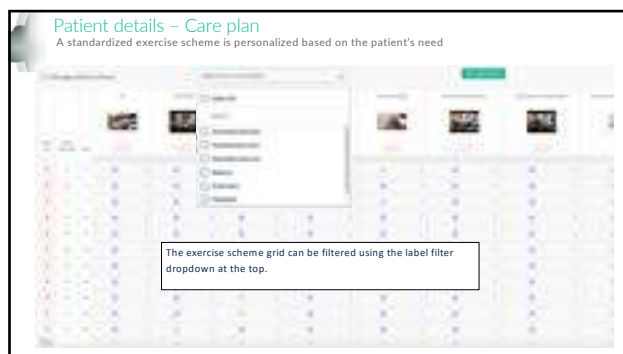90

**Patient details – Care plan**  
A standardized exercise scheme is personalized based on the patient's need

When selecting a label in the dropdown the displayed exercise columns will be filtered according to the selected exercise labels. Following filter options are available

- **Activated exercises:** Only show exercises that are currently active in the patients scheme
- **Standard exercises:** Only show exercises that are part of the care plans core exercise list
- **Extended exercises:** Only show exercises that are part of the care plans extended exercise list
- **List of labels enabled for the care plan (e.g. Balance, Endurance, ...):** Only show exercises matching the selected labels.

91

**Patient details – Care plan Step goal**

92

**Patient details – Care plan Step goal**

Navigate to previous and coming days

93

**Patient details – Care plan Step goal**

See and adapt the step maximum

94

**Patient details – Care plan Step goal**

By clicking on "Save", you can adjust the number of steps for that day (example: day 5)

95

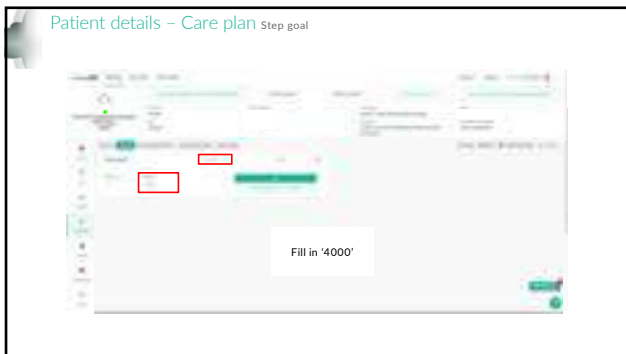
**Patient details – Care plan step goal**

By clicking on "Save & update previous + future days" you can adjust the number of steps for the previous and future days. This way you can change the number of steps for the next 10 days for example.


For example you go 10 days further (in this case you navigate to day 15) and set i.e. step maximum to 4000. Then the number of steps escalate progressively until day 15. (See next slides)

96



97

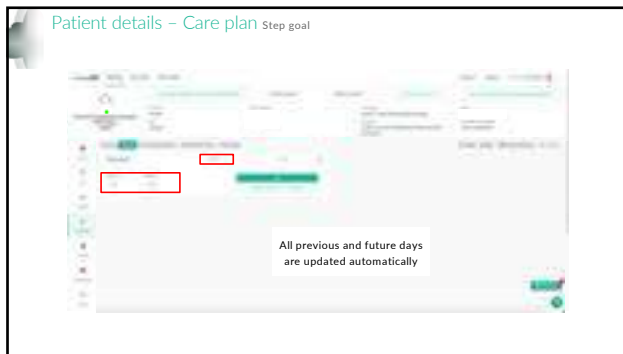
98

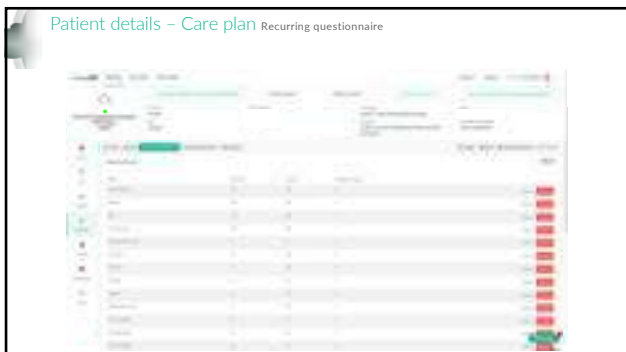
99

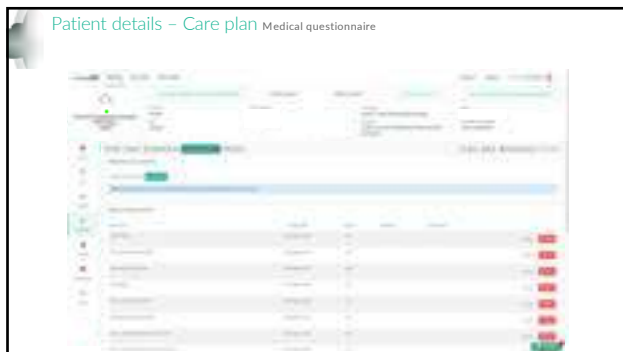100

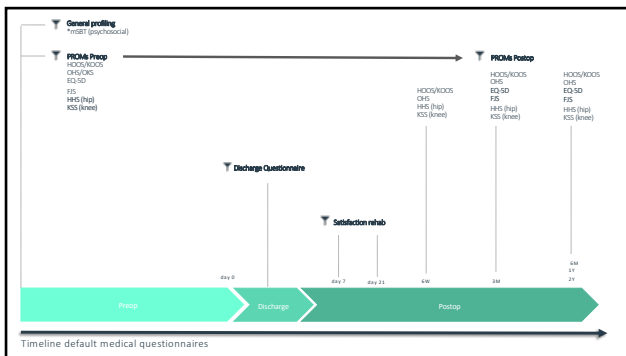
101

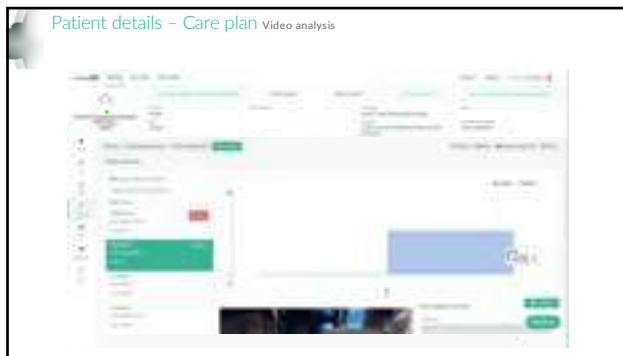
102

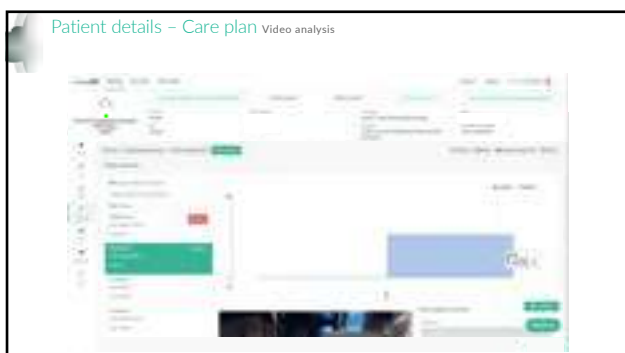
103

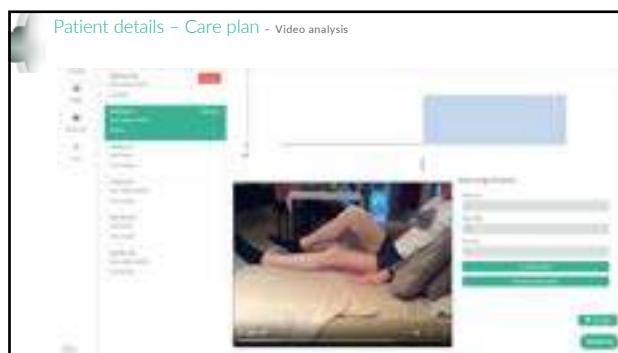
104

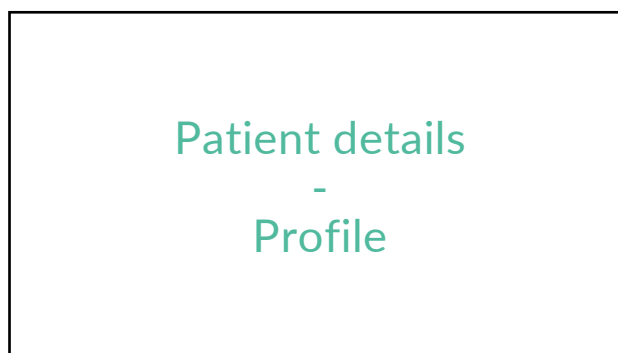
105

106

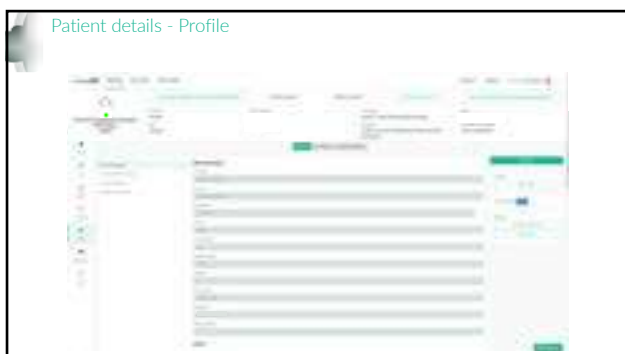107

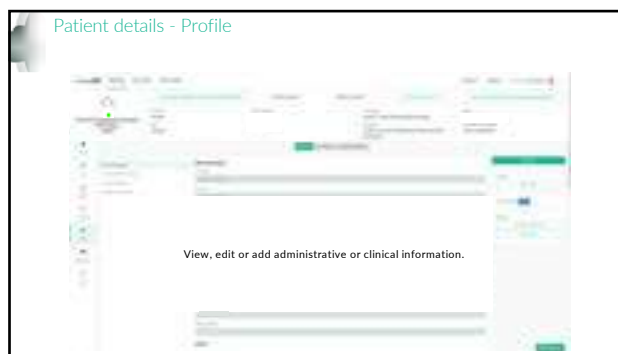108


## Patient details - Documents

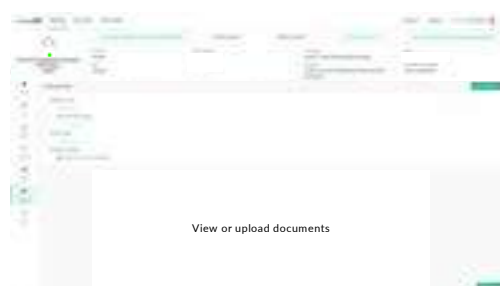109

### Patient details- Documents

110

## Patient details - More

111

### Patient details- More

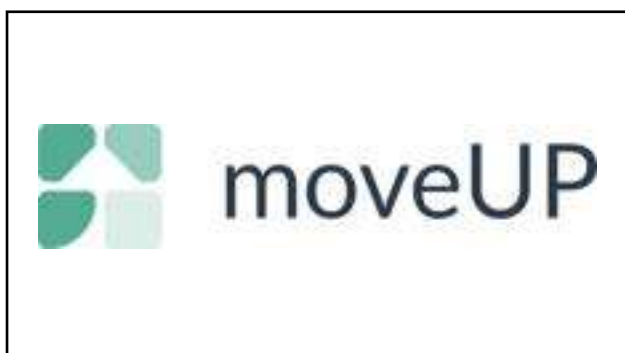
112


moveUP

113


1

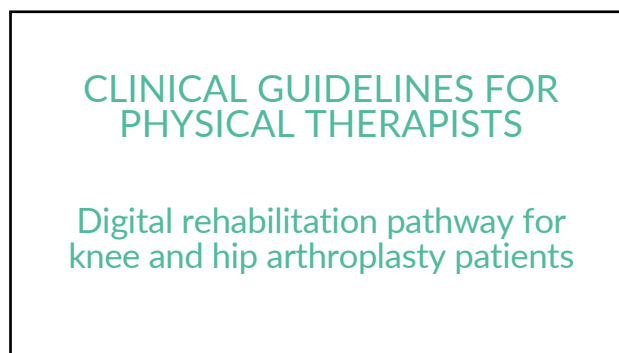
2

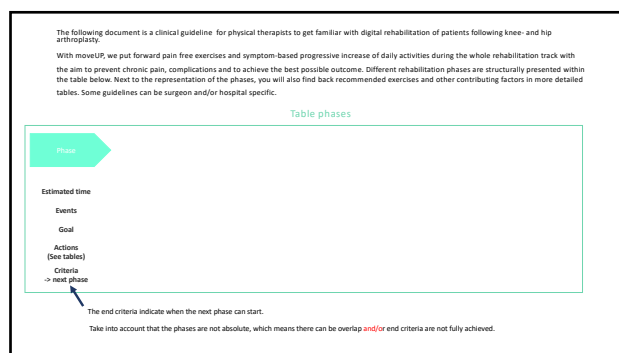
3

4

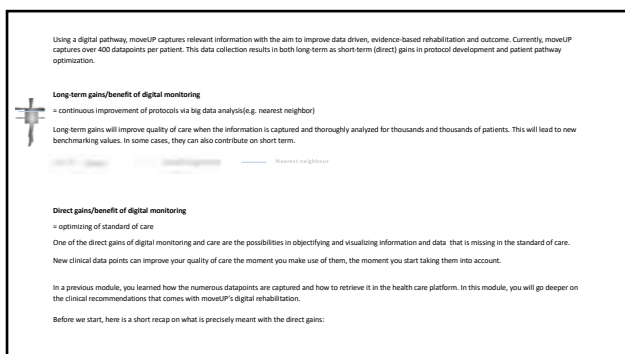5

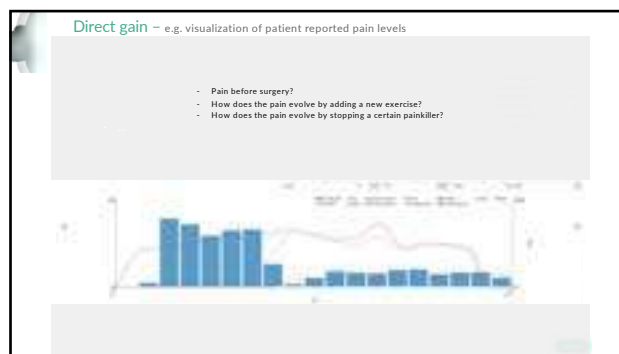6

**Direct gain** - e.g. visualization of patient reported parameters: Joint warmth, swelling

- Patient's reported warmth and swelling of the operated joint can provide usefull insights on the healing proces.
- See below a decrease in patient's reported swelling (green).

7

**Direct gain** - e.g. insight in patient's activity captured by a wrist worn activity tracker

**Insight in postoperative activity levels**

- Which activity level does the patient reach?
- Does the patient gradually increase activity?
- Does the patient follow the advised step maximum?
- ...

**Insight in preoperative activity levels**

8

**Direct gain** - e.g. consider daily reported activities, medication-intake and crutches

**Table overview last 3 days: activities, medication and crutches**

- Check compliance on the prescribed medication  
Questions about medication are escalated to the doctors
- Compliance on the use of walking aids  
Give advice about transition 2 -> 1 -> 0 crutches/walker  
See which activities the patient has already picked up

9

**Escalation to 2nd line doctors**

- Picture of wound
- Alarming or atypical parameters
- High pain levels and possibility to adapt medication
- Non compliance of medication intake
- Question about medication
- Extra motivation
- ...

Flag is automatically reported in patients log:

10

**Direct gain** - Track your patient's range of motion over time

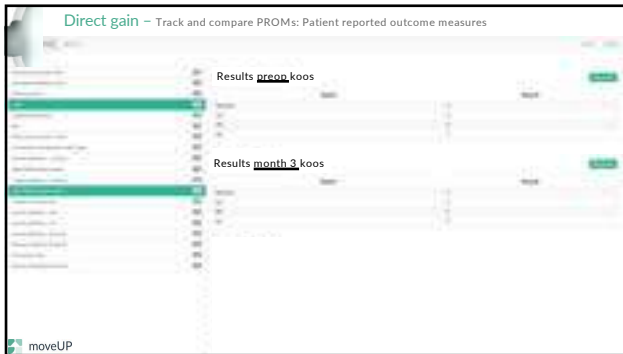
11

**Direct gain** - Evaluate specific PROMs answers

12

13

14

# Knee arthroplasty

Total knee replacement  
Uni knee replacement

15

Preparation	Hospitalization Protection post-surgical knee	Return to daily activity	Return to active lifestyle
<b>Estimated time</b>	Day 0 - 21	Day 7 - 60	Day 95 - ...
<b>Events</b>	Surgery Hospital stay Discharge First week home Removal of staples	First postoperative consult Week 6 PROMs evaluation	Second postoperative consult Month 3 PROMs evaluation Back to work Driving car
<b>Goal</b>	Protection post-surgical knee Prevent complications Pain & symptoms control Regain proprioception Knee movement within pain limits	Gradually decrease in pain <2/10 Gradual stop of medication Gain in pain free basic strength ROM 0-90° week 4 Normalized gait pattern Basic balance / coordination	ROM 0 - 110° week 8 Reach full ROM 0 crutches/walker device Regain functional strength
<b>Actions (See tables)</b>	Educate & support patient Supervised use of crutches/walker Supervise medication intake Provide pain-free exercises Control activity level Check wound Check mobility	Educate & support patient Support transfer 2 crutches > 1/0 crutches Adapt exercise scheme Ask pictures of swelling/haematoma Add non default exercises	Adapt exercise scheme Resume normal progression Add non default exercises Finalize rehabilitation
<b>Criteria -&gt; next phase</b>	Protection & education Pain & symptom control	A minimum of / no pain A minimum feeling of warmth of the knee Painkilling medication stopped or limited to occasional intake of WHO type 1 medication No more NSAIDs	For some /a/ or exercises surgeon approval is advised e.g. driving, biking, lunges...

Phases knee

moveUP

16

Preparation	Hospitalization Protection post-surgical knee	Return to daily activity	Return to active lifestyle
<b>Isometric quadriceps setting</b> - 3 x 10	Ice - 8 x 15 min. Circulation - 8 x 2 min. Heel raises - 3 x 10	Hiq/Knee flexion/extension standing - 3 x 15 Half rounds stationary bike - 2 x 5 min. Hiq/Knee flexion to hip/knee extension - 3 x 10 Hip flexion and knee extension stability exercises on the chair - 2 x 15 Mini squat - 3 x 10 Side toe taps - 3 x 15 Standing march - 3 x 4 min	Stationary bike - 3 x 10 min Squat - 3 x 15 Hip abduction standing - 3 x 15 Step up - 3 x 10 Single leg stance - 3 x 10 Small lunge - 3 x 10 Forward lunge - 3 x 10 Side lunge - 3 x 10 Heeling - 3 x 10

Table knee: Exercises

moveUP

17

Preparation	Hospitalization Protection post-surgical knee	Return to daily activity	Return to active lifestyle
<b>Activity</b>	1. Preop avg < 4000 + Max steps 1000/day 2. Preop avg 4000 - 5000 Max steps 1500/day 3. Preop avg > 5000 + Max steps 2000/day From day 5: + 100 steps/day	Gradual increase until 50-100% preop steps <b>Positive evaluation</b> +100-200 steps / day <b>Negative evaluation</b> a. Max steps on hold b. Decrease in Max steps	Gradual increase until 75% - 100% or > preop steps <b>Positive evaluation</b> +100-200 steps / day <b>Negative evaluation</b> a. Max steps on hold b. Decrease in Max steps Stop PK No NSAID
<b>Medication</b>	PK: WHO 1 - 2 NSAID 2/day - 2 weeks Blood thinning medication	PK: WHO 1 NSAID 1/day - 2 weeks	No NSAID
<b>Mobility</b>	Pain free mobility exercises without forcing	ROM 0° - 90° week 4 Focus on full extension	ROM 0 - 130° week 8 Full ROM
<b>Crutches</b>	2 crutches	2 or 1 crutch	No crutches
<b>Stationary bike</b>	No bike	Stationary bike no resistance No biking	Stationary bike with resistance Biking after surgeon approval
<b>Car</b>		Driving (when feeling safe to do so) *Check the need for surgeon or car insurance approval	

Table knee: Activity - Medication (see addendum) - ROM - ...

moveUP

18

Preparation	Hospitalization Protection post-surgical knee	Return to daily activity	Return to active lifestyle
Activity	High activity >50% prep Disappointed unilateral weightbearing Jumping Running Biking Loading activities or sports	High activity >100% prep Jumping Running Biking "High" loading activities or sports	Jumping Running "High" loading activities or sports → Surgeon dependent
Exercises	Pain provoking exercises Exercises against heavy resistance	Pain provoking exercises Exercises inducing pain after execution	Pain provoking exercises Exercises inducing pain after execution
Mobility	Forcing towards flexion / extension	Forcing towards flexion / extension	Forcing towards flexion / extension
Crutches	No use of crutches	No use of crutch(es) for long distances	
Pain	Pain provoking movements or exercises	Pain provoking movements or exercises	Pain provoking movements or exercises
Sleep	Rest/sleep with a fixed knee (e.g. pillow under operated joint)	Rest/sleep with a fixed knee (e.g. pillow under operated joint)	Rest/sleep with a fixed knee (e.g. pillow under operated joint)

Table knee: Don'ts – contra-indications

19

### Hospitalization & Protection post surgical knee replacement

is a first postoperative phase.

- Focus on protection of the post-surgical knee and education of the patient on how to do so: the patient should take in prescribed medication, use crutches, put ice and remove and reposition of the operated leg.
- Symptom control: Aim to get pain, warmth and swelling under control as soon as possible. Persisting swelling, pain and warmth will prohibit a smooth rehab. So, control and prevention of these parameters is key.
- Low load mobility: Motivate the patient to execute low load mobility exercises frequently. Changing position regularly will reduce stiffness.

**How?**

**Exercises:** See the exercises in the table. The priority are circulatory and light mobility exercises. Circulatory exercises help drain postoperative blood and swelling and prevent thrombosis. Quadriceps setting stimulates proprioception and maintenance of muscle tone. These exercises are automatically active in the exercise scheme. Exercises against resistance or those which provoke pain, swelling, stiffness and warmth have to be avoided, including those during the first 48 hours.

**Activity:** Guidance in activity is provided by a daily, personalized step maximum. (see table for reference)

**The chat:** is used to educate, motivate the patient, address concerns, answer questions and clarify therapeutic actions.

**The use of crutches/walkers:** should be supervised. Walking aids are important to prevent falling and to lower the load on the knee so soft tissue get time to heal.

**Gait pattern:** Check on heel-to-toe gait pattern and on neutral foot positioning during video analysis.

**Knee range of motion:** is recorded by weekly video inquiries. The first mobility check is done at day 2, this first check is a reference point to track improvement. During RDM analysis, also look for movement flunty (e.g. signs of fear or discomfort), early soft knee symptoms and compensation. Reaching full extension without overpressure has high priority in the beginning of the rehabilitation.

**What if...**

- If a patient doesn't manage to get the pain under control? → In the first 48 hours can be triggered to give advice on medication. Most patients are advised to start with third step 2 paracetamol. Step 2 paracetamol can temporarily help high pain persisting patients.
- If a patient reports a 'red flag'? Always flag 2nd line doctors in cases of a bleeding wound, picture of wounds, genetic problems, reports of fever, postop nausea, visit to general practitioner, medication advice or any other sign of complication.
- If regarding the wound care: a picture of the wound after removal of electroclips should be asked. Most wounds are stapled. The staples are removed starting from 18 days to 21 days postoperatively, until staples are removed a patch should cover the wound. In most cases the patch should be changed until this day (unless specified by surgeon).

20

### Common actions Hospitalization & Protection post surgical knee

Exercises (see table)	Messages	Check
Default scheme	Questions	Therapy Compliance
High activity → no more need for circulatory exercises	<ul style="list-style-type: none"> <li>How did surgery go?</li> <li>Medicall when home?</li> <li>How are the exercises going?</li> </ul>	
	Feedback	Picture Video RDM Early RDM (progression RDM not necessary) Swelling Warmth
	Stimulate	Use of crutches Medication intake Daily activity and step max
	<ul style="list-style-type: none"> <li>To remove</li> <li>Adherence of step limit</li> <li>Adherence of use of walking aids</li> <li>Swelling &amp; stiffness</li> <li>To not provoke pain</li> <li>To take medication as prescribed</li> </ul>	

Actions Hospitalisation & protection post-surgical knee

21

### Protection post-surgical knee

Protection & education  
Pain & symptom control

Return to daily activity

#### CRITERIA

- The patient is compliant to the postoperative rehabilitation guidelines.
- The patient found a balance between low load, regular, basic exercises and rest to decrease/control post-operative pain and swelling.
- Transition to the next phase is advised when the patient has control over the postoperative complaints.

Transition phase protection post-surgical knee → Return to daily activity

22

### Return to daily activity:

is a second postoperative phase.

- Pursue a continuous decrease in symptoms and pain while reducing WHO type 1-2 medication to a minimum next to increasing load by new exercises and allowing higher activity.
- Support the patient finding the right balance between rest and activity.

**How?**

**Exercises:** Find back the exercises in the table. Personalize (advance or delay) them in the exercise scheme based on the daily parameters. A maximum of 4 exercises / day is advised. With movement, we strive to control and gain the exercises during the whole rehabilitation. Therefore, when returning exercises from previous phase and/or adding new exercises, a positive emotion should still be noticeable. Performing any of the exercises should not provoke pain, induce warmth and/or swelling, ting and reversion of the operated leg is still advised to give these parameters fully under control.

**Activity:** Guidance in activity is still provided by a daily, personalized step maximum.

- During a positive evolution, the patient is allowed to increase activity by 100-200 steps/day.
- During a negative evolution, the patient should remain on its activity level or have more rest (decrease in activity).
- Avoid pain activity during the whole rehabilitation (see table for reference).

**The messaging function:** is still commonly used to educate the patient, address concerns, answer questions and clarify therapeutic actions.

**Gait pattern:** Check on heel-to-toe gait pattern and on neutral foot positioning during video analysis. Check on compensation.

**The use of crutches:** Transition from 2 to 1 and to no crutches should be motivated when patient is safe.

**Knee range of motion:** is very important to see continuous improvement in the active ROM. A minimum of 90° active flexion should be achieved before the end of week 4. If not, contact the treating surgeon and provide him/her with this information. Full knee extension is stimulated daily by knee extension posture aided with instructions to also focus on full extension while walking and standing. During active or passive flexion mobilization, it is not advised to force and/or to go into the pain zone.

Besides daily step collection and visualization, during this phase, patient's evolution is tracked by postoperative PROM collection. Compare the pre to postoperative results and give feedback to the patient. In most cases reassurance of time of progression is sufficient. At the end of this phase, the biggest gains have been made but naturally, patients should still experience improvements. Most patients perceive improvement over one year post-op.

23

### Common actions Return to daily activity

Exercises (see table)	Messages	Check
Adapt exercise scheme: advance or delay phase 2 exercises	Questions	Reaction on adaptations in the exercise scheme wound
	Feedback	
Basic strength Proprioception Adaptation	<ul style="list-style-type: none"> <li>Gradual increase in exercises and activity</li> <li>Paracetamol medication lowered to occasional use of NSAIDs type 3 medication</li> <li>Reassurance and supporting when there are ups and downs</li> <li>PROM results</li> <li>Trend 2 → 2 Crutches</li> <li>Heading towards excellent result</li> </ul>	PROM: prep vs week 6
Delay exercises which increases pain, warmth, swelling	Stimulate	Knee ROM (RT) before week 4 Gait pattern
	<ul style="list-style-type: none"> <li>Gradual build up basic strength and ADL without pain</li> <li>Motivate ROM in ADL</li> </ul>	

Actions Return to daily activity

24

Return to daily activity

- ✓ Minimum of pain
- ✓ Painkilling stopped or limited use

Return to active lifestyle

**CRITERIA**

- ✓ A minimum of / no pain
- ✓ A minimum feeling of warmth of the knee
- ✓ Painkilling medication stopped or limited to occasional intake of WHO type 1 medication
- ✓ No more NSAIDs
- ✓ for some /a/ or exercises surgeon approval e.g. driving, biking, lunges, etc.

Transition Return to daily activity -> Return to active lifestyle

25

### Return to active lifestyle

in a 2<sup>nd</sup> postoperative phase.

- Focus on functional strengthening exercises.

**Exercises:** Find back the exercises in the table. Gradually add the exercises one by one, while closely following the patient's parameters in the graph or table. Do not forget to remove exercises from exercises phase. A maximum of 4 exercises / day is advised.

**Before starting the functional strengthening exercises, pain, warmth and swelling are stable and at a low level. Medication is limited or stopped. During and after the functional strengthening exercises, these parameters should stay stable and/or at a minimum. There should be no need to increase medication intake.**

**During this phase, leg can still be swollen at the end of a busy day.**

**Activity:** Stimulate a gradual increase in activity. Give recommendations about physical activity in general.

**The messaging function:** During this phase, reassurance of normal progression is often needed. Point out the fact that most patients reach a positive evolution up to 3 year after surgery.

**Gait pattern:** Check on heel-toe gait pattern and on neutral foot positioning during video analysis. Check on compensation.

**Range of motion:** While swelling and pain are reduced to a minimum, mobility should have evolved to 0 - 110° at around week 8-10. Consider that the achievable ROM greatly depends on the patient's preop mobility.

**Finalize rehabilitation** by checking if all expectations are met. You can check patient's satisfaction and outcome at the month 3 PROM collection.

26

### Common actions Return to active lifestyle

Exercises	Messages	Check
Adapt exercise scheme: advance or delay certain exercises	<b>Questions</b> <ul style="list-style-type: none"> <li>• How was the 3m course?</li> <li>• How far from your goals/expectations?</li> <li>• Any movements/activities which are still difficult/painful?</li> </ul>	Evaluate response on higher load
Functional strength	<b>Feedback</b> <ul style="list-style-type: none"> <li>• Right timing 1 -&gt; 0 crutches</li> <li>• PROMs 3 month</li> </ul> <b>Stimulate</b> <ul style="list-style-type: none"> <li>• Back to work</li> <li>• Full ACL update</li> </ul>	Videos ROM Videos gait Patient satisfaction
		<ul style="list-style-type: none"> <li>• Functional ROM (110° + full extension)</li> <li>• Normal gait pattern</li> </ul>

Actions Return to active lifestyle

27

### APPROACH knee arthroplasty

**Preop day -4 to day -1:** Functional strength & mobility: Abduction, quadriceps setting, isometric gluteus, stationary bike, knee extension posture

**Surgery - hospital stay: day 0 to day 5-6:** Preop hospital (DCLP) starts  
Elevation and circulation

**Symptom control, proprioception and mobility: day 0 to day 21:**  
 - Ice application: Rest and Regular Low Load Movement  
 - Gait correction: Quadriceps setting

**Basic strength and mobility: day 7 to day 60:**  
 - Functional ROM (Swim/bathing and back-up of ACL)  
 - Basic mobility: Heeling, single foot, extension posture  
 - Basic strength: Side toe taps, mini squat...

**Functional strength from day 35:**  
 - Low load to higher load  
 - Functional strength (single leg, stairs, jumps, lunges, bridging)

**Timeline:** Day 0 (Surgery) - Day 7-10 (Return to daily activity) - Day 35 (Return to active lifestyle)

28

## Hip arthroplasty

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- DAA
- ALA

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29

Preparation	Hospitalization Protection post-surgical hip	Return to daily activity	Return to active lifestyle
<b>Estimated time</b>	Day 0 - 21	Day 10 - 60	Day 35 and on...
<b>Events</b>	Surgery Hospital stay Discharge First week home Removal of staples	First postoperative consult	Second postoperative consult Back to work Driving car
<b>Goal</b>	Protection post-surgical hip Prevent complications Pain & symptoms control Proprioception	Gradually have no more pain <2/10 Gradual stop medication Basic strength Normalized gait pattern Prevent/minimize groin pain Basis balance / coordination	Full ROM 0 crutches Functional strength
<b>Actions (See tables)</b>	Educate & support patient Supervise use of crutches/walker Supervise medication intake Provide pain free exercises Control activity level Check wound Gait check	Educate & support patient Advise 2 crutches -> 1/0 crutches Adapt exercise scheme	
<b>Criteria next phase</b>	Pain & symptom control Protection & education	A minimum of / no pain Painkilling medication stopped or limited to occasional intake of WHO type 1 medication No more NSAIDs	For some /a/ or exercises surgeon approval is advised e.g. driving, biking, lunges

Phases hip

30

Prep	Hospitalization Protection post-surgical hip	Return to daily activity	Return to active lifestyle
Ice - 4 x 15 min. Circulation - 4 x 2 min. Heart rates - 3x10	Isometric gluteus settings - 3 x 15 Isometric quadriceps settings - 3 x 10 Side toe taps - 3 x 15 Triple flexion lying - 3 x 10	Half rounds stationary bike - 2 x 5 min External rotation - 3 x 15 Hip abduction standing - 3 x 15 Hip flexion to hip extension - 3 x 10 Lower limb standing - 3 x 15 Hip flexion and knee extension mobility exercise on the stairs - 3 x 15 Mini squat - 3 x 15 Balancing - Single leg stance - 3 x 10 Standing march - 3 x 2 min.	Stationary bike - 3 x 10 min Squat - 3 x 15 Single leg stance - knee off - 3 x 15 Small lunge - 3 x 10 Forward lunge - 3 x 10 Side lunge - 3 x 10 Bringing - 3 x 10 Step up - 3 x 10

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31

## Differences in surgical approach

Anterior protocol	Posterior protocol
<p><b>More prone for hip flexor irritation</b></p> <ul style="list-style-type: none"> <li>Avoid extension past 20 degrees</li> <li>Avoid external rotation past 50 degrees</li> </ul> <p><b>Anterior approach in daily exercise scheme:</b></p> <p>Less focus on hip flexor exercises in acute phase, e.g.:</p> <ul style="list-style-type: none"> <li>Triple flexion (with assistance) 3x8: d7-d11-d15-d19 ...</li> <li>Hip flexion to extension 3x8: d9-d13-d17-d21 ...</li> </ul> <p>More focus on abductor and glute exercises, e.g.:</p> <ul style="list-style-type: none"> <li>Side toe taps (3x15): d5-d7-d9</li> <li>Abduction (3x15): d11-d13-d15...</li> <li>Mini squat (3x10): d10-d12-d14...</li> </ul>	<p><b>More prone for abductor weakness and/or irritation</b></p> <ul style="list-style-type: none"> <li>Avoid hip flexion past 90 degrees</li> <li>Avoid external rotation past neutral</li> <li>Avoid abduction past neutral</li> </ul> <p><b>Posterior approach in daily exercise scheme:</b></p> <p>More focus on hip flexor exercises, e.g.:</p> <ul style="list-style-type: none"> <li>Triple flexion (without assistance) 3x10: d5-d7-d9-d11 ...</li> <li>Hip flexion to extension 3x10: d6-d8-d10-d12 ...</li> </ul> <p>Less focus on abductor and glute exercises in acute phase, e.g.:</p> <ul style="list-style-type: none"> <li>Side toe taps (3x10): d7-d11-d13</li> <li>Abduction (3x10): d15-d17-d19 ...</li> <li>Mini squat (3x10): d14-d16-d18...</li> </ul>

32

Preparation	Hospitalization Protection post-surgical hip	Return to daily activity	Return to active lifestyle
<p><b>Activity</b></p> <ol style="list-style-type: none"> <li>Preop avg &lt; 4000:           <ul style="list-style-type: none"> <li>Max steps 1000/day</li> </ul> </li> <li>Preop avg 4000-10000:           <ul style="list-style-type: none"> <li>Max steps 1500/day</li> </ul> </li> <li>Preop avg &gt; 10000:           <ul style="list-style-type: none"> <li>Max steps 2000/day</li> </ul> </li> </ol> <p>From day 5: + 100 steps/day</p> <p><b>Medication</b></p> <p>PK WHO 1-2 NSAID 2/day - 2 weeks Blood thinning medication</p> <p><b>Mobility</b></p> <p>Pain free mobility exercises without forcing</p> <p><b>Crutches</b></p> <p>2 crutches</p> <p><b>Stationary</b></p> <p>No stationary bike</p> <p><b>Biking</b></p> <p>Car</p>	<p>Gradual increase until 50-100% preop steps</p> <p><b>Positive evolution:</b> +100-200 steps / day</p> <p><b>Negative evolution:</b> a. Max steps on hold b. Decrease in Max steps</p> <p>PK WHO 1 NSAID 1/day - 2 weeks</p> <p>Pain free mobility exercises without forcing</p> <p>2 or 1 crutch</p> <p>Stationary bike no resistance</p> <p>Driving (when feeling safe to do so) *Check the need for surgeon or car insurance approval</p>	<p>Gradual increase until 75%- 100% or &gt; preop steps</p> <p><b>Positive evolution:</b> +200-200 steps / day</p> <p><b>Negative evolution:</b> a. Max steps on hold b. Decrease in Max steps</p> <p>Stop PK No NSAID</p> <p>Pain free mobility exercises without forcing</p> <p>No crutches</p> <p>Stationary with resistance</p> <p>Biking after surgeon approval</p>	<p>Jumping Running "high" loading activities or sports → Surgeon dependent</p> <p>Pain provoking exercises Exercises inducing pain after execution</p> <p><b>Anterior don'ts:</b> Repetitive hip flexion</p> <p>a. Excessive ranges b. <b>POSTERIOR don'ts:</b> Hip flexion &gt;90° Extension &gt; neutral Adduction &gt; midline</p> <p>No use of crutches with dysfunctional (limping) gait pattern or for long distances</p> <p>No use of crutches with dysfunctional (limping) gait pattern</p> <p>Pain provoking movements or exercises</p>

Table hip: Activity - Medication (\*see addendum) - Crutches -

33

Preparation	Hospitalization Protection post-surgical hip	Return to daily activity	Return to active lifestyle
<p><b>Activity</b></p> <p>High activity &gt; 50% preop</p> <p>Biking</p> <p><b>Exercises</b></p> <p>Pain provoking exercises Exercises against heavy resistance Straight leg raises (SLR)</p> <p><b>Anterior don'ts:</b> Repetitive hip flexion</p> <p><b>Mobility</b></p> <p>a. Excessive ranges b. <b>POSTERIOR don'ts:</b> Hip flexion &gt;90° Extension &gt; neutral Adduction &gt; midline</p> <p><b>Crutches</b></p> <p>No use of crutches</p> <p><b>Pain</b></p> <p>Pain provoking movements or exercises</p> <p><b>Sleep</b></p> <p>Sleeping on non-op and/or on side (surgeon dependent)</p>	<p>High activity &gt; 100% preop</p> <p>Biking</p> <p>Pain provoking exercises Exercises inducing pain after execution</p> <p><b>Anterior don'ts:</b> Repetitive hip flexion</p> <p>a. Excessive ranges b. <b>POSTERIOR don'ts:</b> Hip flexion &gt;90° Extension &gt; neutral Adduction &gt; midline</p> <p>No use of crutches with dysfunctional (limping) gait pattern or for long distances</p> <p>Pain provoking movements or exercises</p> <p>Sleeping on non-op and/or on side (surgeon dependent)</p>	<p>Jumping Running "high" loading activities or sports → Surgeon dependent</p> <p>Pain provoking exercises Exercises inducing pain after execution</p> <p><b>Anterior don'ts:</b> Repetitive hip flexion</p> <p>a. Painful excessive ranges</p> <p>No use of crutches with dysfunctional (limping) gait pattern</p> <p>Pain provoking movements or exercises</p>	<p><b>Table hip: Don'ts - Contra-indications</b></p>

34

## Hospitalization & Protection post surgical hip

In a first postoperative phase,

- Focus on protection of the post surgical hip and education of the patient on how to do so: the patient should take its prescribed medication, use crutches, put ice and elevate rest and elevation of the operated leg
- Symptom control: Aim to get pain, warmth and swelling under control
- Low load mobility: Low load and light mobility exercises are advised, motivating the patient to move his/her leg regularly without loading and regularly changing position
- Attention:
  - Hip anterior approach: See table don'ts
  - Hip posterior approach: See table don'ts

How?

**Exercise:** See the exercises in the table. Focus is on circulatory and light mobility exercises. Circulatory exercises help drain postoperative blood and swelling and prevent thrombosis. Warming and stretching exercises help maintain range of motion and muscle tone. These exercises are automatically active in the exercise scheme. Exercise against resistance or that provoke pain, swelling, stiffness and warmth are to be avoided, including during a knee-to-face stretch. Guidance in activity is provided by a KPI, personalized step maximum. (see table for reference)

The message is used to inform, motivate the patient, address concerns and answer questions.

The use of crutches/walkers should be supervised. Walking aids are important to prevent falling and lower the load on the hip so each visit get time to hear and subsidence is avoided.

**Wound pattern:** Check on heel-toe gait pattern and an overall floor posturing during video analysis.

What to do...

- A patient doesn't manage to get his pain under control? And the doctor can be triggered to give advice on medication. Most hip patients are advised to start with 1x100mg 3x paracetamol. Step 2 paracetamol can temporarily help high pain perceiving patients.
- A patient reports a "red flag"? Always flag 2nd line doctors in cases of a bleeding wound, picture of wound, gastric problems, reports of fever, postop nausea, visit to general practitioner, medication advice.

1) Regarding the wound care a picture of the wound after removal of stitches/staples should be asked. Most wounds are stapled. The staples are removed starting from 24 days to 21 days postoperatively. Small staples are removed a patch above over the wound in most cases the patch should not be changed until the day (unless specifically noted to change). Some wounds are

35

## Common actions Hospitalization & Protection post surgical hip

Exercises	Messages	Check
<p>Default scheme</p> <p>High activity &gt; no more need for circulatory exercises</p>	<p><b>Questions</b></p> <ul style="list-style-type: none"> <li>How did surgery go?</li> <li>Message when home</li> <li>Removal staples</li> <li>How are the exercises going?</li> </ul> <p><b>Feedback</b></p> <ul style="list-style-type: none"> <li>Pain is under control</li> <li>You have difficulties at night</li> <li>Warmth, stiffness, swelling</li> <li>Transfer from 2 -&gt; 1 crutch: safe and pain free?</li> </ul> <p><b>Stimulate</b></p> <ul style="list-style-type: none"> <li>To move</li> <li>Adherence of step limit</li> <li>Adherence of use of crutches</li> <li>Long &amp; elevation</li> </ul>	<p>Therapy Compliance</p> <p>Picture hematoma Swelling Warmth</p> <p>Use of crutches Medication intake Daily activity and step max</p>

Actions Hospitalization & Protection post-surgical hip

36

Protection post-surgical hip → Return to daily activity

- ✓ Protection & education
- ✓ Pain & symptom control

### CRITERIA

- ✓ The patient is compliant to the postoperative rehabilitation guidelines.
- ✓ The patient found a balance between low load, regular, basic exercises and rest to decrease/control post-operative pain and swelling.
- ✓ Transition to the next phase is advised when the patient has control over its postoperative complaints.

Transition protection post-surgical hip -> Return to daily activity

37

### Return to daily activity

In a second postoperative phase,

- Pursue a continuous decrease in symptoms and pain while reducing WHO type 1-2 medication to a minimum, need to increasing load by new exercises and allowing higher activity.
- Now having a clear overview on the patient's evolution during the first postoperative period, support the patient finding the right balance between rest and activity.

• **Activities:**

- Hip anterior approach: See table doc/1
- Hip posterior approach: See table doc/1

How?

**Exercise:** Find back the exercises in the table. Personalize (advance or delay) them in the exercise scheme based on the daily parameters. With in case, we believe in controlled and passive exercises during the whole rehabilitation. Therefore, adding new exercises, a positive evolution should still be noticeable and performing the exercises should not provoke pain, induce warmth and/or swelling. Stop and reassess if the operating leg is still swollen to get these parameters fully under control.

• **Hip Anterior:** Focus on hip abductors, provoke anterior region less.

- Groin pain is often seen after hip arthroscopy, temporary, avoid exercises which provoke groin pain.

**Activity:** Guidance in activity is still provided by a daily, personalized step maximum.

- During a positive evolution, the patient is allowed to increase activity by 100-200 steps/day.
- During a negative evolution, the patient should remain on its activity level or have more rest (decrease in activity).
- Avoid pain activity during the whole rehabilitation (see table for reference).

**The messaging function:** is still commonly used to motivate the patient, solve concerns and answer questions.

**The use of crutches:** Transition from 2 to 1 and to no crutches should be motivated as pain free and safe.

**Gait pattern:** Check on heel-toe gait pattern and on neutral foot positioning during video analysis. Check on abductor strength or compensations.

Under daily data collection and visualization, during this phase, patient's evolution is tracked by postoperative PROM collection. Comparison pre to post can be made. Interpret the results and give feedback to the patient. In most cases reassurance of normal progression is sufficient. Of course there is still room for improvement there most patients still perceive evolution until 1 year postop.

38

### Common actions Return to daily activity

Exercises	Messages	Check
<p>Adapt exercise scheme: Advance or delay phase 2 exercises</p> <p>Basic strength Preparation Mobilization</p> <p>Delay exercises which increases pain, stiffness, swelling</p>	<p><b>Questions</b></p> <ul style="list-style-type: none"> <li>• How was first postoperative consult?</li> </ul> <p><b>Feedback</b></p> <ul style="list-style-type: none"> <li>• Gradual increase in exercises and activity</li> <li>• Paracetamol medication (avoided to occasional use of WHO type 1 medication before starting next level exercises)</li> <li>• Reassurance and supporting when there are ups and downs</li> <li>• PROMs results</li> <li>• Timing 2 =&gt; 1/3 crutches</li> <li>• Heading towards excellent result</li> </ul> <p><b>Stimulate</b></p> <ul style="list-style-type: none"> <li>• Gradual build up basic strength and ADL without pain</li> <li>• Motivate RCM in ADL</li> </ul>	<p><b>Check</b></p> <ul style="list-style-type: none"> <li>• Reaction on adaptations in the exercise scheme</li> <li>• Wound</li> <li>• Gradual increase in activity/load with decrease of symptoms and pain</li> <li>• PROMs: prep vs week 6</li> <li>• Gait pattern</li> </ul>

Actions Return to daily activity

39

Return to daily activity → Return to active lifestyle

- ✓ Minimum of pain
- ✓ Painkilling stopped or limited use

### CRITERIA

- ✓ A minimum of / no pain
- ✓ A minimum feeling of stiffness & swelling of the hip
- ✓ Painkilling medication stopped or limited to occasional intake of WHO type 1 medication
- ✓ No more NSAIDs
- ✓ for some /a/ or exercises surgeon approval e.g. driving, biking, lunges, etc.

Transition Return to daily activity -> Return to active lifestyle

40

### Return to active lifestyle

In a 3rd postoperative phase,

- focus on functional strengthening exercises.

**Exercises:** Gradually add the exercises one by one, while closely following the patient's parameters in the graph or table.

Before starting the functional strengthening exercises, pain, warmth and swelling are stable and at a minimum. Medication is limited or stopped. During and after the functional strengthening, these parameters should remain stable and/or at a minimum. There should be no more need for (extra) medication also.

During this phase, icing can still be useful at the end of a busy day.

**Activity:** Stimulate a gradual increase in activity.

**The messaging function:** During this phase, reassurance of normal progression is often needed. Point out the fact that most patients notice a positive evolution up till 1 year after surgery.

**Gait pattern:** Check on heel-toe gait pattern and on neutral foot positioning during video analysis. Check on abductor strength or compensations.

**Finalize rehabilitation** by checking if all expectations are met. You can check patient's satisfaction and outcome at the month 3 PROM collection.

41

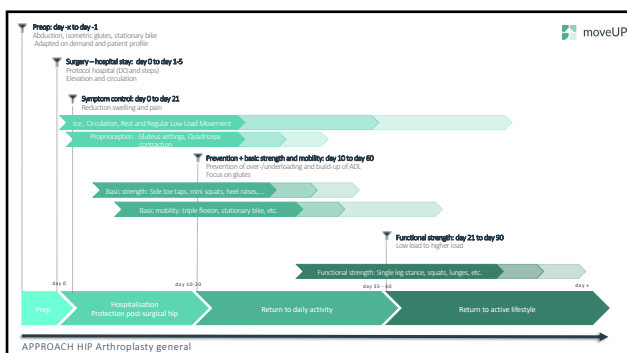
### Common actions Return to active lifestyle

Exercises	Messages	Check
<p>Functional strengthening</p>	<p><b>Questions</b></p> <ul style="list-style-type: none"> <li>• How was first consult?</li> <li>• How far from your goal/expectations?</li> <li>• Any movement/activities which are still difficult/painful?</li> </ul> <p><b>Feedback</b></p> <ul style="list-style-type: none"> <li>• Right timing 1 =&gt; 0 crutches</li> <li>• PROMs 3 month</li> </ul> <p><b>Stimulate</b></p> <ul style="list-style-type: none"> <li>• Back to work</li> <li>• Full ADL uptake</li> </ul>	<p><b>Check</b></p> <ul style="list-style-type: none"> <li>• Video gait</li> <li>• Patient satisfaction</li> <li>• Functional RCM</li> <li>• Normal gait pattern</li> </ul>


Actions Return to active lifestyle

42



43

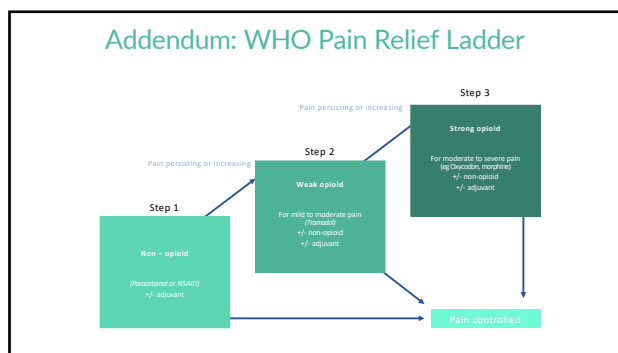44

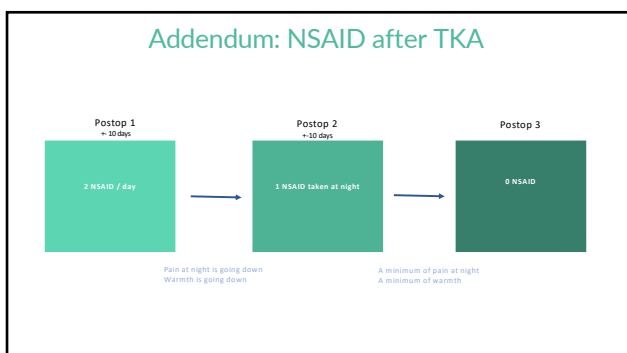45

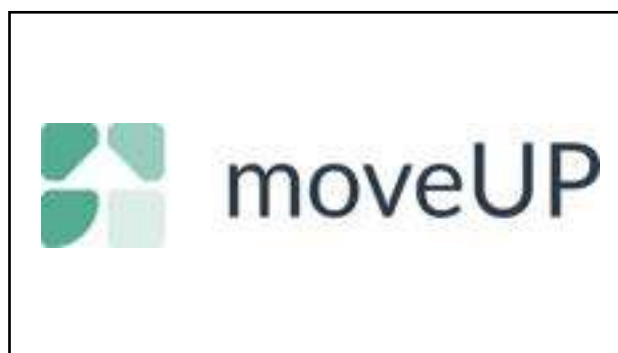46

## How to onboard a new patient on the moveUP platform?

- ✔ Onboarding is the process of registering and starting a new patient in the moveUP platform. After registering a patient in the platform, an invitation mail can be send. In this mail, the patient will receive all the needed information to install and activate the moveUP application.

## 1. Create a new patient in the onboarding functionality

- Surf to: <https://patient-onboarding-admin.moveup.care>.
- Fill out your moveUP login credentials and click “Login”.
- Click “New patient”.

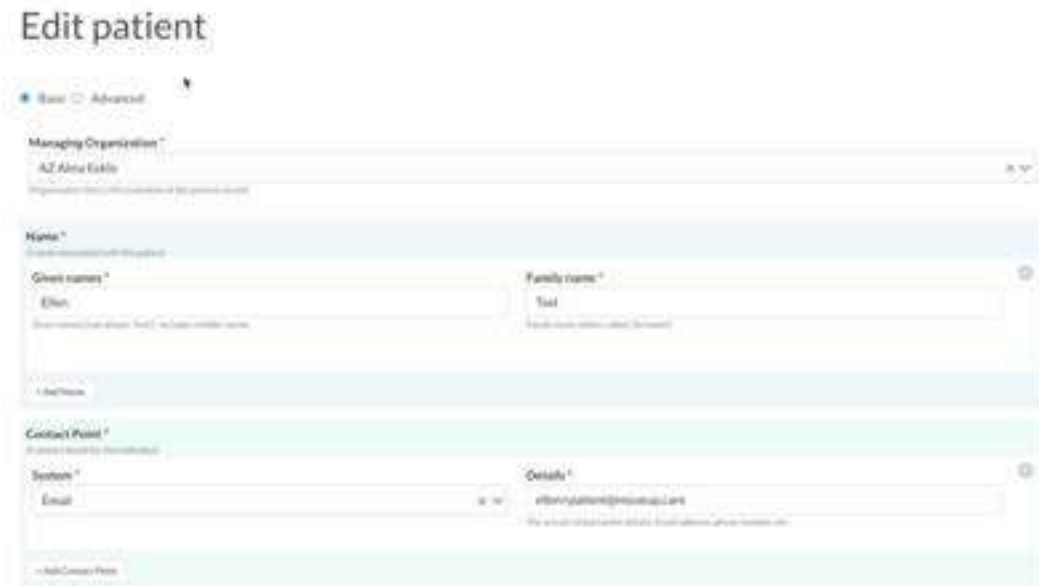
*Create new patient*


## 2. Fill out the patient's details

- Fill out the patient's details.
- Click 'save'.

*Edit patient details*

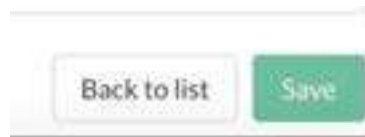
*Save patient details*

- ⓘ Only one account can be coupled to an e-mail address. It is not possible to reuse an e-mail address that is already known within the moveUP database, e.g.: reusing the same e-mail address for the account of a partner or parent.

### 3. Add a new care plan to the patient

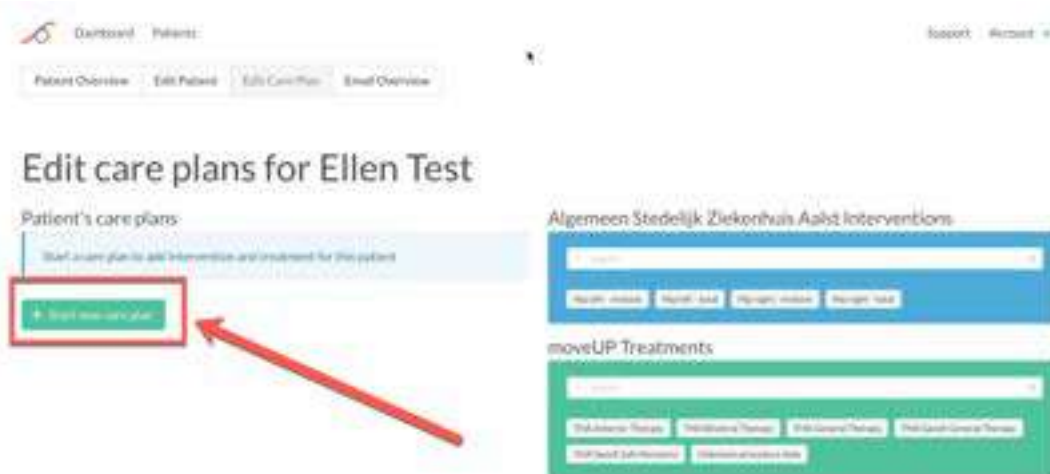
- ✓ A care plan is a crucial part of the digital care journey of your patient. A care plan determines what will be visible for your patient in the moveUP application, e.g.: type of exercises, information modules, message functionality, etc. A care plan usually consists out of an intervention, a moveUP treatment and (a) care team(s).

- The “Edit Care Plan” tab will automatically open after entering patient’s details.
- Click “Start new care plan”.

*Create a new care plan*

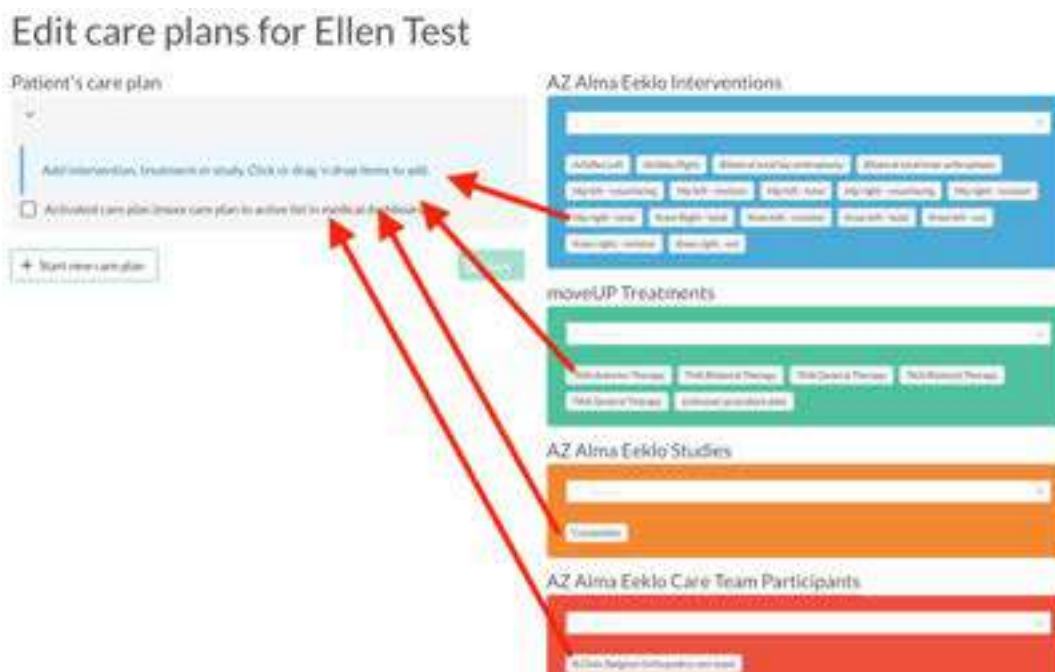- Drag and drop an intervention to the care plan.
  - Fill out the intervention date (e.g.: operation date) and the name of the treating doctor (e.g.: surgeon).

*Drag and drop to create a care plan*

# Edit care plans for Ellen Test

Patient's care plan

Knee Right - total, TKA General Therapy

Intervention: Knee Right - total

Date: dd/mm/yyyy

Doctor

Treatment: TKA General Therapy

Start Date: dd/mm/yyyy

Code

ICD: List Paradigm

Activate care plan (press care plan to activate for medical dashboard)

+ Start new care plan

AZ Alma Eekko Interventions

moveUP Treatments

AZ Alma Eekko studies

*Add type of intervention, date of intervention and doctor*

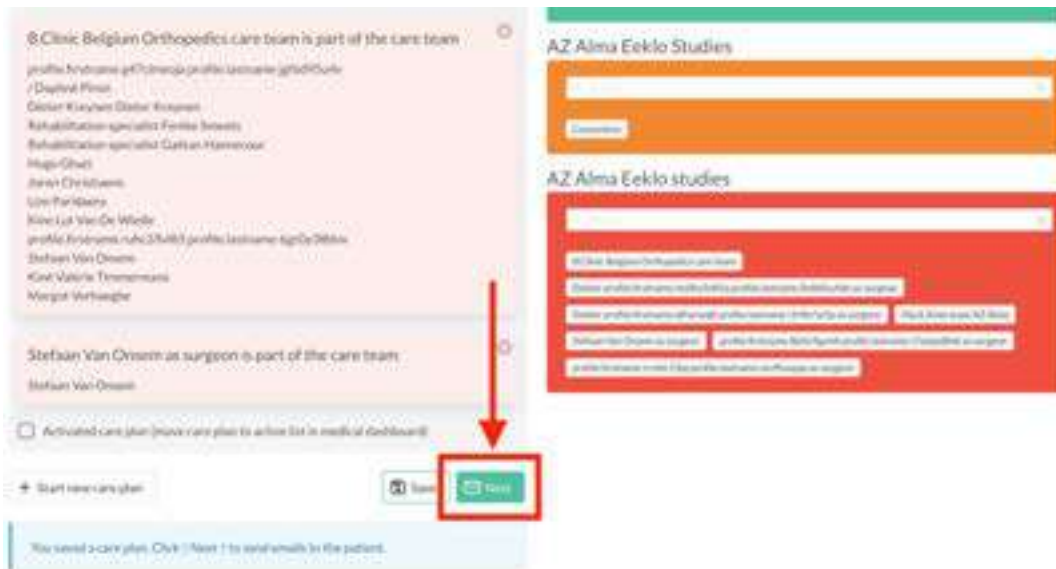
- Drag and drop a moveUP treatment.
  - REMARK: The treatment's start date is set automatically to today's date if you don't fill it out. In most cases, it is not needed to set a date. This is only done in case the intervention or intervention date is not yet known.
- Drag and drop a study if applicable.
- Drag and drop care team(s) or care team participants.
- Click 'save'.

- ✓ Care team participants determine which healthcare providers will have access to the patient's account on the moveUP platform. The type of moveUP healthcare providers accounts determines which data this healthcare provider can access. Someone with a support role (e.g.: moveUP support staff) will not have access to the same data as the treating surgeon for example.

Care team participants are added automatically based on the type of intervention, however it is possible to manually add or remove a healthcare provider from the patient's care team if needed.

## 4. Send an invitation email to patients

- Once a care plan is saved, click on “Next”.


*Click on 'Next'*

- A template of the email(s) that will be sent to the patient is shown.
- Click on “Send selected email(s)” to send the emails to the patient.

*Email template proposal*

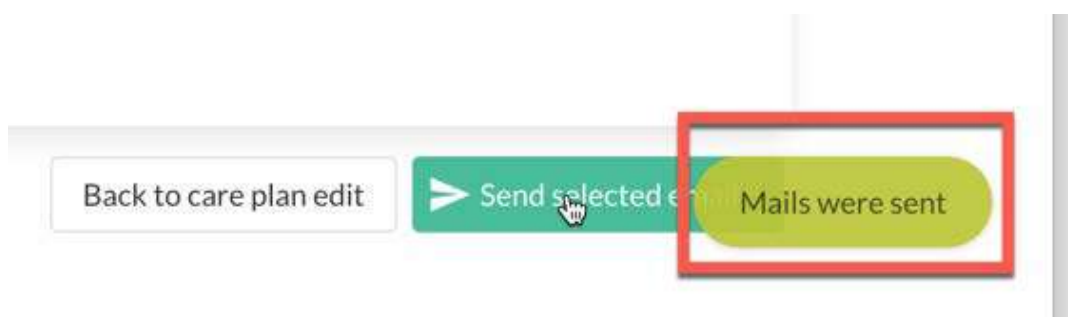- A message “mails were sent” will appear for a few seconds so you know the mails are successfully sent.

*Confirmation mail(s) sent*

## 5. What happens next?

The patient will receive an **invitation email** with

- **Instructions how to install moveUP**– With a short introduction of moveUP, information link on pricing and instructions on how to install moveUP with login details, and a link to the quick start guide.

In case the moveUP treatment of your patient also uses an **activity tracker**, the patient will receive a second email with :

- **Link to order activity tracker** and confirm his/her subscription– With an order form for the activity tracker.

## How to stop the treatment of a patient?

To stop the follow-up of your patient, **there are two mains steps** to follow.

### Table of Contents

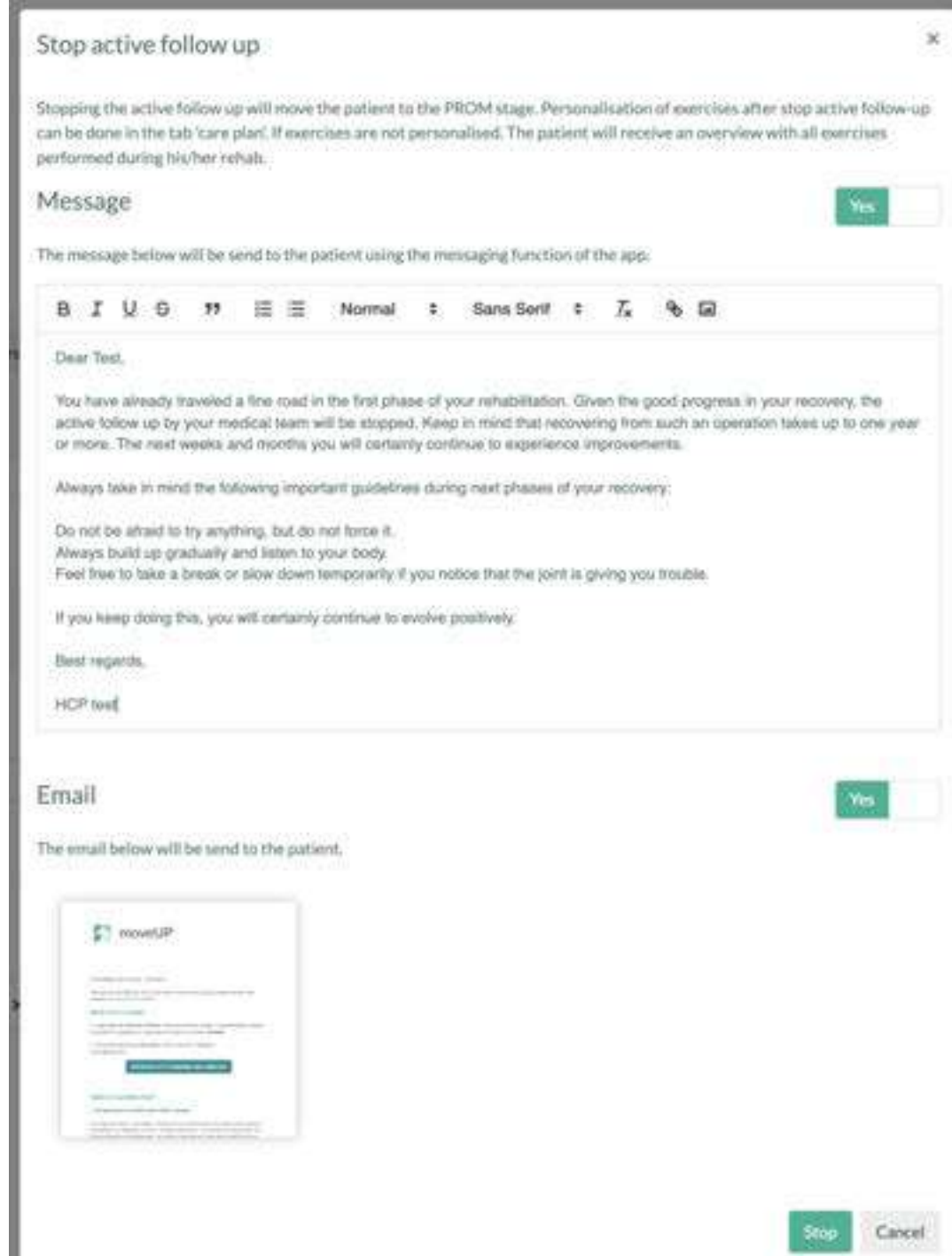
1. Stop active follow-up in care plan tab
2. Fill out the stop form
3. Optional: Adjust exercises for the patient if you want to personalize them
  - Personalize general exercise message for patient
  - Personalize the available exercises to a patient

## 1. Stop active follow-up in care plan tab

1. Go to the Care plan tab
2. Click on “Stop active follow up” in the top right of the Care plan tab.


3. The following pop up will appear.

4. A default chat message will be set on “Yes” and below it you may read the chat message that will be sent to the patient when clicking “stop” at the bottom. You can also edit this message if you want to personalize it. If you don’t want to send a chat message you can toggle this option to “No”.

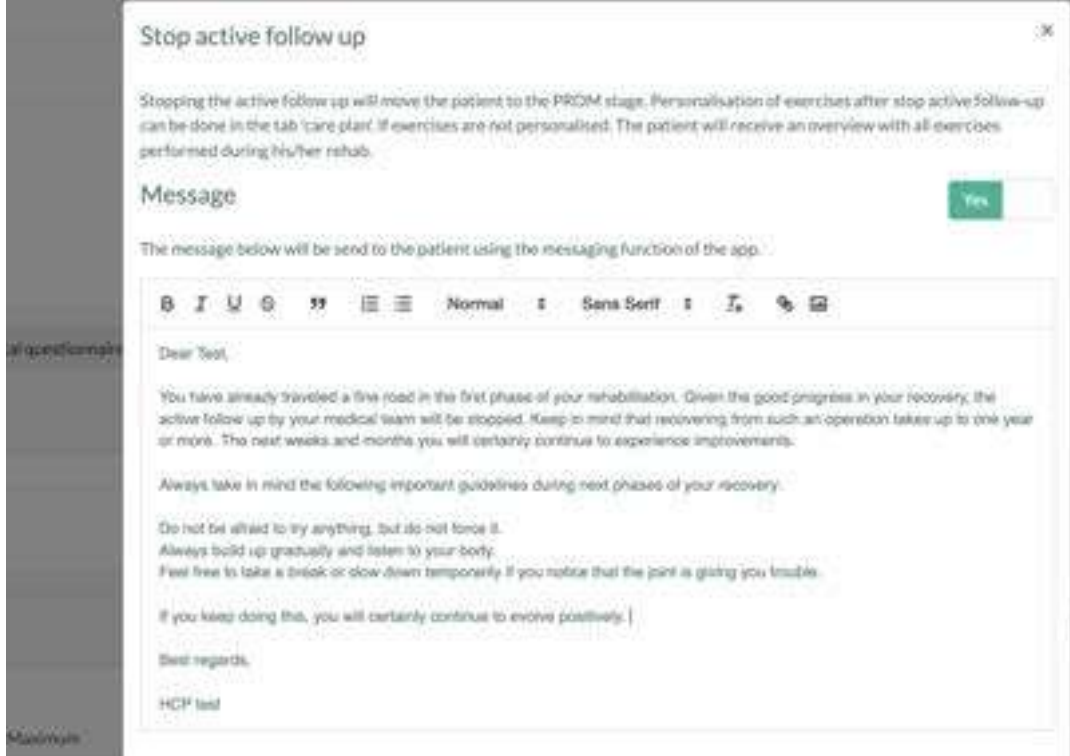 The message is designed for a patient that is doing well at the end of his rehabilitation. In case of opt-out, early end of follow-up, etc., this message really needs to be adapted or toggled off.

5. In addition, a default email with administrative info will be sent to the patient (e.g. how reimbursement and invoicing are arranged, how to send back the activity tracker etc.). If you don't want to send this e-mail, you can toggle it off.

6. If you are ready to stop the active follow-up, you can click on "Stop".

✓ The patient will no longer be listed in the “Therapy” tab. You can now find him in the “PROM” tab or also in the “All” tab.


i In case of opt-out, early end of follow-up or if your patient does not wish to receive any more emails from moveUP, you must complete his follow-up. The patient will appear in the “Completed” tab.

## 2. Fill out the stop form

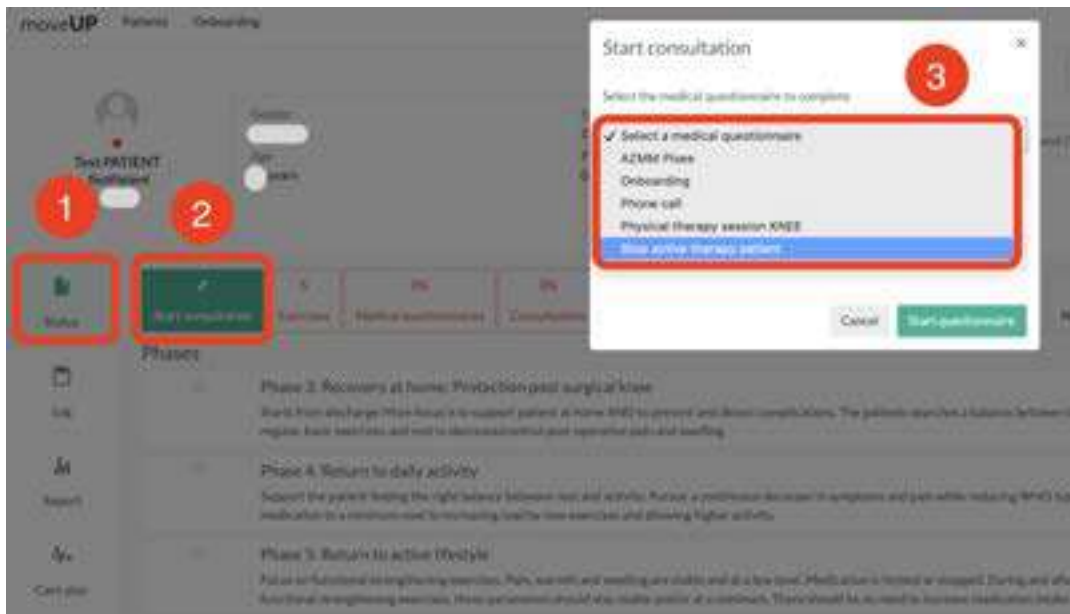To fill out the form:

1. Go to the status tab
2. Click “start consultation”
3. Choose “Stop active therapy patient”

### 3. Optional: Adjust exercises for the patient if you want to personalize them

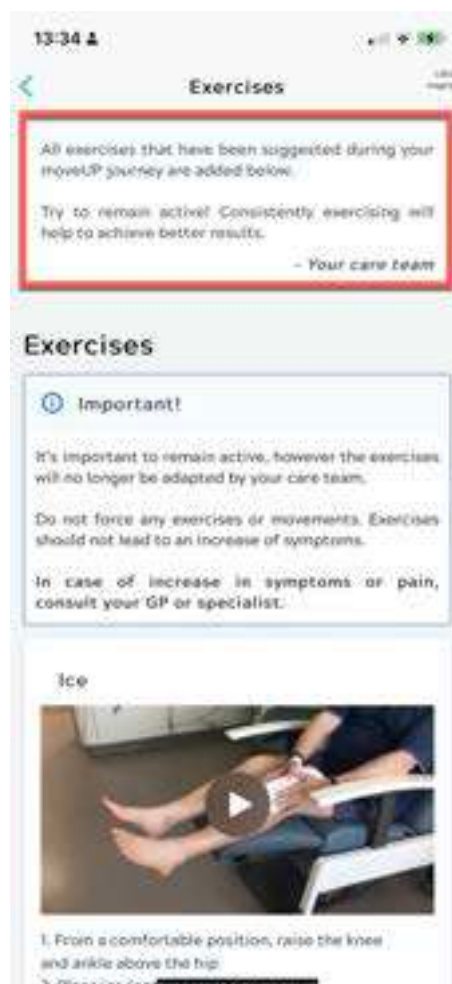
Once the active follow-up has been stopped, we still want the patient to have access to the exercises executed during their rehabilitation. By default, the patient will see an overview of these exercises.

### Personalize general exercise message for patient

If you want to personalize the message the patient sees in his app, you can do this by adding your message in the medical dashboard in the care plan tab.


The patient will see this message like this in the app:

## Personalize the available exercises to a patient

To make sure the patient has the best experience possible, try to personalize which exercises the patient will see (by default it will be all exercises that were active during active follow-up). It is also possible to include a message related to the specific exercises.

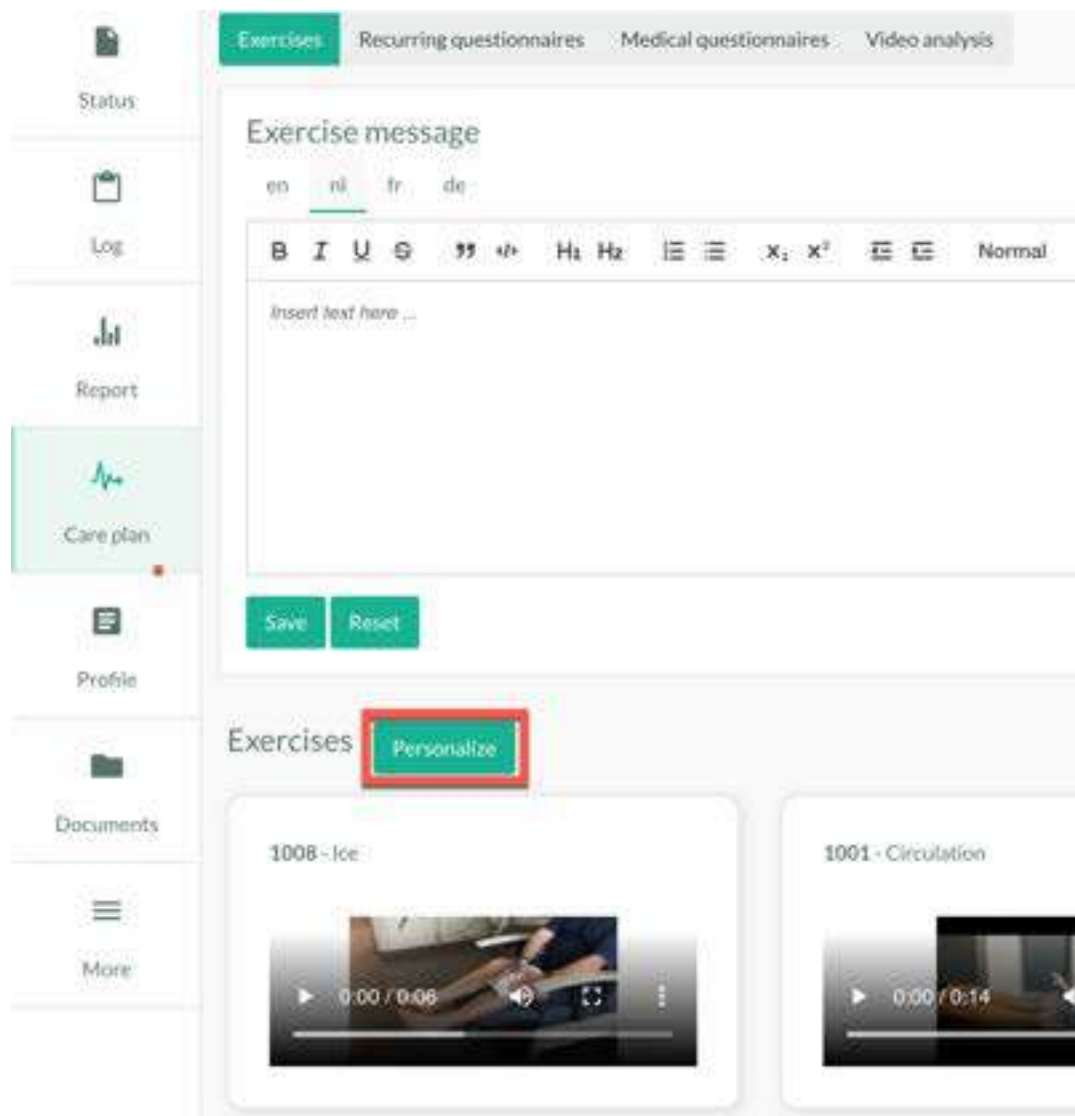To personalize the exercises click “personalize” in the care plan tab.

To change the available exercises you can:

- Add exercises: click on the + (1)
- Remove exercise: click on the x (2)
- Add personal message to an exercise: click edit (3)

In the pop-up menu that appears you can type a personal message that goes with that exercise:

**Add exercise** ✕

Patient app preview

Ice

0:00 / 0:06

1. Froos a comfortable position, raise the knee and ankle above the hip
2. Place ice (not directly on the skin!)
3. Ice is important to reduce swelling, warmth and pain.

Available exercises

1008  
Ice

Message:

**B I U**     Normal 

Sans Serif  

Type a message

**Add** Cancel

The patient will see this message in the exercise overview:

### Drop and dangle

1. Sit high enough so your feet are not touching the ground. Sit for a minute or so before starting the exercise.
2. Gently dangle your leg back and forth within your comfort zone.
3. This exercise should be relatively painless.

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PT [status icons] Surgeon [status icons] Technical [status icons] Jan Coppens

**Elza Van Der Knaap** [status icons] ON

**Patient details**  
 Patient number 1436  
 Gender Female  
 Birthdate 12 Jun 1967 Age 52  
 Head Physical Therapist Roland De Saer  
 Phone Number 00324000000  
 PaymentScheme 3rd Party 25

**Surgery**  
 Lieb Knee right - uni  
 Date 11 Dec 2019 (R)  
 Surgeon Other  
 Hospital GZA

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18/12/19

Type a message [Send] [More]

- Manage treatment 3
- Daily questionnaire ✓
- Medical questionnaires 100 %
- Therapeutical links 11
- Sync activity data
- More =

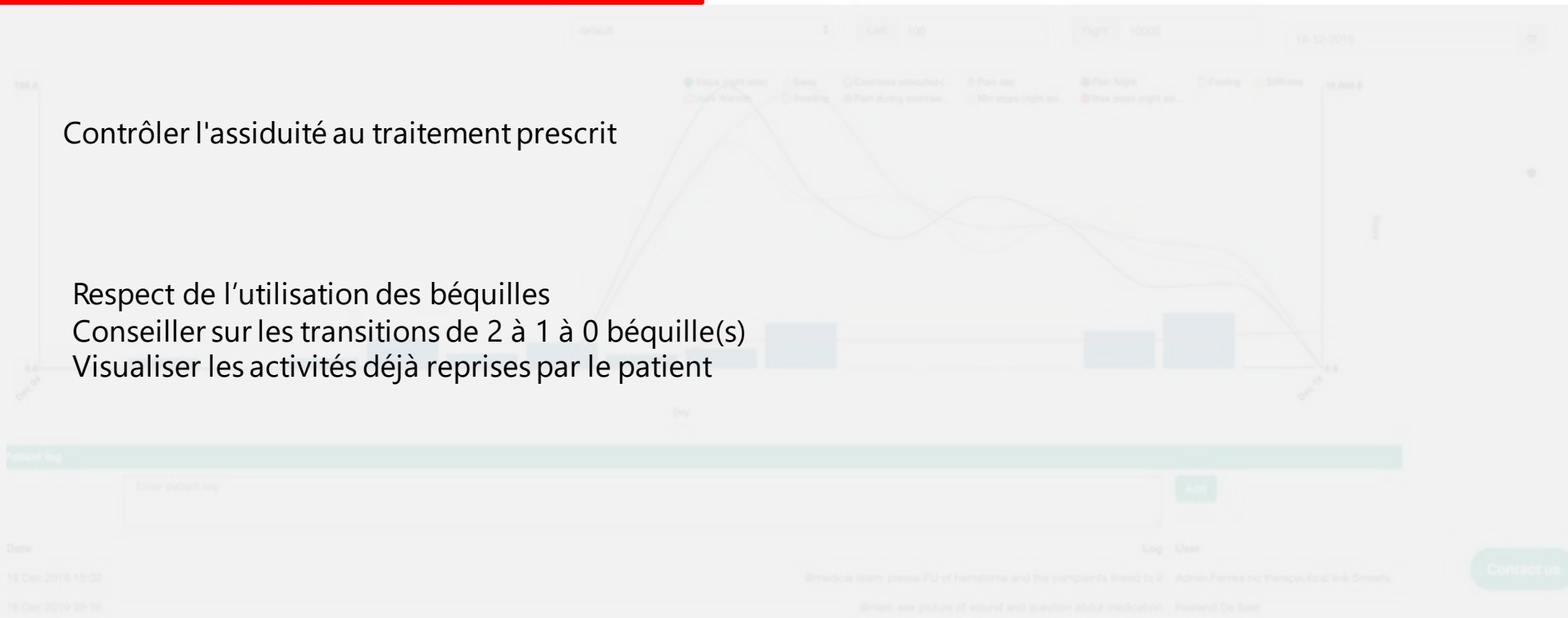
Question	Two days before yesterday	A day before yesterday	Yesterday
Activities	Not filled in	Not filled in	Not filled in
Painkillers	Dafolgen/Paracetamol: 4/day	Dafolgen/Paracetamol: 3/day	Dafolgen/Paracetamol: 3/day
Inflammatories	No	No	No
Crutches	Two crutches	Two crutches	Two crutches

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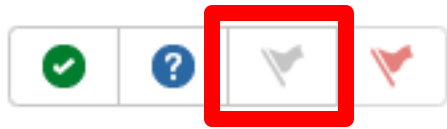
Patient log

Enter patient log

Date	Log	User
31 jan. 2020 10:09	@medical: Please your advice on medication (high pain levels, low intake on PK)	Roeland De Saer
31 jan. 2020 10:08	@medical: See picture of wound	Roeland De Saer

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## Stop active follow up ✕

Stopping the active follow up will move the patient to the PROM stage. Personalisation of exercises after stop active follow up can be done in the tab 'care plan'. If exercises are not personalised, The patient will receive an overview with all exercises performed during his/her rehab.

### Message

The message below will be send to the patient using the messaging function of the app:

**B I U G** **Normal** **Sans Serif**

Dear Test,

You have already traveled a fine road in the first phase of your rehabilitation. Given the good progress in your recovery, the active follow up by your medical team will be stopped. Keep in mind that recovering from such an operation takes up to one year or more. The next weeks and months you will certainly continue to experience improvements.

Always take in mind the following important guidelines during next phases of your recovery:

Do not be afraid to try anything, but do not force it.  
Always build up gradually and listen to your body.  
Feel free to take a break or slow down temporarily if you notice that the joint is giving you trouble.

If you keep doing this, you will certainly continue to evolve positively.

Best regards,

HCP test

### Email

The email below will be send to the patient.

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