



750 Garcia Ave, Pittsburg  
CA 94565  
+1.925.427.7800  
[HSIheadwalls.com](http://HSIheadwalls.com)

## Patient Service Console Compact Wall Unit

100% USA Manufactured

Union Made

12 Year Product Warranty

All specifications are nominal and subject to change without notice. Hospital Systems, Inc. makes no warranty, expressed or implied, except for the express written warranty extended on the sale of its products.

CONSOLE basic unit shall be manufactured by Hospital Systems, Inc., Pittsburg CA 94565 USA, in accordance with shop drawings and documents. The following is a general specification, and components listed may not be present or required in final product.

The CONSOLE is listed by Underwriters Laboratories Inc. (UL) in Canada and the United States of America. This product also complies with the seismic requirements of the State of California's Office of Statewide Health Planning and Development (OSPHD). HSI manufactures all products in accordance with the National Fire Protection Association (NFPA), NFPA-99 (current edition) and National Electric Code (NEC).

The installation contractor is responsible for compliance with all local, state, and federal codes applicable to the installation of medical gas and electrical systems.

### 1. Submittals and Approvals:

Job specific shop drawings shall be produced for each project. These shop drawings will clearly indicate

the area of Medical Gas termination, and electrical connection points inside or outside the CONSOLE. Drawings of Record will be produced, and emailed at the time of shipment, on request.

HSI will manufacture equipment as per signed approval drawings and verifications documents, which are provided in the job specific submittal package.



### 2. Basic Construction

Backbox shall be a code gauge, formed aluminum trough with a combination of 1/2" – 3/4" inch combination knockouts on top and bottom of the backbox. A preformed aluminum end cap shall be permanently attached to the backbox. An extruded anodized aluminum fascia panel shall be field attached to the backbox assembly by the installing contractor using screws provided by HSI. The fascia includes a high-pressure plastic laminate insert available in a variety of colors and patterns. Electrical conduit runs to the knockouts and wiring within the console shall be the responsibility of the electrical contractor. Medical gas outlets shall



be installed in the consoles by the mechanical contractor. Individual fixed barriers in the backbox shall be pre-installed by HSI at the factory to separate different voltages and mechanical services.

### 3. Components

#### A. Electrical

##### ELECTRICAL RECEPTACLES

Receptacles shall be the type and quantity as shown on the drawings and specified. Unless otherwise noted 2 pole, 3 wire, rated at 20 amps, 120 Volt, Hospital Grade shall be provided. Also available: BS, DIN, and others.

- **Duplex receptacles** shall be NEMA style 5-15R or 5-20R, color Ivory for use on normal power circuits, and color Red for use on critical branch power circuits.
- **Simplex receptacles** shall be NEMA style 5-15R or 5-20R, color Ivory for use on normal power circuits, and color Red for use on critical branch power circuits.
- **Safety receptacles** (if required) shall be duplex type, be NEMA style 5-15R or 5-20R, color Ivory for use on normal power circuits, and color Red for use on critical branch power circuits. Receptacles shall limit proper access to energized contacts.

#### SWITCHING

Switches shall be Industrial Grade 120/277

volt, 20 amps. Switch type options include SPST, 3-Way or Momentary, and shall be provided as shown on submittal shop drawings. Low Voltage Switching will be 0-12 volts, 15 amps unless otherwise noted. HSI shall furnish, pre-install and wire all switches.

#### B. Med Gas

##### MEDICAL GAS PIPING

Oxygen, vacuum, medical air, evacuation, and nitrous oxide outlets shall be as indicated in plans and specifications. All outlets when provided by HSI will be shipped loose. The mechanical contractor will install and bring the primary supply lines of the medical gases to the terminal connections as shown on the drawings. In situ, all Medical Gas Outlets and piping shall be brazed and tested in accordance with NFPA 99c.

#### C. Provisions

##### NURSE CALL EQUIPMENT

All nurse call equipment shall be furnished and installed by others as indicated in the plans and specifications. A labeled pull-cord shall be provided by HSI for use by the nurse call installer.

##### MONITORING EQUIPMENT

Monitor equipment shall be provided and installed by others. HSI shall provide a matching blank plate for the monitor



750 Garcia Ave,  
Pittsburg  
CA 94565  
+1.925.427.7800





750 Garcia Ave,  
Pittsburg  
CA 94565  
+1.925.427.7800



receptacle to the monitor supplier when requested. The CONSOLE wall unit will include a factory installed raceway and labeled pull cord to accommodate the monitor wiring. Where specified a monitor slide and optional pivot/tilt arm shall be provided to support the physiological monitor.

#### **COMPUTER INTERCONNECT**

A computer system furnished by others shall be installed at locations as shown on the plans and specifications. HSI shall provide a matching blank plate for the computer jack when requested. The CONSOLE unit will include a factory installed raceway and labeled pull cord to accommodate the computer system wiring.

#### **4. Installation**

HSI shall provide ceiling mounting plate (for units with vertical services chases) and rail mounting bracket. The installer shall mount the headwall units and make all primary medical gas and electrical connections, as well as connections between the rails and the chase. The installer or contractor shall make connection of building services to pre-wired terminal enclosure as shown on the electrical drawings and as herein specified. Customer shall coordinate with HSI for equipment supplied by others to ensure compatibility of this equipment with the headwall. The installer or contractor shall

provide primary connections to the medical gas system in the access area of headwall unit as detailed on drawings. It shall be the responsibility of the mechanical contractor to perform and certify all pressure tests as required by NFPA 99c, and to install the gas outlet finishing assemblies. Please refer to the installation instructions for further details.

#### **5. Ordering Information**

All HSI headwalls are Build-to-Order and will be assigned their own unique model numbers during the submittal phase.

Please visit our website at [www.HSIheadwalls.com](http://www.HSIheadwalls.com) for the contact information of your local sales representative